

Measure your child	Choose the appropriate seat size	Consider your child's strength			Choose gas strut kit
Back height		Able to activate the dynamic movement	Too easy to activate the dynamic movement	Can't activate the dynamic movement	Weak = 50 N (8911050) Standard = 100 N (8911100)
Back height = 27-37 cm	x:panda 1 (891011x-01)	Standard gas strut = 100 N is sufficient	Choose a stronger gas strut = 150 N	Choose a weaker gas strut = 50 N	Strong = 150 N (8911125)
Measure your child	Choose the appropriate seat size	Consider your child's strength			Choose gas strut kit
Back height				1	Weak = 50 N (8911050)
	1000	Able to activate the dynamic movement	Too easy to activate the dynamic movement	Can't activate the dynamic movement	Standard = 100 N (8911100)
Back height = 32-42 cm	x:panda 2 (891012x-01)	Standard gas strut = 100 N is sufficient	Choose a stronger gas strut = 150 N	Choose a weaker gas strut = 50 N	Strong = 150 N (8911125)
Measure vous child	Chance the appropriate cost size	Consider your child's strength			Chanca das strut kit
Measure your child	Choose the appropriate seat size	Consider your child's strength			Choose gas strut kit
Measure your child Back height	Choose the appropriate seat size	Consider your child's strength			1
	Choose the appropriate seat size				Weak = 150 N (8911150)
Back height		Able to activate the dynamic movement	Too easy to activate the dynamic movement	Much too easy to activate the dynamic movement	Weak = 150 N (8911150) Standard = 250 N (8911250)
	Choose the appropriate seat size X:panda 3 (891013x-01)		Too easy to activate the dynamic movement Choose a stronger gas strut = 250 N	Much too easy to activate the dynamic movement Choose an even stronger gas strut = 300 N	Weak = 150 N (8911150)
Back height		Able to activate the dynamic movement			Weak = 150 N (8911150) Standard = 250 N (8911250)
Back height Back height = 35-47 cm	X:panda 3 (891013x-01)	Able to activate the dynamic movement Standard gas strut = 150 N is sufficient			Weak = 150 N (8911150) Standard = 250 N (8911250) Strong = 300 N (8911300) Choose gas strut kit
Back height Back height = 35-47 cm Measure your child	X:panda 3 (891013x-01)	Able to activate the dynamic movement Standard gas strut = 150 N is sufficient			Weak = 150 N (8911150) Standard = 250 N (8911250) Strong = 300 N (8911300)