740/840 mm (28<sup>3</sup>/<sub>4</sub>"/32<sup>3</sup>/<sub>4</sub>"\*)

<50 kg (110 lb)

## Stingray frame User Guide

EN 12182    EN1021-1  
EN 12183    EN1021-2  
ISO 7176-19

2015.09-rev.03

	3		21
	4 - 5		24
	6	<b>mm (inch)</b>	23
	7		24
	8	ENGLISH	25 - 29
	9	SVENSKA	50 - 54
	10	SUOMI	55 - 59
	11		
	12		
	13		
 Stingray	14 - 15		
 x:panda Panda Futura	16 - 17		
 Alternative 1	18 - 19		
 Alternative 2	20		

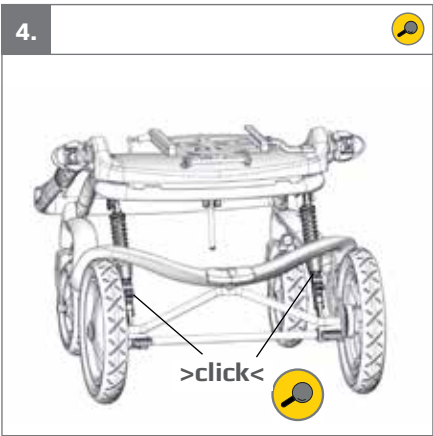
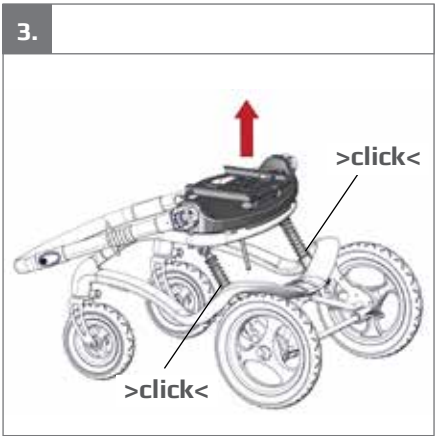
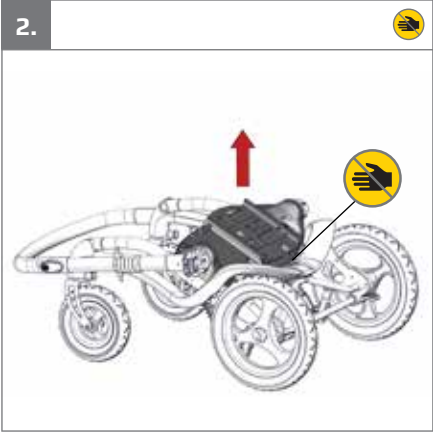
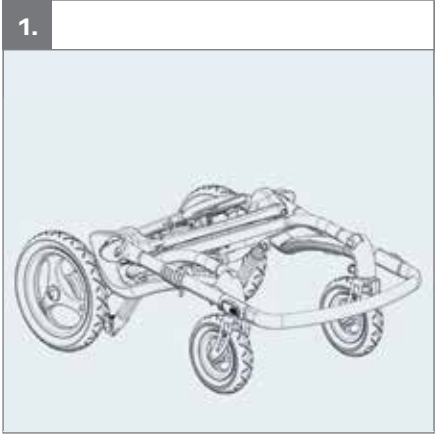
### Stingray 1


### Stingray 2

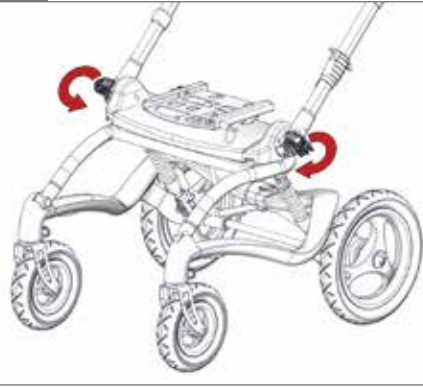
7.

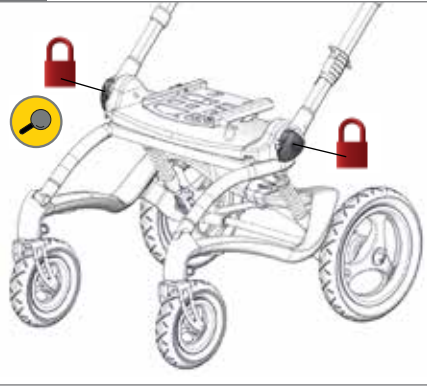8.

9.

10.

11.

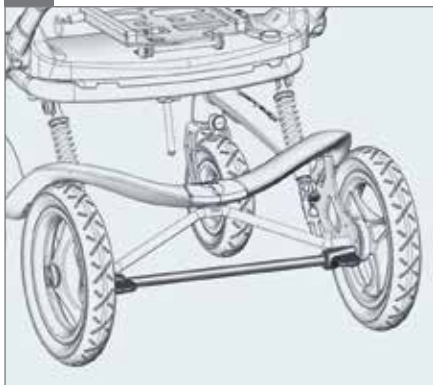
12.



1.

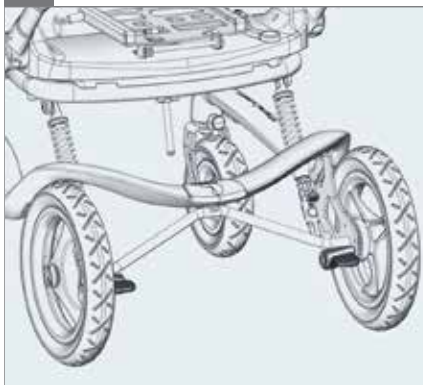
2.

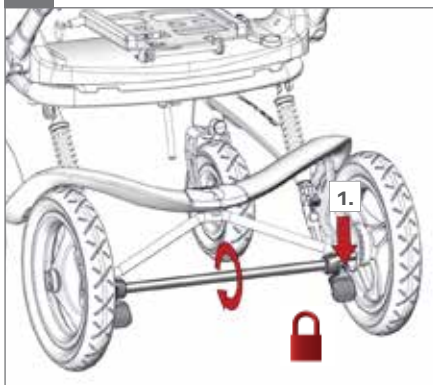
3.

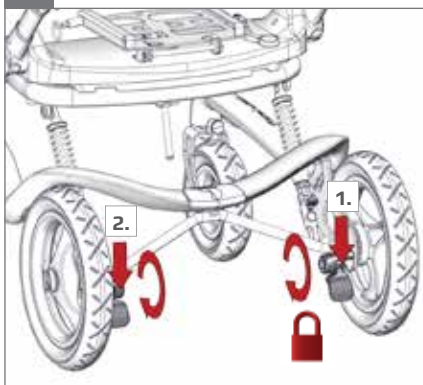
4.

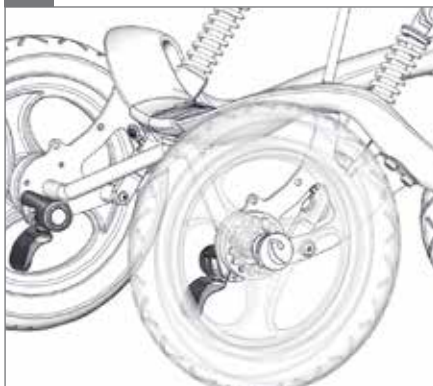5.

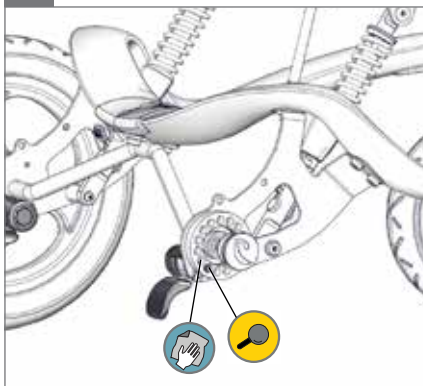6.


1.


2.

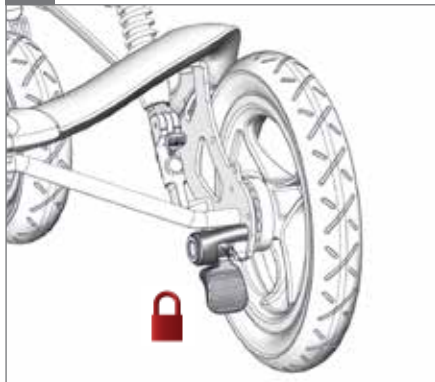3.

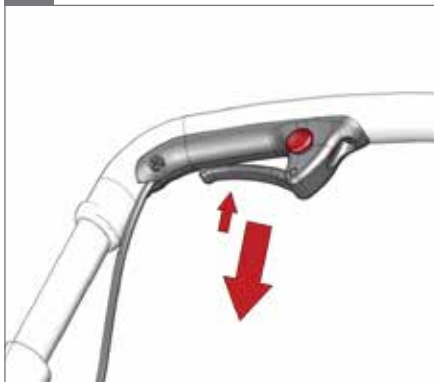
4.

5.

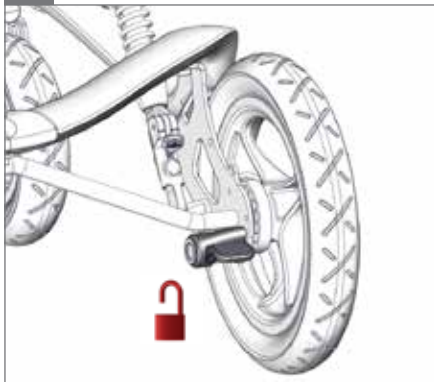6.


1.

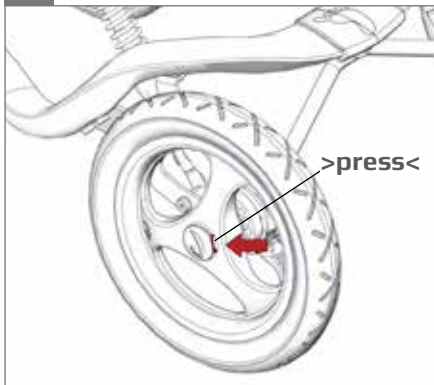
2.


3.

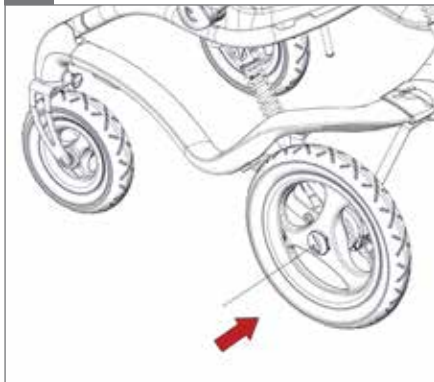4.

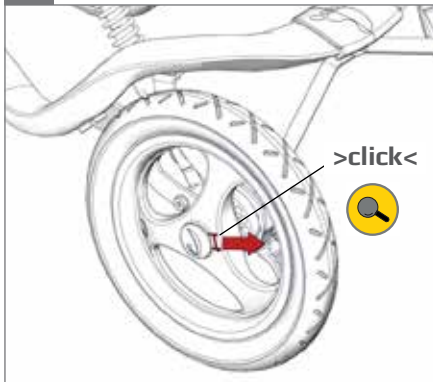5.


1.

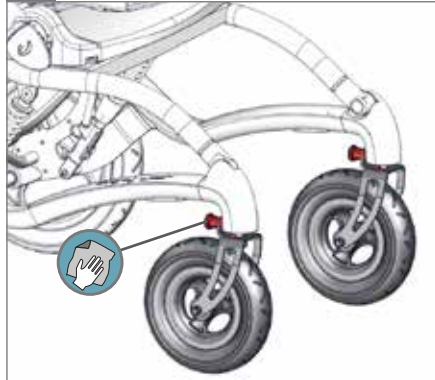2.

3.

4.

5.

6.

1.

2.

3.


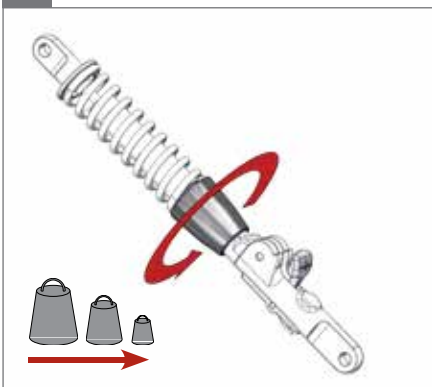
1.


2.

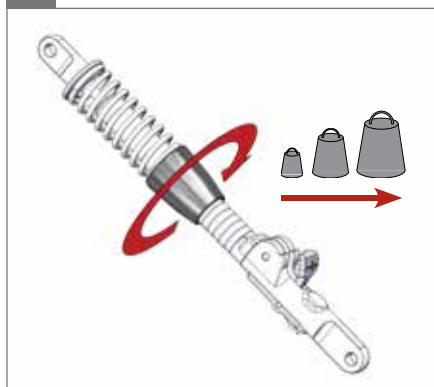
3.

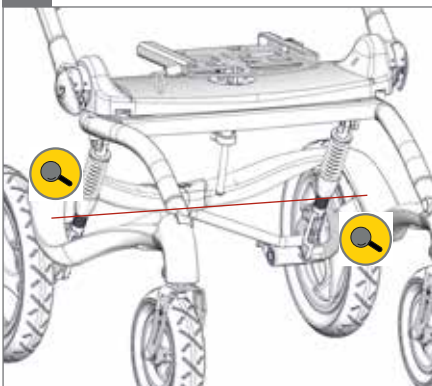4.

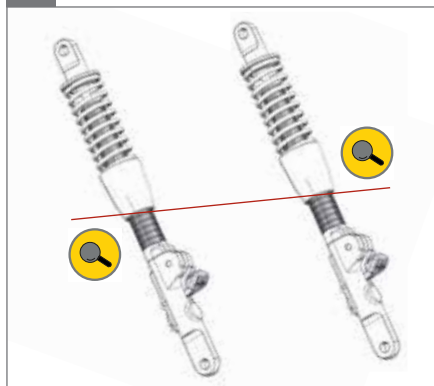5.

1.

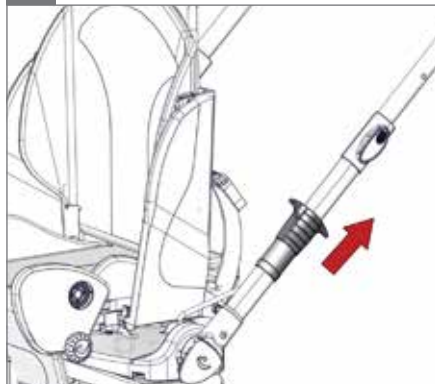
2.

3.

4.

5.

6.

1.

2.

3.

4.

5.

6.


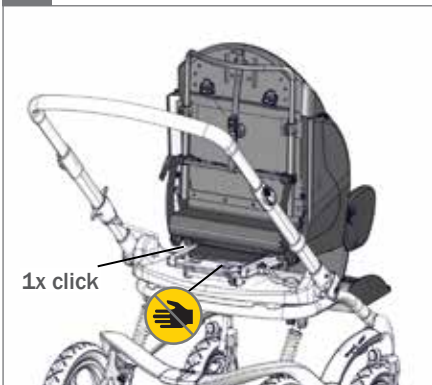
1.

 Stingray


2.

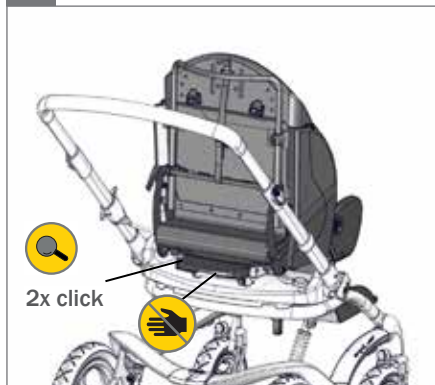3.

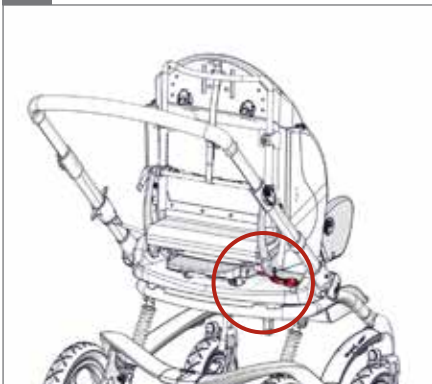4.

5.

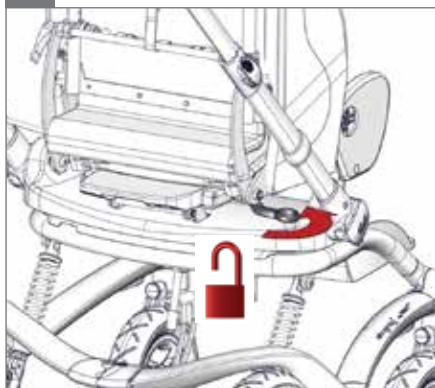
1.

 Stingray


2.

3.

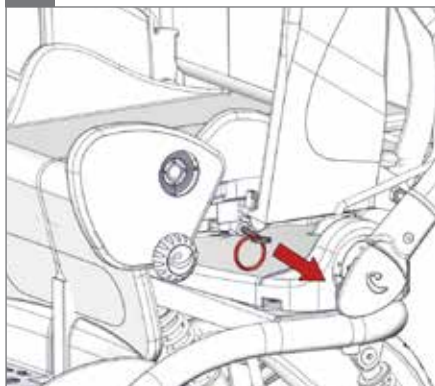4.

5.

1.

x:panda &amp; Panda Futura

Panda Futura 1 ✓

Panda Futura 2 ✓

Panda Futura 2½ ✓

x:panda 1S ✓

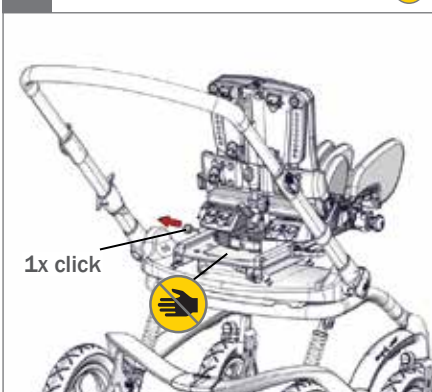
x:panda 1M ✓

x:panda 1L ✓

x:panda 2S ✓


x:panda 2M ✓

2.


1x click

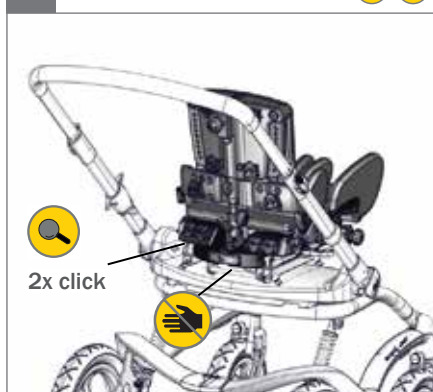3.


2x click

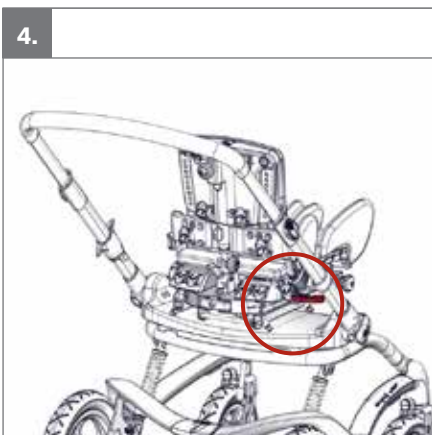4.

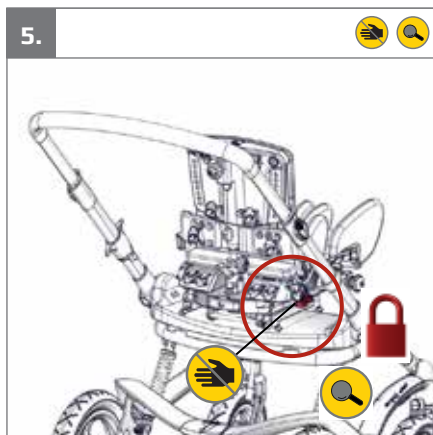5.


1.

 x:panda & Panda Futura

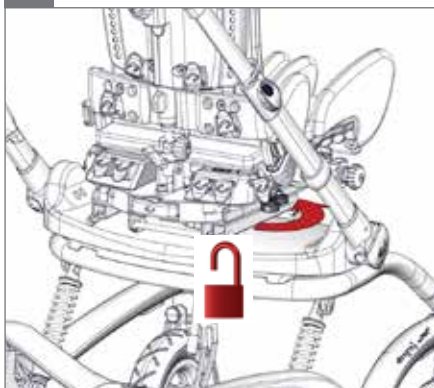
2.

3.

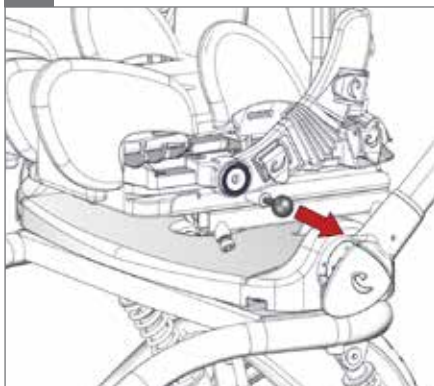4.


5.

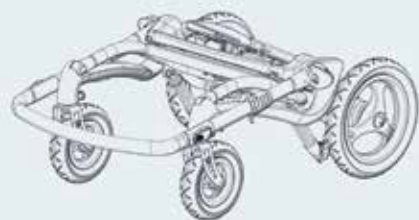1.

Alternative 1 - 

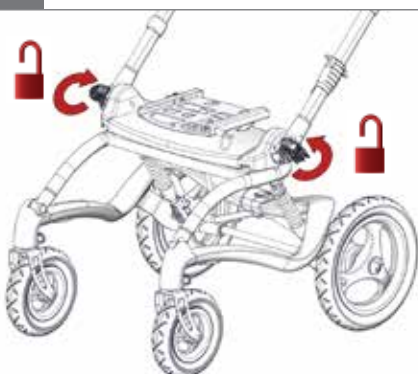
2.

3.

4.

5.

6.


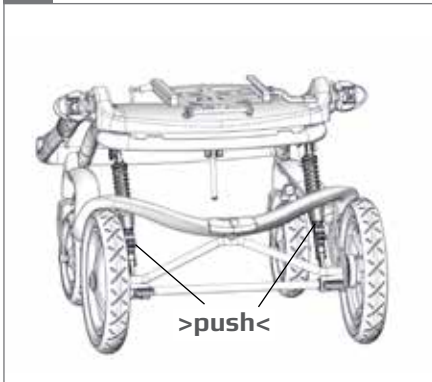
7.


8.

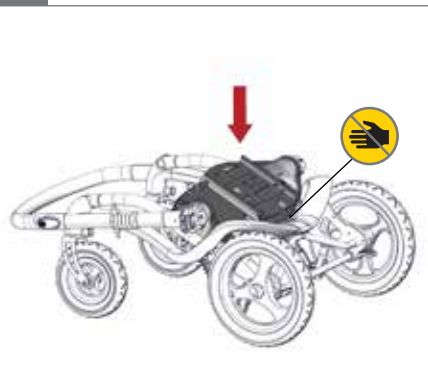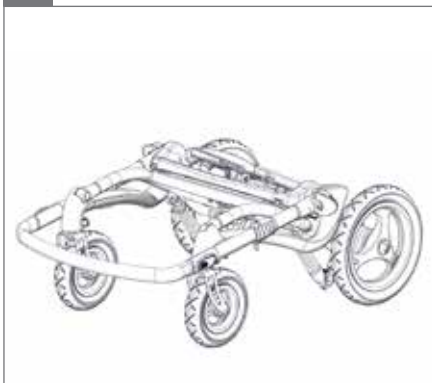9.

10.

1.

Alternative 2 -  

2.

3.

4.

5.

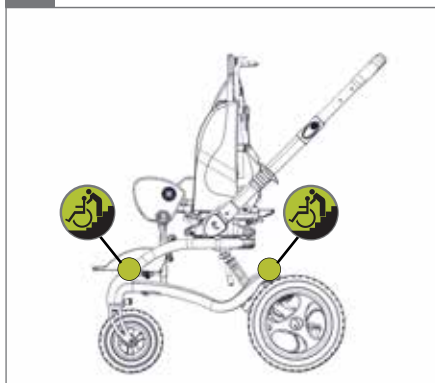
6.


1.

2.

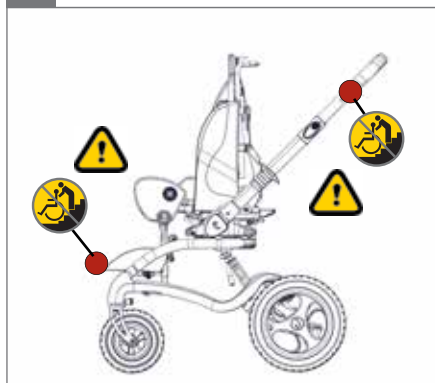3.

4.

5.

A

Date: YYYY-MM-DD Max.Load: XX KG  
 SN: XXXXXX Ver.: XX  
 Art.No: XXXXXX  
 Product - Size X

(01)0570729530134(21)256829

B

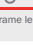
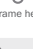
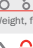
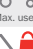
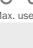
**R82** 

Parallelsvej 3  
 DK-8751 Gedved

www.R82.com  
 Made in Denmark

## mm (inch)

	 1	 2
 Frame width	590 (23")	590 (23")
 Frame length	740 (28¾") 840 (32¾")	740 (28¾") 840 (32¾")
 Frame height	1000,1030,1060 (39",40",41¼")	1000,1030,1060 (39",40",41¼")
 Lower leg length	140-450 (5½-17½")	140-450 (5½-17½")
 Leg rest angling	-30° -> 30°	-30° -> 30°
 Seat height above floor	520 (20¼")	520 (20¼")
 Tilt in space	45°	45°
 Turning radius	1020 (39¾")	1020 (39¾")
 Frame length folded	900 (35")	900 (35")
 Frame height folded	490 (19")	490 (19")
 Weight, frame	13.5 kg (29.5 lb)	13.5 kg (29.5 lb)
 Max. load, frame	50 kg (110 lb)	50 kg (110 lb)
 Max. user weight/load, seat + frame	40 kg (88 lb)	40 kg (88 lb)
 Max. user weight/load, transportation	40 kg (88 lb)	40 kg (88 lb)
 10°	 10°	 10°

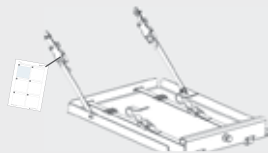
9996097176

9996097178

9996097181

9996097186

9996097187

## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this manual before use and save it for further reference.

### Intended use

The Stingray has been designed for children who need good comfort and support. The seat depth and back height are easy to adjust, making it possible to expand the seat as when the child grows. The Stingray is suitable for users needing practical outdoor transportation, e.g. children with CP, GMFCS level 3-5. The Stingray comes in 2 sizes.

### Tools

Enclosed tools; 3, 4 & 5 mm Allen keys.

### Declaration of Conformity -

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to ISO 7176/19, EN12182, EN12183, EN1021-1 and EN1021-2

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or daily maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the manual. For further information, we refer to the R82 homepage/download.

The warranty can only be sustained if the R82 product is in use in the same country where it was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if non-original R82 parts/accessories are used, or if the product has not been repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the instructions for use (IFU). R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. The warranty is given by R82 or, subsequently, an R82 dealer.

## ENGLISH

## Symbols

For indoor and outdoor use  
Never leave the user unattended in this product

Consult instructions  
The latest versions of the instructions are always available on the R82 website and can be printed in larger sizes

Caution  
Incorrect use of the product may cause serious damage to the product and the environment and may cause injury

Caution  
Be careful not to get your fingers caught

Correct lifting points for safe moving and handling

Caution  
Do not pull/lift the chair by the push brace/handles, especially on stairs

Caution  
Do not use the foot support for standing transfers

Inspection  
Carry out inspections frequently

Cleaning  
Keep clean to maintain optimum function

## Disposal -

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

## ENGLISH

**Safety -** 

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

---

**Caregiver**

---

Read all of the instructions carefully before use and keep them for future reference. Incorrect use of this product may cause serious injury to the user

---

Always use the correct lifting techniques and aids for this purpose

---

Never leave the user unattended in this product. Ensure permanent supervision by an adult

---

Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier

---

If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

---

**Environment**

---

For the purpose of safety and comfort, always move the product away from the sun and allow to cool before use

---

Be aware of reduced maneuverability on slippery roads

---

**User**

---

If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

---

**Product**

---

Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children

---

Activate the brakes before placing the user into the product

---

Ensure the stability of the product is maintained before placing the user in it

---

We recommend you notify the user before angling the seat and back

---

Ensure the wheels are fully-functional and secure before use

---

Inspect the product and all of its accessories and replace any worn out parts before use

---

## ENGLISH

**Maintenance responsibilities table**

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1st Day	Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference
Daily	Store the enclosed tools on/at the product
	Use a dry cloth to clean the product
	Visual inspection. Repair or replace any damaged or worn-out parts
	Ensure that all fasteners are in place and fitted correctly
Weekly	Check all fittings and buckles for signs of worn out parts
	Be aware of any max mark indications before adjustment
	Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
	Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent and dry before use
Monthly	All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures
	Oil the swing-away parts. We recommend you use a professional lubricant system, e.g. Veidec Dry Lube
Annually	Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty

**Accessories and spare parts**

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

**Service interval**

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is issued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

Contact your local dealer for instructions on repairs e.g. dealing with tyre punctures

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

## ENGLISH

**Transport in motor vehicles**

This product is dynamically tested according to ISO 7176-19.

**Warnings** - 

The following instructions must be followed to maintain the safety

The approval is not valid on 'custom made' chairs

This product is approved for transporting the user in motor vehicles. Please look at MEASUREMENTS in the back of this manual for the max load indication

The user should transfer to the vehicle seat and use the vehicle-installed restraint system whenever possible

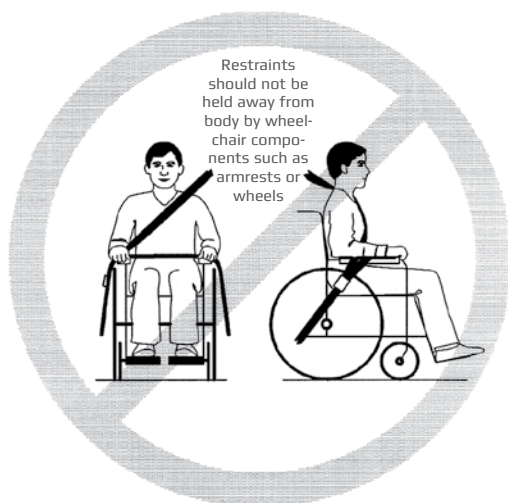
The product must be placed in a forward-facing position, when used as a seat in a motor vehicle. Use an approved 4-point strap-type tiedown system according to ISO 10542-2

Use an approved 3-point belt according to ISO 10542-1 in the wheelchair. Restraints should not be held away from the body by wheelchair components such as armrests or wheels. It is not enough ONLY to use a hipbelt

All accessories must be removed from the wheelchair and secured separately in the vehicle. Accessories which can not be removed, must be secured to the wheelchair but positioned away from the user with energy-absorbing padding placed between the accessories and the user

The product should be inspected by the dealer before reuse following involvement in any type of vehicle collision

The instructions on how to prepare the seat and chair, must be carried through before transportation

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Supplier for:  
Rest of the world

#### R82 UK Ltd.

Unit D4A,  
Coombswood Business Park East  
Coombswood Way,  
Halesowen  
West Midlands B62 8BH  
England  
Tel. +44 (0)121 561 2222  
Fax +44 (0)121 559 5437  
R82uk@R82.com  
R82.uk  
facebook.com/R82UK

Supplier for:  
United Kingdom, Ireland

#### R82 GmbH

An den Wiesen 10  
55218 Ingelheim  
Germany  
Tel +49 (0)6132 710 70  
Fax +49 (0)6132 710 710  
R82de@R82.com  
R82.de  
facebook.com/R82.de

Supplier for:  
Germany, Switzerland, Austria

#### R82 Nederland B.V.

Tinweg 8  
8445 PD Heerenveen  
The Netherlands  
Tel +31 (0)513 68 16 86  
Fax +31 (0)513 68 19 84  
R82nl@R82.com  
R82.nl  
R82.be

Supplier for:  
The Netherlands, Belgium

#### R82 CZ, s.r.o.

Bohunicka 52  
CZ-619 00 Brno  
Czech Republic  
Tel: +420 777 969 999  
R82cz@R82.com  
R82.cz  
facebook.com/R82.cz

Supplier for:  
Czech Republic, Slovakia

#### Snug Seat Inc.

12801 E. Independence Blvd.  
P. O. Box 1739  
Matthews NC 28106  
USA  
Tel +1 800 336 7684  
Fax +1 704 882 0751  
Information@SnugSeat.com  
SnugSeat.com  
facebook.com/SnugSeatInc

Supplier for:  
USA, Canada

#### R82 au Pty Limited

Unit 1, Level 2, 13 Lord Street  
Botany NSW 2019  
Australia  
Tel +61 (02) 8213 6666  
Fax +61 (02) 8213 6655  
R82.com.au  
facebook.com/R82Australia

Supplier for:  
Australia

#### R82 China 中国

Room 920 Tower 2  
China Central Place  
79 Jianguo Road  
Chaoyang District  
Beijing 100025, China  
Tel +86 10 5920 4399  
Fax +86 10 5920 4222  
R82cn@R82.com  
R82.com.cn

Supplier for:  
China

Find your local dealer

**R82.com**

DISTRIBUTOR: