180-400 mm (7-15½")

<40 kg (88 lb)

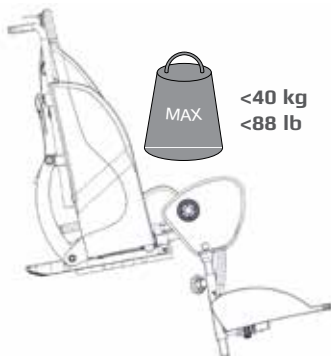
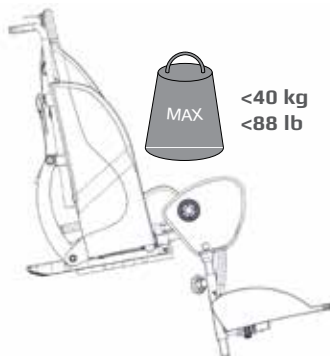
Stingray seat User Guide

EN 12182 EN1021-1
EN 12183 EN1021-2
ISO 7176-19


2015.09-rev.03

	3		19
	4		20
	5	mm (inch)	21
	6 - 7		22 - 23
	8	ENGLISH	25 - 29
	9	SVENSKA	50 - 54
	10 - 11	SUOMI	55 - 59
	12		
	13		
	14 - 15		
	16		
	17		
	18		


**Stingray 1****Stingray 2**

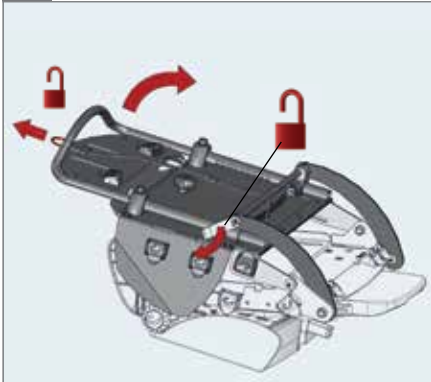
1.

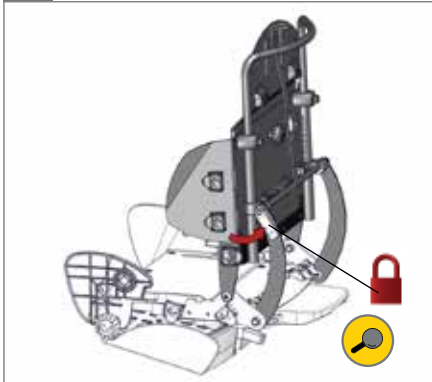
2.

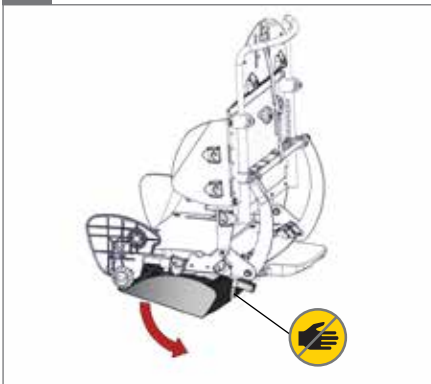
3.

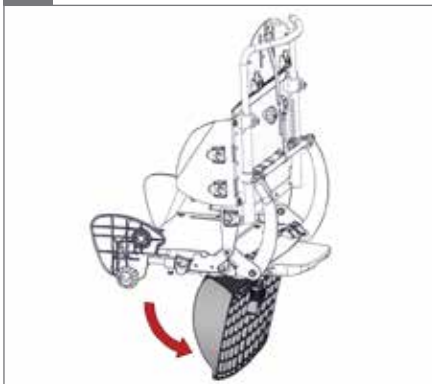
4.

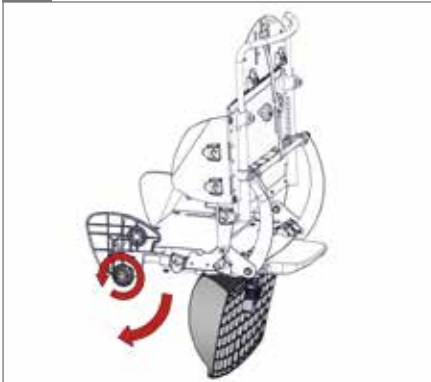5.

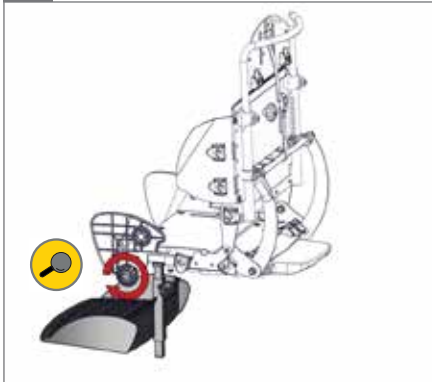6.

1.

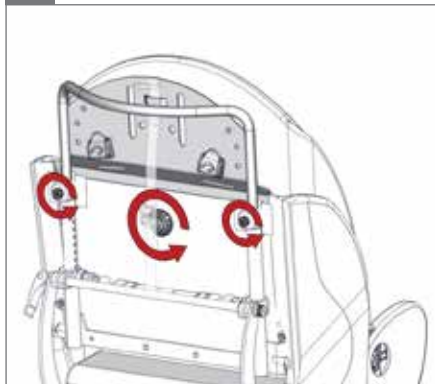
2.

3.

⊙ 5 mm


4.

5.

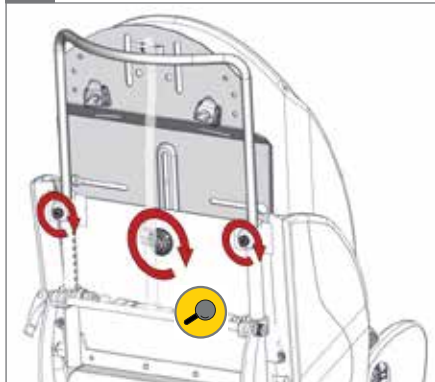⊙ 5 mm - 2,5 Nm

6.

1.

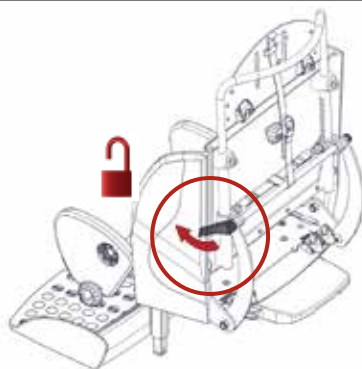
2.

3.


4.

5.

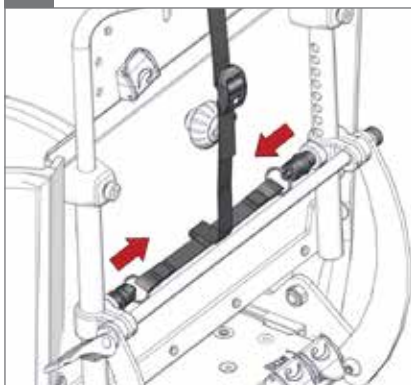
6.

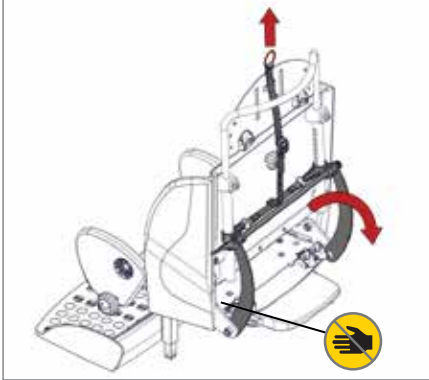
1.

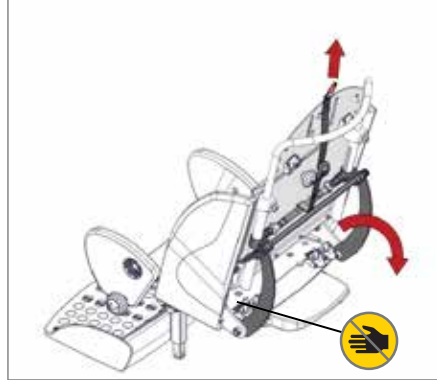
2.

3.

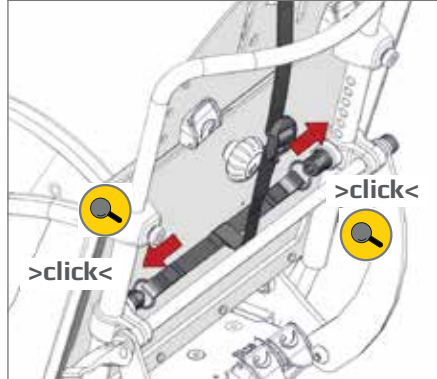
4.

5.

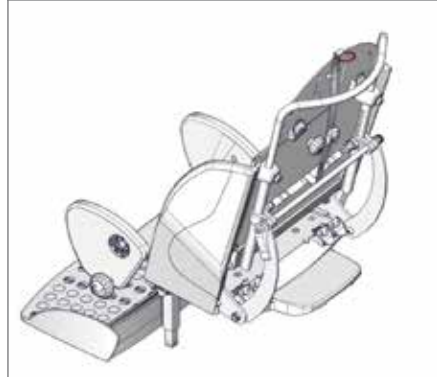
6.

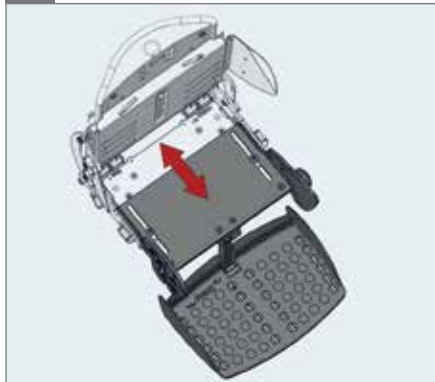1.


2.

⬡ 5 mm

3.

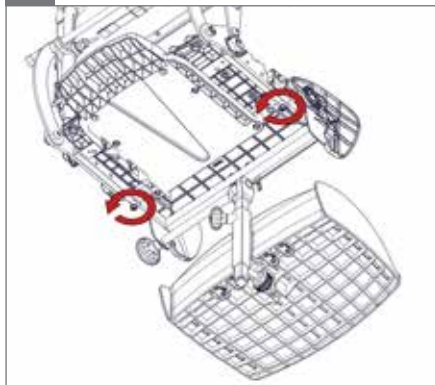⬡ 5 mm

4.

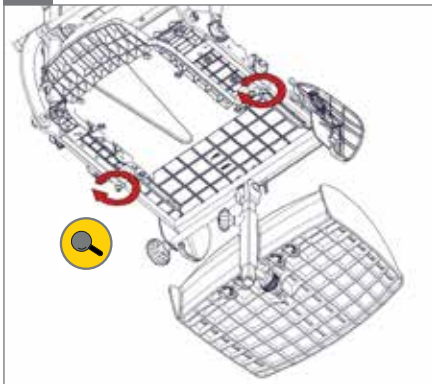
5.

⬡ 5 mm - 2,5 Nm 🔍

6.

⬡ 5 mm - 2,5 Nm 🔍



1.

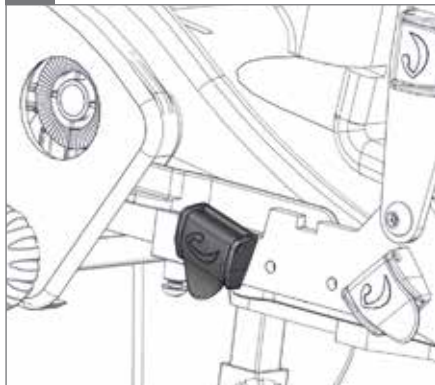
2.

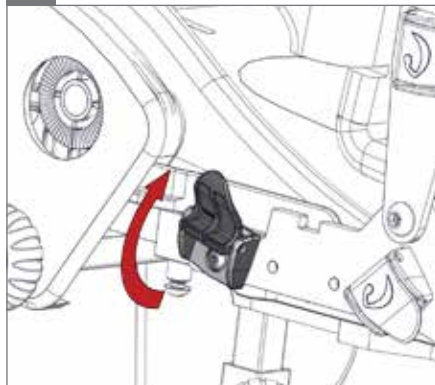
3.

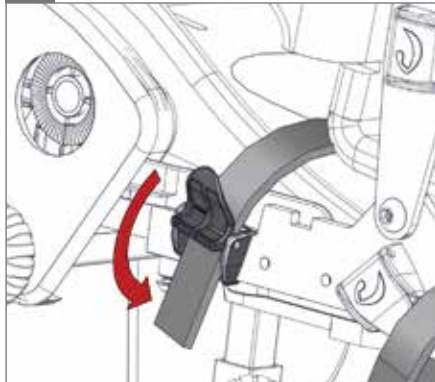4.

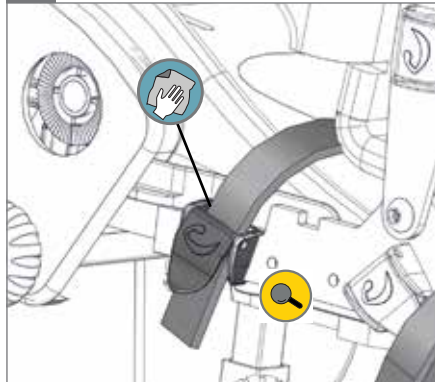5.


1.

2.

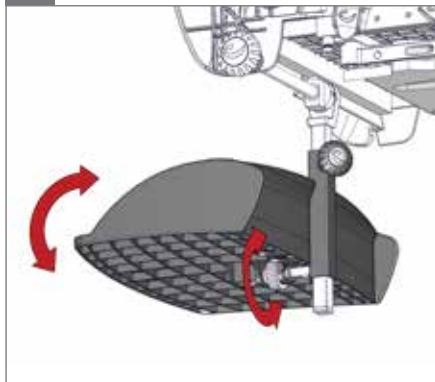3.


4.

5.

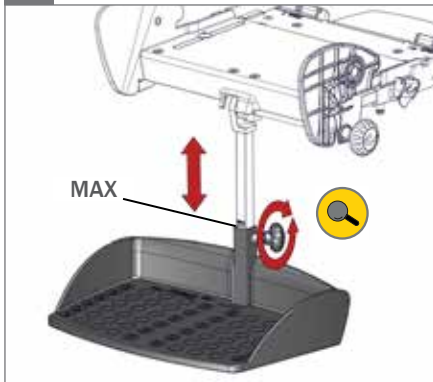
6.

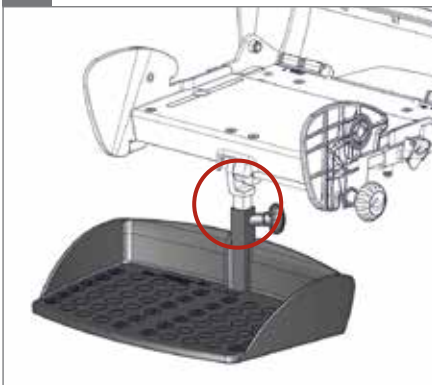
7.

8.

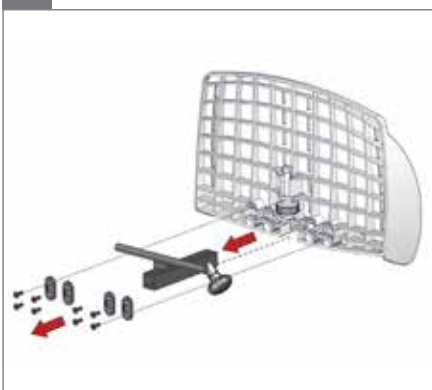9.


10.

11.

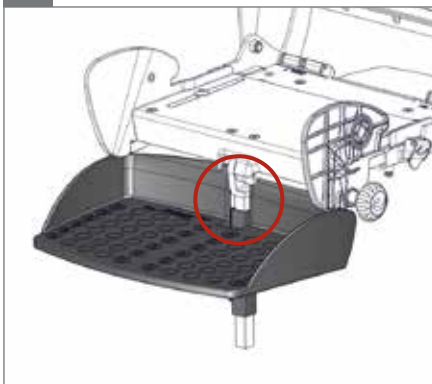12.


1.

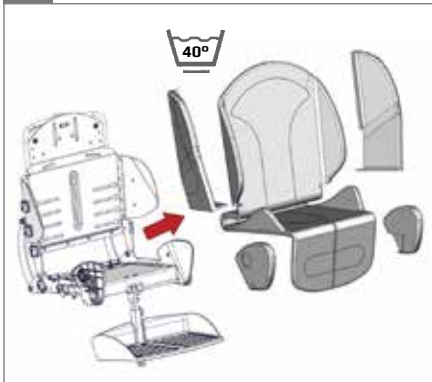
2.


3.

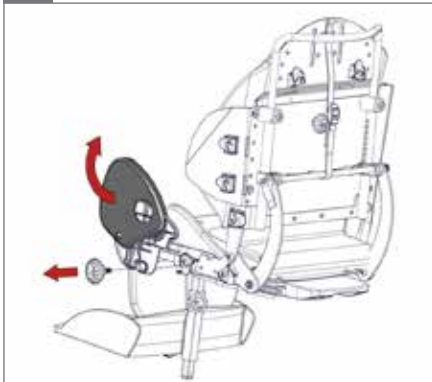
4.

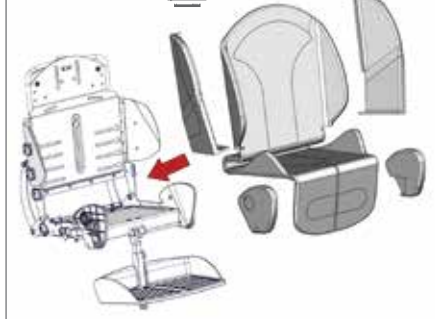5.

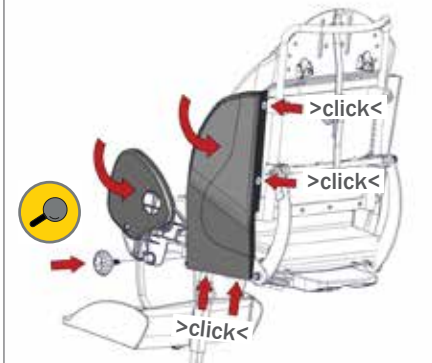6.

1.

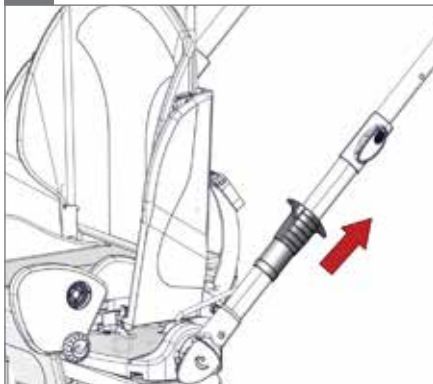
2.

3.

4.

5.

6.

1.

2.

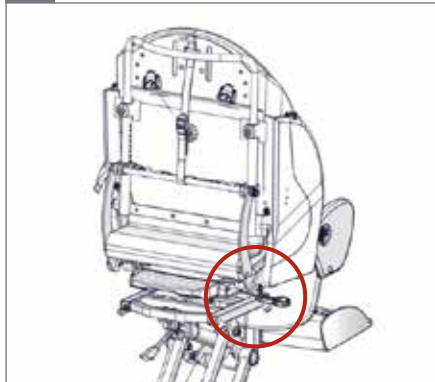
3.


4.

5.

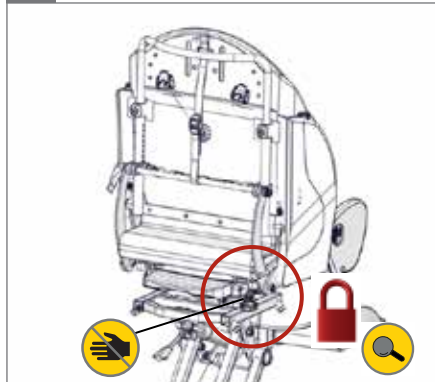6.


1.

2.

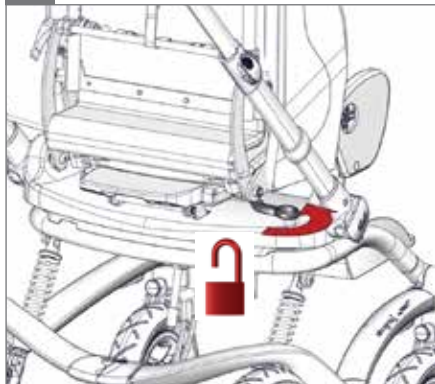
3.

4.

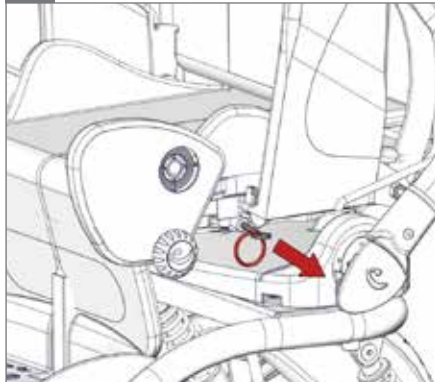5.

6.

1.

2.

3.

4.

5.

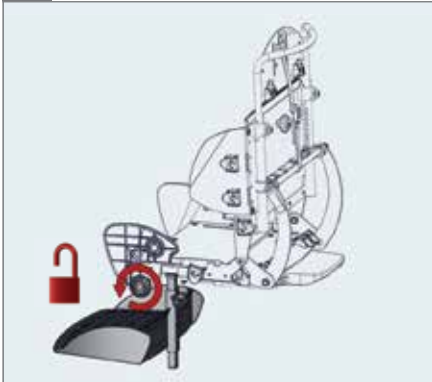
6.



1.

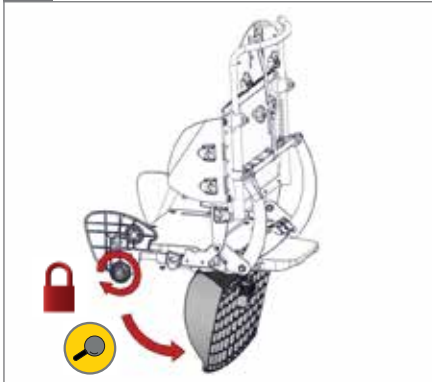2.

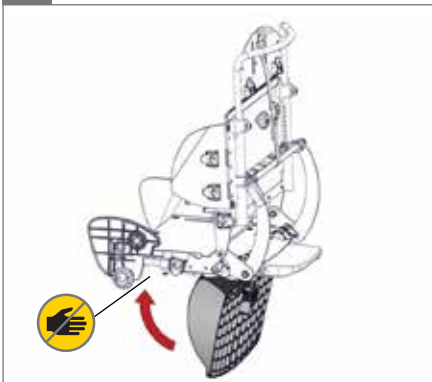3.


4.

5.

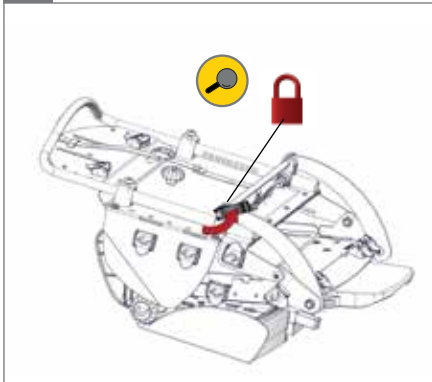6.


1.


<40 kg
<88 lb

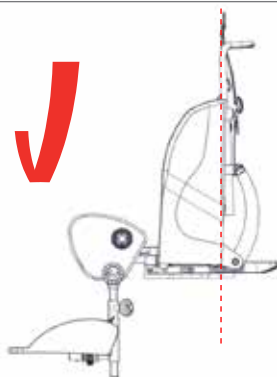2.

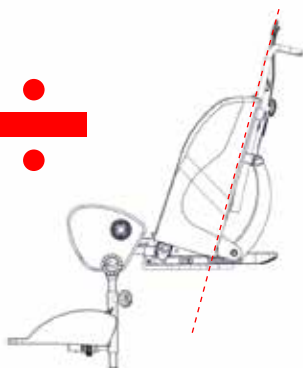
3.


4.

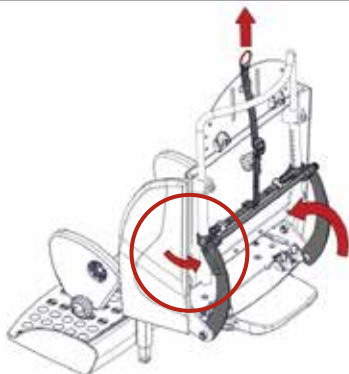5.

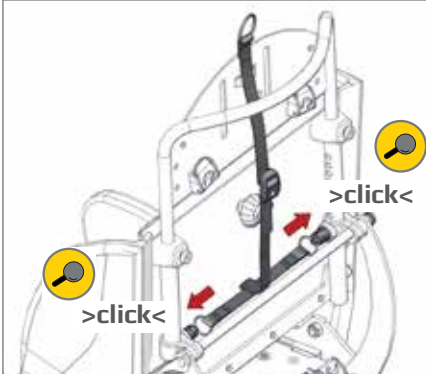
6.


1.

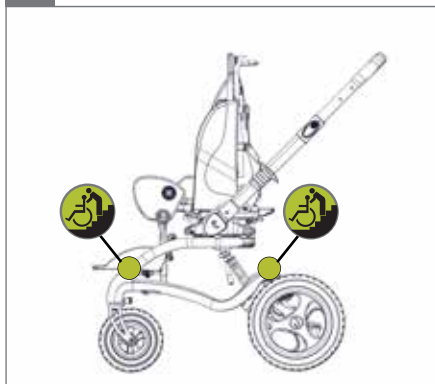2.

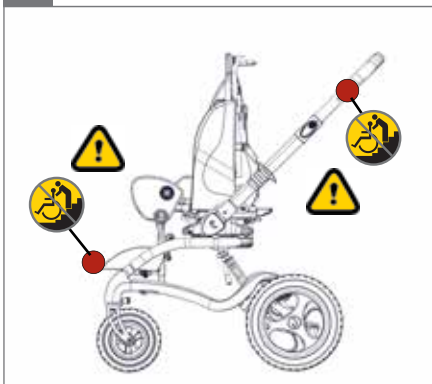3.


4.

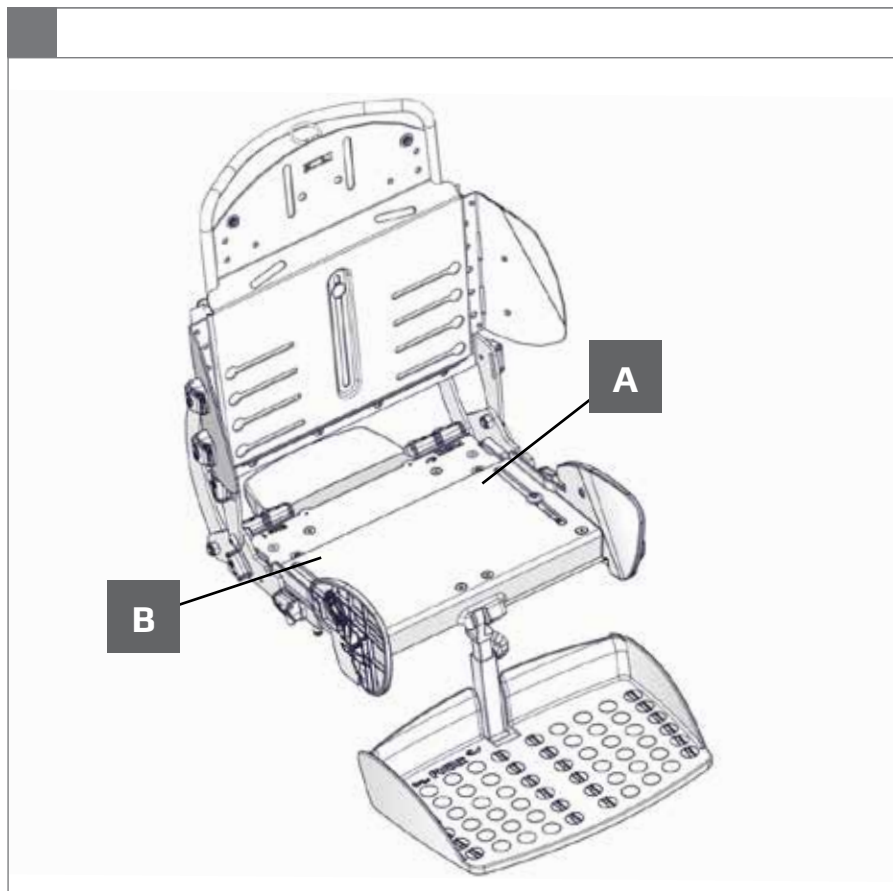5.

A

Date: YYYY-MM-DD Max.Load: XX KG
 SN: XXXXXX Ver.: XX
 Art.No: XXXXXX
 Product - Size X

(01)0570729530134(21)256829


B

R82 

Parallelsvej 3
 DK-8751 Gedved
www.R82.com
 Made in Denmark

mm (inch)

	1	2
 Seat width	300 (11¾")	350 (13¾")
 Seat depth	180-300 (7-11¾")	250-400 (9¾-15½")
 Back height	410-550 (16-21½")	500-650 (19½-25¼")
 Seat width, folded	420 (16½")	475 (18½")
 Seat length, folded	575 (22½")	575 (22½")
 Seat height, folded	285 (11")	305 (12")
 Lower leg length	140-450 (5½-17½")	140-450 (5½-17½")
 Leg rest angling	-30° -> 30°	-30° -> 30°
 Back recline	170°	170°
 Weight, seat	5.5 kg (12 lb)	6.5 kg (14.5 lb)
 Max. user weight/load, seat	40 kg (88 lb)	40 kg (88 lb)
 Max. user weight/load, transportation	40 kg (88 lb)	40 kg (88 lb)

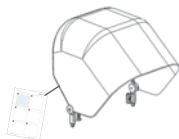
9996097170


9996097171

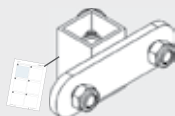
9996097172


9996097173

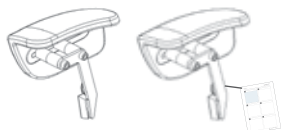
9996097174

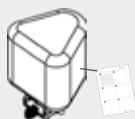
9996097175

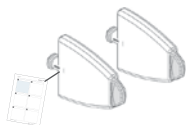9996097177

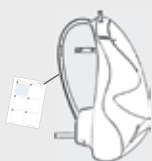9996097179

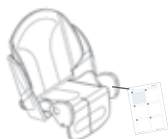9996097180

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this manual before use and save it for further reference.

Intended use

The Stingray has been designed for children who need good comfort and support. The seat depth and back height are easy to adjust, making it possible to expand the seat as when the child grows. The Stingray is suitable for users needing practical outdoor transportation, e.g. children with CP, GMFCS level 3-5. The Stingray comes in 2 sizes.

Tools

Enclosed tools; 3, 4 & 5 mm Allen keys.

Declaration of Conformity -

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to ISO 7176/19, EN12182, EN12183, EN1021-1 and EN1021-2

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or daily maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the manual. For further information, we refer to the R82 homepage/download.

The warranty can only be sustained if the R82 product is in use in the same country where it was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if non-original R82 parts/accessories are used, or if the product has not been repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the instructions for use (IFU). R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase. The warranty is given by R82 or, subsequently, an R82 dealer.

ENGLISH

Symbols

For indoor and outdoor use
Never leave the user unattended in this product

Consult instructions
The latest versions of the instructions are always available on the R82 website and can be printed in larger sizes

Caution
Incorrect use of the product may cause serious damage to the product and the environment and may cause injury

Caution
Be careful not to get your fingers caught

Correct lifting points for safe moving and handling

Caution
Do not pull/lift the chair by the push brace/handles, especially on stairs

Caution
Do not use the foot support for standing transfers

Inspection
Carry out inspections frequently

Cleaning
Keep clean to maintain optimum function

Disposal -

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

ENGLISH

Safety -

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

Read all of the instructions carefully before use and keep them for future reference. Incorrect use of this product may cause serious injury to the user

Always use the correct lifting techniques and aids for this purpose

Never leave the user unattended in this product. Ensure permanent supervision by an adult

Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier

If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

Environment

For the purpose of safety and comfort, always move the product away from the sun and allow to cool before use

Be aware of reduced maneuverability on slippery roads

User

If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children

Activate the brakes before placing the user into the product

Ensure the stability of the product is maintained before placing the user in it

We recommend you notify the user before angling the seat and back

Ensure the wheels are fully-functional and secure before use

Inspect the product and all of its accessories and replace any worn out parts before use

ENGLISH

Maintenance responsibilities table

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1st Day	Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference
	Store the enclosed tools on/at the product
Daily	Use a dry cloth to clean the product
	Visual inspection. Repair or replace any damaged or worn-out parts
	Ensure that all fasteners are in place and fitted correctly
	Check all fittings and buckles for signs of worn out parts
	Be aware of any max mark indications before adjustment
Weekly	Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
	Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent and dry before use
Monthly	All nuts and bolt on this product must be checked and tightened to avoid unnecessary failures
	Oil the swing-away parts. We recommend you use a professional lubricant system, e.g. Veidec Dry Lube
Annually	Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be sub-standard or faulty

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is issued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

Contact your local dealer for instructions on repairs e.g. dealing with tyre punctures

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

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Supplier for:
Rest of the world

R82 UK Ltd.

Unit D4A,
Coombswood Business Park East
Coombswood Way,
Halesowen
West Midlands B62 8BH
England
Tel. +44 (0)121 561 2222
Fax +44 (0)121 559 5437
R82uk@R82.com
R82.uk
facebook.com/R82UK

Supplier for:
United Kingdom, Ireland

R82 GmbH

An den Wiesen 10
55218 Ingelheim
Germany
Tel +49 (0)6132 710 70
Fax +49 (0)6132 710 710
R82de@R82.com
R82.de
facebook.com/R82.de

Supplier for:
Germany, Switzerland, Austria

R82 Nederland B.V.

Tinweg 8
8445 PD Heerenveen
The Netherlands
Tel +31 (0)513 68 16 86
Fax +31 (0)513 68 19 84
R82nl@R82.com
R82.nl
R82.be

Supplier for:
The Netherlands, Belgium

R82 CZ, s.r.o.

Bohunicka 52
CZ-619 00 Brno
Czech Republic
Tel: +420 777 969 999
R82cz@R82.com
R82.cz
facebook.com/R82.cz

Supplier for:
Czech Republic, Slovakia

Snug Seat Inc.

12801 E. Independence Blvd.
P. O. Box 1739
Matthews NC 28106
USA
Tel +1 800 336 7684
Fax +1 704 882 0751
Information@SnugSeat.com
SnugSeat.com
facebook.com/SnugSeatInc

Supplier for:
USA, Canada

R82 au Pty Limited

Unit 1, Level 2, 13 Lord Street
Botany NSW 2019
Australia
Tel +61 (02) 8213 6666
Fax +61 (02) 8213 6655
R82.com.au
facebook.com/R82Australia

Supplier for:
Australia

R82 China 中国

Room 920 Tower 2
China Central Place
79 Jianguo Road
Chaoyang District
Beijing 100025, China
Tel +86 10 5920 4399
Fax +86 10 5920 4222
R82cn@R82.com
R82.com.cn

Supplier for:
China

Find your local dealer

R82.com

DISTRIBUTOR: