<560 mm (22")

<70 kg (154.3 lb)







Wombat Solo User Guide

EN 12182

2019-08 - rev.4

	3		32-36
	4-8		37-38
	7	ENGLISH	39-41
	8-9	SVENSKA	55-57
	10-13	SUOMI	58-60
	14		
	15-16		
	17		
	18-19		
	20-21		
	22		
	23-25		
	26-27		
	28		
	29		
	29		
	29		
mm (inch)	30-31		

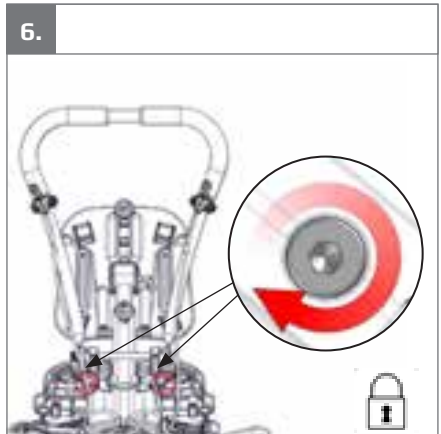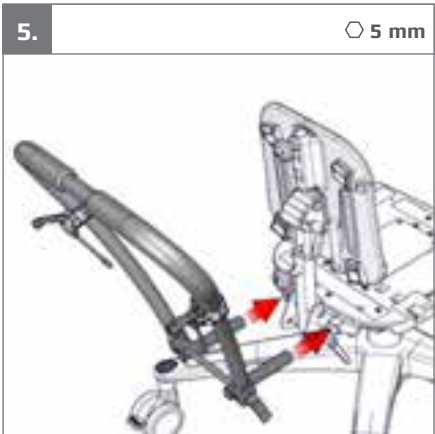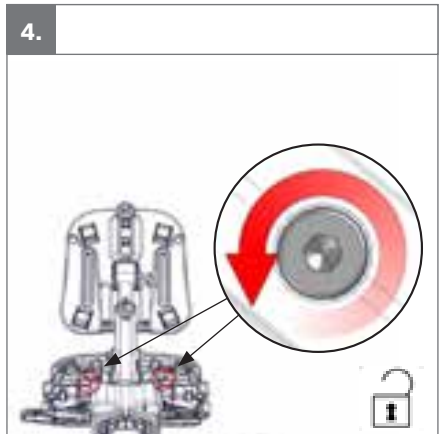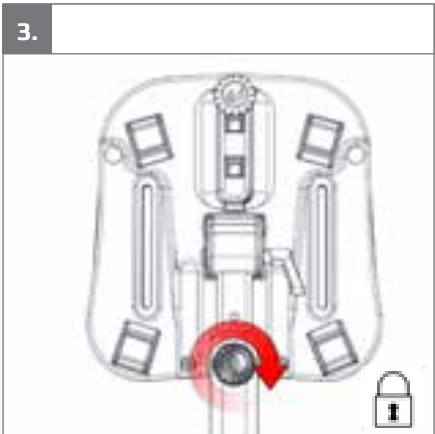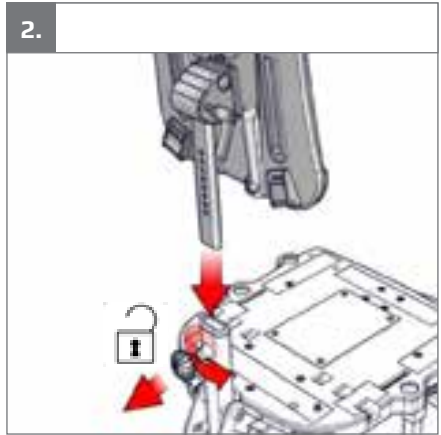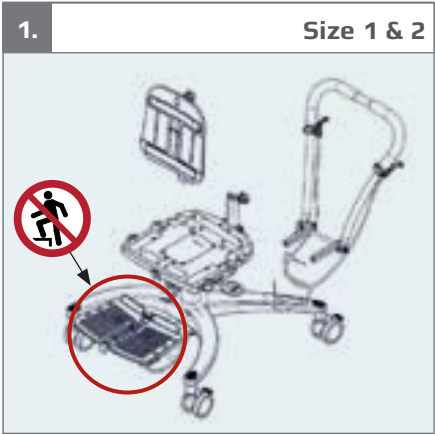
**WOMBAT SOLO 1+2****WOMBAT SOLO 3**


1.

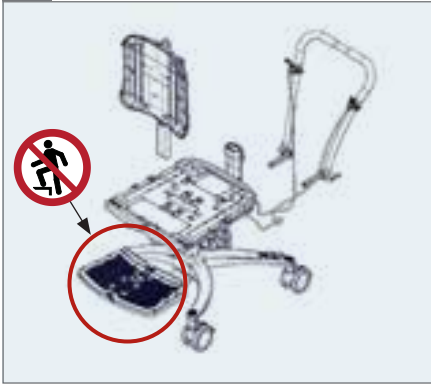Size 3

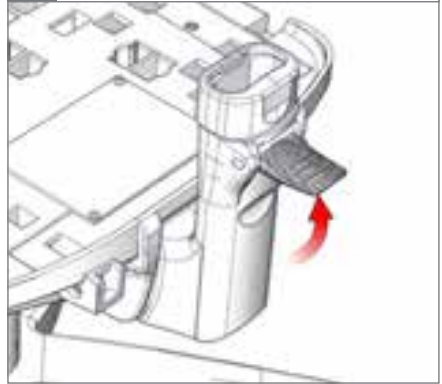2.


3.

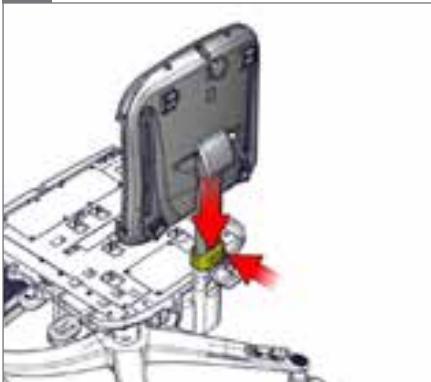⊕ 5 mm  10 mm

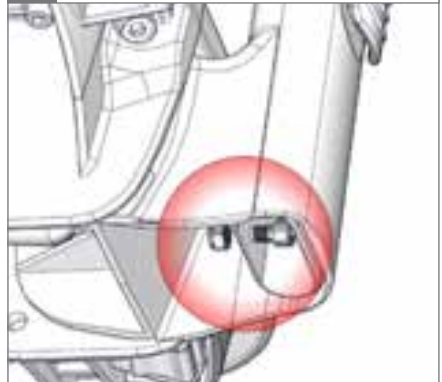
4.






5.


6.

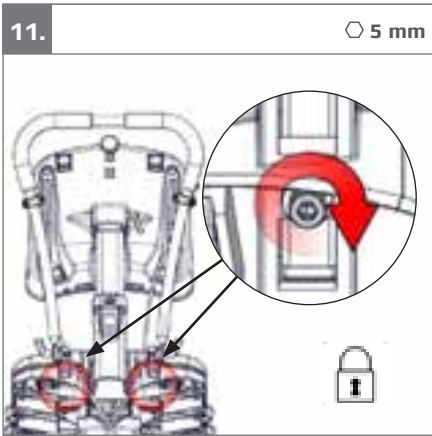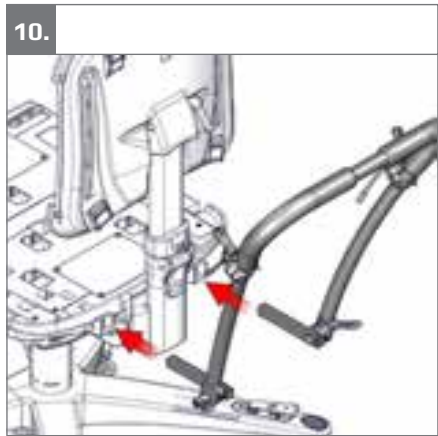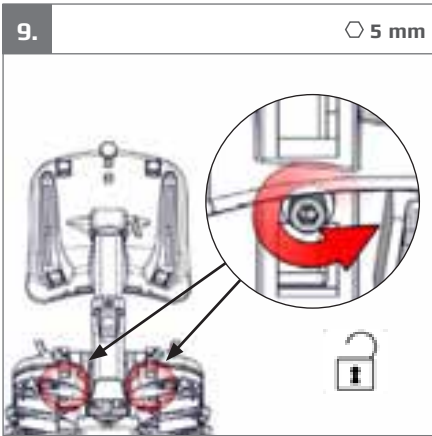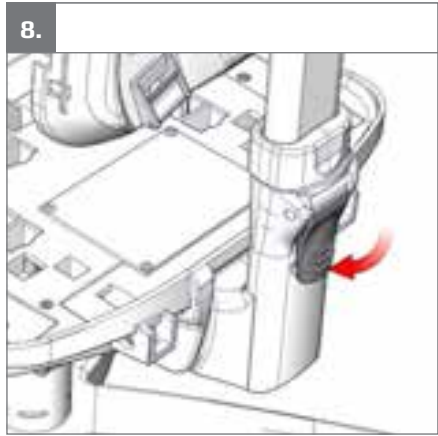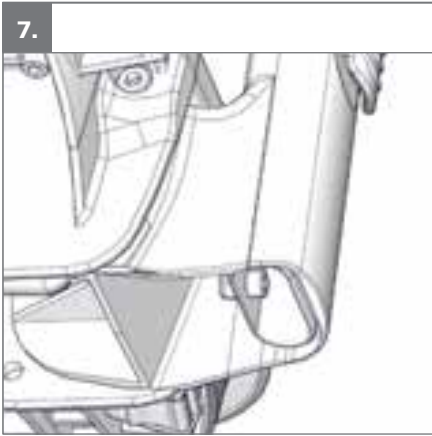⊕ 5 mm  10 mm

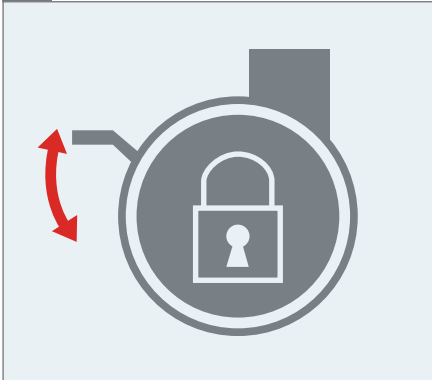1.

2.

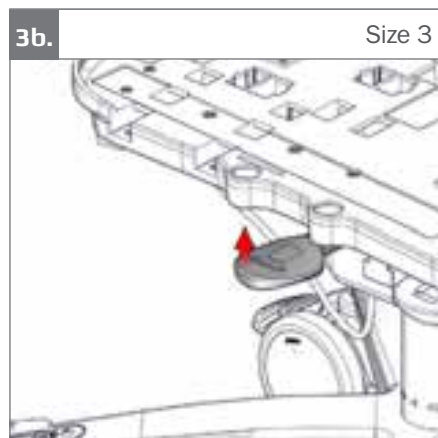
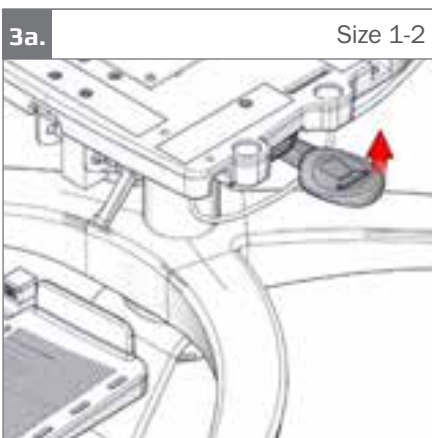
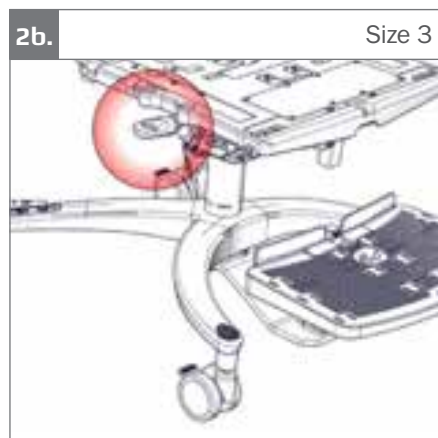
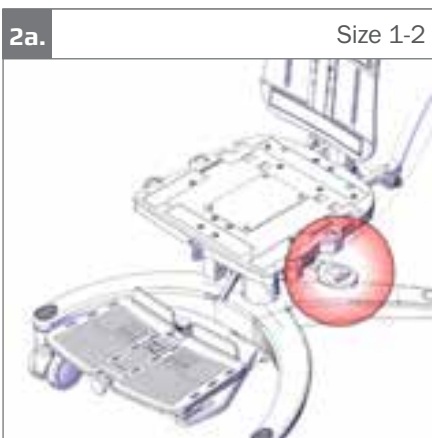
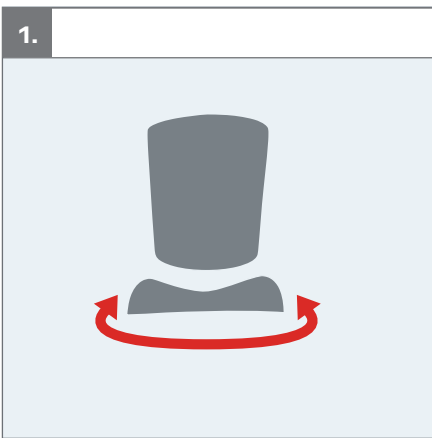
3.


4.

4a.

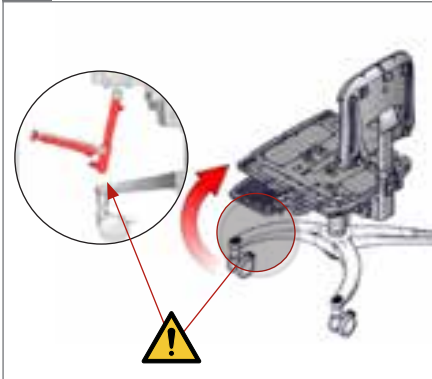 05

4b.

 05

5a.


5b.


6a.



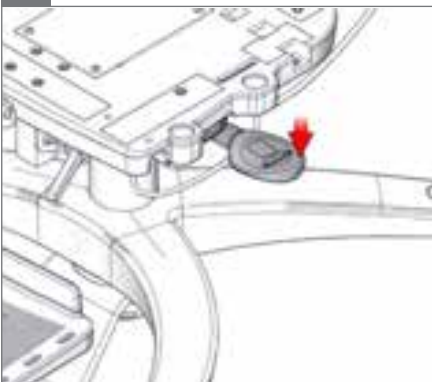
Size 1-2

6b.

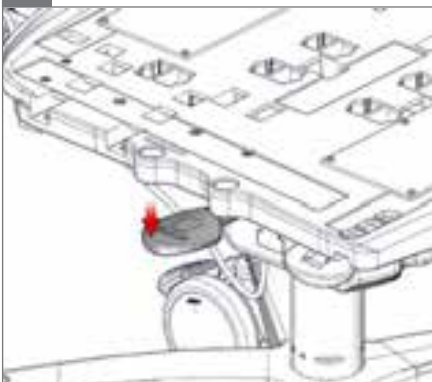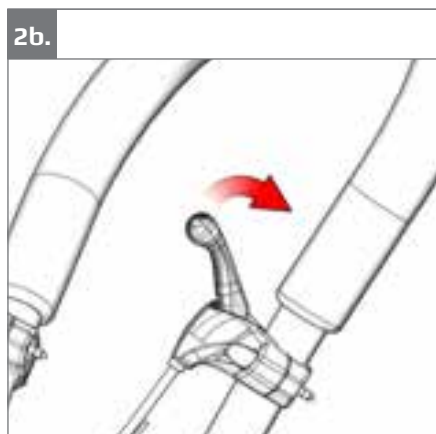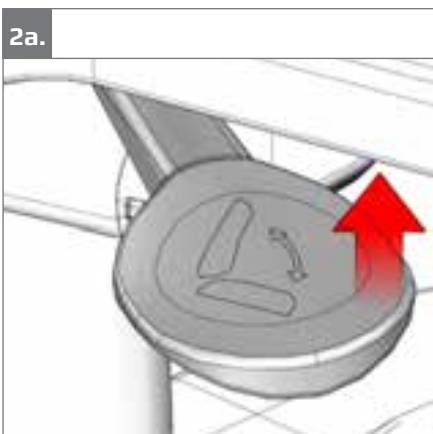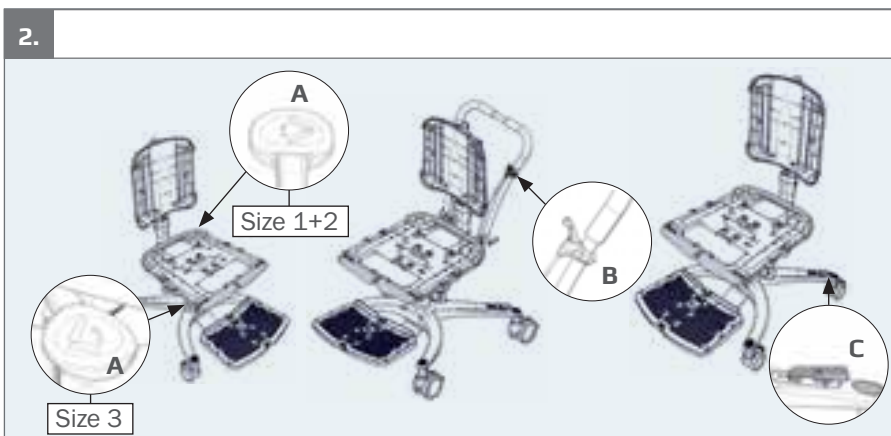
Size 3



2c.

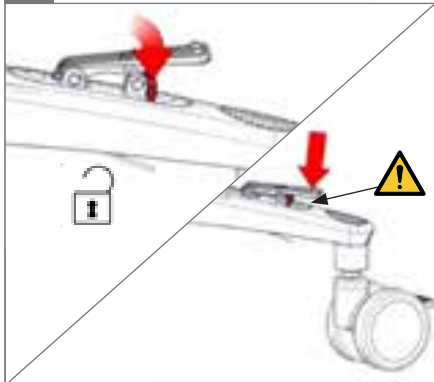 05

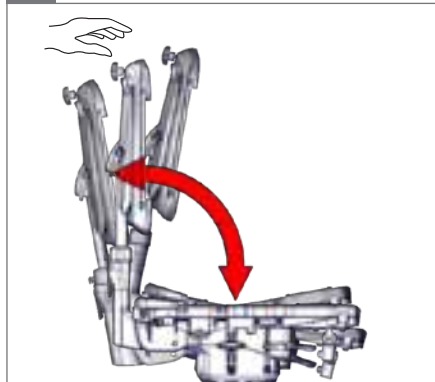
3.


4a.

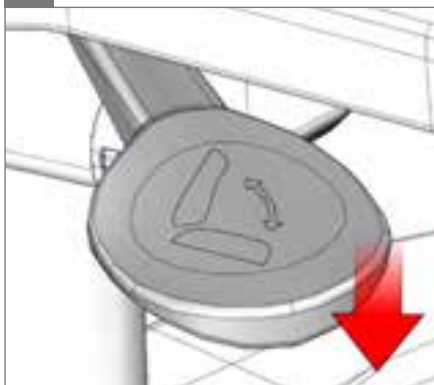4b.

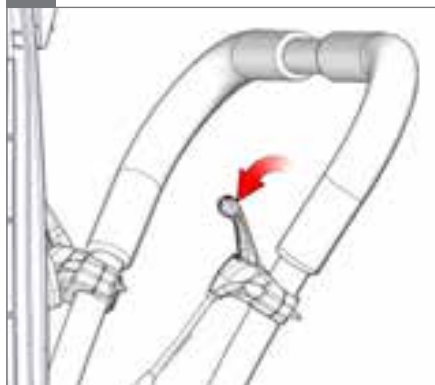
4c.

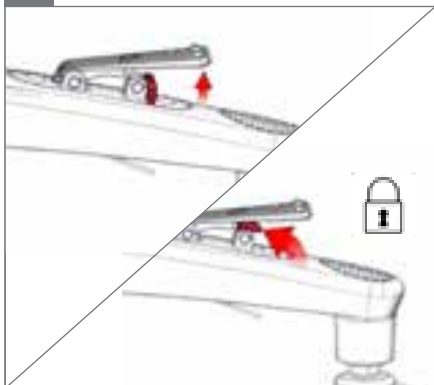
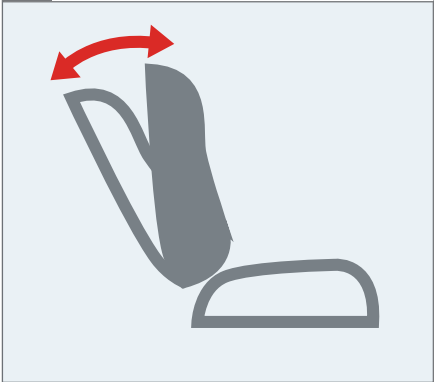
1. Size 1-2

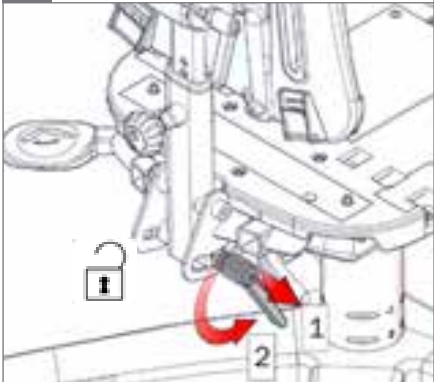
2.

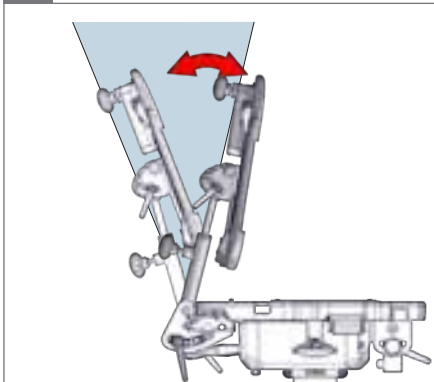
3.

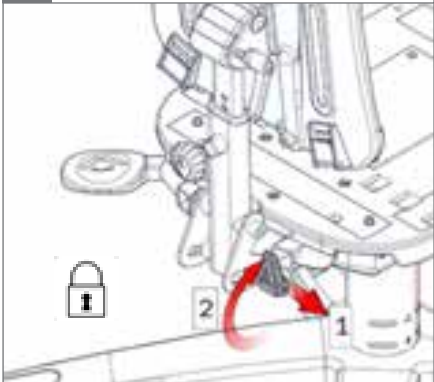4.

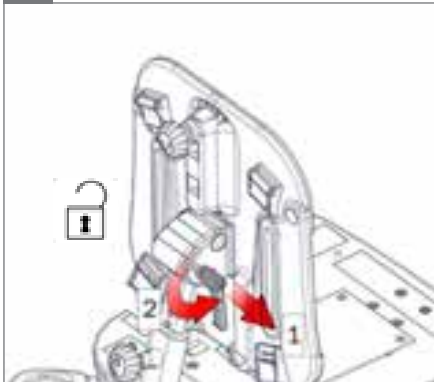
5.

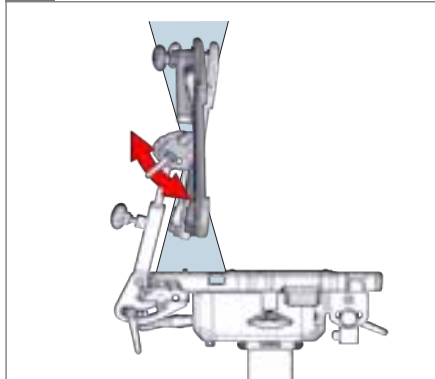
6.

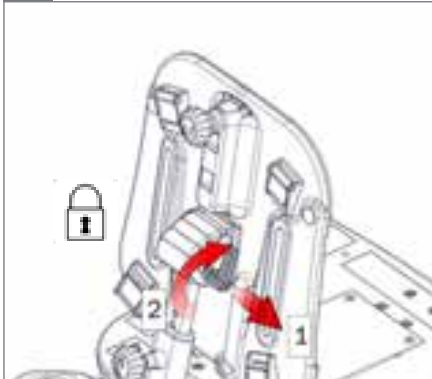
7.


1.

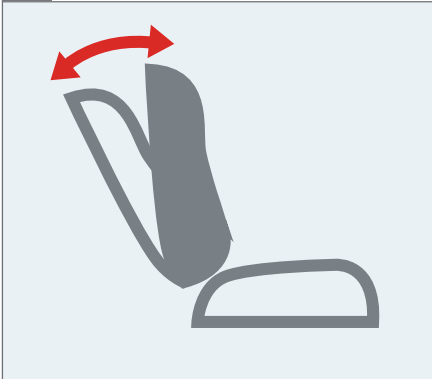
Size 3

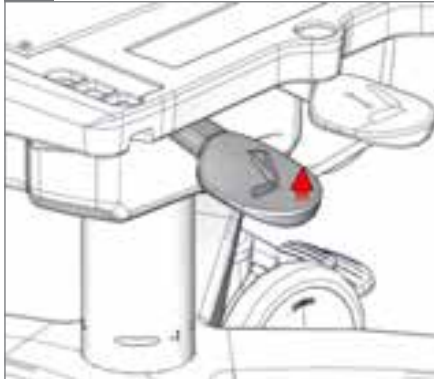
2.

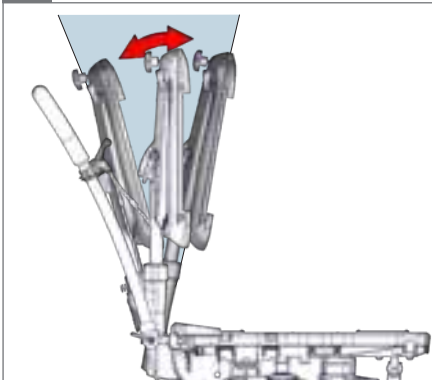
3.

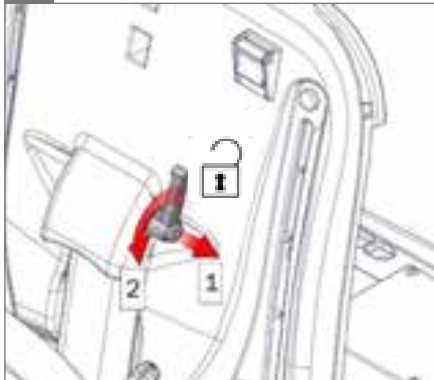4.




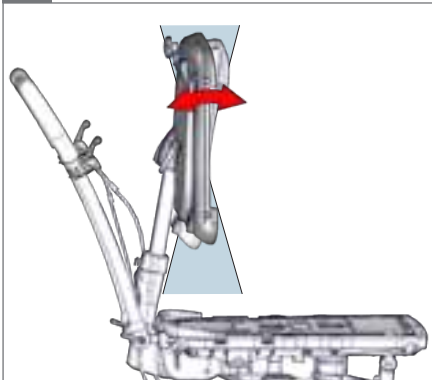
5.

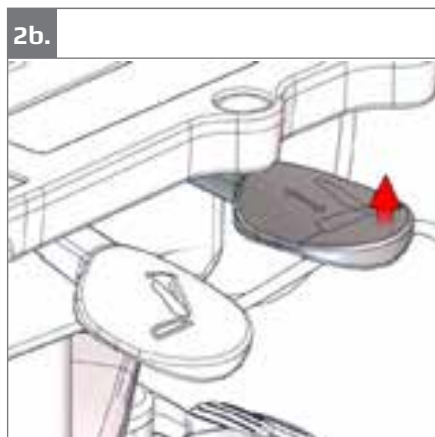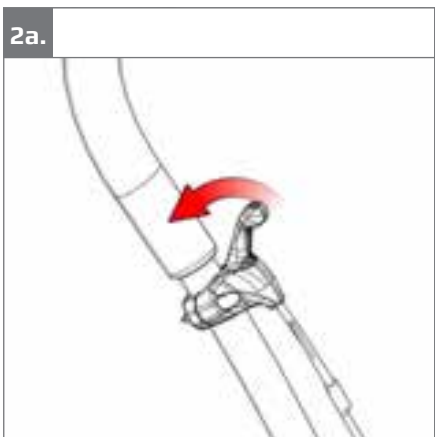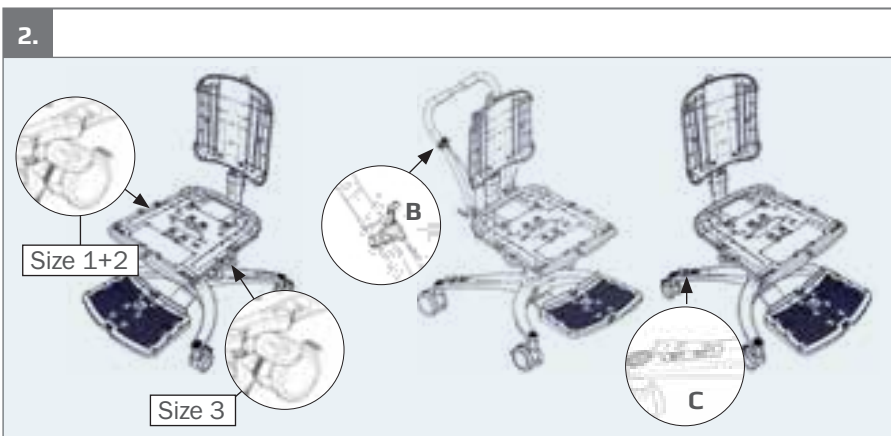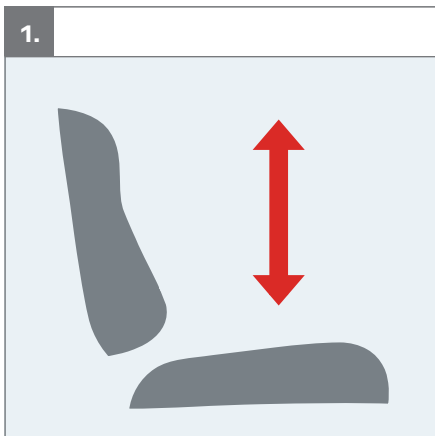
6.



2c.

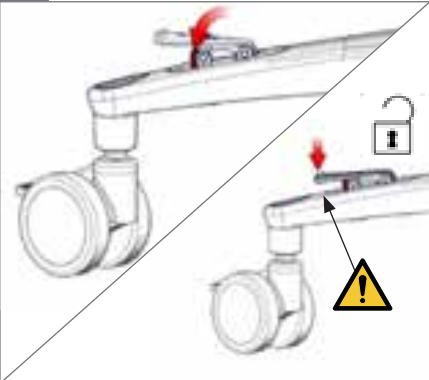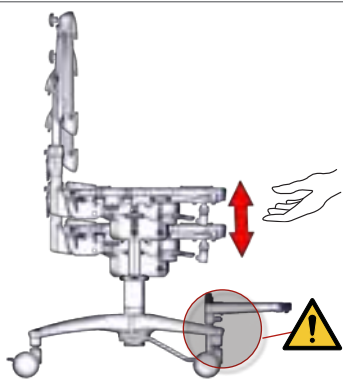
3.


4a.









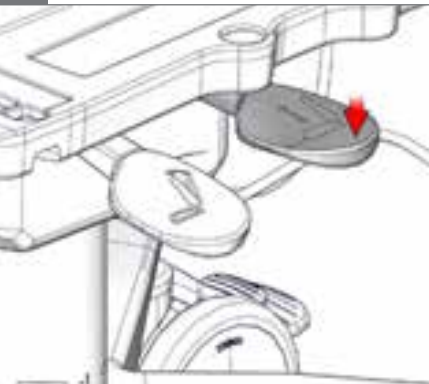
4b.








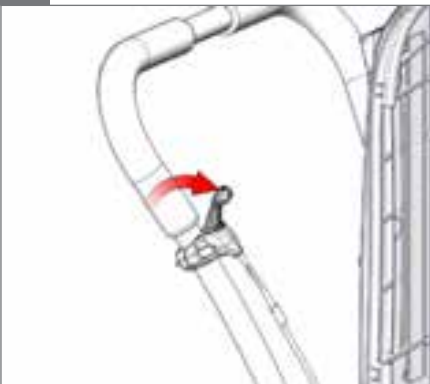
4c.

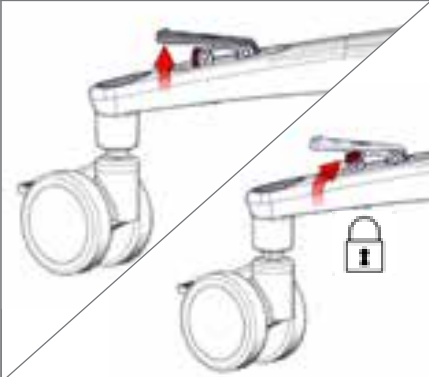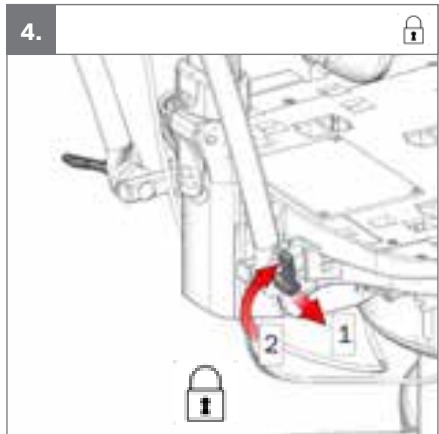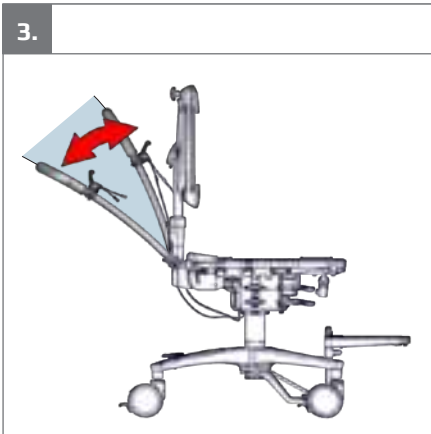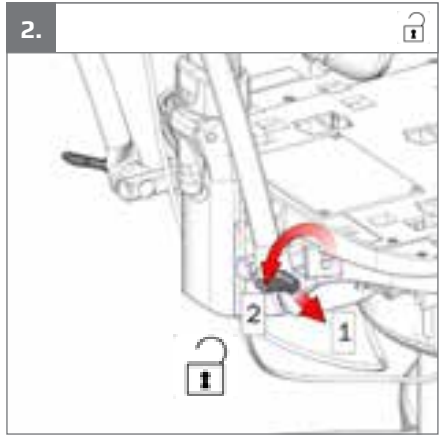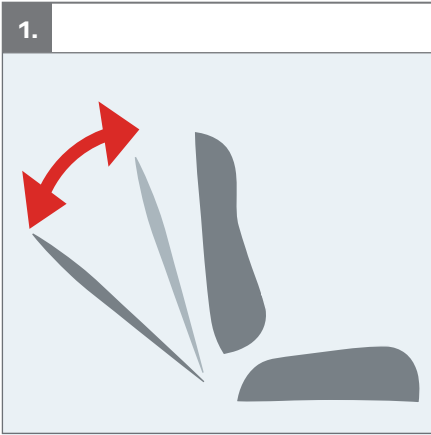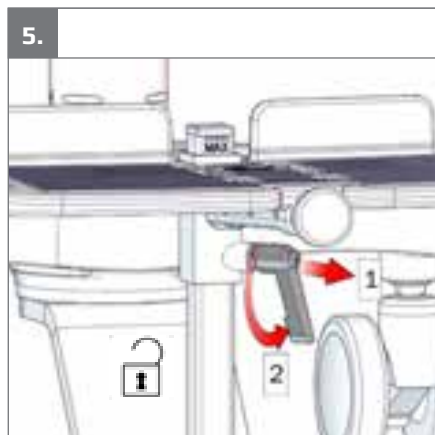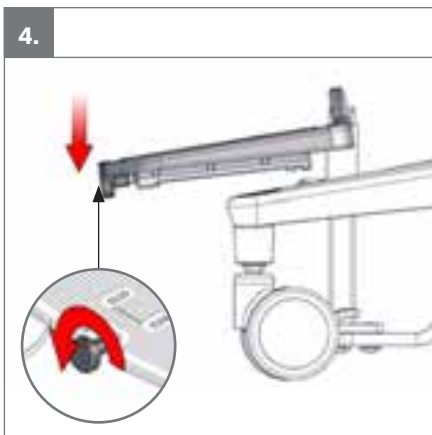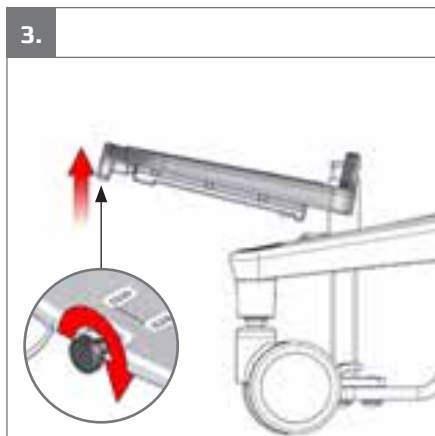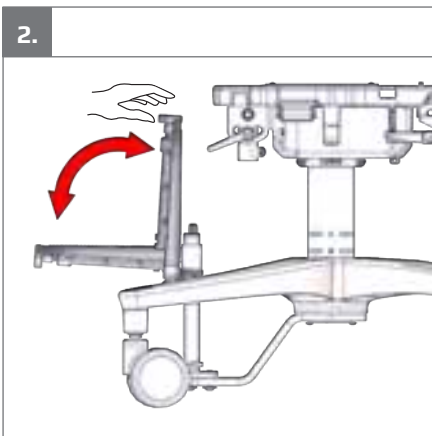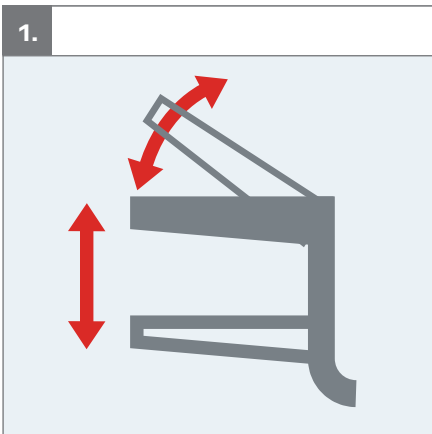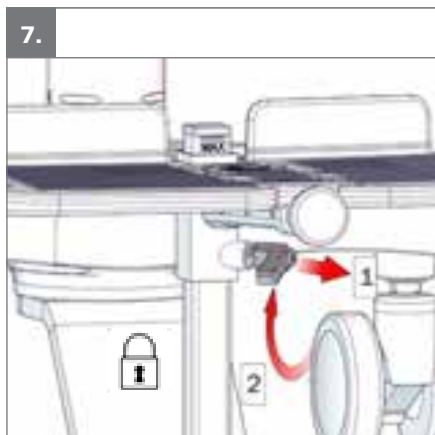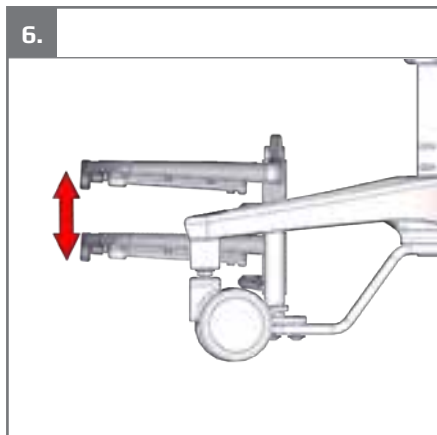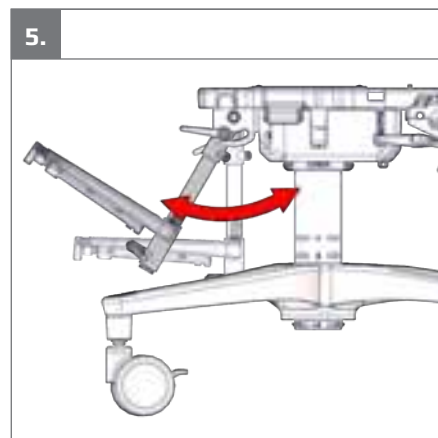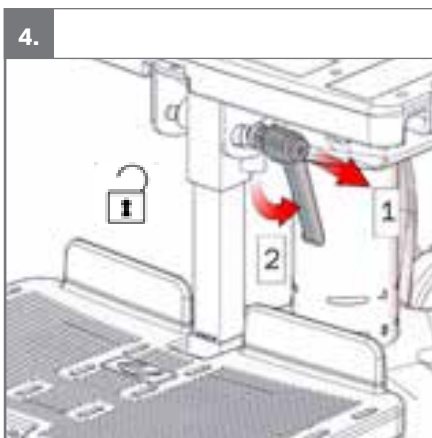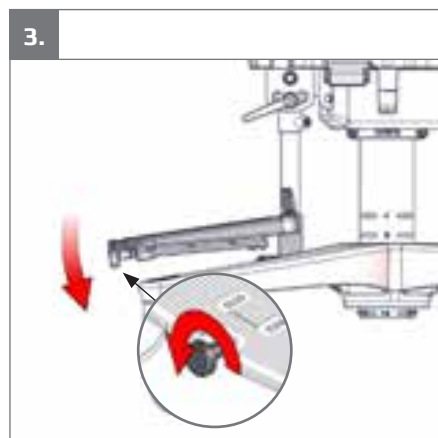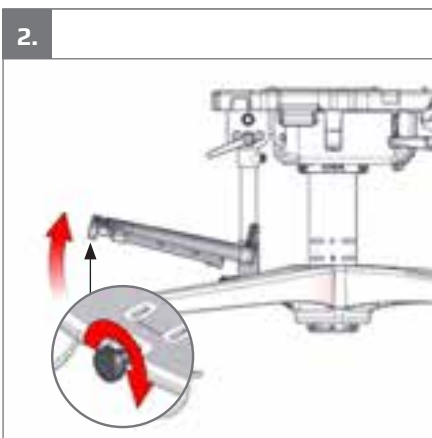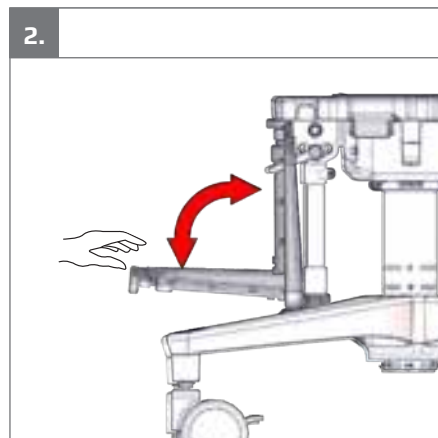
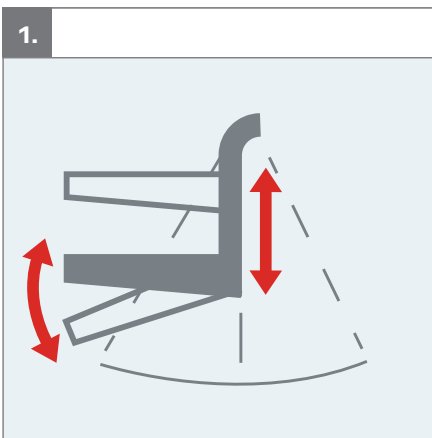
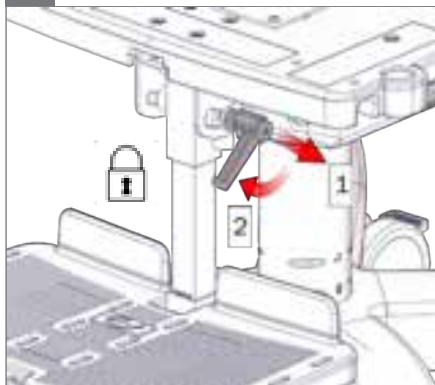
6.

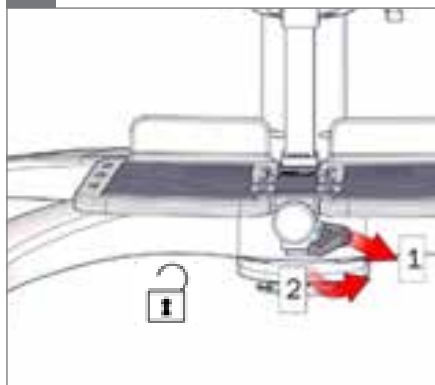
7.

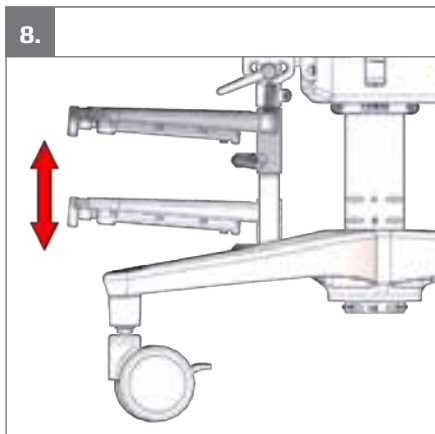8.

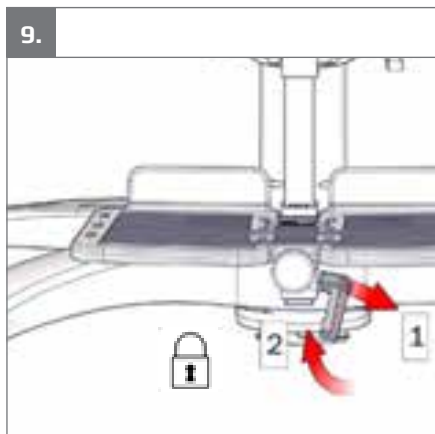9.


1.


2.

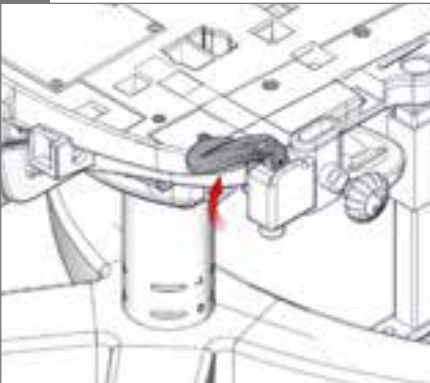
3a.


3b.









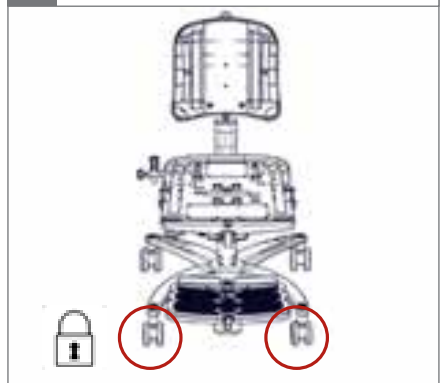
4a.

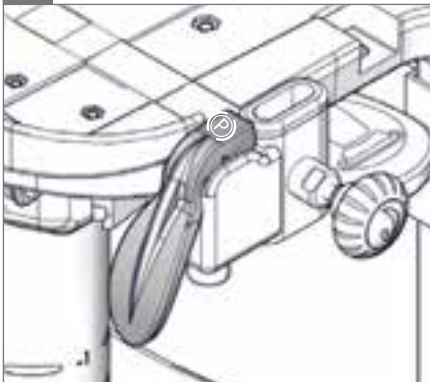4b.

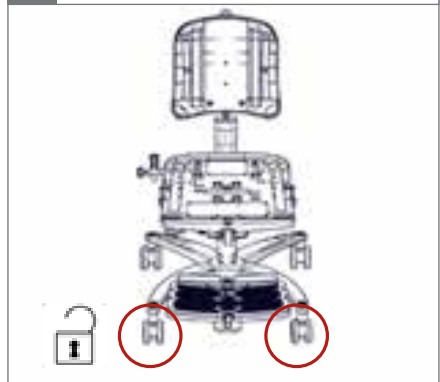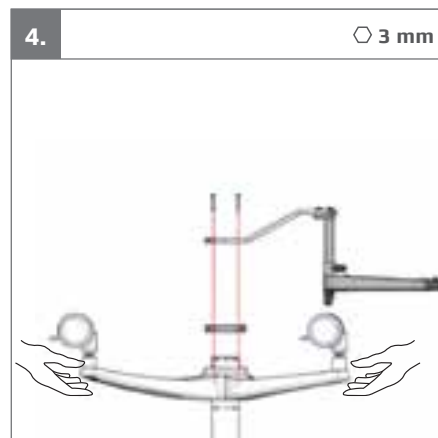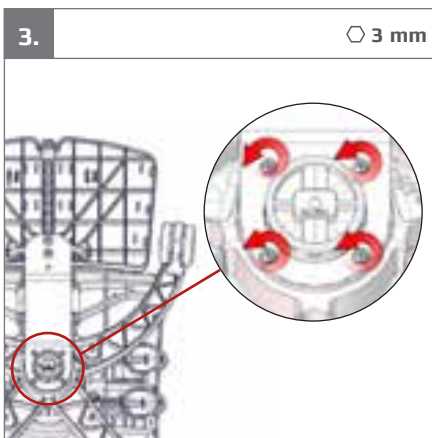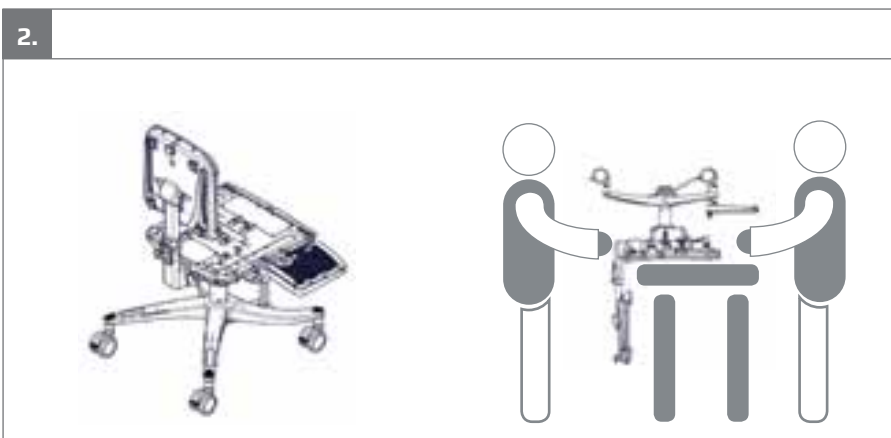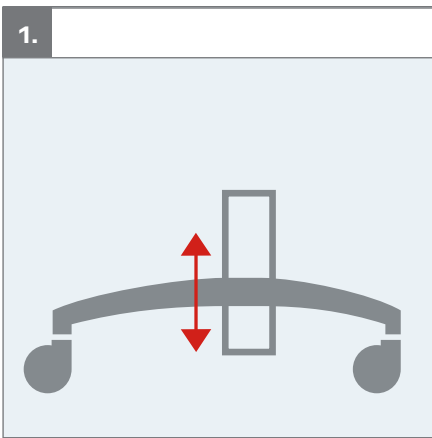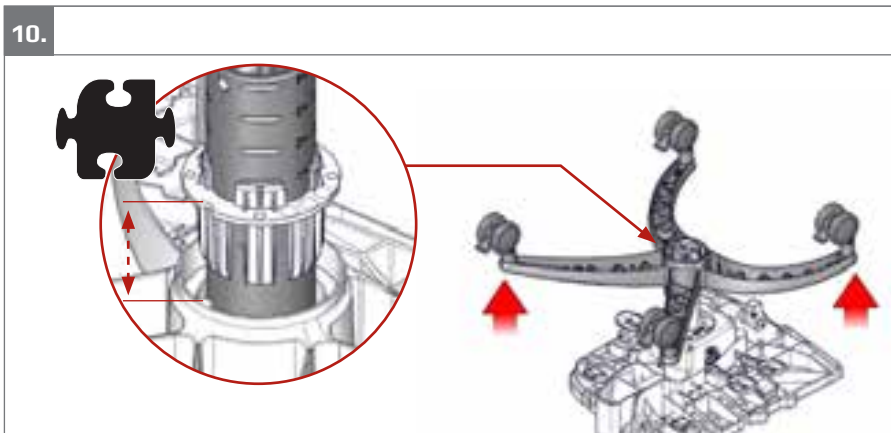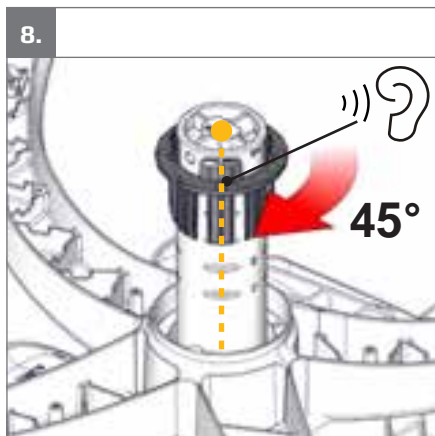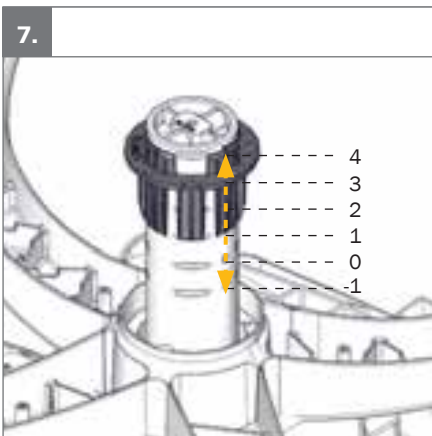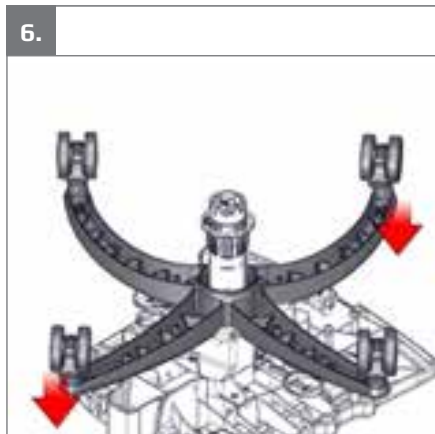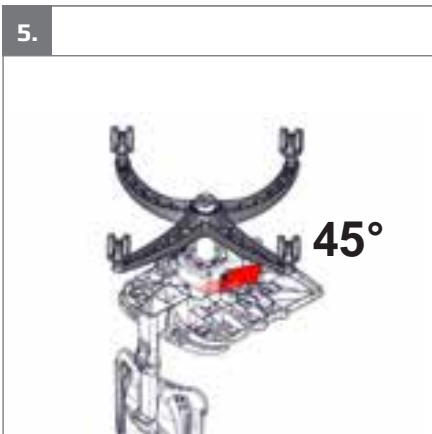
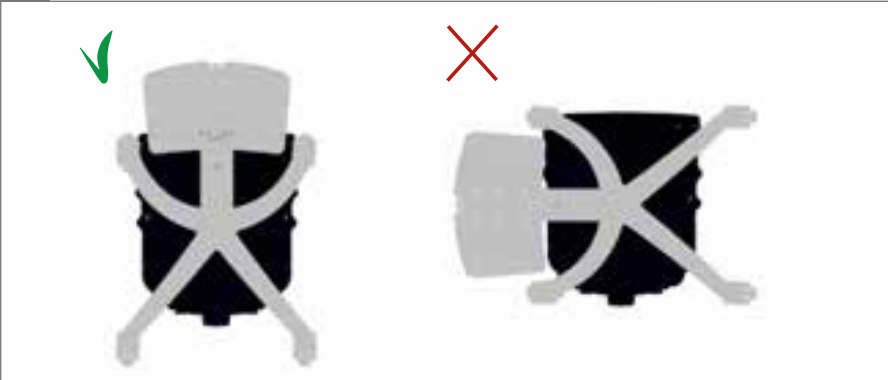
11.


12.

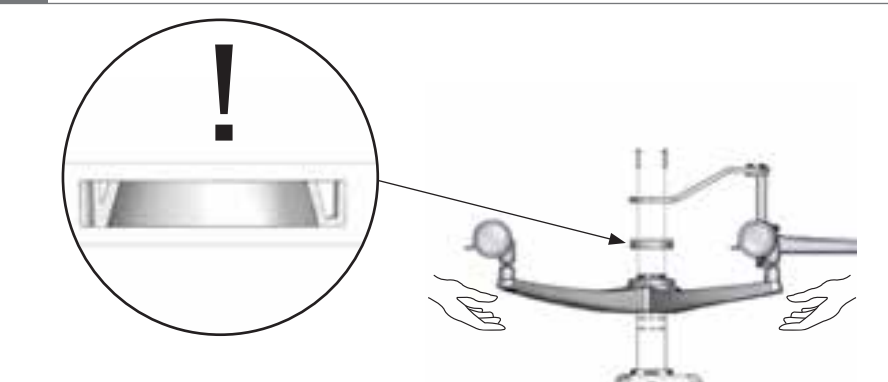3 mm







13.

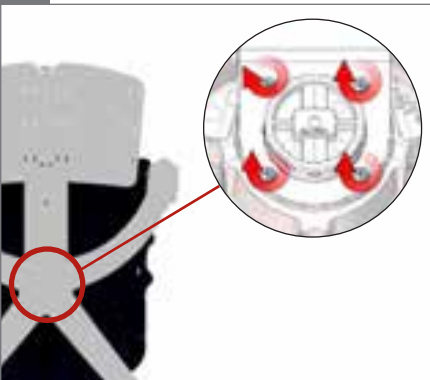3 mm

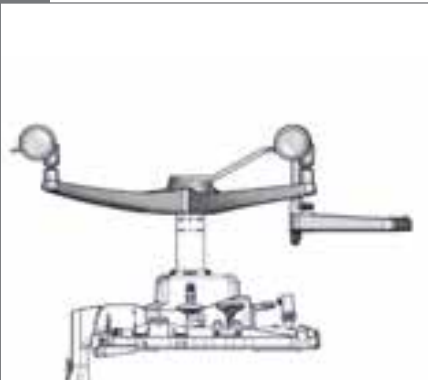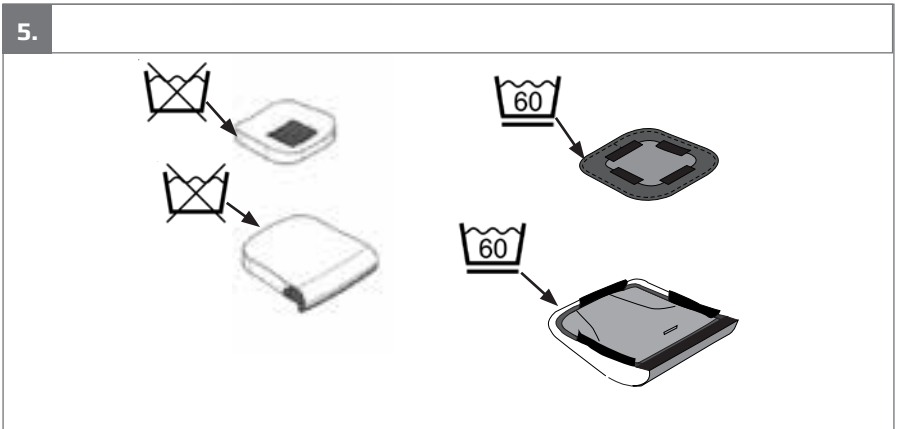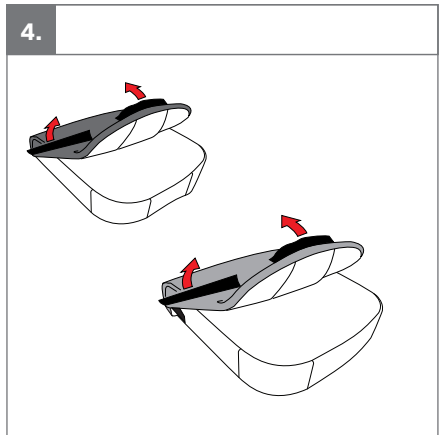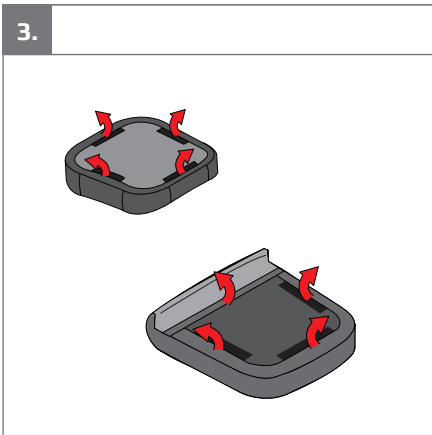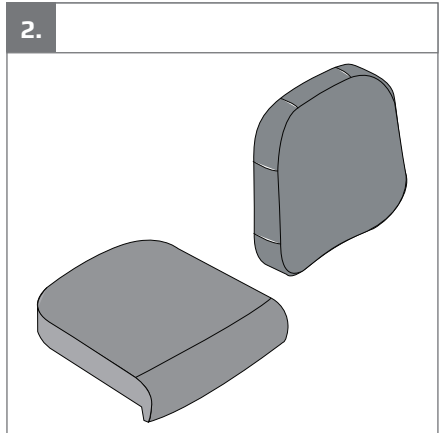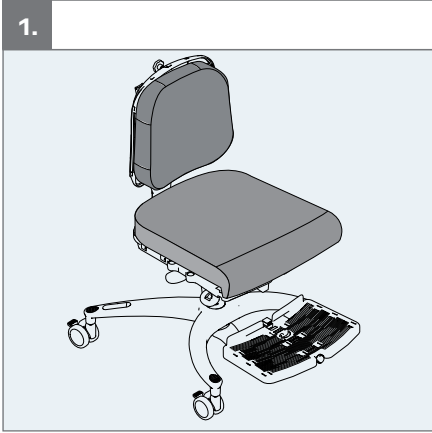
14.


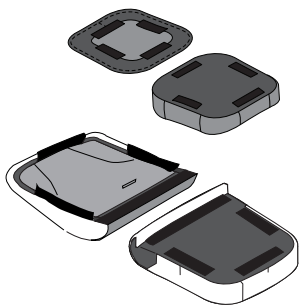
6.

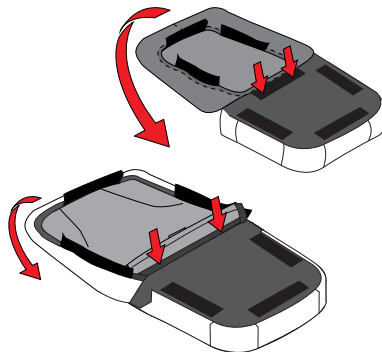
7.

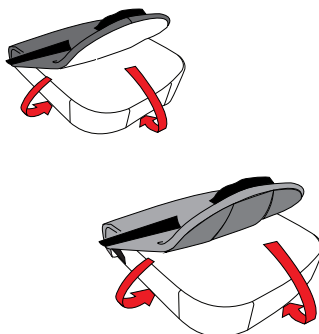
8.

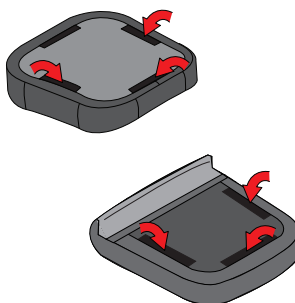
9.

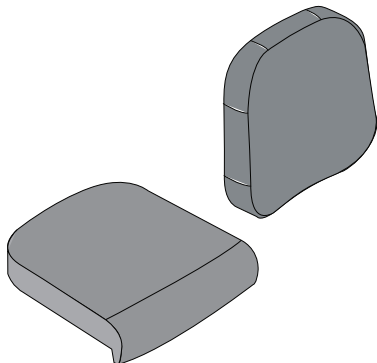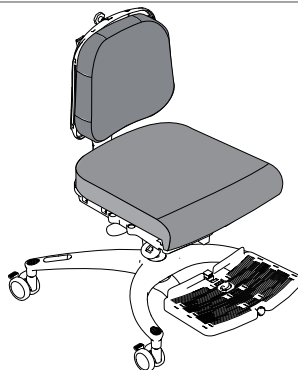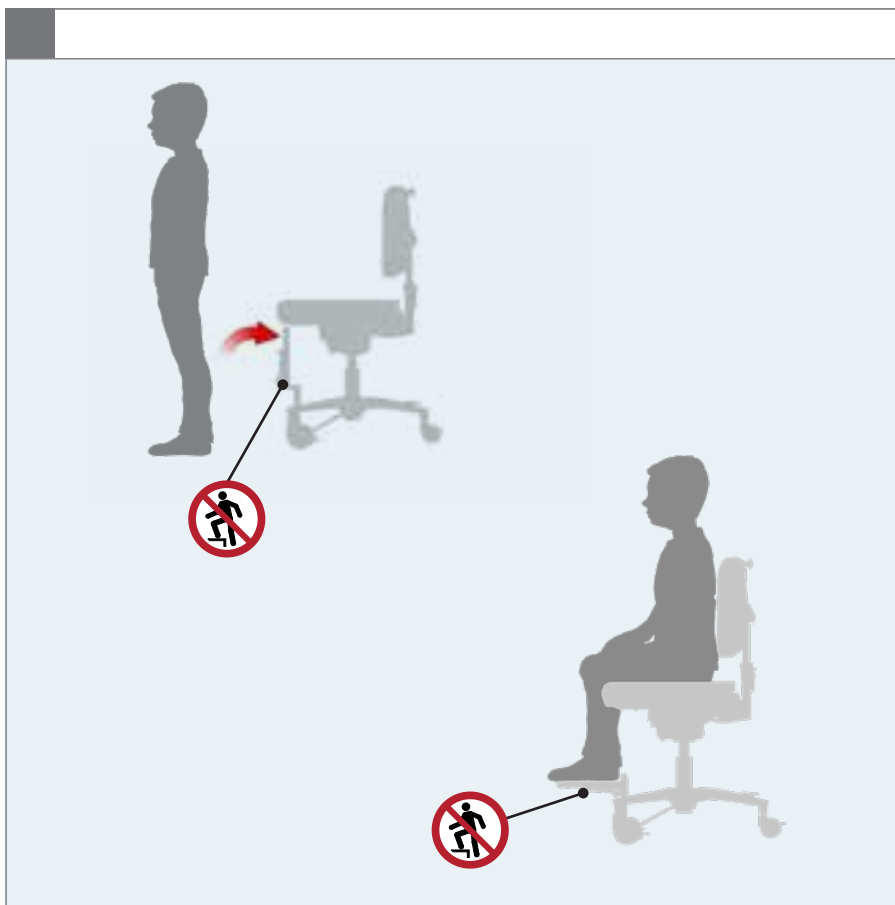
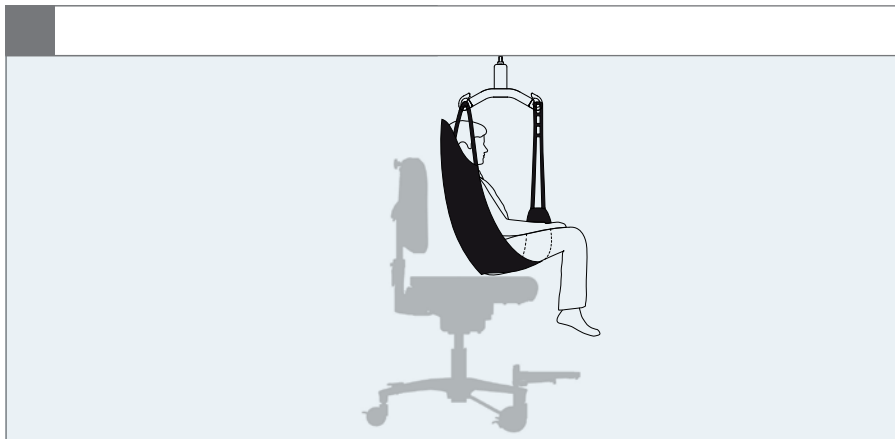
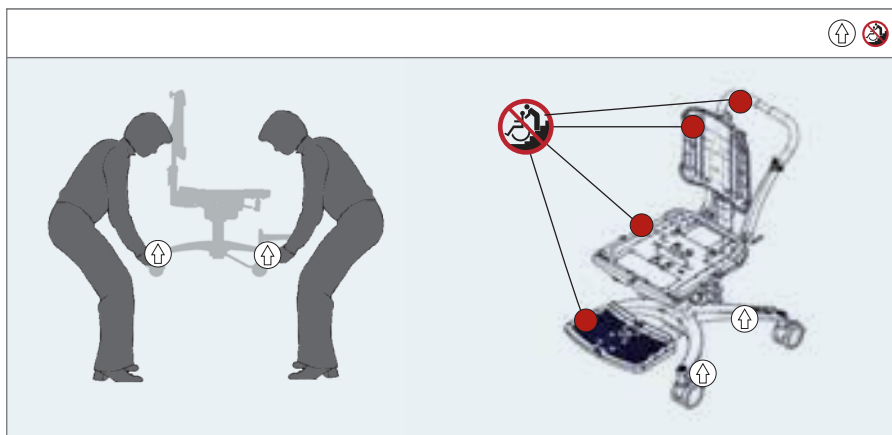
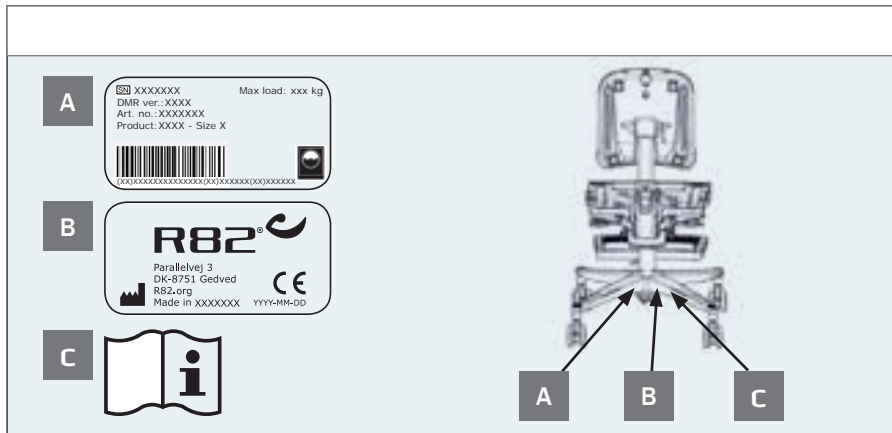
10.

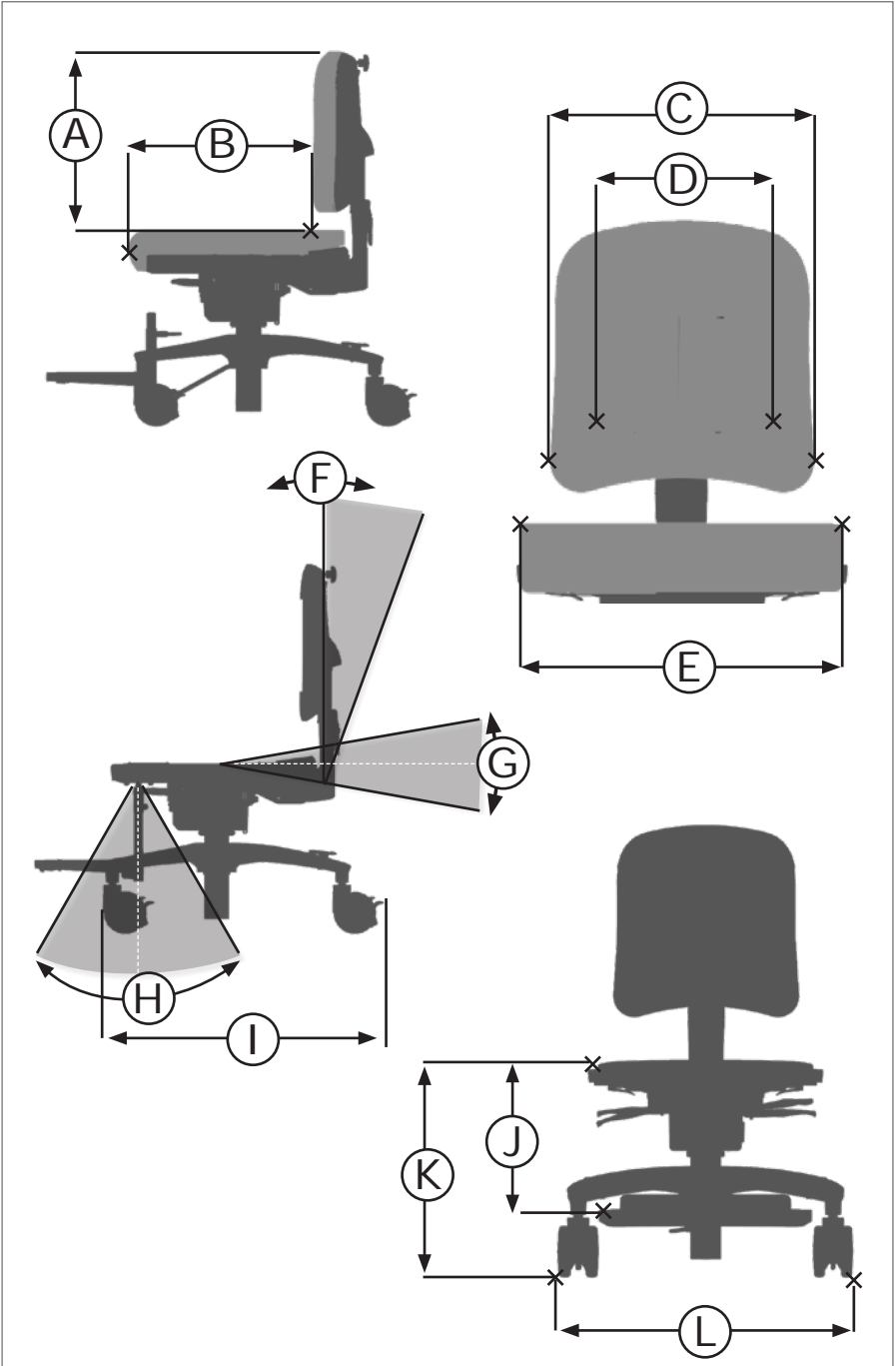
10.


mm (inch)	 1	 2	 3
 Backrest height (A)	280-340 (11-13½")	310-380 (12¼-15")	390-500 (15¼-19¾")
 Effective seat depth (B)	180-310 (7"-12¼")	250-380 (9¾-15")	320-500 (12½-19¾")
 Back width, flat cushion (C)	250 (9¾")	300 (11¾")	350 (13¾")
 Back width, contoured cushion (D)	170 (6¾")	230 (9")	270 (10½")
 Effective seat width (E)	300 (11¾")	370 (14½")	420 (16½")
 Backrest angle (F)	0-28°	0-28°	0-33°
 Seat plane angle (G)	-10-15°	-10-15°	-10-10°
 Angle foot support bar (H)	-30-30°	-30-30°	-30-30°
 Length (I)	645 (25½")	645 (25½")	645 (25½")
 Footrest to seat distance, seat mounted (J) **	85-185 (3¼-7¼")	85-285 (3¼-11¼")	85-375 (3¼-14¾")
 Footrest to seat distance frame mounted (J) **	95-320 (3¾-12½")	95-320 (3¾-12½")	245-395 (9¾-15½")
 Seat surface height at front edge (K)*	310-430 (12¼-17")	310-430 (12¼-17")	375-491 (14¾-19¾")
 Seat surface height at front edge with cushion	375-495 (14¾-19½")	375-495 (14¾-19½")	440-556 (17¼-22")
 Overall width (L)	560 (22")	560 (22")	560 (22")
 Max load	45 kg (99.2 lb)	45 kg (99.2 lb)	70 kg (154.3 lb)
 Total mass	12,5 kg (27.5 lb)	13 kg (28.7 lb)	16 kg (35.3 lb)

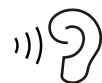* Measured without cushions. There are 6 positions for height adjustment in the center column. Each increment adds 22 mm to measure starting from position -1.
 ** Add 65 mm to measure with cushion.
 A-E Measured with cushion
 I Add 15 mm to measure when 100 mm castors are mounted.
 L Add 10 mm to measure when 100 mm castors are mounted.
 K Add 25 mm to measure when 100 mm castors are mounted.

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EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger
01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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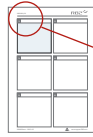
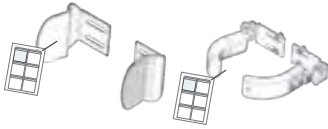
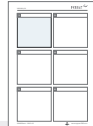
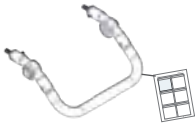
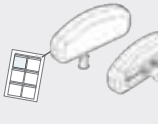
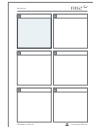
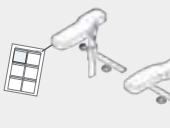
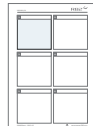
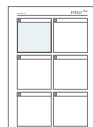
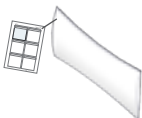
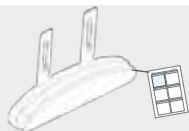
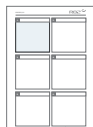
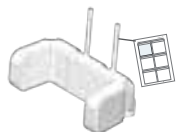
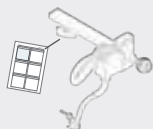
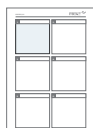
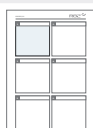
9996097357

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Wombat Solo is a chair for children with disabilities.

Contra indications

There are no known contraindications.

Caution! When used for:

- Children with limitation in their ability to maintain the position of their head, use the product with a headrest.
- Children with limitation in their ability to maintain a sitting position, use the product with accessories (e.g. side supports or vest).
- Children who experience severe or complete impairments in involuntary movement functions of the trunk, consider a modular seating system with a dynamic back.

Application areas

The product is for indoor use

Enclosed tools:

- 3+4+5 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- support.r82.org

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Users who have difficulties reading the user guide can have someone read the user guide aloud to understand how to use the product.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Firmly grab hold and support the back of the chair before adjusting it to prevent a sudden recline.
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- The handles underneath the seat plate can be operated by the user.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Be aware of any MAX mark indications before adjustment

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces

- When using on inclines, please refer to measurements

User

- Never carry the product with a user seated in it
- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Wombat Solo comes with multiple adjustment options. When adjusting the product, take care that body parts are not trapped in holes and openings intended for adjustment.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Lock the castors before placing the user in the product
- Ensure the stability of the product is maintained before placing the user in it
- For independent transfer ensure the castors are locked sufficiently.
- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- The footplate is for support only! The foot support does not have a weight bearing function
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This product is tested for resistance to ignition at level non-ignition in accordance with EN 1021-1, EN 1021-2 and ISO 7176-16
- The product might lose its flame resistant characteristics if using other than original R82 Cushioning
- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference
- Store the enclosed tools on/at the product

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use
- Disinfect the product. please check the disinfection section in this User Guide for more information

Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

Washing

The cushion covers can be washed at 60 °C using a mild detergent in a washing machine. It is not recommended to wash the foam.

The foam inserts can be washed at 40 °C using a mild detergent in a washing machine.

During and after washing the foam, do not bend, squeeze or wring it as it may negatively affect the shape

Warewashing machine

This product can be washed at 60 ° using a mild detergent in 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.

Disinfection

The product without accessories and cushion can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Frame is adjustable in height
- Adjustment options for the foot rest, it is possible to lock the angle and fasten the telescopic tubes in position
- Seat back is adjustable and can be locked in position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

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© 2018 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 A/S