SELKÄTUET

AIR
Physio

POSTURE
Physio

KID
Physio

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1 Introduction

The PHYSIO backrest is designed and developed to provide a suitable support for a wide variety of wheelchair users.

That is possible thanks to a wide choice among models, sizes and shapes combined to light weight, comfort, and functionality.

Choosing the correct backrest helps the user to maximize function and increase the sitting tolerance.

Rehateam s.r.l. recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a PHYSIO backrest is appropriate. The PHYSIO backrest, by virtue of its hardware, perfectly adapts itself to most folding or rigid wheelchairs.

FOR THE DEALER: this manual MUST be handed to the user of this wheelchair.

FOR THE USER and/or FOR THE ATTENDANT: read this manual in all its parts and keep it for possible future reference.

For visually impaired people, this document is also available in PDF format on our web site www.rehateamprogeo.com.

2 Symbols

It indicates a potentially hazardous situation, which, if not avoided, may cause minor personal injuries or property damage.

It indicates a prohibited action.

It indicates important instructions or special information.

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3 Safety

Physio backrests have to be assembled and adjusted only by personnel who is qualified and authorized by Rehateam s.r.l.

Installing a back support on a wheelchair may affect the centre of gravity of the wheelchair. It is suggested to check its stability and, if necessary, install an anti-tip device.

Maximum load 120 kg

Check your skin conditions very frequently after the installation of a new postural system and possibly consult your doctor or therapist.

If the backrest positioning does not correspond to personal needs, as appropriate, consult your doctor, therapist, or device supplier.

BURN HAZARD! The components of the wheelchair may heat up if they are exposed to strong solar radiation.

Do not use the backrest, side support pads and swing-away laterals, in direct contact with the skin; it is recommended that the user wears clothing with the aim of avoiding possible allergies and/or irritations of the skin.

Do not use the backrest if this is not well fixed in the position of use, in fact, that may lead to damage or injury. If the backrest is not well fixed, consult device supplier.

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Never use the backrest to push or lift the wheelchair. An improper use may cause damage to the backrest and injury to the user.

The instructions in this manual refer to the models:
Physio Air (carbon shell)
Physio Posture (aluminum shell)
Physio Kid (aluminum shell).

Notify the manufacturer in case of serious accidents that occurred in relation to the device.

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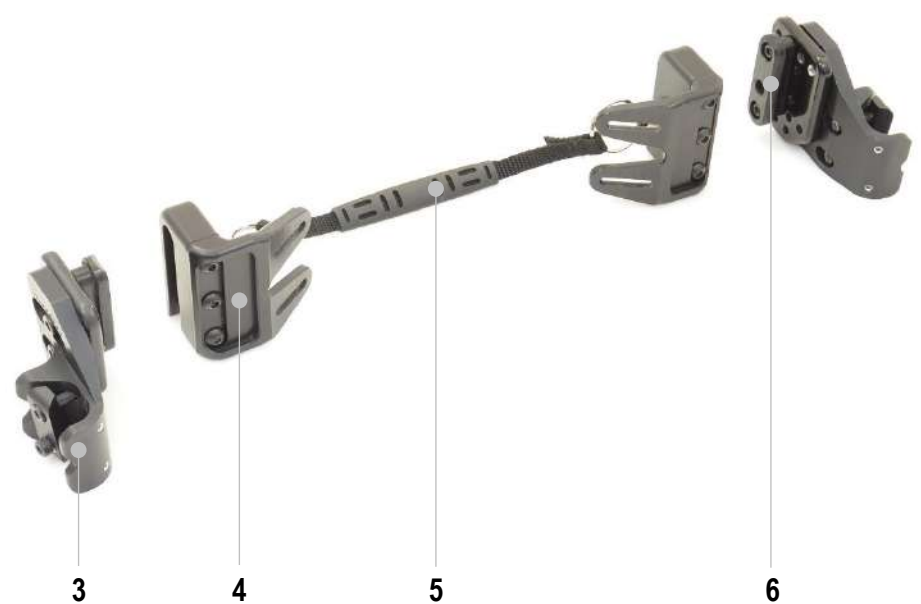
4 Description of parts

- 1 backrest upholstery
- 2 shell (aluminium or carbon)
- 3 hardware
- 4 clamp
- 5 release string
- 6 adjustment plate and locking wedge

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5 Backrest upholstery

Removing the upholstery

- 1) Insert your hand between the upholstery and the shell until you reach one of the two double tabs **A**. Join the two tabs and push them out.
- 2) Lift the padding and, now with greater ease, release the second double tab fixing point.
- 3) The 52 cm high back has a double tab fastening in the lower part and a fitting fastening in the upper part. Once the double tab has been released as described above, the padding comes off immediately.

Mounting the upholstery

- 4) Pay attention to the upper and lower part of the upholstery which must correspond with that of the shell.
- 5) Join the double tab **A** of one of the two flaps, insert it in the respective slot and spread it apart to fasten it.
- 6) Accompany the upholstery towards the other end, bring the flap towards the outside of the shell, join the two tabs, pass them through the slot and keep them in place.
- 7) Insert your hand between the upholstery and the shell until you reach the double tab and spread it apart to fasten it.
For the 52 cm high back, fasten the lower part with the double tab as described above and then put on the upper part.

Removing and inserting the foam

- 8) To remove it, open the pocket in the back of the cover to insert a hand.
- 9) Grasp the foam rubber, bring the two side ends of it together and pull it out.
- 10) To insert it, open the pocket, roll up the foam and push it inside the cover.
- 11) Evenly distribute the foam around the inside perimeter of the cover.

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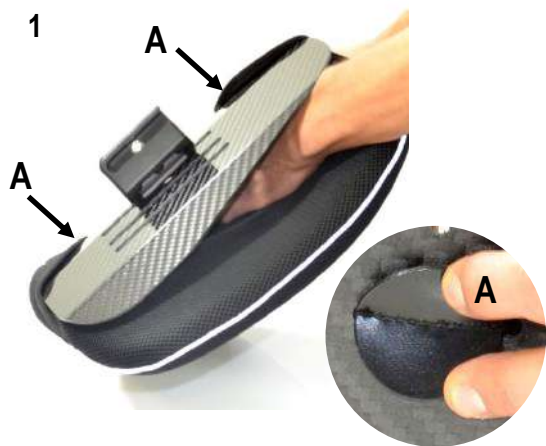
6 Mounting / removing the backrest

The Physio backrest is designed with a very simple and safe mounting/removing system.

Hooking

- 1) Take hold of the backrest on its upper and lower sides.
- 2) Align the two locking blocks with the two lock wedges.
- 3) Scroll the two locking blocks down until the systems automatically locks.

Removal

- 4) With one hand, take hold of the release string in the middle and pull it upwards to release the locking pins.
- 5) At the same time, taking hold of the upper side of the backrest, guide the vertical movement until the two locking blocks get off the lock wedges.
- 6) Finally, remove the backrest. The removal results very easy if the vertical movement of the backrest is parallel with the backrest tubes.

Always check the backrest is locked before using the wheelchair. To do so, take hold of the backrest and, while pulling it upward, it should not move.

Do not grab the release string to move the wheelchair with or without the person sitting on it, the backrest may release and cause injury or damage.

If mounting/removing operation results difficult, consult your device supplier.

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7 Accessories

Physio accessories are specially designed and manufactured to be used in combination with Physio Air, Physio Posture and Physio Kid backrests. Accessories designed by other manufacturers have not been tested by Rehateam s.r.l.. Therefore, it is not recommended to use them. The installation and use of non-original accessories make the warranty on the product void.

Side support deep pad (00)

The side support pad (ambidextrous) allows you to bring the side support to 15 cm on one or both sides of the backrest. The side support pad for all backrests fits the same way.

Spinte laterali swing-away (01)

1) To turn the swing-away lateral outwards, take hold of it in its lower part, pull it upwards and then rotate it.

2) To put the lateral in the position of use, pull it upward and swing it inside until it will lock with a downward movement.

Always check the pad is locked before using the wheelchair.

Headrest (02)

3) To mount the headrest, loosen the lever **A** and insert the tube **B** into the support until the stop ring **C** leans on the support. Tighten lever **B**.

4) To remove the headrest, loosen the lever **A** and remove the entire headrest.

Always check the lever A securely tightens the tube B before using the wheelchair.

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8 Installation and adjustments

The installation and any adjustment can be carried out exclusively by qualified and authorized by Rehateam s.r.l. personnel.

It is forbidden to carry out any modifications, even when possible, to the original design.

Any adjustments and/or any modification that is carried out by non-authorized personnel will immediately void the warranty on the product and it relieves Rehateam s.r.l. from any responsibility on any malfunctioning and/or damage due to such adjustments/modifications.

Always contact Rehateam s.r.l. and its technicians for any non-standard requirements or modifications to allow them to evaluate such modifications and verify that they will not compromise the normal and safe use of the wheelchair.

After every adjustment made to the backrest, check carefully that all parts are correctly fixed. Check that all screws and nuts are tightened and that all moving parts are functioning correctly.

After any adjustment, always test the wheelchair before normal use, possibly assisted by an attendant or a technician.

The possible adjustments for all Physio backrests are:

HEIGHT – DEPTH – ANGLE

The instructions to properly carry out the installation and the adjustments, are included in the corresponding SERVICE MANUAL that is available on the web site www.rehateamprogeo.com

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9 Maintenance and cleaning

Rehateam s.r.l. recommend making a general check of the whole system once a week to ensure the backrest can be used in complete safety.

In case of any anomaly or damaged or broken parts, do not use the product and immediately contact the device supplier.

To clean the cover, remove the foam pad following the instructions on the chapter 5 "Backrest upholstery" and turn the cover inside out. Hand wash in lukewarm water (30°). This textile product must not be treated with chlorine, must not be ironed, should be dried in moderate warm air (never in drier machine).

The foam can be washed with cold water and neutral detergent and should be dried in the open air.

When the cover and the foam are dry, insert the foam pad as described in chapter 5 "Backrest upholstery".

The shell and the hardware can be wiped clean with soap and water. Do not immerse any part of the hardware underwater.

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10 Disinfection and reuse

While using the Physio system, all surfaces that come into contact with the user should be treated frequently, or whenever the need arises, with a spray disinfectant. It is not normally possible to guarantee safe disinfection on the sewing.

Therefore, we recommend that the seat and back covers be changed and appropriately discarded in the event of bacterial contamination, in compliance with current regulations.

In case of reuse of the backrest, the same type of treatment must be carried out very carefully before using the backrest again.

The first thing you should know is that **cleaning is not the same as disinfecting**. By cleaning we remove germs and dirt from surfaces, but we do not kill germs, although removing them decreases the amount and risk of spreading infections. On the other hand, with a disinfection, what we do is kill microorganisms such as bacteria through using chemicals.

This process does not necessarily clean dirty surfaces, but by killing germs on the surface, it can further reduce the risk of spreading the infection.

Keeping that in mind, the correct process to follow with the wheelchair is to first perform **routine cleaning and then disinfect it** using a sanitary surface disinfectant.

These products are bactericidal and virucidal.

You can find more information about these products on the website of the Ministry of Health or other reliable sources.

We also recommend that you carefully read the manufacturer's instructions for correct use and disinfection as well as to avoid possible contraindications.

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11 Storage

If the wheelchair is not used for a medium-long period (over 4 months), it is advisable to store it clean in a dry and covered place and inside a box (preferably that of the original packaging). At the time of re-use, it is necessary to carry out a general check of the wheelchair following the same points of paragraph 9 "Maintenance and cleaning".

12 Disposal/recycling of materials

When the backrest is to be disposed of (end of life of the product), it is necessary to consider any local regulations in force for waste disposal or recycling. This includes cleaning or disinfecting the backrest before disposal.

Below, a description of the materials used.

- Aluminum: hardware, shell
- Steel: bolts and nuts, pins
- Plastic: lock wedge
- Carbon fiber: shell
- Upholstery: textile with expanded foam
- Packaging: plastic bags, cardboard

Contact your authorized disposal centre for disposal or recycling. Alternatively, the backrest can be returned to the retailer for disposal.

In the case of disposal, be aware of any broken or damaged parts which could create a situation of potential danger of cutting or pinching injuries. It is recommended to use suitable protections.

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Min. 0.65 kg
 Max. 1.15 kg

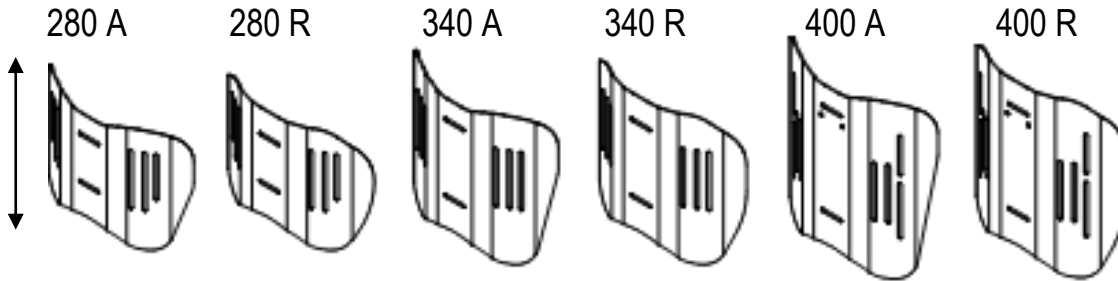
13 Techical data

PHYSIO AIR – carbon fibre shell
 PHYSIO POSTURE – aluminium shell
 PHYSIO KID – aluminium shell
 HARDWARE
 SIDE DEEP PAD SUPPORT (1)
 SWING AWAY LATERAL (2)
 HEADREST (3)

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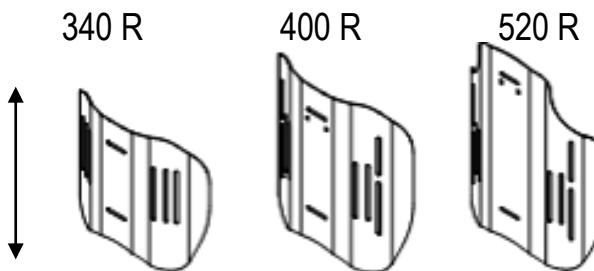
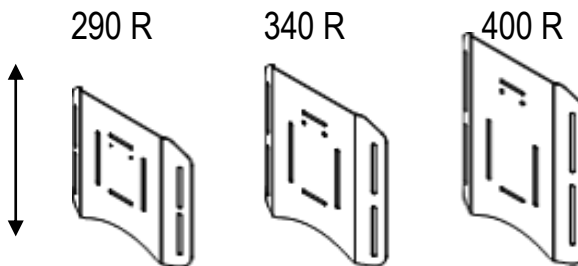
14 Warranty

Every Physio backrest is carefully inspected and tested to provide optimum performance. Each backrest is guaranteed to be free from defects for a period of 24 months from the date of purchase. Should a defect in material or workmanship occur within 24 months of the original date of purchase, Rehateam s.r.l., at its own option, repair or replace it free of charge. This warranty does not apply to punctures, tears or burns, nor to the removable cushion cover.

All warranty claims must be made through the dealer/retailer where the cushion was purchased. Except for express warranties made herein, all other warranties of fitness or use are excluded. There are no warranties that extend beyond the warranties provided above.

Remedies for breach of this warranty are limited to repair or replacement of goods. In no event shall damages for breach on any warranty include any consequential damages or exceed the original cost of the backrest.

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<http://www.rehateamprogeo.com> info@rehateamprogeo.com

Azienda certificata TÜV Rheinland secondo la norma UNI CEI EN 13485 : 2016

Certified company TÜV Rheinland in compliance with directive UNI CEI EN 13485 : 2016

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Rehateam S.r.l.