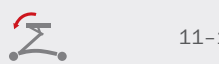
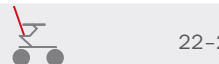
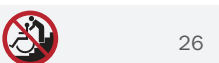
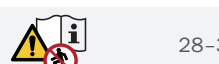
High-low:xo User Guide

With gas spring height adjustment
High-low:xo indoor: EN 12182
High-low:xo outdoor: EN 12182 & 12183

2021-10-rev.2.0


www.etac.com

| | | | |
|---|-------|----------------------------------|--------------------|
|  | 3-4 | ENGLISH DEUTSCH | 41-44 45-49 |
|  | 5 | NEDERLANDS DANSK | 50-53 54-57 |
|  | 6-7 | NORSK SVENSKA | 58-61 62-65 |
|  | 8 | SUOMI ESPAÑOL | 66-69 70-73 |
|  | 9-10 | FRANÇAIS ITALIANO | 74-77 78-81 |
|  | 11-13 | PORTUGUÊS PORTUGUÊS DO BRASIL | 82-85 86-89 |
|  | 14-15 | РОССИЯ POLSKI | 90-95 96-99 |
|  | 16-20 | ČESKY 中國 | 100-103 104-108 |
|  | 21 | 日本 ΕΛΛΗΝΙΚΗ | 109-115 116-120 |
|  | 22-23 | | |
|  | 24-25 | | |
|  | 26 | | |
|  | 27 | | |
|  | 28-32 | | |
| mm (inch) | 33-38 | | |
|  | 39-40 | | |
| | | | |

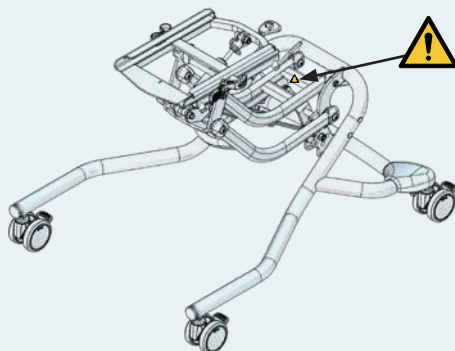! 05

Size 1


Size 2


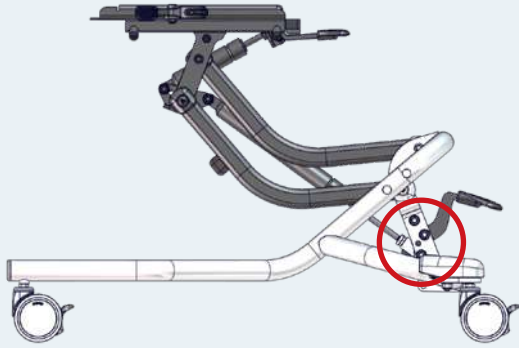
1.

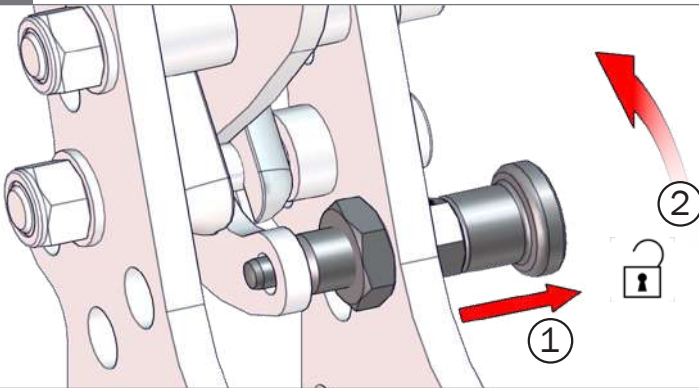2.

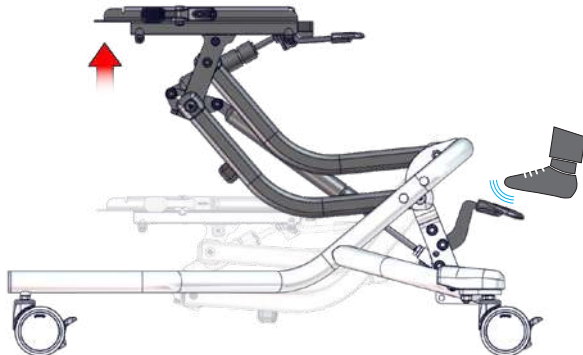
3.


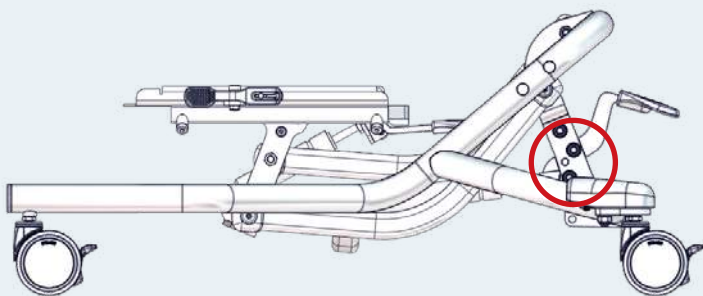
1.

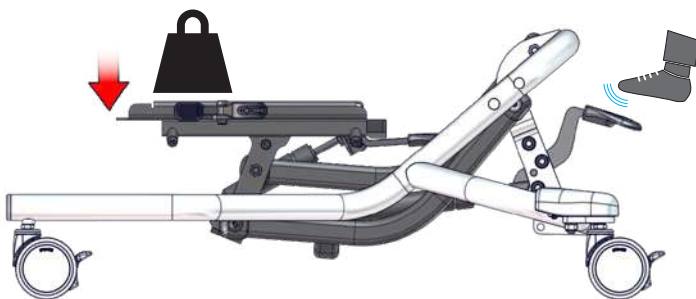2.

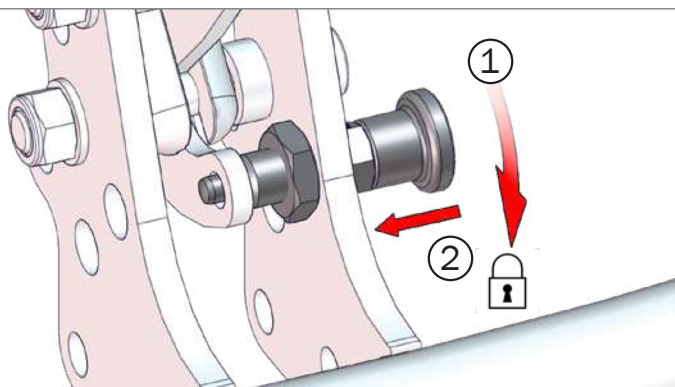3.

⚠ 05

Size 1

<70 kg
<154 lb

Size 2

<75 kg
<165 lb

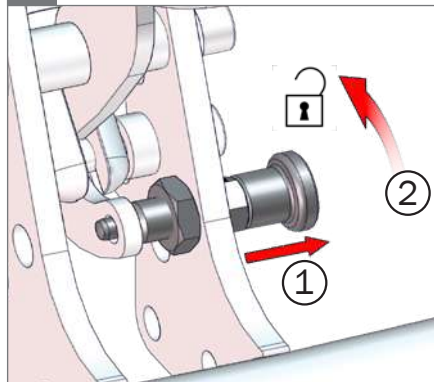
1.

2.

3.


4.


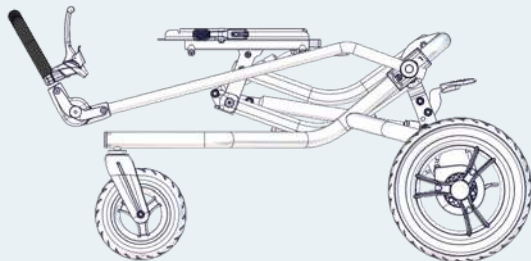
1.

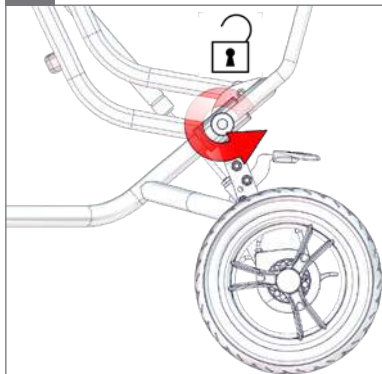
2.

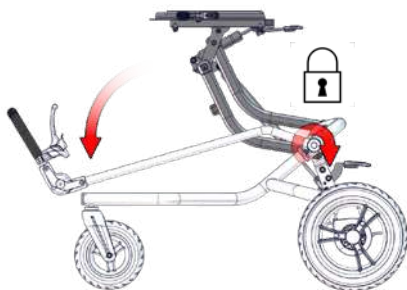
3.

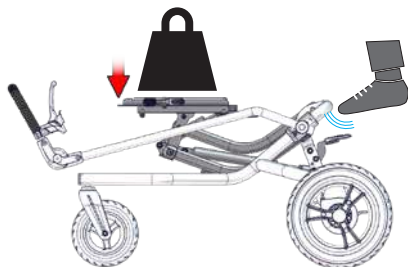4.

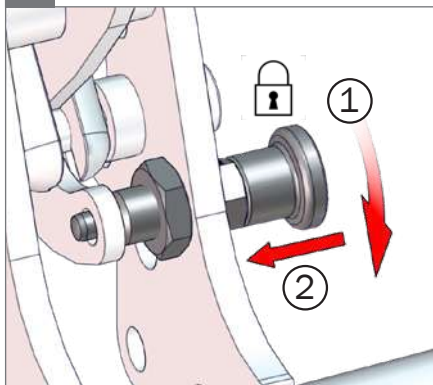
5.


1.

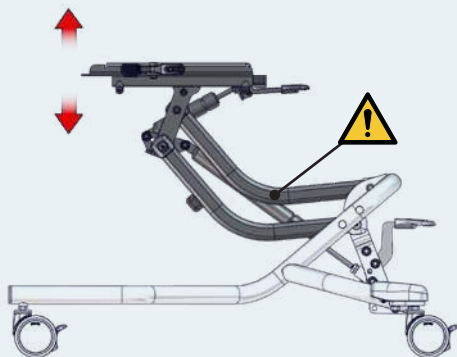
05



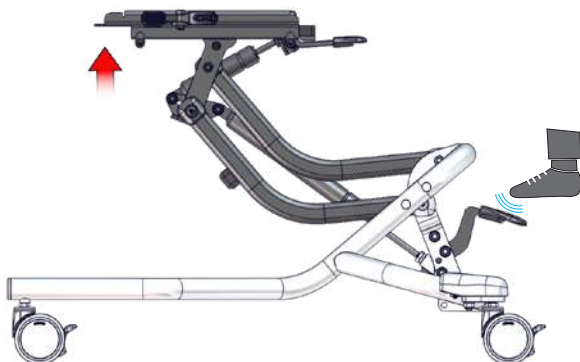
2.

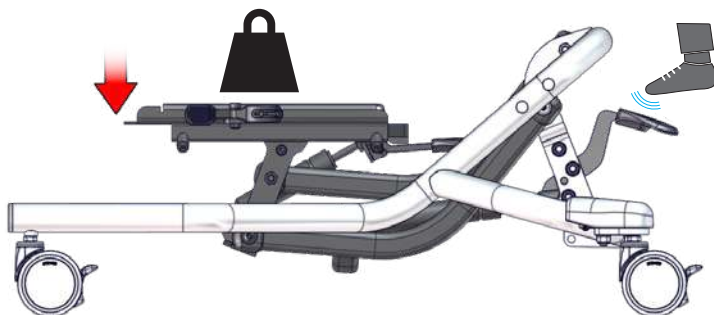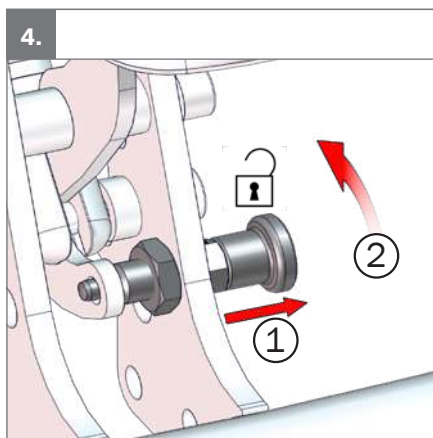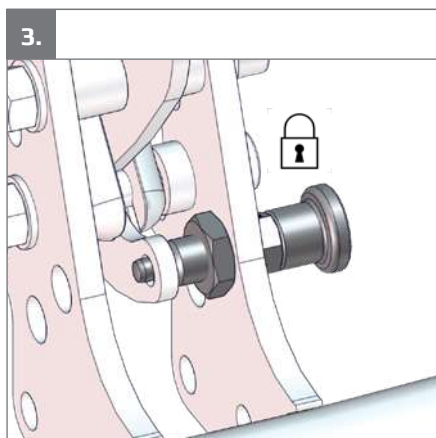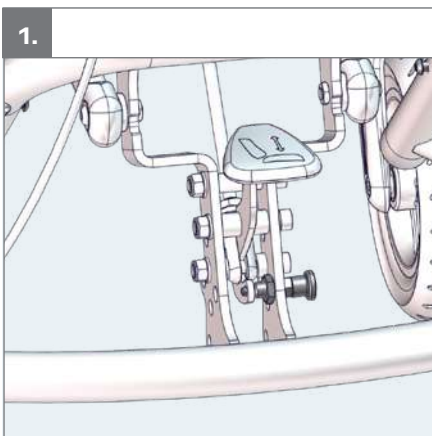
3.


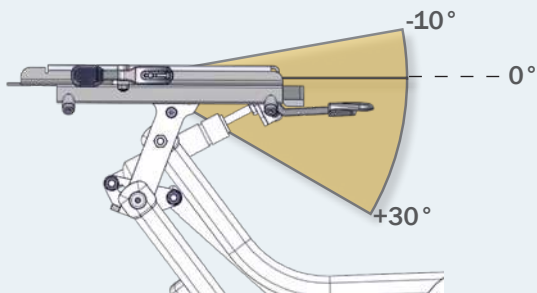
1.

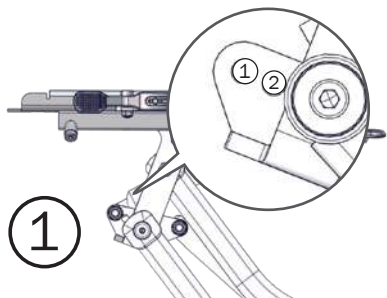
2.

 $\angle = 0-30^\circ$ 

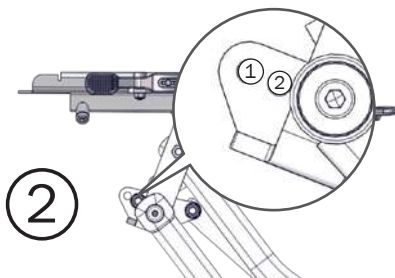
3.

 $\angle = -10-20^\circ$ 

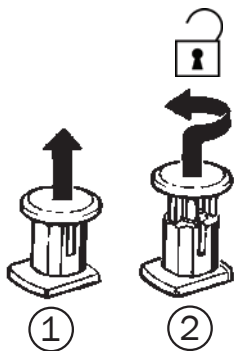4.

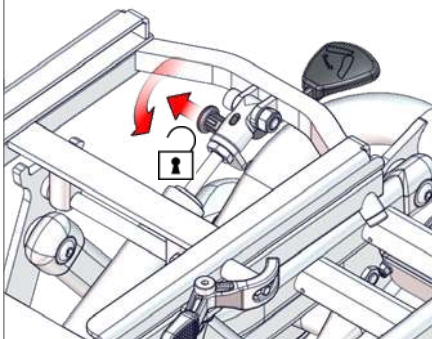
5.


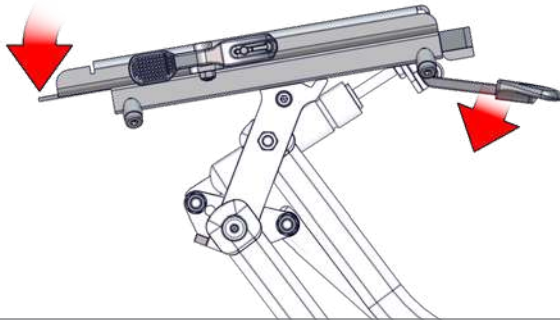
6.

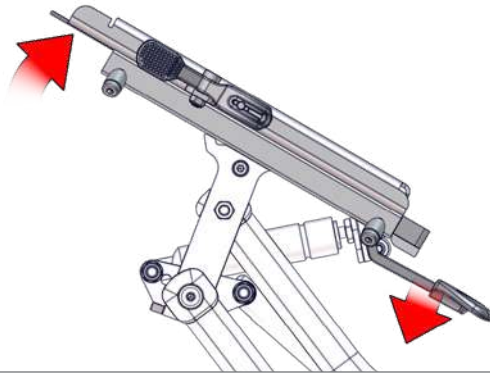
7.

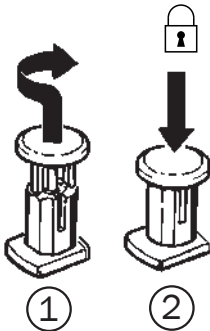8.

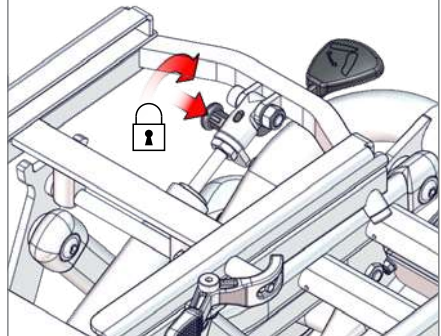
9.


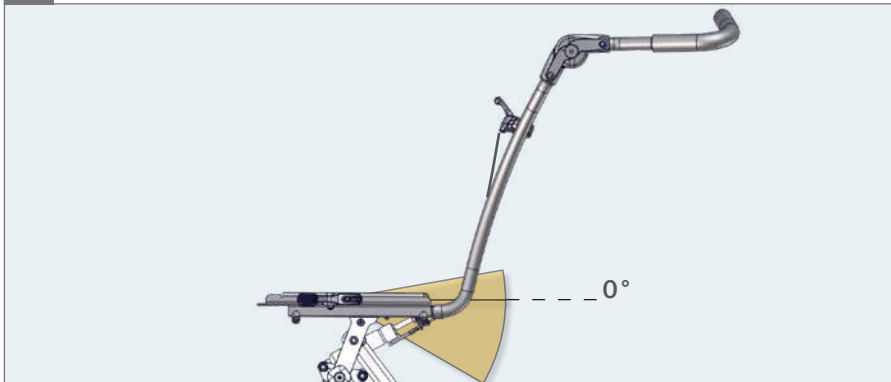
1.

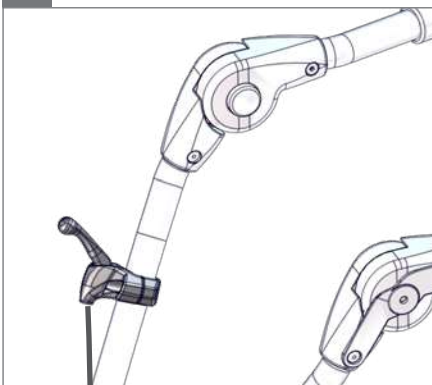
2.

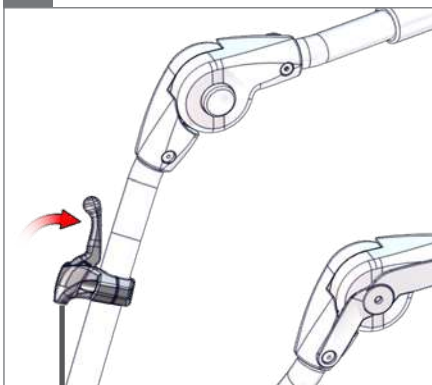
3.

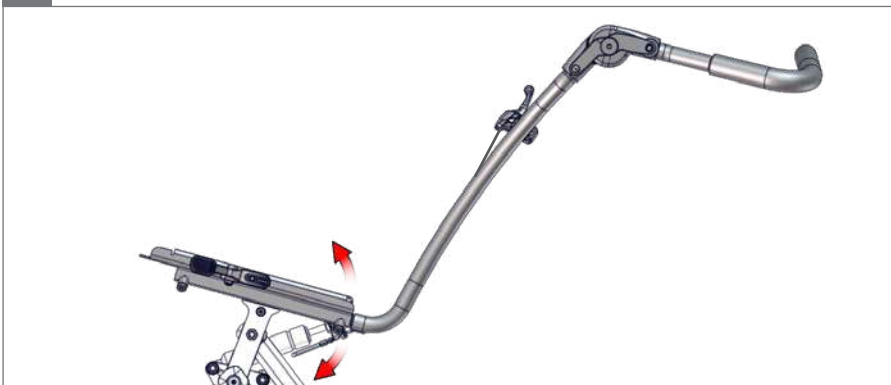
4.

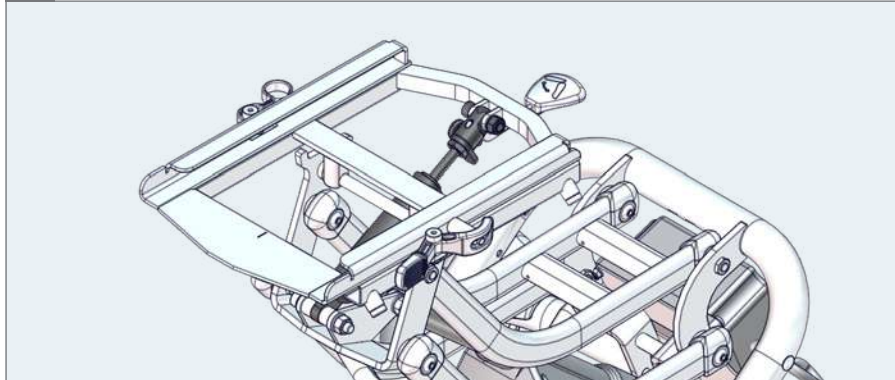
1.

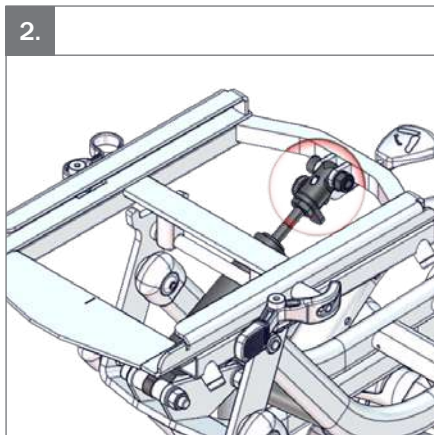
2.

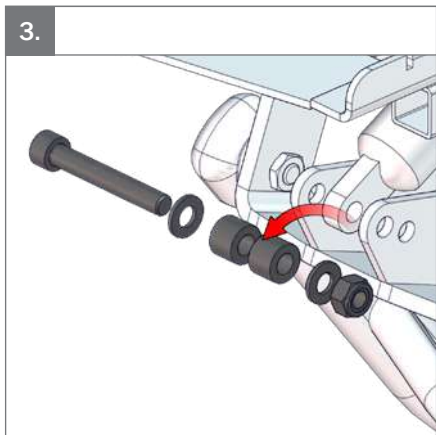3.

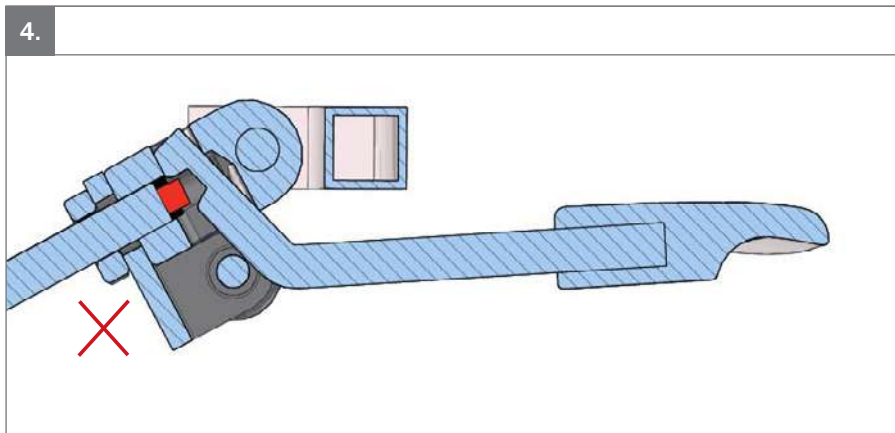
4.


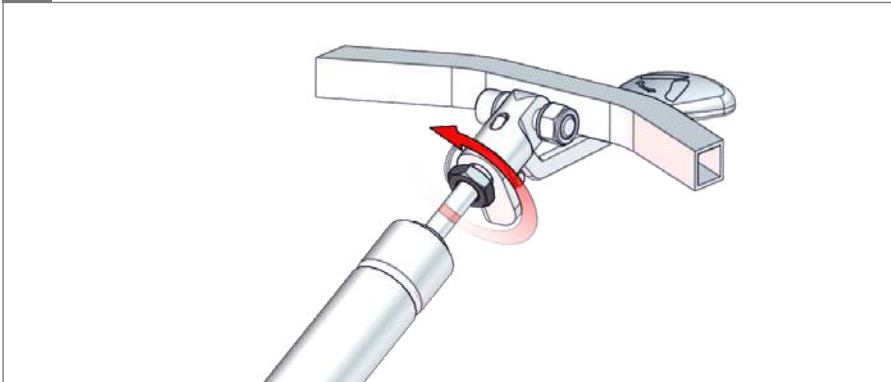
5.

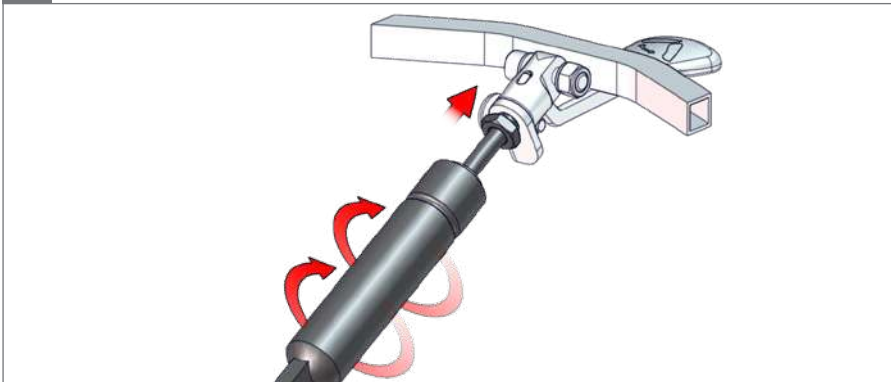
6.

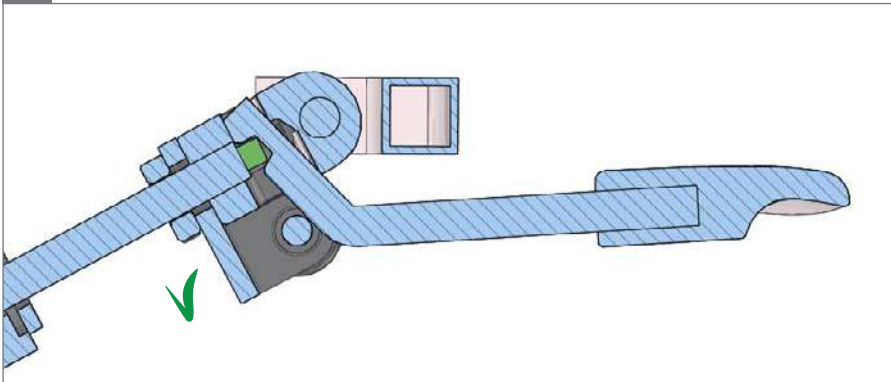
7.

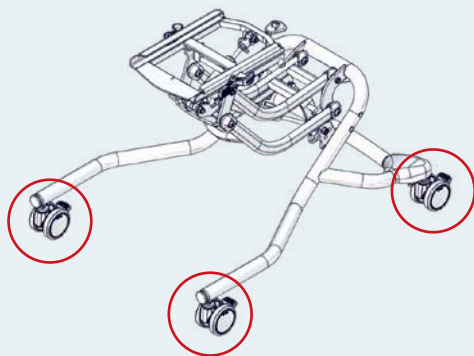1.



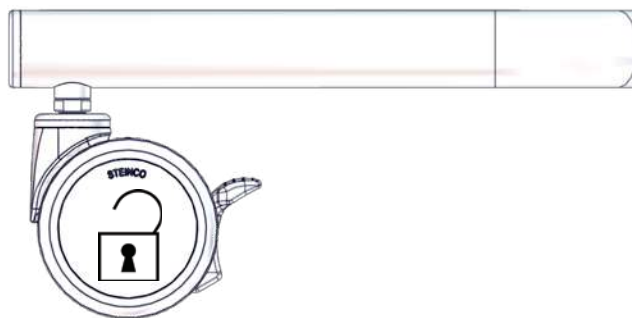
2.

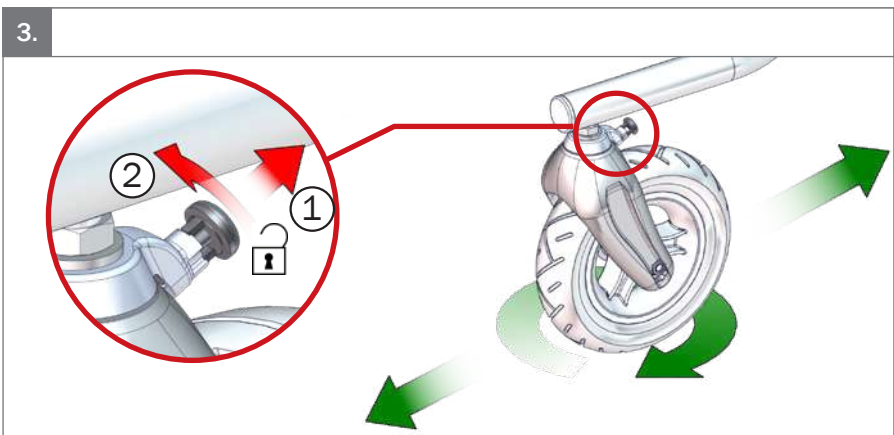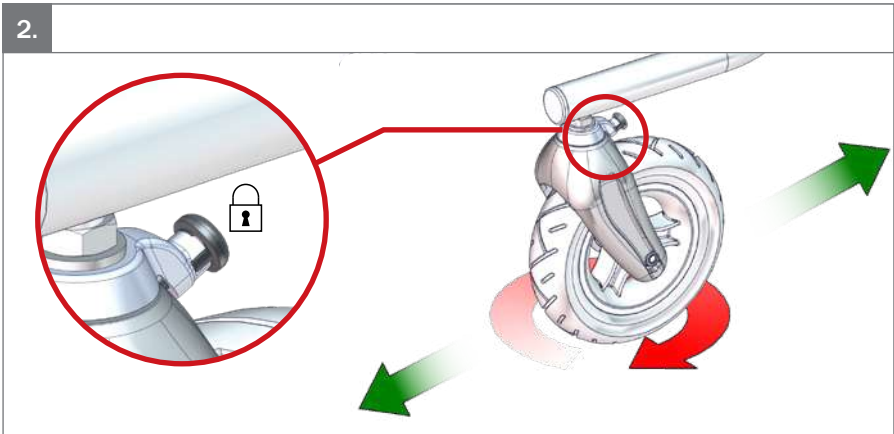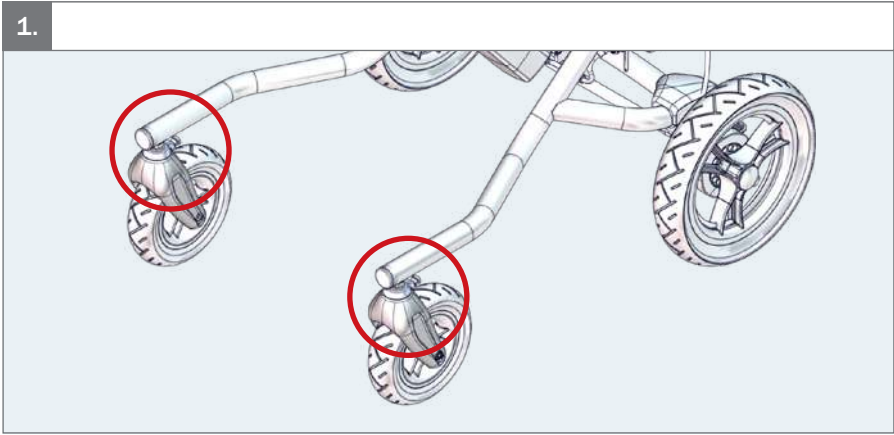
3.


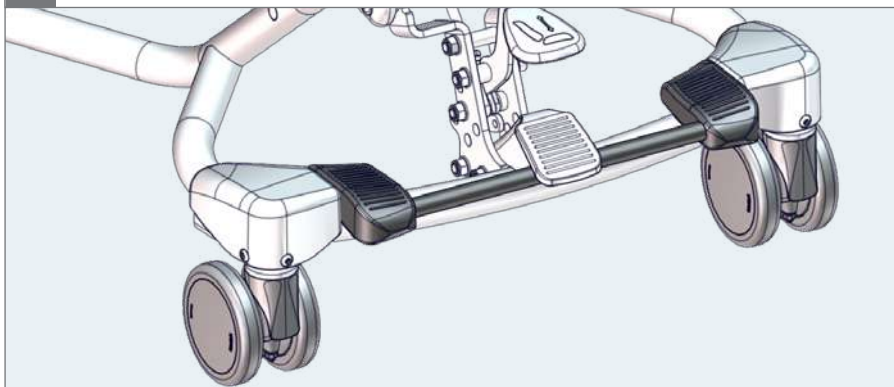
1.

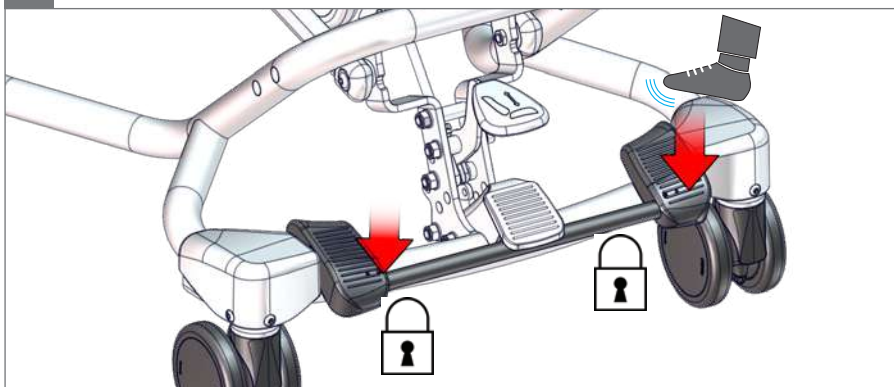2.

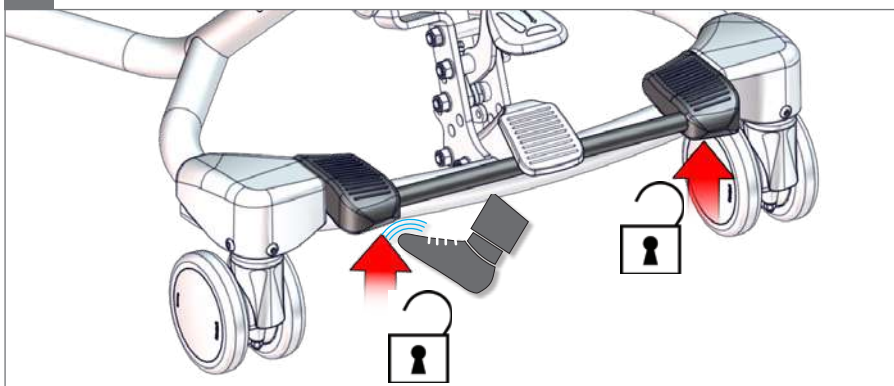
3.


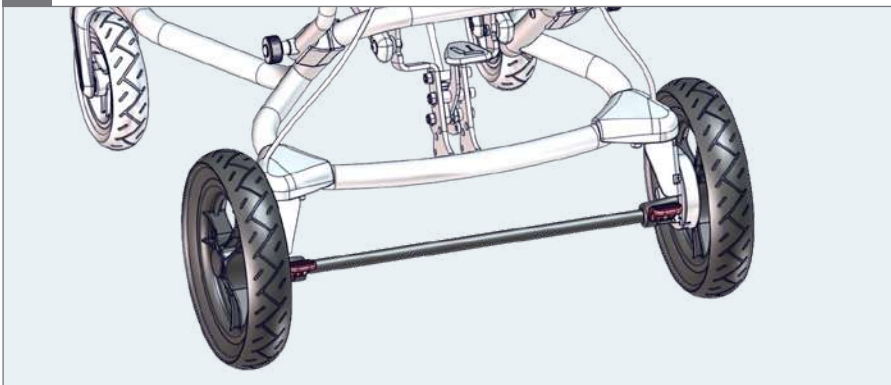
1.

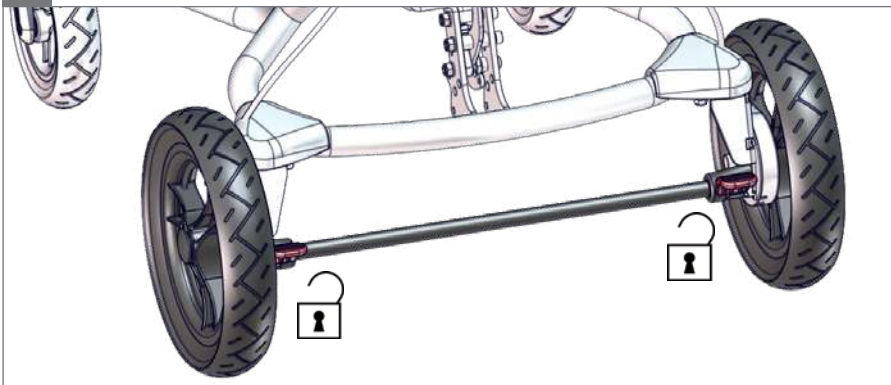2.

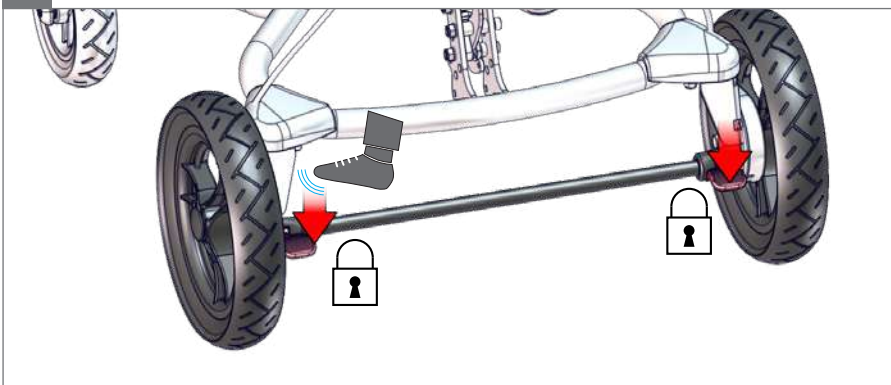
3.


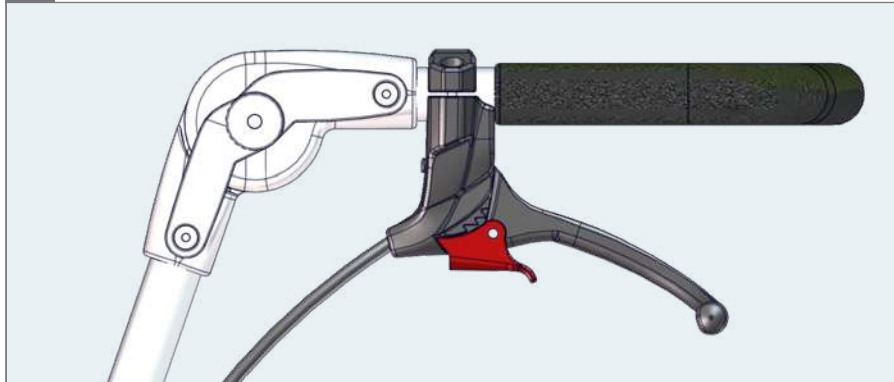
1.

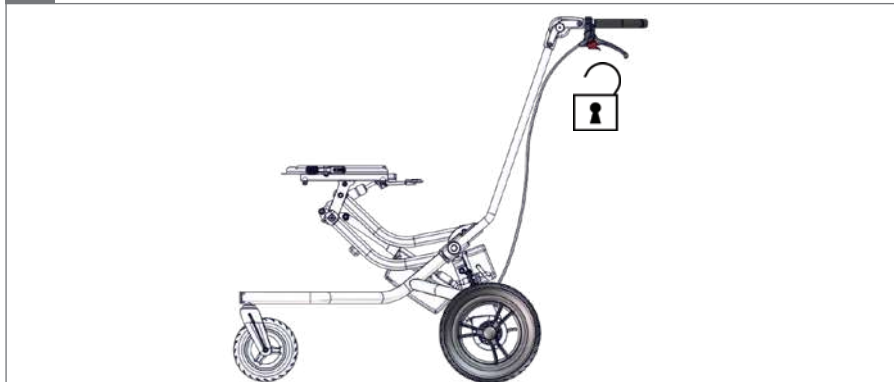2.

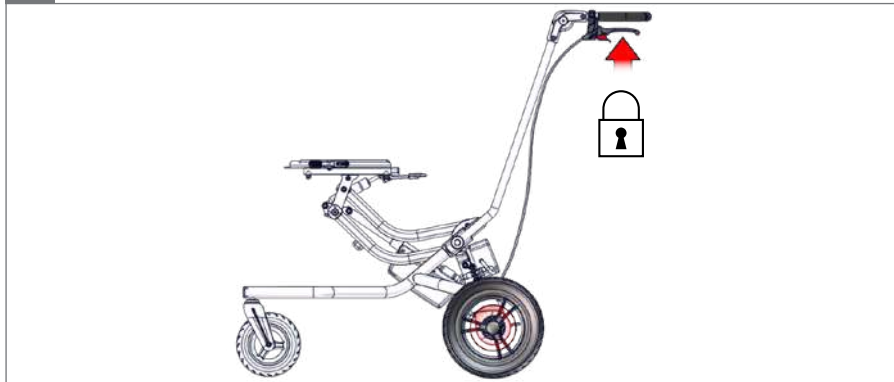
3.

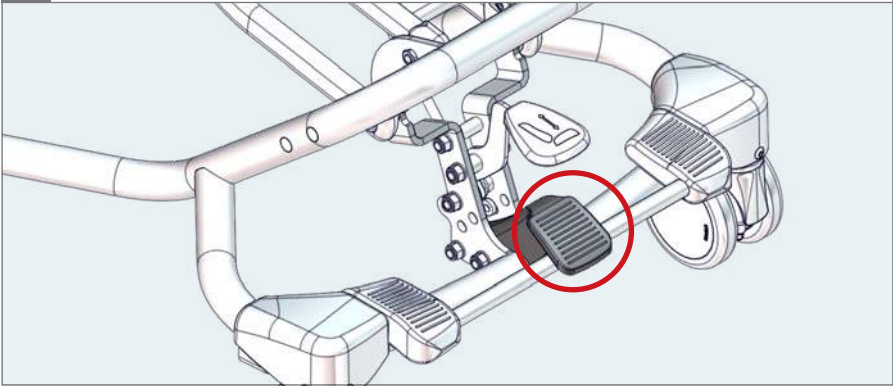
1.


2.

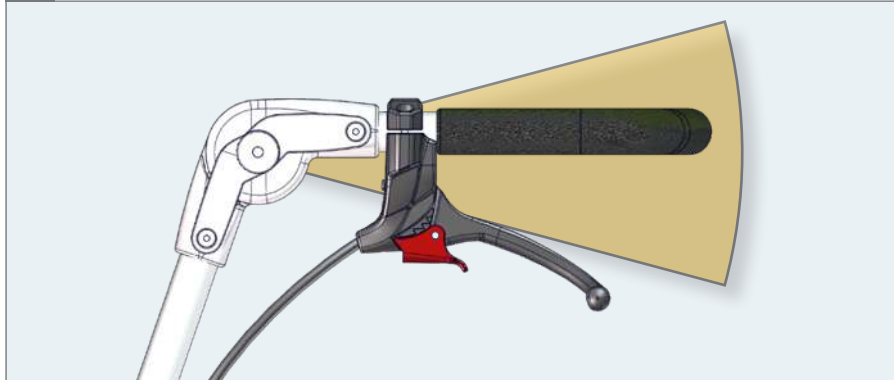1.

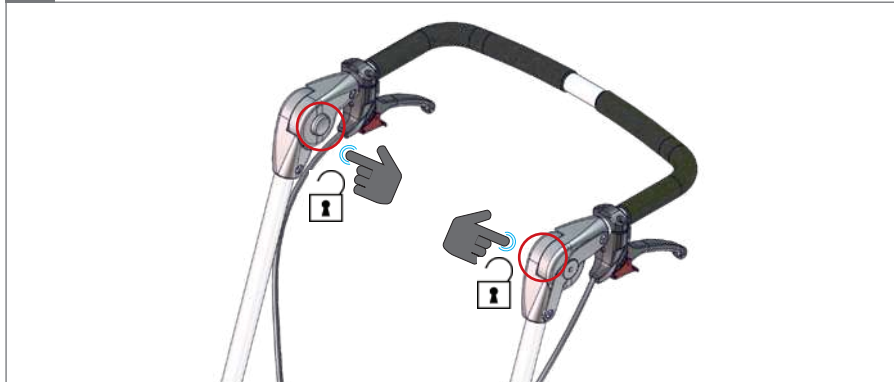2.

3.

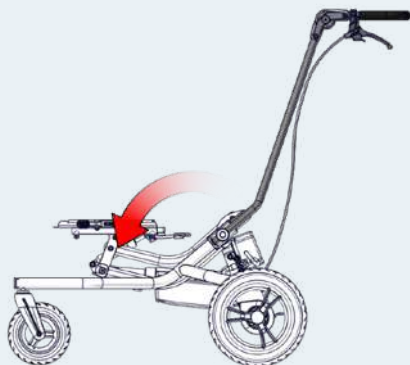
4.



1.

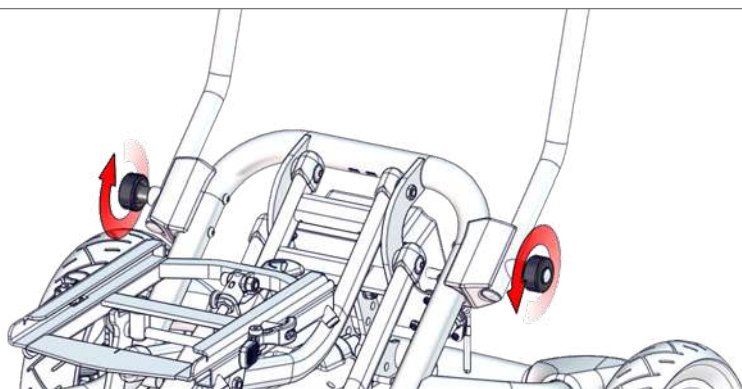2.

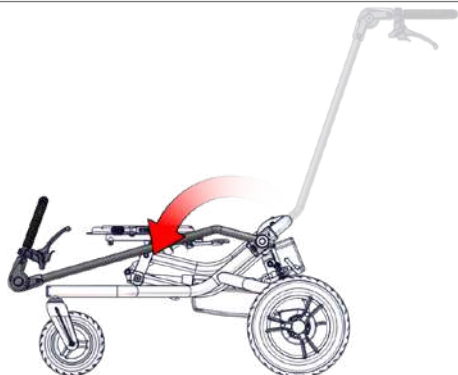
3.


1a.

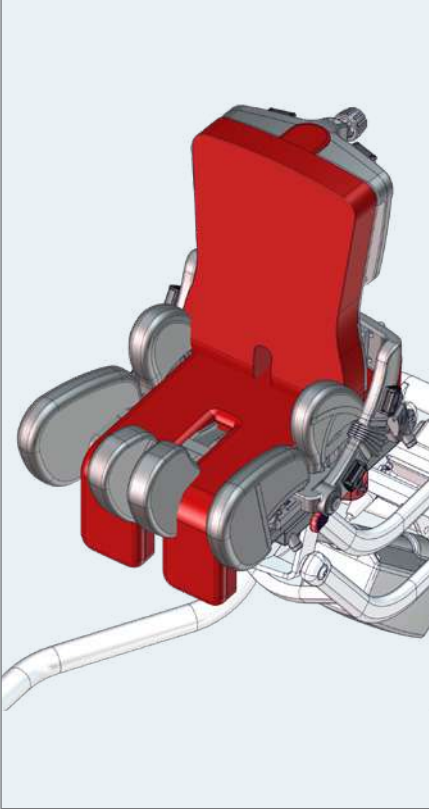Size 1 with x:panda size 1 - 2
Size 2 with x:panda size 3
Size 1 with Panda Futura size 1 - 4


1b.

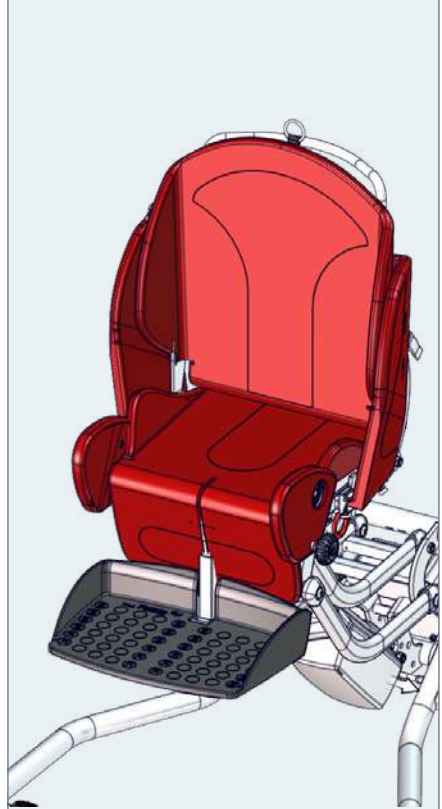
Size 1 with Stingray size 1
Size 2 with Stingray size 2

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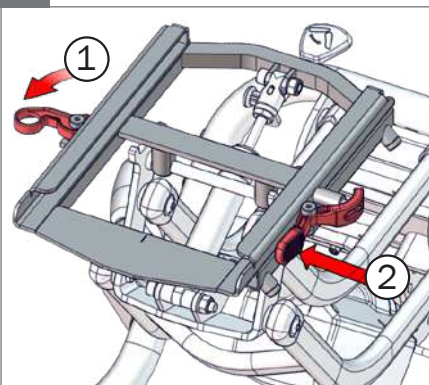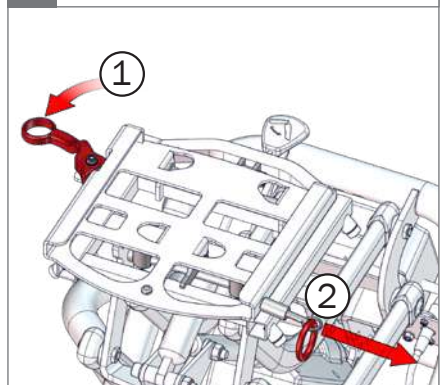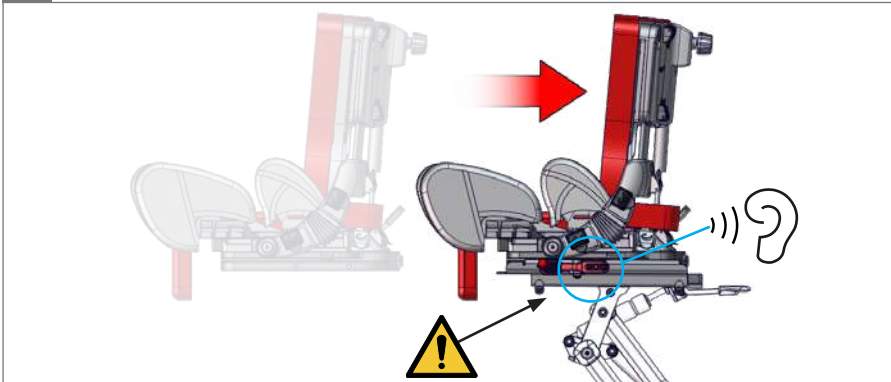
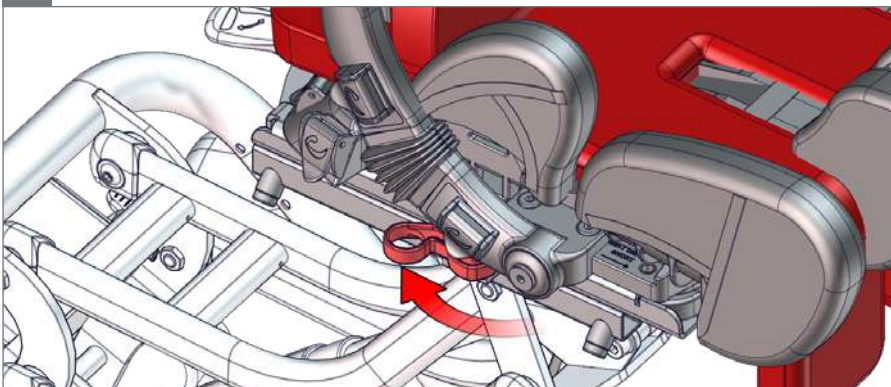
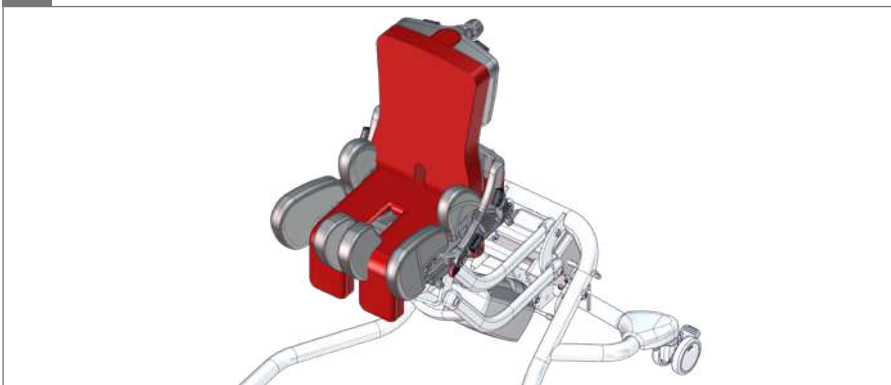
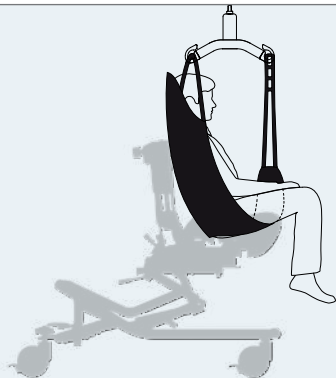
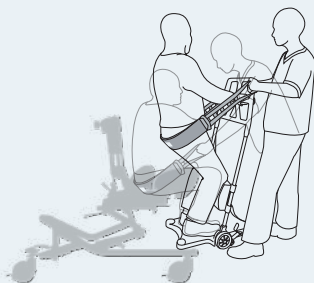
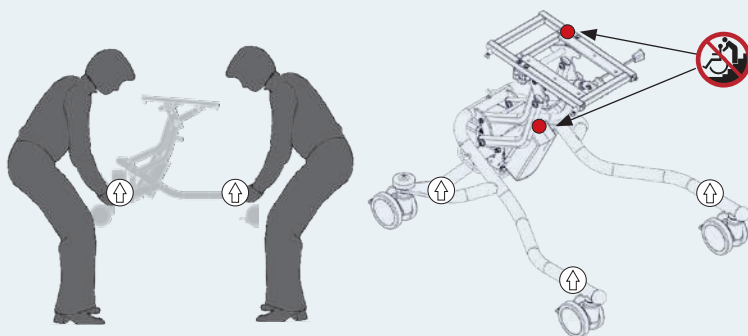
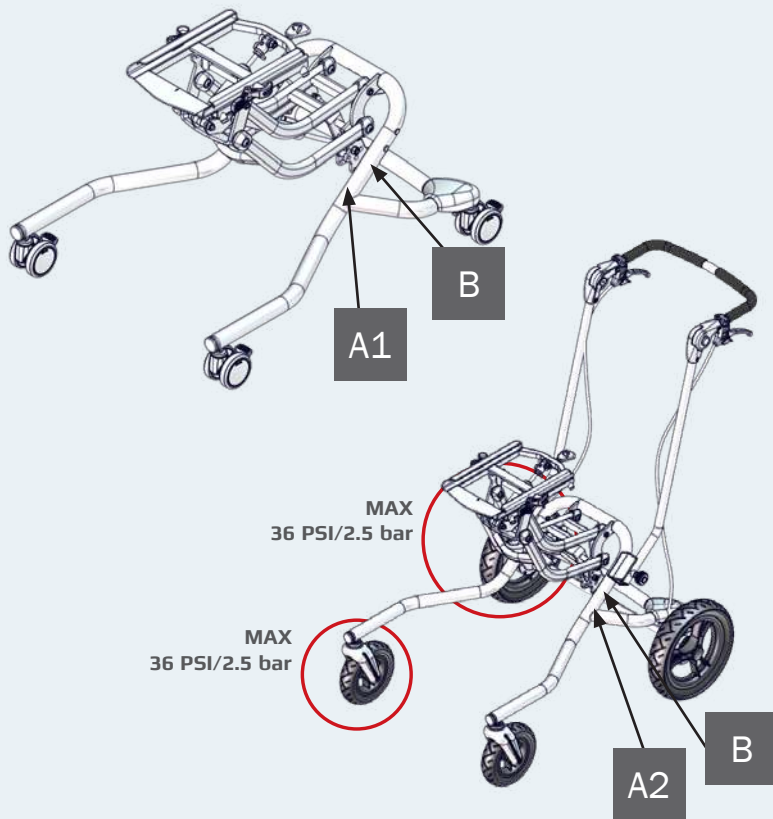
EN: Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger

01 = Entrapment Hazards
02 = Tilting Hazards
03 = Transportation in motor vehicles
05 = Squeezing Hazard

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Size 1 - 2

| Indoor | 1 | 2 |
|---|------------------------|------------------------|
|  Overall height (A), to push brace* | 1100 (43¼") | 1100 (43¼") |
|  Seat surface height at front edge (B) (adapter plate) with 75 mm castors | 170–590 (6¾–23¼") | 215–590 (8½–23½") |
|  Seat surface height at front edge (B) (adapter plate) with 100 mm castors | 200–620 (7¾–24½") | 245–620 (9¾–24¾") |
|  Seat surface height at front edge (B) (adapter plate) with 125 mm castors | 240–660 (9½–24") | 285–660 (11¼–24") |
|  Seat plane angle (adapter tilt) (C) | (-10)–30° | (-10)–30° |
|  Length (D)* | 655 (25¾") | 795 (31¼") |
|  Overall length (E) | 670 (26¾") | 830 (32¾") |
|  Overall width (F) | 560 (22") | 620 (24¾") |
|  Castors diameter | 75/100/125 (3/4/5") | 75/100/125 (3/4/5") |
|  Minimum turning radius | 400 (15¾") | 475 (18¾") |
|  Max. load | 70 kg (154 lb) | 75 kg (165 lb) |
|  Mass device* | 13 kg (29 lb) | 13 kg (29 lb) |
|  Max. rated load* | 85 kg (187 lb) | 90 kg (198 lb) |
|  Static stability Sideways, downhill and uphill | 10° | 10° |

EN: *Measurements: The High-low:xo measurements above are based on the use of 75 mm castors with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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

| Seat plane angle (C) in combination with: |  |  |
|---|---|---|
|  x:panda size 1 - 2 | (-10)-30° | N/A |
|  x:panda size 3 | N/A | (-10)-30° |
|  Panda Futura size 1 - 4 | 0-30° | N/A |
|  Stingray size 1 | (-10)-20° | N/A |
|  Stingray size 2 | N/A | (-10)-20° |

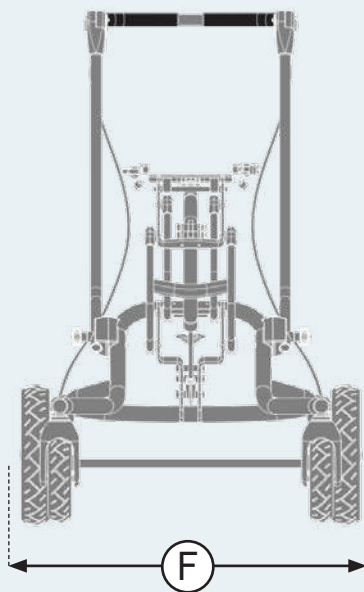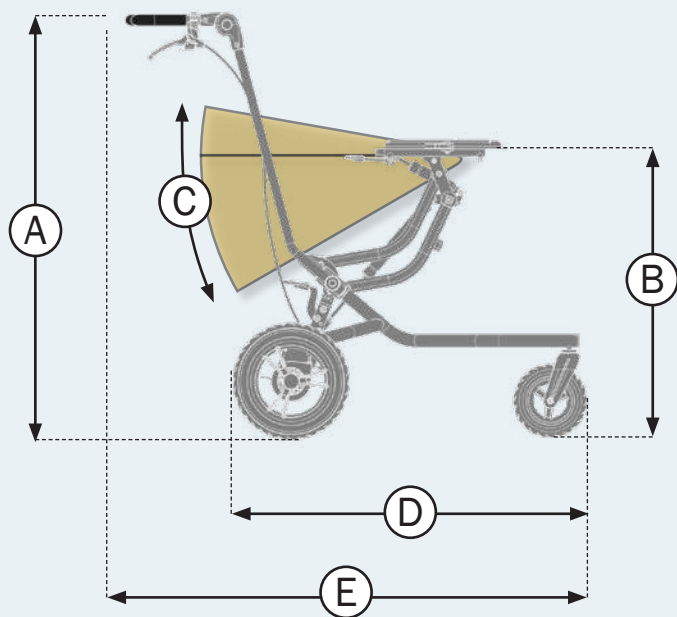Size 1 - 2

| | Outdoor | 1 | 2 |
|---|---------|--------------------|--------------------|
| Overall height (A), to push brace | | 1045 (41¼") | 1045 (41¼") |
| Seat surface height at front edge (B) (adapter plate) | | 300–720 (11¾–28¾") | 345–720 (13½–28¾") |
| Seat plane angle (adapter tilt) (C) | | (-10)–30° | (-10)–30° |
| Length (D)* | | 795 (31¼") | 800 (31½") |
| Overall length (E) | | 1070 (42½") | 1170 (46½") |
| Overall length with leg rest | | 1115 (43⅞") | 1190 (46⅞") |
| Overall width (F) | | 660 (26") | 700 (27½") |
| Foot support to adapter plate | | 50-290 (2-11½") | 50-290 (2-11½") |
| Leg to seat surface angle | | (-30)–30° | (-30)–30° |
| Front wheel h×w | | 190×50 (7½×2") | 190×50 (7½×2") |
| Rear wheel h×w | | 290×60 (11½×2¼") | 290×60 (11½×2¼") |
| Folded height | | 460 (18") | 460 (18") |
| Minimum turning radius | | 875 (34½") | 930 (36½") |
| Horizontal location of axle | | 200–210 (7¾–8¼") | 255–265 (10–10½") |
| Max. load | | 70 kg (154 lb) | 75 kg (165 lb) |
| Min. user mass in transportation | | 22 kg (49 lb) | 22 kg (49 lb) |
| Max. user mass in transportation | | 57 kg (126 lb) | 57 kg (126 lb) |
| Mass device* | | 22 kg (49 lb) | 22 kg (49 lb) |
| Max. rated load* | | 94 kg (207 lb) | 99 kg (218 lb) |
| Mass of the heaviest part | | 18 kg (40 lb) | 18 kg (40lb) |
| Pivot width | | 1180 (46½") | 1250 (49¼") |
| Maximum slope (for use of parking brake) | | 7° | 7° |
| Static stability Sideways, downhill and uphill | | 10° | 10° |

EN: *Measurements: The High-low:xo Outdoor measurements above are based on the use of standard solid wheels with the frame in the most basic configuration. Different settings may result in another set of measurements. As we supply various types of accessories, the weight is listed without.

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
| Seat plane angle (C) in combination with: |  |  |
|---|---|---|
|  x:panda size 1 - 2 | (-10)–30° | N/A |
|  x:panda size 3 | N/A | (-10)–30° |
|  Panda Futura size 1 - 4 | 0–30° | N/A |

9996097791

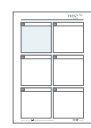9996097792

9996097124

9996097796

9996097731

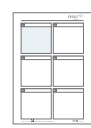
9996097793

Indoor
9996097790

Outdoor
9996097795

9996097799

9996097703

9996097786

9996097743 Height
9996097744 Tilt

9996097794

9996097798

9996097797 size 1
9996097785 size 2 & 3

9996097118

9996097755

ENGLISH

General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The High-low:xo is a frame for a seating system for children and adults with disabilities.

Indications

The device is used together with a seating system (i.e. a modular seating system like R82 x:panda or R82 Panda Futura). It can be used by children and adults with disabilities, who lack the ability to maintain a seated position and move around (Within the room or between rooms / outdoors) without an assistive device. The target group for the device is based on functional ability and anthropometry. It is not based on a specific health condition or age.

The user is the person sitting in the device. The caregiver is the person operating the device. Never leave the user unattended in this device. Ensure permanent supervision by a caregiver.

The use of the device will be discontinued more times during the day where the user will be lying down, standing or sitting in another assistive device or a conventional product.

The outdoor version of the device is approved for transportation for children and adults in the weight range of 22 kg to specified max user weight for transportation.

Contra indications

None.

Precautions

None.

Application areas

The product is for indoor and outdoor use.

Enclosed tools

None.

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745). Further, this product conforms to the requirements according to:

- High-low:xo indoor: EN 12182
- High-low:xo outdoor: EN 12182 & EN 12183

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings.

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Alignment of user weight/mass of the combined seat and base, approval for transportation in vehicles of both products (ISO16840-4 for seat and ISO7176-19 for base)
- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety.

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories including vests and belts which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the etac.com/support.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Presales information

- www.etac.com

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

For information about product safety notices and product recalls, local dealer information are available at the website etac.com

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by a caregiver
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide and who has been given the necessary instructions by a product advisor.
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- When adjusting the seat angle, keep a hand on the seat mounted on the frame to ensure slow angling
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/accessories.
- Always use the product in a moderate pace. Do not run with the product
- Information about transportation can be found in the document "Transportation in Motor Vehicles" supplied with the product
- Information about roadability can be found in the document "Roadability" supplied with the product
- Information regarding frame and seat combinations including max. load in combination can be found at: etac.com
- This product contains small parts which could present a choking hazard if removed from their intended placing.
- Activate the brakes before placing the user into the product
- Ensure the wheels are fully-functional and secure before use
- The correct tyre pressure is printed on the tyre casing and listed in the section for tyre technical data.
- In case of a tyre puncture do not use the product until remedied
- Ensure the stability of the product is maintained before placing the user in it
- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- If the frame has to be used in the lowest position, remove the footrest before adjustment.
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- When mounting a Panda Futura seat size 4 on any High-low:xo frame, you must mount the anti-tip device that comes as an accessory to the product
- The user must not occupy the product when it is lifted from the ground
- Ease of access to and maneuverability in, motor vehicles can be significantly affected by size of device and turning radius, and the smaller device and/or device with a shorter turning radius will generally provide greater ease of vehicle access and maneuverability to a forward facing position. In accordance with ISO 7176-19 Annex D, the product is evaluated regarding the ability to accommodate vehicle-anchored pelvic and shoulder belts. The device receives an overall score of "Good"
- The device conforms to ISO 7176-8, covering requirements and test methods for static, impact and fatigue strengths
- Unoccupied device is suitable for land and air transport
- Depending on the size, the device is intended for occupant mass groups I and II (ISO 7176-5).

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- Do not place more than one user in the product
- When changing user, clean the product and disinfect it if necessary. Carry out necessary adjustments after cleaning.

Product

- Removable parts without use of tool: Foot support and seat. The removal will have adverse effect on the device
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children

Due to variant and settings the wheelchair may become larger than the recommended dimensions.

Compliance with recommended overall width and overall length ensures unhindered use of emergency escape routes. (See table 1)

Table 1

| Recommended dimensions | Ready for use (mm) |
|----------------------------|--------------------|
| Overall length - maximum | 1200 |
| Overall width - maximum | 700 |
| Overall height - maximum | 1200 |
| Turning diameter - maximum | 1000 |
| Pivot width - maximum | 1300 |
| Ground clearance - minimum | 30 |

Due to variant and settings the wheelchair may not meet the limits for wheelchairs transportable by train. Measures can be taken to verify whether the specific product in question meets the limits. (See table 2)

Table 2

| Wheelchairs transportable by train - limits | |
|--|---|
| Overall length - maximum | 1200 mm + 50 mm for feet |
| Overall width - maximum | 700 mm + 50 mm on each side for hands |
| Overall height - maximum | 1375 mm (including 95th percentile male occupant) |
| Turning diameter - maximum | 1500 mm |
| Smallest wheels | Accommodate a gap of 75 mm horizontal and 50 mm vertical |
| Weight - maximum | 200 kg for manual wheelchair and occupant (including baggage) |
| Obstacle height that can be overcome - maximum | 50 mm |
| Ground clearance - minimum | 60 mm under foot rest with upward slope angle of 10° on top for going forward |
| Safe slope on which the wheelchair remain stable - maximum | 6° dynamic stability in all directions 9° static stability in all directions, including with brake applied |

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product. Contact local dealer for assistance, if needed, in setting up, using or maintenance of the device.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference.

Daily

- Use a dry cloth to clean the product
- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Be aware of any MAX mark indications before adjustment.

Weekly

- Check that all wheels are moving freely and that all wheel locks/brakes are easily activated
- Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent/soap and dry before use.

Monthly

- If equipped with airfilled tyres, check the air pressure
- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system.

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty.

Warewashing machine

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.

Remove the following items and clean them separately: Cushions, Seat and Accessories.

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

A detailed inspection of the product must be made every 12 months (every 6 months if the product is used very often - daily use for more than 7 hours), every time the product is reissued for use and after a long term storage (more than 4 months). The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded. The device may be reissued within its lifetime.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Frame is adjustable in height
- On outdoor versions, the push-brace can be folded over the frame
- If equipped with airfilled tyres, check the air pressure
- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Wheels are moving freely and that all wheel locks/brakes are easily activated. Adjust if required
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact.

Materials

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized.

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© 2021 R82 A/S All rights reserved The R82 logo and the icons are
registered trademarks of R82 A/S
R82 A/S