
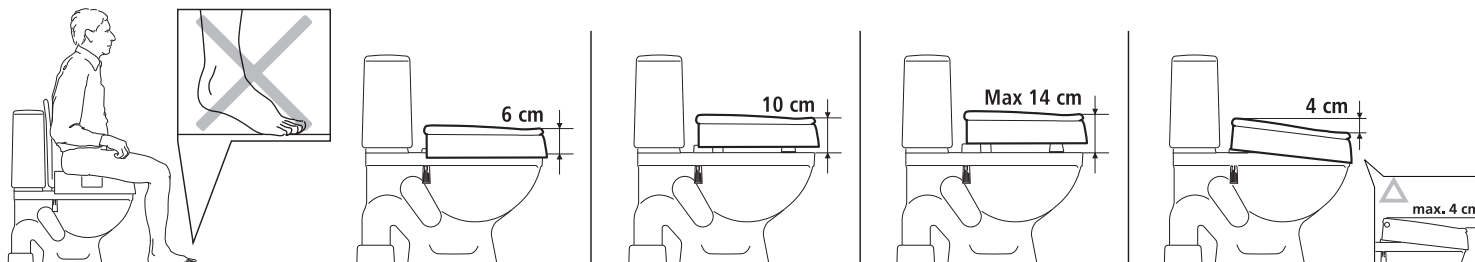
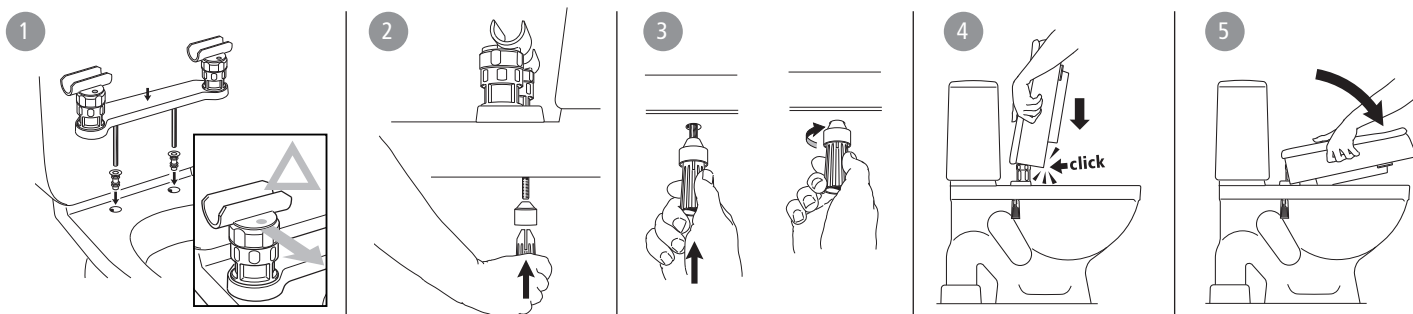
A


B

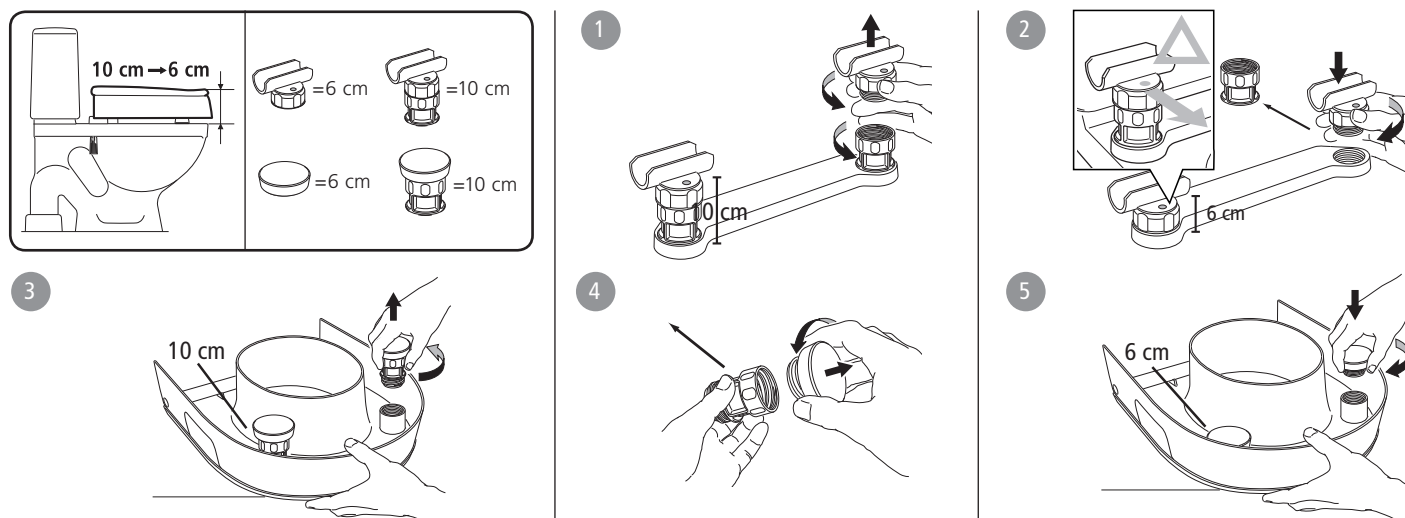
C


D

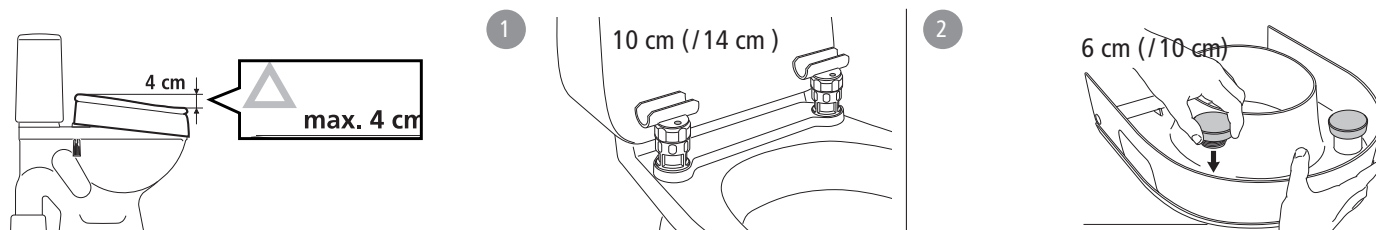
E

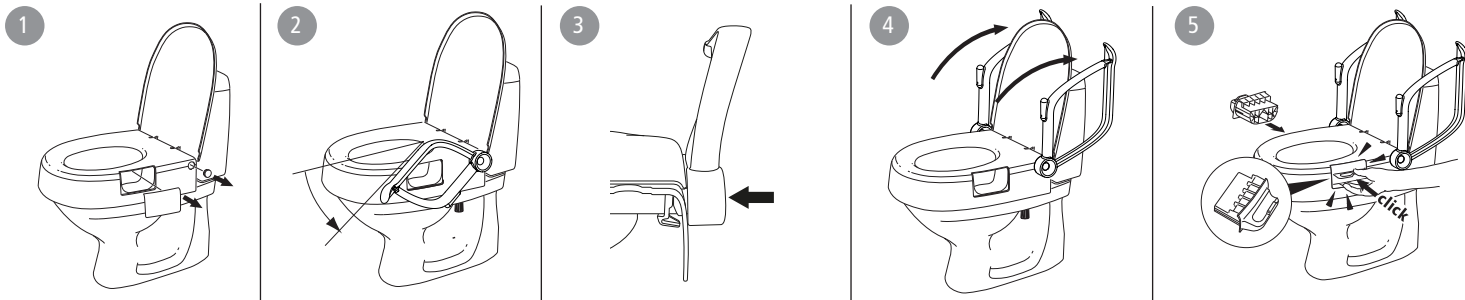
F

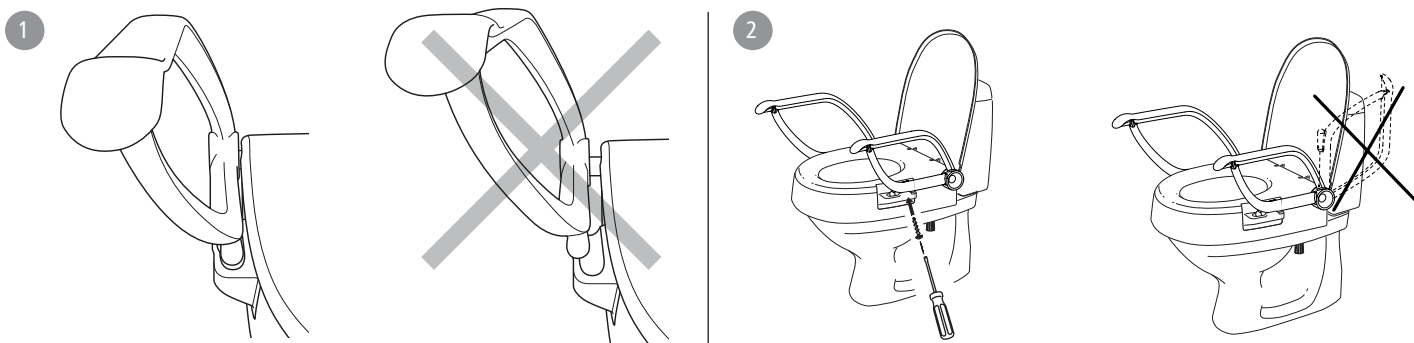G

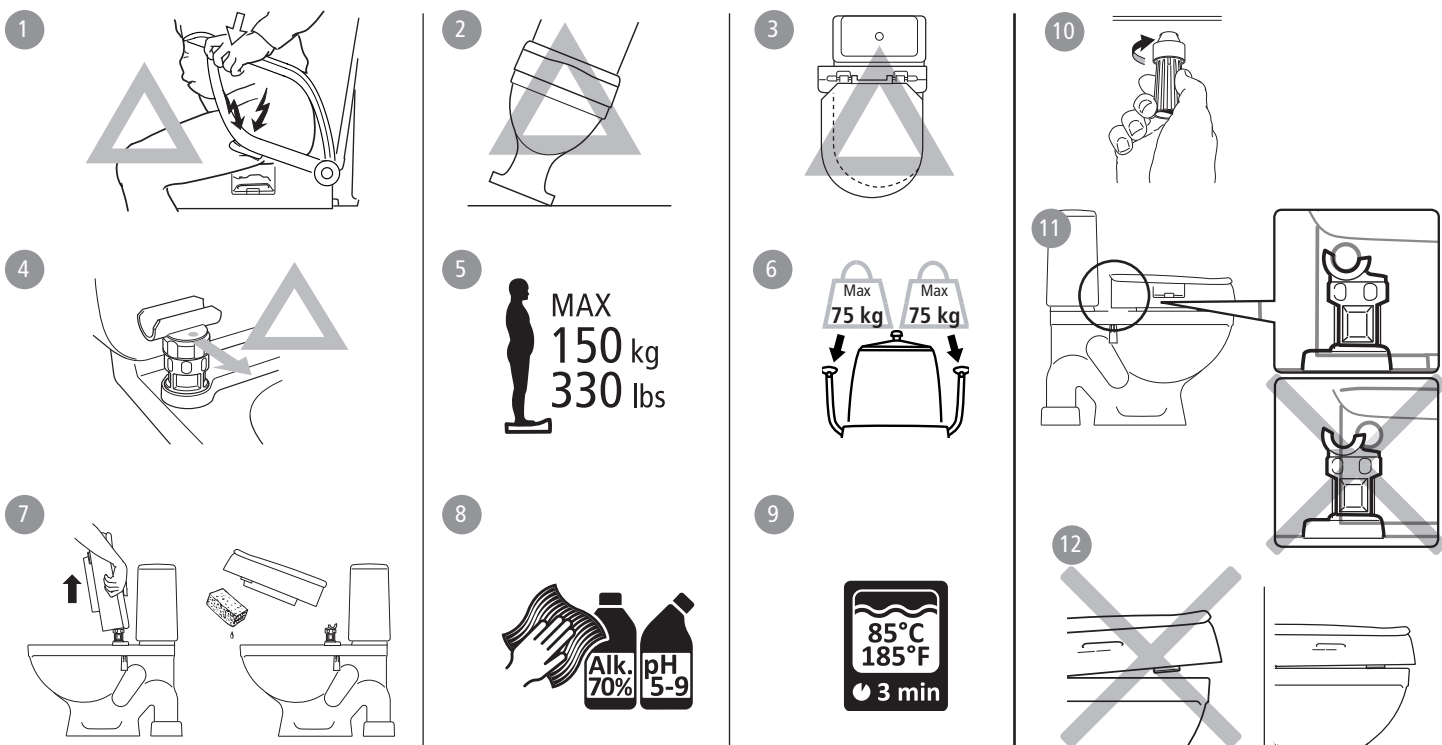H

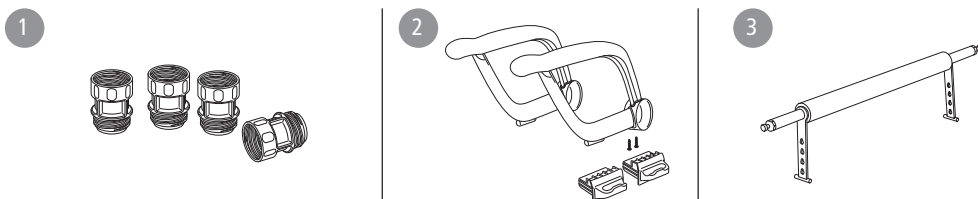SV Svenska

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en English

General information
Cloo is a height-adjustable toilet seat raiser that is fixed securely to the toilet. It is supplied with a 10 cm elevation. When armrests are included in the kit, these are not supplied already fitted. Cloo can also be fitted at an angle.

Warranty: 5 year warranty against material and manufacturing defects. For terms and conditions, see www.etac.com. A defective product must not be used.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

- Choosing your height** *Figure A*
- Mounting** *Figure B*
- Height adjustment** *Figure C*
- Angled fitting** *Figure D*
- Armrest** *Figure E*
- Fixed mounting** *Figure F*
- Safety and Maintenance** *Figure G*

G1. Risk of crushing **G2.** The commode must be securely anchored **G3.** The product should be mounted centrally on the commode **G4.** Mount so that the red dot is pointing forwards. **G5.** Maximum user weight 150 kg **G6.** The armrests have each been tested and approved for half of the maximum user weight **G7.** The seat ring can be removed for cleaning. Make sure it clicks in place completely when refitting **G8.** Clean the product with a solvent-free cleaning agent with a pH level between 5 and 9, or with a 70% disinfectant solution. **G9.** Can be decontaminated at max. 85°C if so required. **G10.** Check the screw attachment.

Accessories *Figure H*
H1. Blocks for 14 cm elevation. **H2.** Armrest kit. Supplied in pairs. **H3.** Crossbar

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R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB