


Etac Cross 5


Spareparts list english

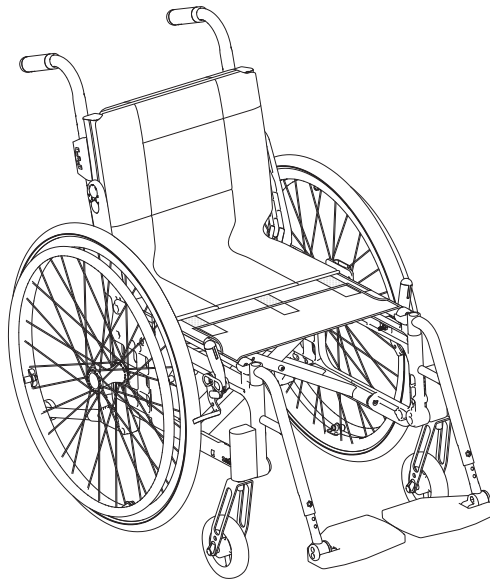

75205.31 2025-01-09



Etac Cross 5

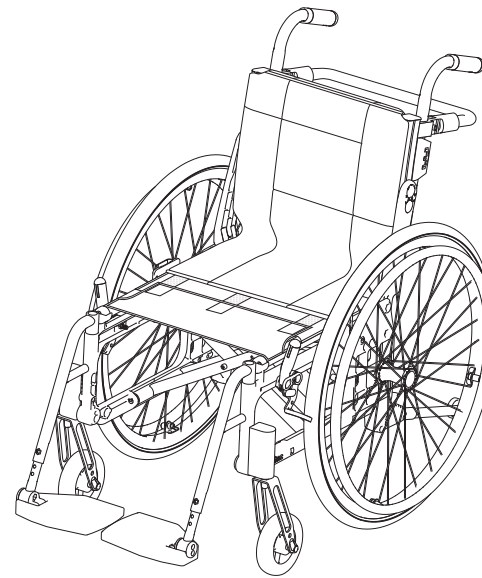

Etac Cross 5 XL


Etac Cross 5 Active

Index

Index	Page
Side frame	3
Cross frame	4
Back support 3A	5-7
Back support tubes for separate back support, ECS (Crossback 2)	8
Fixed back support with fixed push handles	9
Back support 2B	10
Bow handle	11
Back support brace	12
Leg support	13
Leg support elevating mod 1/ Flip-up	14
Leg support elevating mod 3	15
Amputee leg support	16
Plaster leg support	17
Foot support	18-20
Foot support one piece	21
Brake	22
Rear wheel	23-25
Drum brake	26-27
Drum brakes, fixed back support with fixed push handles	28
Single hand drive	29
Tyre and innertube	30-31
Handrim	32-33
Castor	34-35
Width expansion kit	36
Upholstery seat/Seat extension Cross back 3A	37
Solid seat	38
Back support upholstery Cross back mod. 2	39

Back support upholstery Cross back 3A	40
Back support upholstery Cross back 2B	41
Back support upholstery fixed back	42
Back support cover Cross back mod. 2	43
Back support cover Cross back 3A	44-46
Back support cover Cross back 2B	47
Arm support	48-49
Sideguard	50
Mudguard	51
Thigh support	52
Trunk support Cross back 3A	53-54
Tray	55
Tray Hemi	56
Tray Hemi/Communication	57
Anti tip	58-60
Spoke guard	61
Tetra quick release adapter	62
Cane holder	63
Head support	64
IV stand	65


Frame colors:

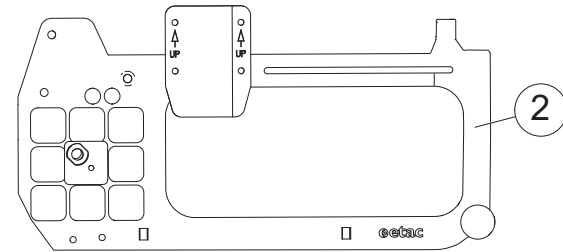

7	Schnorr washer 17x24x1.3 zink plated	13084
8	Nut M16x 1 NV 24 B8	12052
9	Locking nut M 6	12035
10	Attachment arm support Cross/Transit	25535
11	Screw M 6x 20 MC6S 8.8 black	11392
12	Wire attachment anti-tip	82571
13	Touch up paint	80517-xx

Frame colors:
01= matte black, 92=white, 93=dark blue


Cross frame

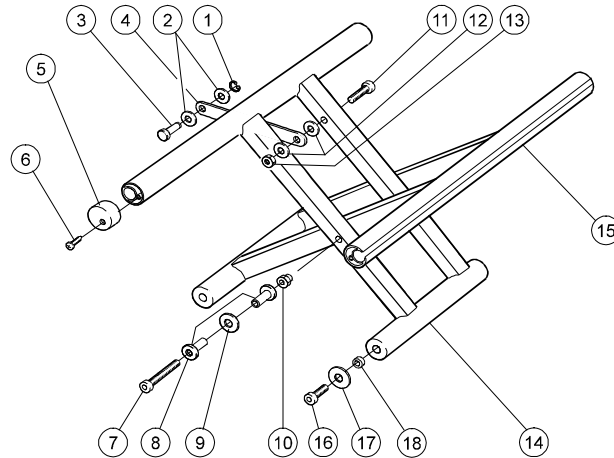

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Cir clip RSM 4 stainless	13054	●	●
2	Joint sleeve Cross	50285	●	●
3	Axle plate Cross frame	20507	●	●
1-3, 5-6, 16-18	Attachment kit	25765	●	●
4	Plate cross frame 35 cm (14")	25148	●	
	Plate cross frame 37,5 cm (15")	25149	●	
	Plate cross frame 40 cm (16")	25150	●	
	Plate cross frame 42,5 cm (17")	25151	●	
	Plate cross frame 45 cm (18")	25152	●	
	Plate cross frame 47,5 cm (19")	25725	●	●
	Plate cross frame 50 cm (20")	25451	●	●
	Plate cross frame 52,5 cm (21")	26035		●
	Plate cross frame 55 cm (22")	26036		●
	Plate cross frame 57,5 cm (23")	26037		●
	Plate cross frame 60 cm (24")	26899		●
5	Plug seat tube Cross	50406	●	●
6	Screw B8x16FXS black	11401	●	●
7	Screw M 6x 55 MC6S 12.9	11351	●	●
8	Cross brace washer Cross	20383	●	●
9	Washer D20 d6,4 t=3,5	51028	●	●
10	Domed nut M 6 black	12005	●	●
11	Screw M 5x 35 MC6S zink plated	11290	●	●
12	Joint bushing	50285	●	●
13	Locking nut M 5	12036	●	●

			Cross 5	Cross 5 XL
4-15	Cross short 35 compl (14) mod 4	27832-01	●	
	Cross short 37.5 ((15) compl mod 4	27833-01	●	
	Cross short 40 (16) compl mod 4	27834-01	●	
	Cross short 42.5 (17) compl mod 4	27835-01	●	
	Cross short 45 (18) compl mod 4	27840-01	●	
	Cross short 47.5 (19) compl mod 4	27841-01	●	●
	Cross short 50 (20) compl mod 4	27842-01	●	●
	Cross short 52.5 (21) compl mod 4	27843-01		●
	Cross short 55 (22) compl mod 4	27844-01		●
	Cross short 57.5 (23) compl mod 4	27845-01		●
	Cross short 60 (24) compl mod 4	27846-01		●
	Cross long 35 (14) compl mod 4	27836-01	●	
	Cross long 37.5 (15) compl mod 4	27837-01	●	
	Cross long 40 (16) compl mod 4	27838-01	●	
	Cross long 42.5 (17) compl mod 4	27839-01	●	
	Cross long 45 (18) compl mod 4	27847-01	●	
	Cross long 47.5 (19) compl mod 4	27848-01	●	●
	Cross long 50 (20) compl mod 4	27849-01	●	●
	Cross long 52.5 (22) compl mod 4	27850-01		●
	Cross long 55 (22) compl mod 4	27851-01		●
	Cross long 57.5 (23) compl mod 4	27852-01		●
	Cross long 60 (24) compl mod 4	27853-01		●
16	Screw M 6x 25 MC6S black esloc 8.8 DIN 912	11636	●	●
17	Washer 25/10/1.5 black pa	50281	●	●
18	Joint cross frame	20322	●	●



Back support 3A

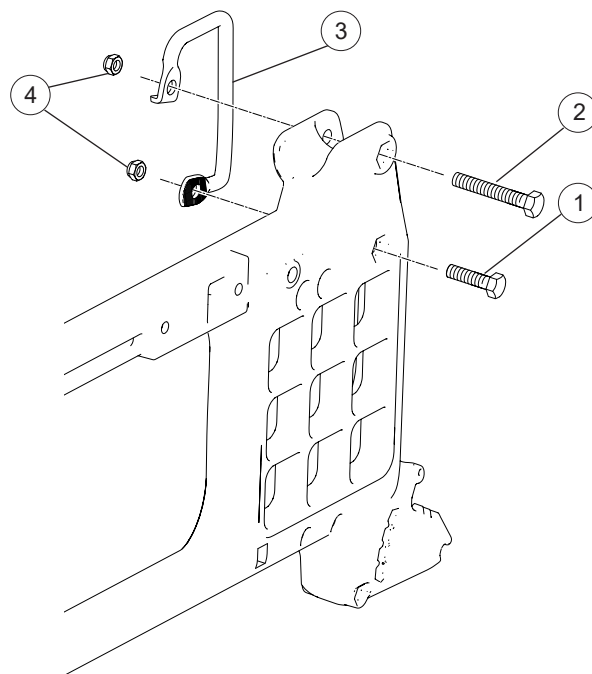

Pos.	Description	Item. no.
1	Screw M8x30 M6S	11363
2	Screw M8x55 M6S	11432
3	Securement points Cross 5 ri Crossback 3A	82573
	Securement points Cross 5 le Crossback 3A	82574
4	Locking nut M8x1 DIN 985	12051

Cross 5	Cross 5 XL
●	●
●	●
●	●
●	●

Back support 3A cont.

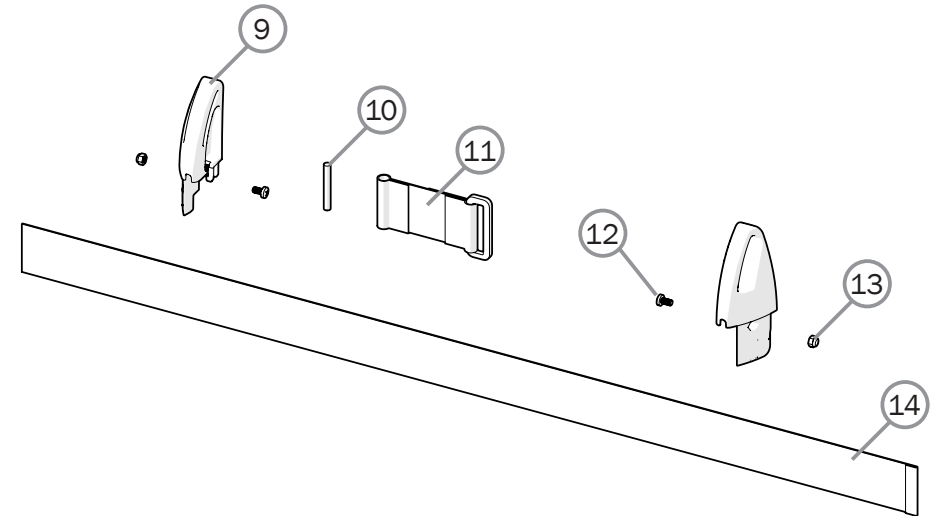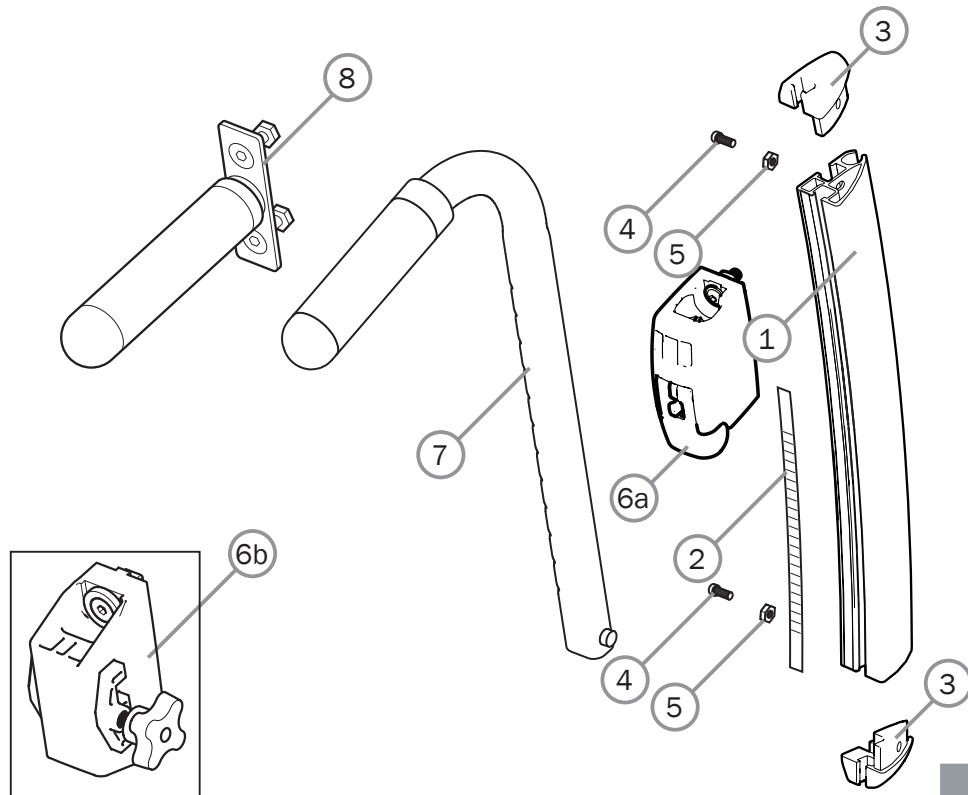


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Back rail Cross 3A right	83013-1	●	●
	Back rail Cross 3A left	83013-2	●	●
2	Label height adjustment	71377	●	●
	Label height adjustment high	71378	●	●
3	Dust cover 1 Cross back mod 4	51130	●	●
	Dust cover 2 Cross back mod 4	51131	●	●
4	Screw MRX M4x8	11441	●	●
5	Locking nut M 4 fzb	12026	●	●
6a	Attachment push handle height adj 3A mod 4	28576	●	
6b	Attachment push handle ri height adj Cross mod 3	27371-1	●	●
	Attachment push handle le height adj Cross mod 3	27371-2	●	●

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
7	Push handles h adj Cross 100 mm	27588-01	●	●
	Push handles h adj Cross XL 100 mm	27589-01		●
	Push handles h adj Cross XL 75 mm	27341-01		●
8	Push handle fixed Crossback 3A	27644	●	●
9-15	Extension back 3A 35-40 cpl	28658	●	●
	Extension back 3A 42,5-50 cpl	28659	●	●
	Extension back 3A 52,5-60 cpl	28660	●	●
9	Extension back R	51223-1	●	●
	Extension back L	51223-2	●	●
10	Tension rod	51222	●	●
11	Loop extension back 3A cpl	63294	●	●
12	Screw M4x8 MRX black 4.8 pos	11441	●	●
13	Locking nut M4 zink plated	12026	●	●
14	Seat depth extension 35-40 cm	63291	●	●
	Seat depth extension 42,5-50 cm	63292	●	●
	Seat depth extension 52,5-60 cm	63293	●	●

Back support tubes for separate back support, ECS (Crossback 2)


Pos.	Description	Item. No.		
			Cross 5	Cross 5 XL
1	Locking nut M 8x 1 DIN 985	12051	●	●
2	Screw M 8x 1x30 black	11363	●	●
3	Screw M8x1x55 M6S 10.9 black	11432	●	●
4	Securement points Cross 5 right	82573	●	●
	Securement points Cross 5 left	82574	●	●
5	Joint support backrest	20319	●	●
6	Plug 25-21/23 R12.5	50248	●	●
7	Back tube Cross back 2	25593-01	●	●
8	Label height adjustment	71851	●	●
9	Screw M 6x 22 MC6S fzb	11012	●	●
10	Washer BRB 6x3x10x1.5 fzb	13080	●	●
11	Wedge back 6063A-T6	80851	●	●
12	Dust cover Cross back 2 right	50405	●	●
	Dust cover Cross back 2 left	50407	●	●
13	Square nut high M6	12058	●	●
6-13	Back support tube compl. right	25609-01-1	●	●
	Back support tube compl. left	25609-01-2	●	●
14	Attachement push handles	25676	●	●
15	Washer BRB M 6	13030	●	●
16	Screw M6x30 MC6S 8.8 black esloc	11398	●	●
17	Push handles heightadjustable	27298	●	●
18	Cover for handle	82221	●	●
19	Screw M6 x 25	11041	●	●
17-19	Push handle h adj Cross std	27338-01	●	●

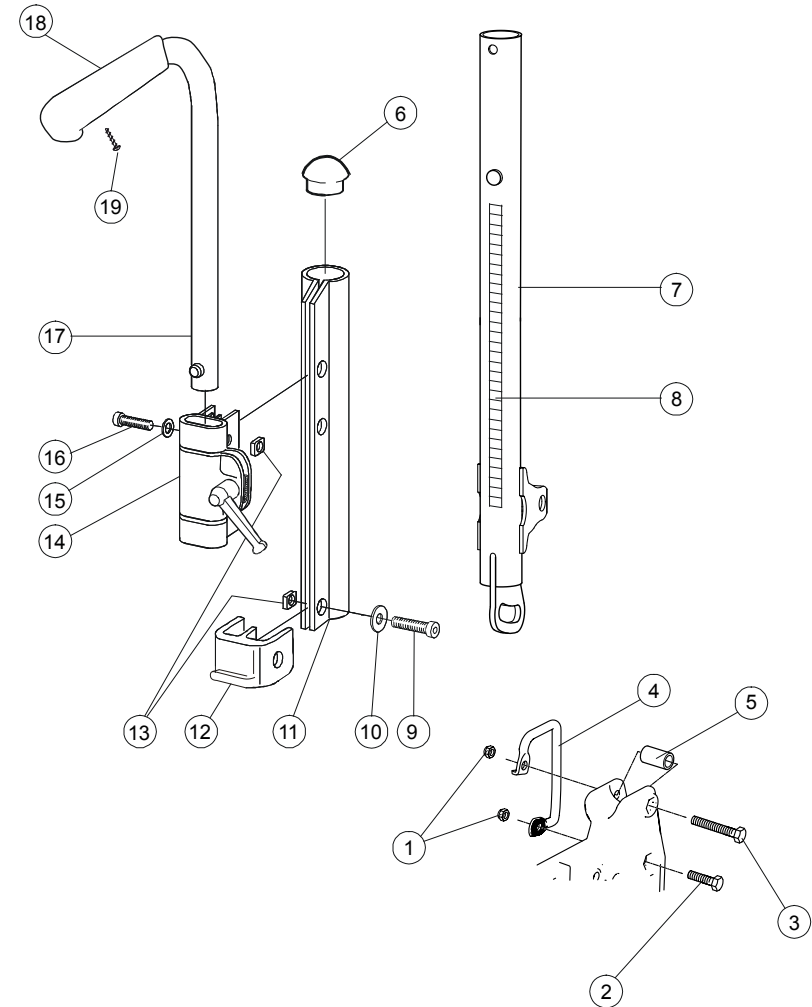

Fixed back support with fixed push handles


Pos.	Description	Item. No.
1	Joint support back support	20319
2	Locking nut M 8x 1 DIN 985	12051
3	Screw M8x1x30 10.9	11363
4	Screw M8x55x1 M6S 10.9 black	11432
5	Securement points Cross 5 right	82573
	Securement points Cross 5 left	82574
6	Back support tube attachment	26571-01
7	Locking nut M5 M6MF	12060
8	Back support tube with push handle Twin	27344-01
9	Screw M 5x 30 M6S	11403
10	Screw B 8x16 RTS st 4.2	11041
11	Cover for handle	82221
12	Screw M 5x 12 K6S 10.9	11208

	Cross 5	Cross 5 XL
1	●	
2	●	
3	●	
4	●	
5	●	
6	●	
7	●	
8	●	
9	●	
10	●	
11	●	
12	●	

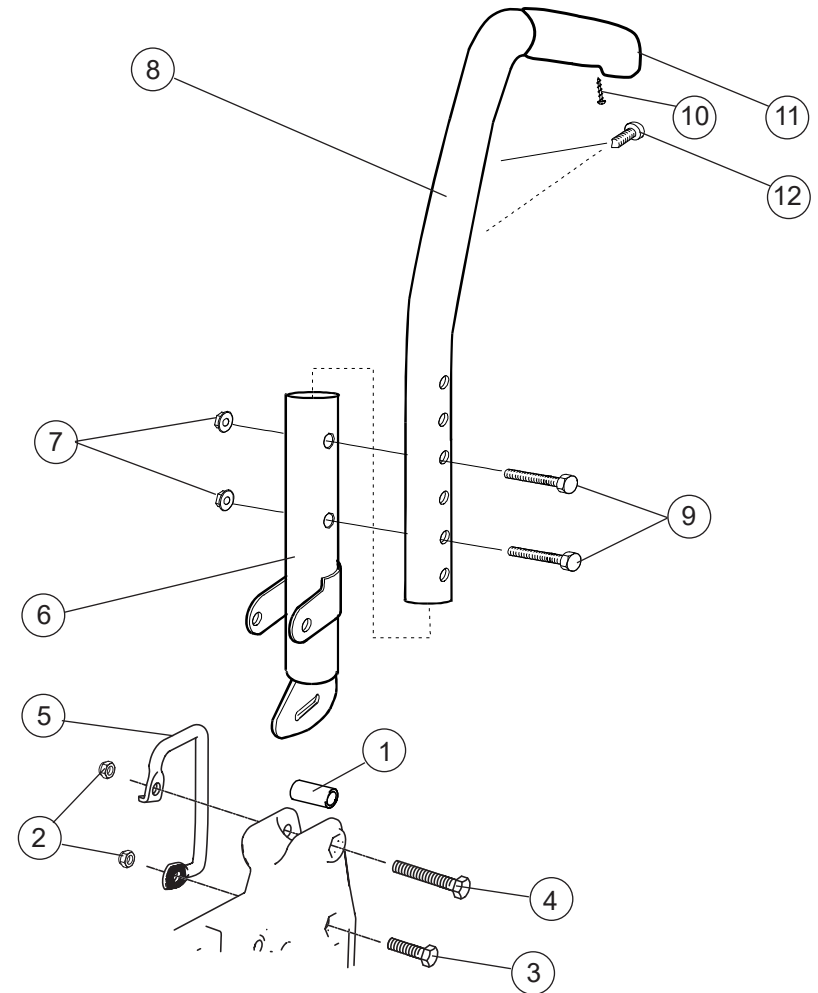


Back support 2B

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Back support attachment B2B cpl	28366-01	●	
2	Screw M 5x 30 M6S zink plated	11403	●	
3	Locking nut M 5 black DIN 985-6	12060	●	
4	Back support tube low			
	Back support tube high			
4-7	Back support tube with push handle B2B low incl cover for handle	28378	●	
	Back support tube with push handle B2B high incl cover for handle	28379	●	
5	Screw B 8x16 RTS zink plated 4.2	11041	●	
6	cover for handle L80	28364	●	
7	End plug black	51174	●	

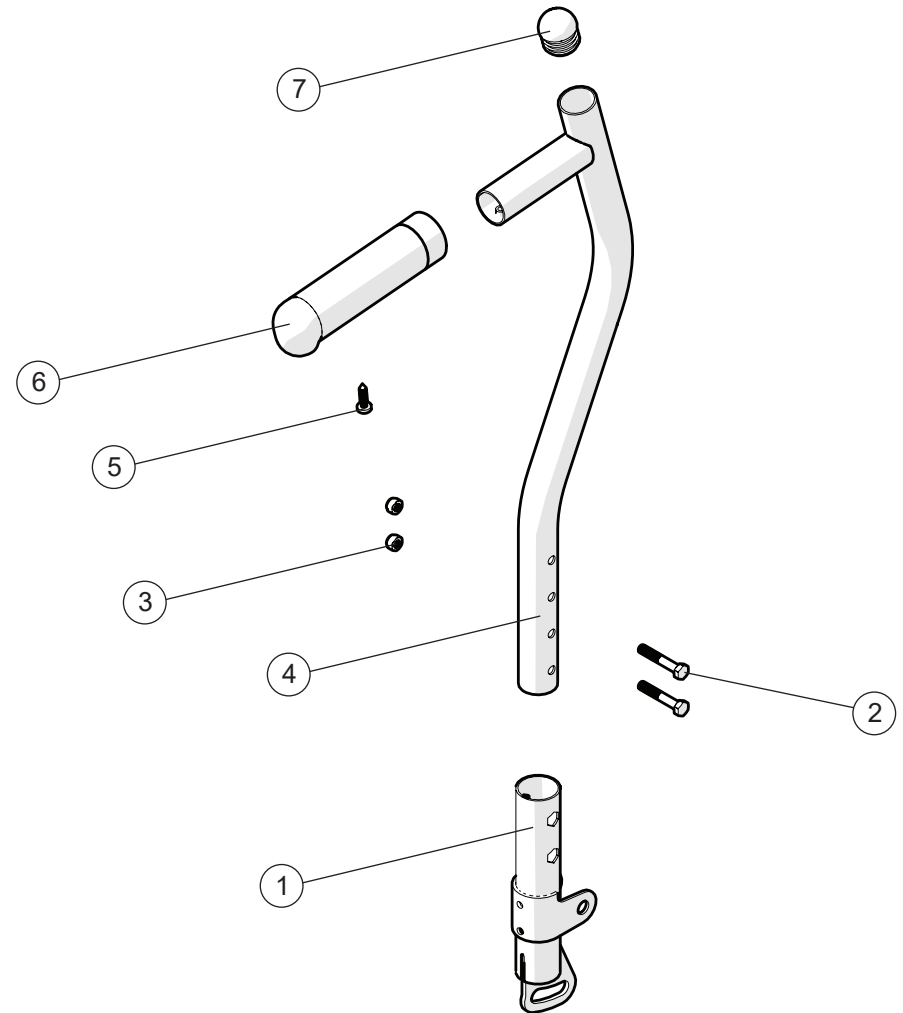


Bow handle

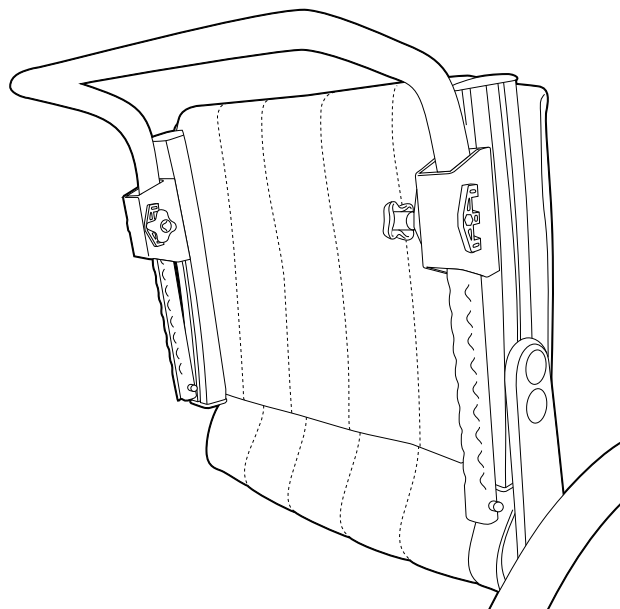

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Bow handle 32.5 cm back 3A 35 cm height adj compl	27433-01	●	
	Bow handle Cross back 2 35 cm back 3A 37.5 cm height adj compl	25578-01	●	
	Bow handle Cross back 2 37.5 cm back 3A 40 cm height adj compl	25579-01	●	
	Bow handle Cross back 2 40 cm back 3A 42.5 cm height adj compl	25580-01	●	
	Bow handle Cross back 2 42.5 cm back 3A 45 cm height adj compl	25581-01	●	
	Bow handle Cross back 2 45 cm back 3A 47.5 cm height adj compl	25582-01	●	
	Bow handle Cross back 2 47.5 cm back 3A 50 cm height adj compl	25730-01	●	
	Bow handle Cross XL 45 cm back 3A 47.5 cm height adj compl	27434-01		●
	Bow handle Cross XL 47.5 cm back 3A 50 cm height adj compl	26063-01		●
	Bow handle Cross XL 50 cm back 3A 52.5 cm height adj compl	26064-01		●
	Bow handle Cross XL 52.5 cm back 3A 55 cm height adj compl	26068-01		●
	Bow handle Cross XL 55 cm back 3A 57.5 cm height adj compl	26069-01		●
	Bow handle Cross XL 57.5 cm back 3A 60 cm height adj compl	26070-01		●

Back support brace

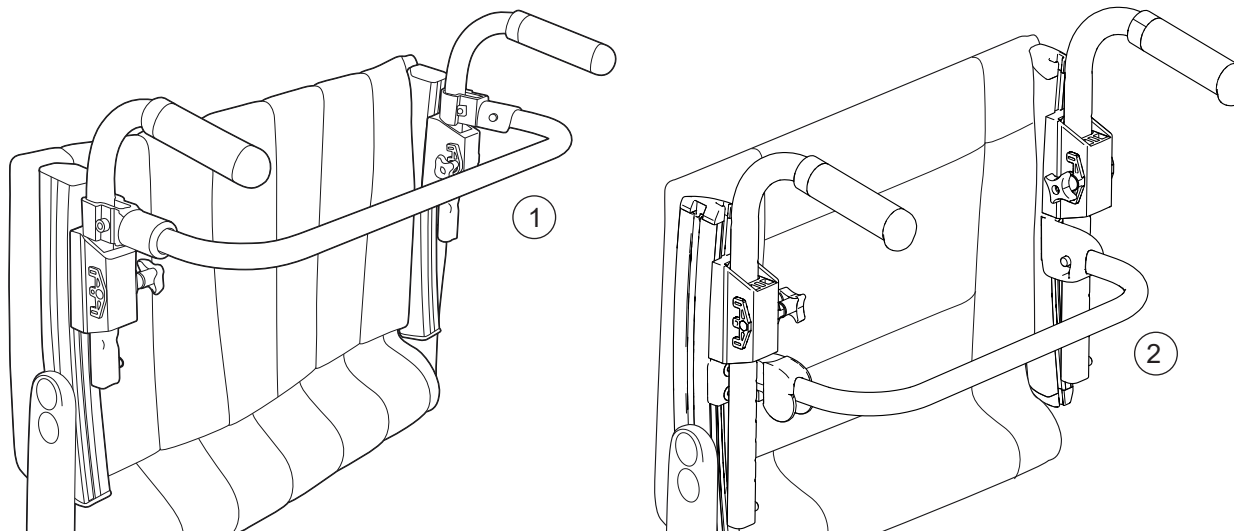


Cross 5
Cross 5 XL

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Back support brace 32.5 cm 35 cm back 3A cpl	27431	●	
	Back support brace 35 cm Cross back 2. 37.5 cm back 3A cpl	25610	●	
	Back support brace 37.5 cm Cross back 2. 40 cm back 3A cpl	25611	●	
	Back support brace 40 cm Cross back 2. 42.5 cm back 3A cpl	25612	●	
	Back support brace 42.5 cm Cross back 2. 45 cm back 3A cpl	25613	●	
	Back support brace 45 cm Cross back 2. 47.5 cm back 3A cpl	25614	●	●
	Back support brace 47.5 cm Cross back 2. 50 cm back 3A cpl	25739	●	●
	Back support brace 50 cm Cross back 2. 52.5 cm back 3A cpl	25615		●
	Back support brace 52.5 cm Cross back 2. 55 cm back 3A cpl	26030		●
	Back support brace 55 cm Cross back 2. 57.5 cm back 3A cpl	26031		●
	Back support brace 57.5 cm Cross back 2. 60 cm back 3A cpl	26032		●

Cross 5
Cross 5 XL

2	Back support brace 3A 35 cm compl mod 4	28114	●	
	Back support brace 3A 37,5 cm compl mod 4	28115	●	
	Back support brace 3A 40 cm compl mod 4	28116	●	
	Back support brace 3A 42,5 cm compl mod 4	28117	●	●
	Back support brace 3A 45 cm compl mod 4	28118	●	●
	Back support brace 3A 47,5 cm compl mod 4	28695	●	●
	Back support brace 3A 50 cm compl mod 4	28696	●	●
	Back support brace 3A 52,5 cm compl mod 4	28697	●	●
	Back support brace 3A 55 cm compl mod 4	28698	●	●
	Back support brace 3A 57,5 cm compl mod 4	28699	●	●
	Back support brace 3A 60 cm compl mod 4	28700		●


Leg support

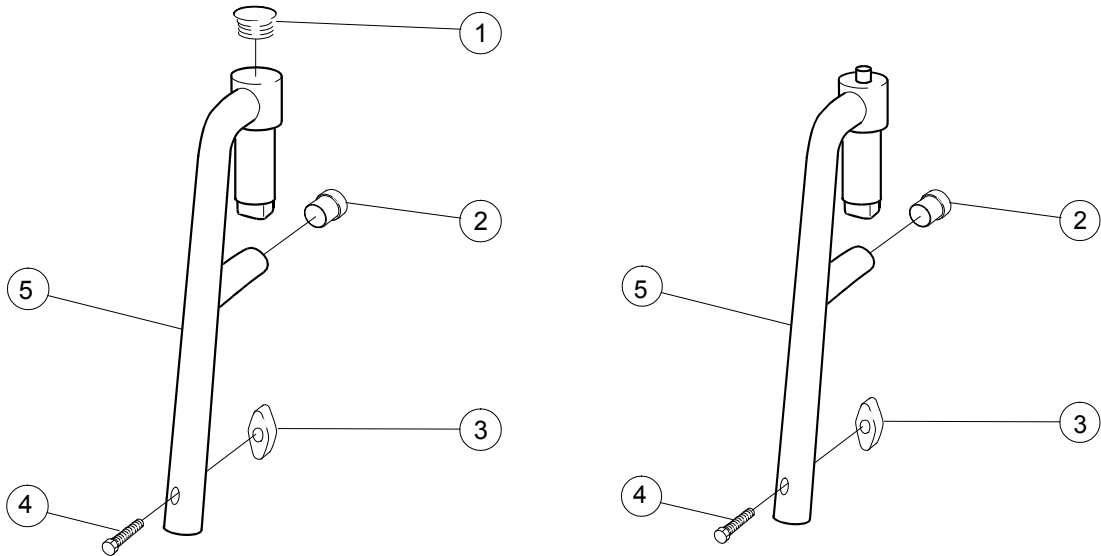

Pos.	Description	Item. No.
1	Dust cover leg support	50287
2	Dust cover 16x 1 black	50240
3	Wing nut M6	80847
4	Screw M6x30 M6S 8.8 zink plated DIN 931	11555
1-5	Leg support narrow angle	25570-01
	Leg support wide angle	25574-01
	Leg support narrow short	25571-01
2-5	Leg support narrow short lockable	25572-01
	Leg support narrow lockable	25573-01
	Leg support wide lockable	25575-01

Cross 5	Cross 5 XL
●	●
●	●
●	●
●	●
●	●
●	●
●	
●	
●	
●	●

Elevating leg support model 1

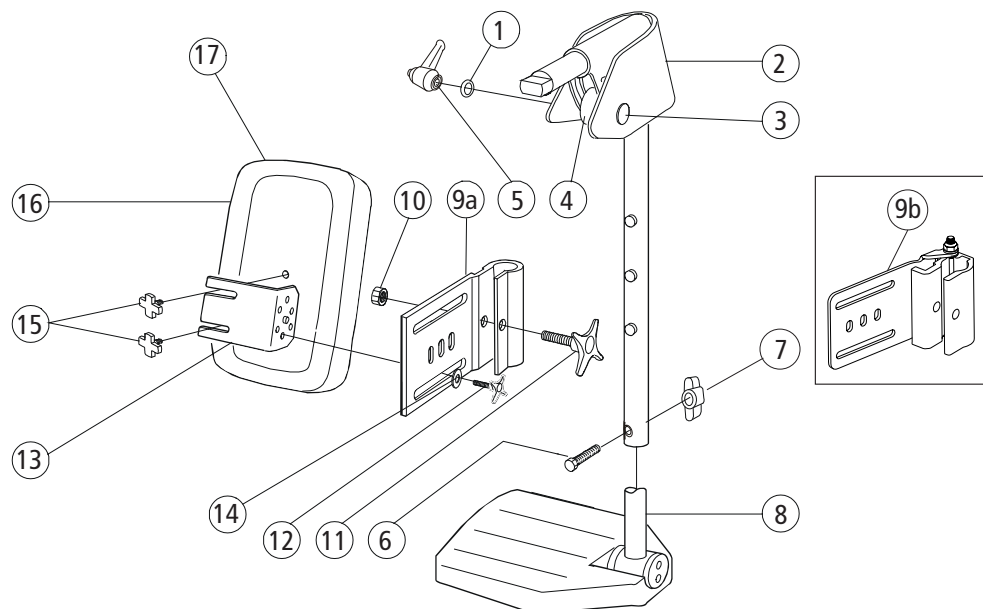


	Cross 5	Cross 5 XL
1	●	●
2	●	●
3	●	●
4	●	●
5	●	●
6	●	●
7	●	●
1-7	●	●
8		
9a	●	●
10	●	●
11	●	●
12	●	●

13	Holder for calf support leg support elevating black	24421-01
14	Washer 6.4x17x3 DIN 7349 steel zink plated	13052
9a-14	Attachment leg support elevating with holder black	24544-01
9b-14	Attachment leg support FlipUp elevating with holder black	27322-01
15	Knob M 5x15 leg support	80807
15-16	Calf support 35-37.5 cm dark grey	62105-60
	Calf support 40-42.5 cm dark grey	62104-60
	Calf support 45-47.5 cm dark grey	62484-60
	Calf support 50-60 cm dark grey	62451-60
17	Cover calf support 35-37.5	62888-60
	Cover calf support 40-42.5	62889-60
	Cover calf support 45-47.5	62890-60
	Cover calf support 50-60	62906-60

	Cross 5	Cross 5 XL
13	●	●
14	●	●
9a-14	●	●
9b-14	●	●
15	●	●
15-16	●	
	●	
	●	●
	●	●
17	●	
	●	
	●	●
	●	●

Pos.	Description	Item. No.
1	Washer NB 6x16x1.0 black crome	13034
2	Joint guard leg support angle adjustable	62176
3	Screw M 6x 40 MVBF zink plated	11234
4	Spacer leg support vrb Cross	20414
5	Handle M 6x25 43 mm black	80488
6	Screw M 6x 30 M6S zink plated	11555
7	Wing nut M 6	80847
1-7	Leg support elevating mod 1 Cross black	25660-01
8	See page 17 (foot supports)	
9a	Attachment leg support elevating Cross	24446-01
10	Nut M6M M 8 zink plated	12006
11	Knob 60x22 M 8x32 black	80295
12	Knob 40 M 6x17 black	80297

Elevating leg support model 3

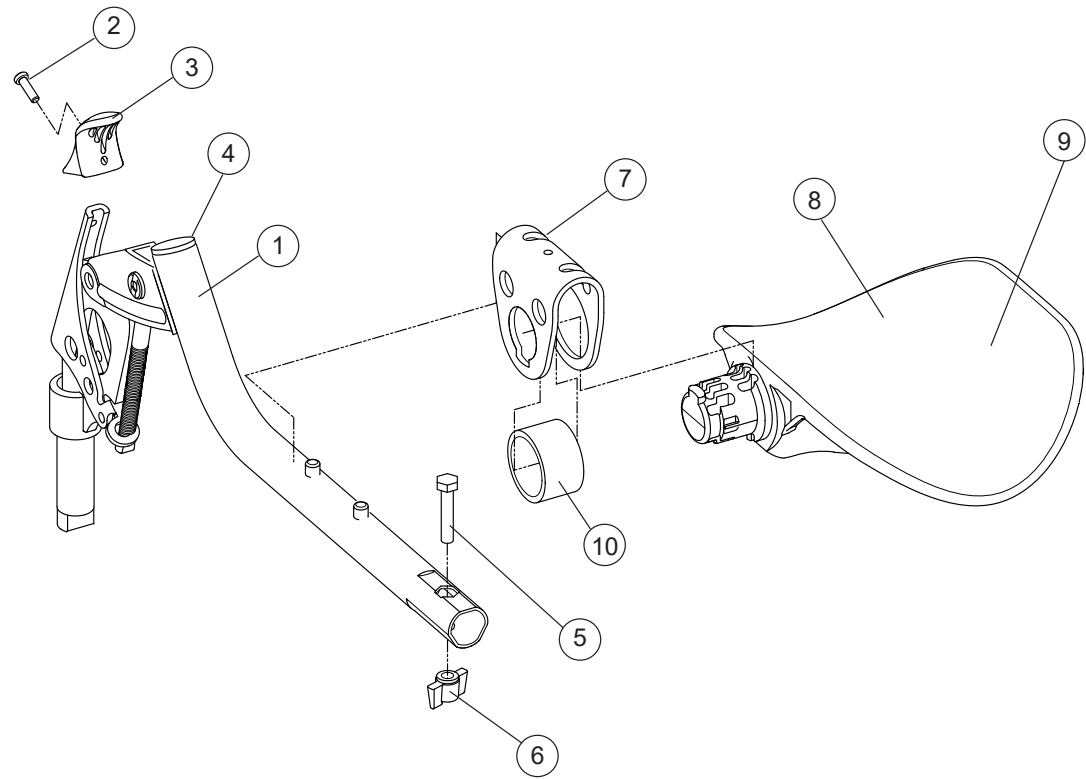

Pos.	Description	Item. No.
1-8, 10	Leg support elevating compl with calf support	26100
2	Screw K40x 10 PT-KX black	11419
3	Handle adj leg support	50842
4	Dust cover 22x2	51020
5	Screw M 6x 30 M6S zink plated	11555
6	Wing nut WN M6	80847
7	Bracket calf support adj leg support	26122


Cross 5	Cross 5 XL
●	●
●	●
●	●
●	●
●	●
●	●
●	●

8	Calf support adj leg support	50844
9	Cover calf support elevating leg support dark grey	62762-60
	Cover calf support elevating leg support Dartex	62762-63
10	PUR-ring	50860
7-8, 10	Calf support compl adj leg support	26136
1-6	Leg support elevation without calfsupport	27149

Cross 5	Cross 5 XL
●	●
●	●
●	●
●	●
●	●
●	●

Amputee leg support

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Knob M 5x15 leg support amputee	80807	●	●
2	Washer BRB 8.4/A2 stainless	13011	●	●
3	Screw M 8x 35 MC6S	11276	●	●
4	Locking tube for leg support amputee	25518-01	●	●
5	Stop screw M 8x 8 black	11388	●	●
6	Axle leg support amputee compl	25519-01	●	●
7	Attachment calf support leg support amputee	80809	●	●
8	Square nut high M8	12056	●	●
9	Support pad leg support amputee dark grey	62335-60	●	●
	Support pad leg support Dartex	62335-63	●	●
	Support pad leg support amputee ri XL dark grey	62452-60-1		●
	Support pad leg support amputee le XL dark grey	62452-60-2		●
	Support pad leg support amputee ri XL dartex	62452-63-1		●
	Support pad leg support amputee le XL dartex	62452-63-2		●
10	Cover leg support amputee Cross dark grey	62893-60	●	●
	Cover leg support amputee Cross Dartex	62893-63	●	●
	Cover leg support amputee XL ri Cross dark grey	62450-60		●
	Cover leg support amputee XL ri Cross Dartex	62450-63		●
	Cover leg support amputee XL le Cross dark grey	62761-60		●
	Cover leg support amputee XL le Cross Dartex	62761-63		●
11	Allen key NV 4 DIN 911	80141	●	●
12	Allen key Nv 6 180 mm	82625	●	●

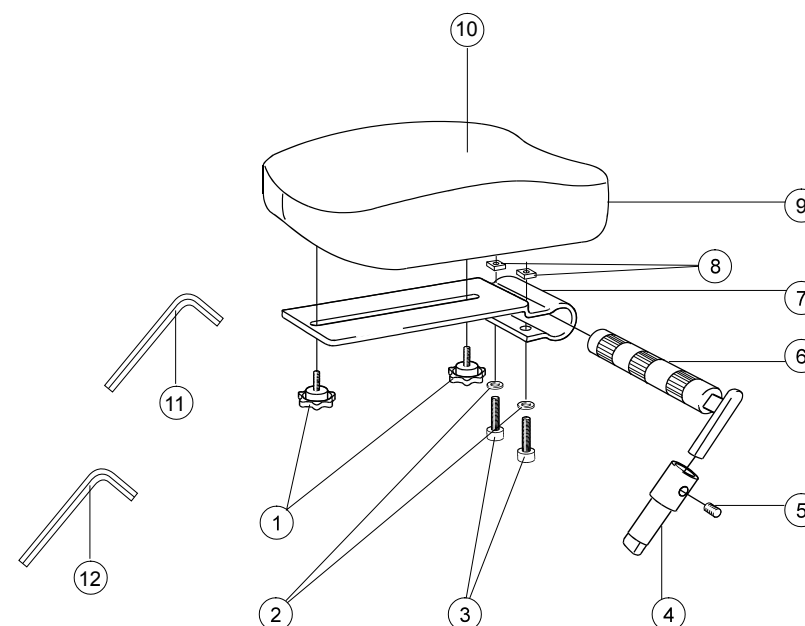


1-12			Cross 5	Cross 5 XL
	Amputee support compl Dark grey	25516-60	●	
	Amputee support Cross compl Dartex	25516-63	●	
	Amputee support XL ri compl dark grey	26210-60-1		●
	Amputee support XL le compl dark grey	26210-60-2		●
	Amputee support XL right compl Dartex	26210-63-1		●
	Amputee support XL left compl Dartex	26210-63-2		●


Plaster leg support

Pos.	Description	Item. No.
1-9	Plaster leg support right	27417-1
	Plaster leg support left	27417-2
1	Plaster leg support	
2	Cushion Plaster leg support	82520
3	End stop	50287
4	Attachment leg support	24446-01
5	Nut M8	12006
6	Knob 60x22 M 8x32 black	80295
7	Knob 40 M6x17 black	80297
8	Holder for calf support	24421-01
9	Washer 6.4x17x3 DIN 7349	13052
4-9	Attachment leg support	24544-01

	Cross 5	Cross 5 XL
	●	●
	●	●
	●	●
	●	●
	●	●
	●	●
	●	●
	●	●
	●	●

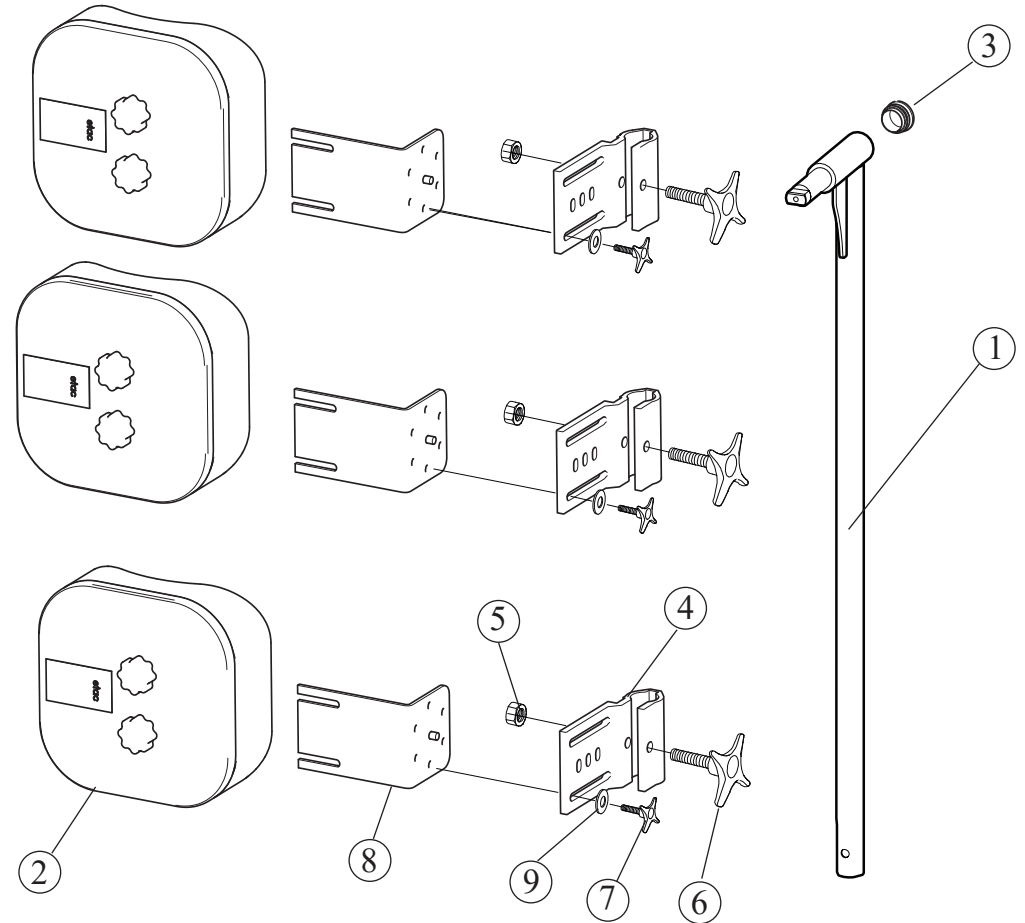


Foot support

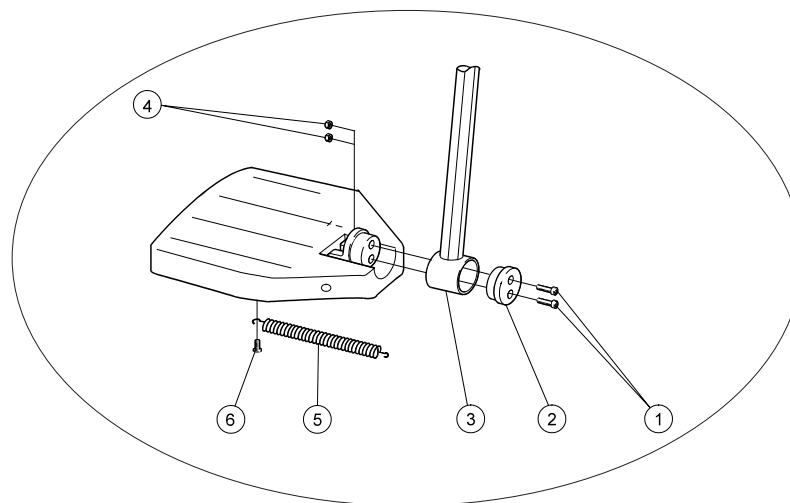

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Screw M6x25 MC6S 8.8 zink plated	11033	●	●
2	Locking block foot support	20324	●	●
3	Attachment tube foot support xshort c/c 15 mm black	82603	●	
	Attachment tube foot support adj short	81999	●	
	Attachment tube foot support adj	81997	●	●
	Attachment tube foot support adj long	81998	●	●
4	Locking nut M 6 zink plated	12017	●	●
5	Spring footplate Cross	80619	●	●
6	Screw K40x8 PT-KX PHIL WN 1442 zink plated	11344	●	●


Foot support cont.

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
7	(Foot support with std attachment tube 25 cm, 81997")			
	Foot support standard 35 cm (14") right	25641-1	●	
	Foot support standard 35 cm (14") left	25641-2	●	
	Foot support standard 37.5 cm (15") right	25642-1	●	
	Foot support standard 37.5 cm (15") left	25642-2	●	
	Foot support standard 40 cm (16") right	25643-1	●	
	Foot support standard 40 cm (16") left	25643-2	●	
	Foot support standard 42.5 cm (17") right	25644-1	●	
	Foot support standard 42.5 cm (17") left	25644-2	●	
	Foot support standard 45 cm (18") right	25645-1	●	
	Foot support standard 45 cm (18") left	25645-2	●	
	Foot support standard 47.5 cm (19") right	25736-1	●	●
	Foot support standard 47.5 cm (19") left	25736-2	●	●
	Foot support standard 50 cm (20") right	25646-1	●	●
	Foot support standard 50 cm (20") left	25646-2	●	●
7	(Foot support with short attachmnet tube -8 cm, 81999)			
	Foot support short 35 cm (14") right	25653-1	●	
	Foot support short 35 cm (14") left	25653-2	●	
	Foot support short 37.5 cm (15") right	25654-1	●	
	Foot support short 37.5 cm (15") left	25654-2	●	
	Foot support short 40 cm (16") right	25655-1	●	
	Foot support short 40 cm (16") left	25655-2	●	
	Foot support short 42.5 cm (17") right	25656-1	●	
	Foot support short 42.5 cm (17") left	25656-2	●	
	Foot support short 45 cm (18") right	25657-1	●	
	Foot support short 45 cm (18") left	25657-2	●	
	Foot support short 47.5 cm (19") right	25738-1	●	
	Foot support short 47.5 cm (19") left	25738-2	●	
	Foot support short 50 cm (20") right	25658-1	●	
	Foot support short 50 cm (20") left	25658-2	●	

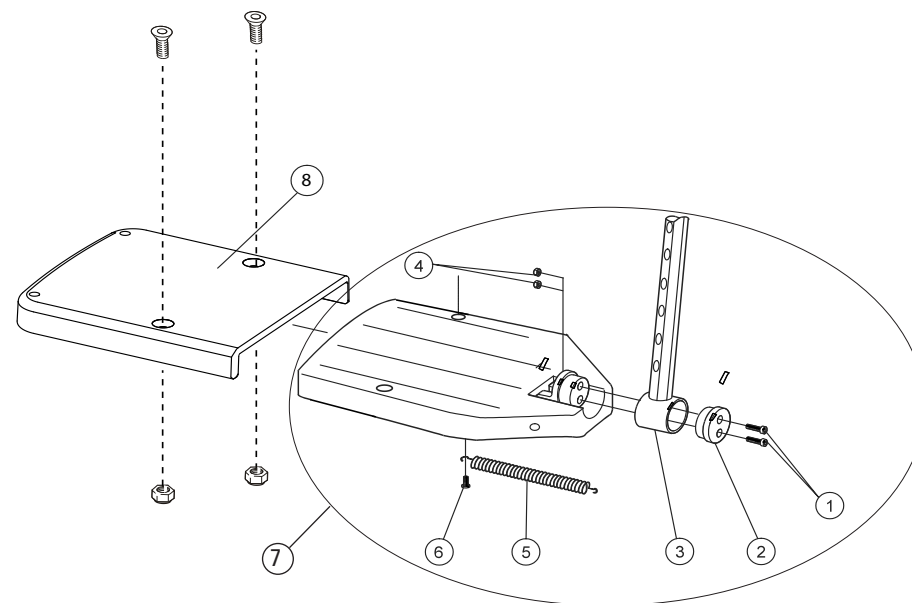


7	(Foot support with long attachment +10 cm, 81998)		Cross 5	Cross 5 XL
	Foot support long 35 cm (14") right	25647-1	●	
	Foot support long 35 cm (14") left	25647-2	●	
	Foot support long 37.5 cm (15") right	25648-1	●	
	Foot support long 37.5 cm (15") left	25648-2	●	
	Foot support long 40 cm (16") right	25649-1	●	
	Foot support long 40 cm (16") left	25649-2	●	
	Foot support long 42.5 cm (17") right	25650-1	●	
	Foot support long 42.5 cm (17") left	25650-2	●	
	Foot support long 45 cm (18") right	25651-1	●	
	Foot support long 45 cm (18") left	25651-2	●	
	Foot support long 47.5 cm (19") right	25737-1	●	●
	Foot support long 47.5 cm (19") left	25737-2	●	●
	Foot support long 50 cm (20") right	25652-1	●	●
	Foot support long 50 cm (20") left	25652-2	●	●
8	Foot support extended Cross XL 52.5-57.5	28443		●

Foot support cont.

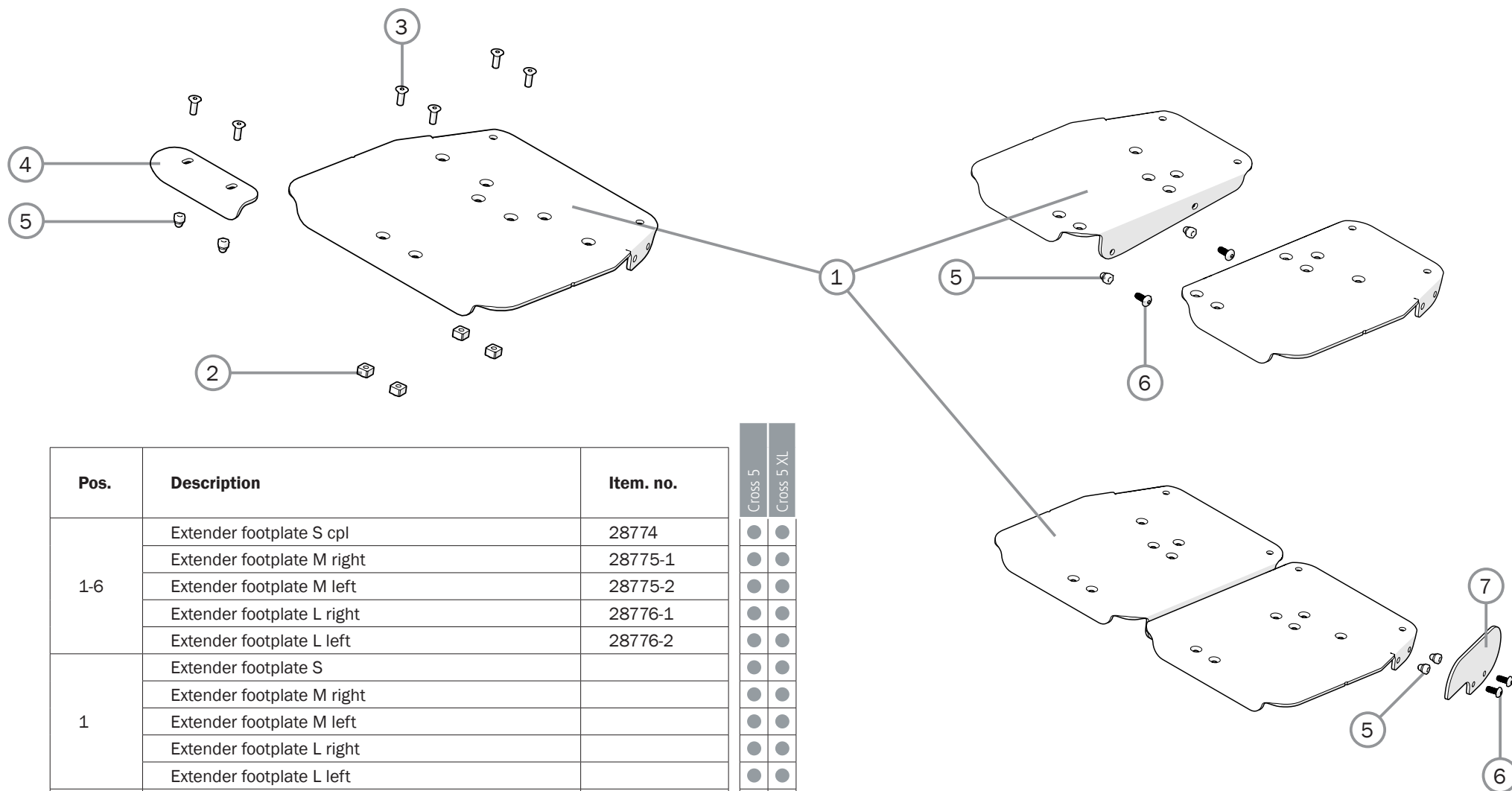


Pos.	Description	Item. no.	Cross 5	Cross 5 XL
1-6	Extender footplate S cpl	28774	●	●
	Extender footplate M right	28775-1	●	●
	Extender footplate M left	28775-2	●	●
	Extender footplate L right	28776-1	●	●
	Extender footplate L left	28776-2	●	●
1	Extender footplate S		●	●
	Extender footplate M right		●	●
	Extender footplate M left		●	●
	Extender footplate L right		●	●
	Extender footplate L left		●	●
2	Stop bar arm support	80661	●	●
3	Screw M5x14 MF6S Zn8/P esloc	11237	●	●
4	Guide one-piece footplate	83421	●	●
5	Locking nut M5 domed zink plated	12168	●	●
6	Screw M5x12 K6S 10.9 zink plated	11208	●	●
7	Side protection footplate	83420	●	●

Foot support one piece

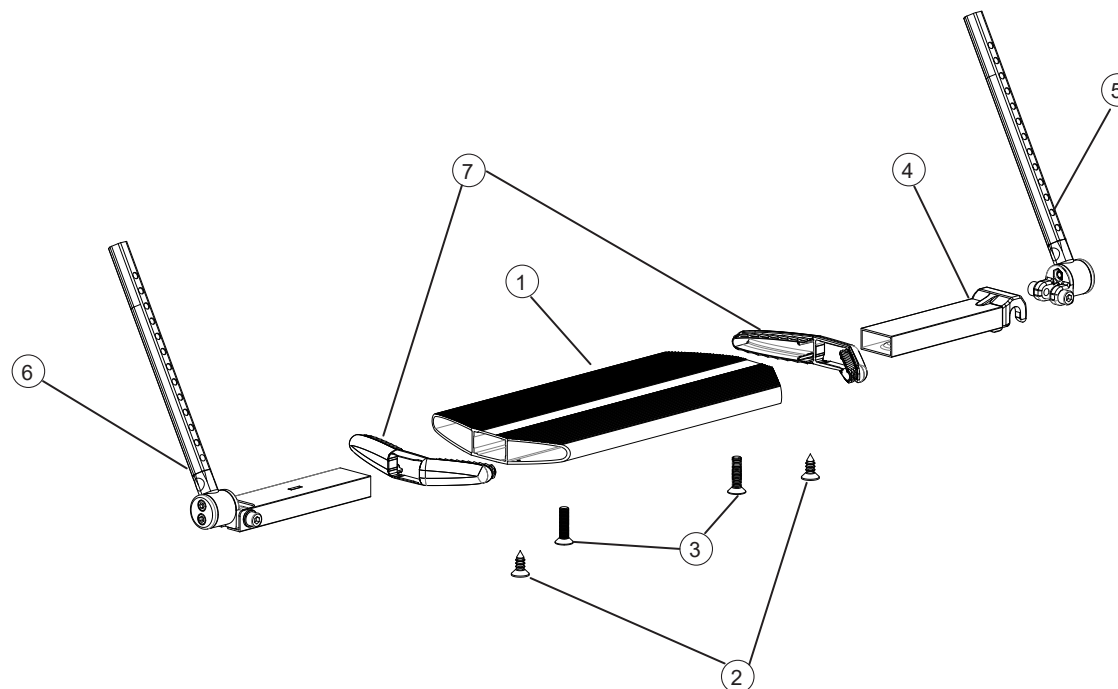

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1-7	Foot support one piece 35-37.5 cm compl	25936	●	
	Foot support one piece 40-50 cm compl	25937	●	●
	Foot support one piece 50-60 cm compl	26853	●	●
1	Foot support one piece 35-37.5 cm	81130	●	
	Foot support one piece 40-50 cm	81131	●	●
	Foot support one piece 50-60 cm	81993	●	●
2	Screw B 6 3.5x 9.5 FXS black	11384	●	●
3	Screw M 5x 20 MF6S black	11526	●	●
4	Locking joint for foot support compl 35-37.5 cm	25931	●	
	Locking joint for foot support comp 40-50 cm, 50-60 cm	25932	●	●
5	Attachment tube foot support le	25934	●	●
6	Attachment tube foot support ri	25935	●	●
7	End caps foot support one piece	50830	●	●

User brakes

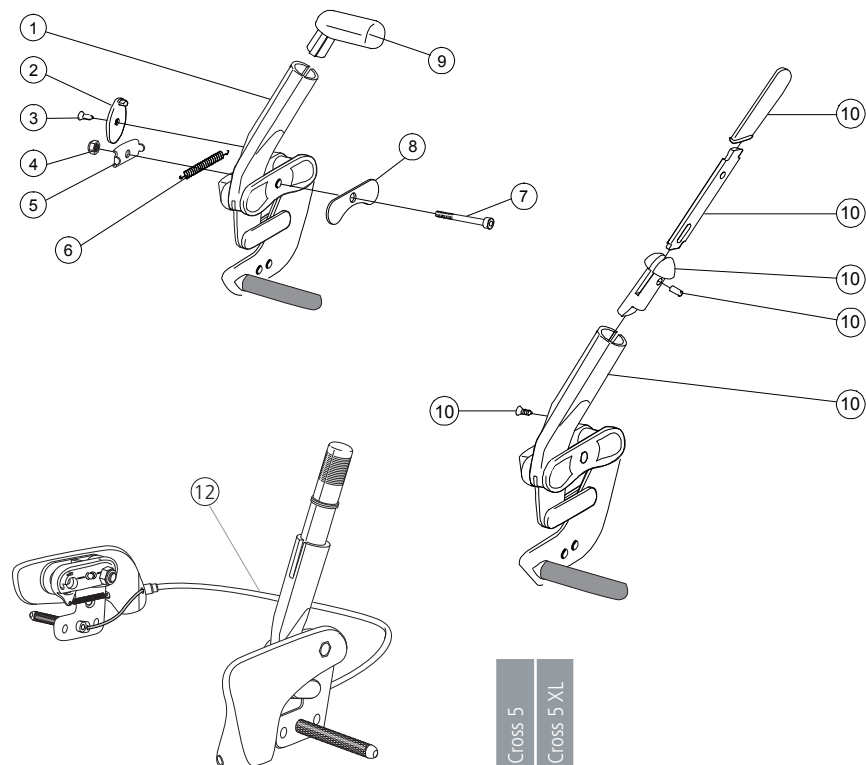[illegible]

Pos.	Description	Item. No.
1-9	Brake with angled red handle right	27528-1
	Brake with angled red handle left	27528-2
2	Stop detail brake Cross mod new	50373
3	Screw B 6 3.5x 9.5 FXS black	11384
4	Locking nut M 6 zink plated	12017
5	Washer inner brake mod 3	82564
6	Spring for brake Cross	80796
7	Screw M 6x 50 MC6S zink plated	11433
8	Washer outer brake mod 3	82575
9	Plug for brake angled red	51012
10	Brake lever extension Cross compl	25515

1-8, 10	Brake Cross extended handle right	27530-1
	Brake Cross extended handle left	27530-2
11	Brake handle angled compl kit red	27574
	Brake handle angled compl kit black	25537
1-8, 11	Brake with angled black handle right	27532-1
	Brake with angled black handle left	27532-2
12	Single hand brake right	27531-1
	Single hand brake left	27531-2

[illegible]

Rear wheel (C5)

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
	20" Rear wheel			
1	Rear wheel 20" C5 high pressure without handrim excl axle	28420	●	●
	Rear wheel 20" C5 high pressure/alu excl axle	28474	●	●
	Rear wheel 20" C5 high pressure/cellular rubber excl axle	28475	●	●
	Rear wheel 20" C5 high pressure/plastic coated excl axle	28476	●	●
	Rear wheel 20" C5 high pressure/stainless steel excl axle	28477	●	●
	Rear wheel 20" C5 solid/without handrim excl axle	27541	●	●
	Rear wheel 20" C5 solid/alu grey excl axle	27542	●	●
	Rear wheel 20" C5 solid/cellular rubber excl axle	27543	●	●
	Rear wheel 20" C5 solid/plastic coated excl axle	27957	●	●
	Rear wheel 20" C5 solid/stainless steel excl axle	27544	●	●
	Rear wheel 20" C5 without tube, tyre, handrim, excl axle	27545	●	●
	Rear wheel 20" C5 solid tube/stainless	27683	●	●
	Rear wheel 20" C5 solid tube/cellular rubber	27684	●	●
	Rear wheel 20" C5 solid tube/plastic coated	27685	●	●
	Rear wheel 20" C5 solid tube without handrim excl axle	27686	●	●
	Rear wheel 20" C5 matt hub high pressure/wo handrim excl axle	28478	●	●
	Rear wheel 20" C5 matt hub high pressure/plastic coated excl axle	28479	●	●
	Rear wheel 20" C5 matt hub high pressure/stainless steel excl axle	28480	●	●
	Rear wheel 20" C5 matt hub solid/wo handrim excl axle	28076	●	●
	Rear wheel 20" C5 matt hub solid/plastic coated excl axle	28077	●	●
	Rear wheel 20" C5 matt hub solid/stainless steel excl axle	28078	●	●
	Rear wheel 20" C5 matt hub wo tyre/tube handrim excl axle	28079	●	●

	22" Rear wheel		Cross 5	Cross 5 XL
1	Rear wheel 22" C5 high pressure/without handrim excl axle	27546	●	●
	Rear wheel 22" C5 high pressure/alu excl axle	27547	●	●
	Rear wheel 22 C5 high pressure handrim cellular rubber excl axle	27548	●	●
	Rear wheel 22" C5 high pressure/plastic coated excl axle	27549	●	●
	Rear wheel 22" C5 high pressure/stainless steel excl axle	27550	●	●
	Rear wheel 22" C5 solid excl axle	27551	●	●
	Rear wheel 22" C5 solid/alu excl axle	27552	●	●
	Rear wheel 22" C5 solid/cellular rubber excl axle	27553	●	●
	Rear wheel 22" C5 solid/plastic coated excl axle	27554	●	●
	Rear wheel 22" C5 solid/stainless steel excl axle	27555	●	●
1	Rear wheel 22" C5 without tube, tyre, handrim excl axle	27556	●	●
	Rear wheel 22" C5 solid tube/stainless	27687	●	●
	Rear wheel 22" C5 solid tube/cellular rubber	27688	●	●
	Rear wheel 22" C5 solid tube/plastic coated	27689	●	●
	Rear wheel 22" C5 solid tube without handrim excl axle	27690	●	●
	Rear wheel 22" C5 matt hub high pressure/wo handrim excl axle	28080	●	●
	Rear wheel 22" C5 matt hub high pressure/plastic coated excl axle	28081	●	●
	Rear wheel 22" C5 matt hub high pressure/stainless excl axle	28082	●	●
	Rear wheel 22" C5 matt hub solid/without handrim excl axle	28083	●	●
	Rear wheel 22" C5 matt hub solid/plastic coated excl axle	28084	●	●
	Rear wheel 22" C5 matt hub solid/stainless excl axle	28085	●	●
	Rear wheel 22" C5 matt hub without tyre/tube handrim excl axle	28086	●	●

Rear wheel cont. (C5)

Pos.	Description	Item. No.				
			Cross 5	Cross 5 XL		
1	24" Rear wheel					
	Rear wheel 24 C5 high pressure without handrim black excl axle	28860	●	●		
	Rear wheel 24 C5 high pressure/alu black excl axle	28861	●	●		
	Rear wheel 24 C5 high pressure/cellular rubber black excl axle	28862	●	●		
	Rear wheel 24 C5 high pressure/plastic coated black excl axle	28863	●	●		
	Rear wheel 24 C5 high pressure/stainless black excl axle	28864	●	●		
	Rear wheel 24 C5 high pressure/titanic black excl axle	28865	●	●		
	Rear wheel 24 high pressure alu Marathon Plus excl axle	28603	●	●		
	Rear wheel 24 high pressure cellular rubber Marathon Plus excl axle	28604	●	●		
	Rear wheel 24 high pressure plastic coated Marathon Plus excl axle	28605	●	●		
	Rear wheel 24 high pressure stainless steel Marathon plus excl axle	28606	●	●		
	Rear wheel 24 high pressure titanic Marathon Plus excl axle	28607	●	●		
	Rear wheel 24 high pressure without handrim Marathon Plus excl axle	28608	●	●		
	Rear wheel 24" C5 solid/without handrim excl axle	27563	●	●		
	Rear wheel 24" C5 solid/alu excl axle	27564	●	●		
	Rear wheel 24" C5 solid/cellular rubber excl axle	27565	●	●		
	Rear wheel 24" C5 solid/plastic coated excl axle	27566	●	●		
	Rear wheel 24" C5 solid/stainless steel excl axle	28844	●	●		
	Rear wheel 24" C5 solid/titanic excl axle	27568	●	●		
	Rear wheel 24" C5 without tube, tyre, handrim excl axle	27569	●	●		
	Rear wheel 24" C5 solid tube/cellular rubber	27692	●	●		
	Rear wheel 24" C5 solid tube/plastic coated	27693	●	●		
	Rear wheel 24" C5 solid tube without handrim excl axle	27694	●	●		
	Rear wheel 24" C5 carbon/plastic coated excl axle	82555	●			
	Rear wheel 24" C5 carbon/titanic excl axle	82534	●			
	Rear wheel 24" C5 matt hub high pressure/wo handrim excl axle	28087	●	●		
1	Rear wheel 24" C5 matt hub high pressure/plastic ctd excl axle	28088	●	●		
	Rear wheel 24" C5 matt hub high pressure/stainless excl axle	28089	●	●		
	Rear wheel 24" C5 matt hub solid/without handrim excl axle	28090	●	●		
	Rear wheel 24" C5 matt hub solid/plastic coated excl axle	28091	●	●		
	Rear wheel 24" C5 matt hub solid/stainless excl axle	28092	●	●		
	Rear wheel 24" C5 matt hub without tyre/tube handrim excl axle	28093	●	●		
	Rear wheel 24" carbon/titan excl axle	28095	●			
	26" Rear wheel					
	Rear wheel 26" C5 high pressure	27570	●	●		
	Rear wheel 26" C5 high pressure	27572	●	●		
	Rear wheel 26" C5 semipneumatic	27992	●	●		
	Rear wheel 26" C5 matt hub high pressure/stainless excl axle	28094	●	●		
	Axle					
	QR axle D12 std steel hub Cross, Prio, Crissy	82483	●	●		
	QR axle D12 light weight wheel	82581	●	●		

Rear wheel 20", 22", 24", 25" (C6, Maxx)


Pos.	Description	Art. No.	Cross 5	Cross 5 XL
	20" Rear wheel			
	Rear wheel 20 C6 high pressure al Speedlite excl axle	83908	●	●
	Rear wheel 20 C6 high pressure stainl Speedlite excl axle	83909	●	●
	Rear wheel 20 C6 high pressure plastic coated Speedlite excl axle	83884	●	●
	Rear wheel 20 C6 solid al excl axle	83885	●	●
	Rear wheel 20 C6 solid stainl excl axle	83910	●	●
	Rear wheel 20 C6 solid plastic coated excl axle	83911	●	●
	22" Rear wheel			
	Rear wheel 22 C6 high pressure al Speedlite excl axle	83880	●	●
	Rear wheel 22 C6 high pressure stainl Speedlite excl axle	83906	●	●
	Rear wheel 22 C6 high pressure plastic coated Speedlite excl axle	83881	●	●
	Rear wheel 22 C6 solid al excl axle	83882	●	●
	Rear wheel 22 C6 solid stainl excl axle	83907	●	●
	Rear wheel 22 C6 solid plastic coated excl axle	83883	●	●

Pos.	Description	Art. No.	Cross 5	Cross 5 XL
	24" Rear wheel			
	Rear wheel 24 C6 high pressure al Speedlite excl axle	83873	●	●
	Rear wheel 24 C6 high pressure stainl Speedlite excl axle	83874	●	●
	Rear wheel 24 C6 high pressure plastic coated Speedlite excl axle	83875	●	●
	Rear wheel 24 C6 solid al excl axle	83876	●	●
	Rear wheel 24 C6 solid stainl excl axle	83877	●	●
	Rear wheel 24 C6 solid plastic coated excl axle	83878	●	●
	25" Rear Wheel			
	Rear Wheel 25 Maxx high pressure hard al Trailblazer ri excl axle	28964-1	●	●
	Rear Wheel 25 Maxx high pressure hard al Trailblazer le excl axle	28964-2	●	●
	Rear Wheel 25 Maxx high pressure plastad Trailblazer ri excl axle	28965-1	●	●
	Rear Wheel 25 Maxx high pressure plastad Trailblazer le excl axle	28965-2	●	●
	Axle			
2	QR axle D12,7x110	83872	●	●

Rear wheel with attendant brake


Cross 5
Cross 5 XL

Pos.	Description	Item. No.
1	20" Rear wheel	
	Rear wheel 20" drum brake solid handrim aluminium	81337
	Rear wheel 20" drum brake solid inner tube/stainless	27695
	Rear wheel 20" drum brake solid inner tube/cellular rubber	27696
	Rear wheel 20" drum brake solid inner tube/plastic coated	27697
	Rear wheel 20" drum brake solid inner tube wo handrim excl axle	27698
	Rear wheel 20" drum brake matt hub solid/stainless	28097
	Rear wheel 20" drum brake matt hub solid/wo handrim excl axle	28096
	Rear wheel 20" drum brake matt hub low pressure/stainless	28098
	Rear wheel 20" drum brake matt hub low pressure/without handrim excl axle	28099
1	22" Rear wheel	
	Rear wheel 22" drum brake high pressure handrim aluminium	81342
	Rear wheel 22" drum brake solid handrim aluminium	81339
	Rear wheel 22" drum brake solid handrim plastic-coated	81340
	Rear wheel 22" drum brake solid handrim stainless	81341
	Rear wheel 22" drum brake high pressure handrim plastic coated	81343
	Rear wheel 22" drum brake tigh pressure handrim stainless	81344
	Rear wheel 22" drum brake solid inner tube/stainless	27255
	Rear wheel 22" drum brake solid inner tube/cellular rubber	27699
	Rear wheel 22" drum brake solid inner tube/plastic coated	27700
	Rear wheel 22" drum brake solid inner tube wo handrim excl axle	27701
	Rear wheel 22" drum brake matt hub solid/wo handrim excl axle	28100
	Rear wheel 22" drum brake matt hub solid/plastic coated	28101
	Rear wheel 22" drum brake matt hub solid/stainless	28102
	Rear wheel 22" drum brake matt hub high pressure/plastic coated	28103
	Rear wheel 22" drum brake matt hub high pressure/stainless	28104
	Rear wheel 22" drum brake matt hub high pr/wo handrim excl axle	28105

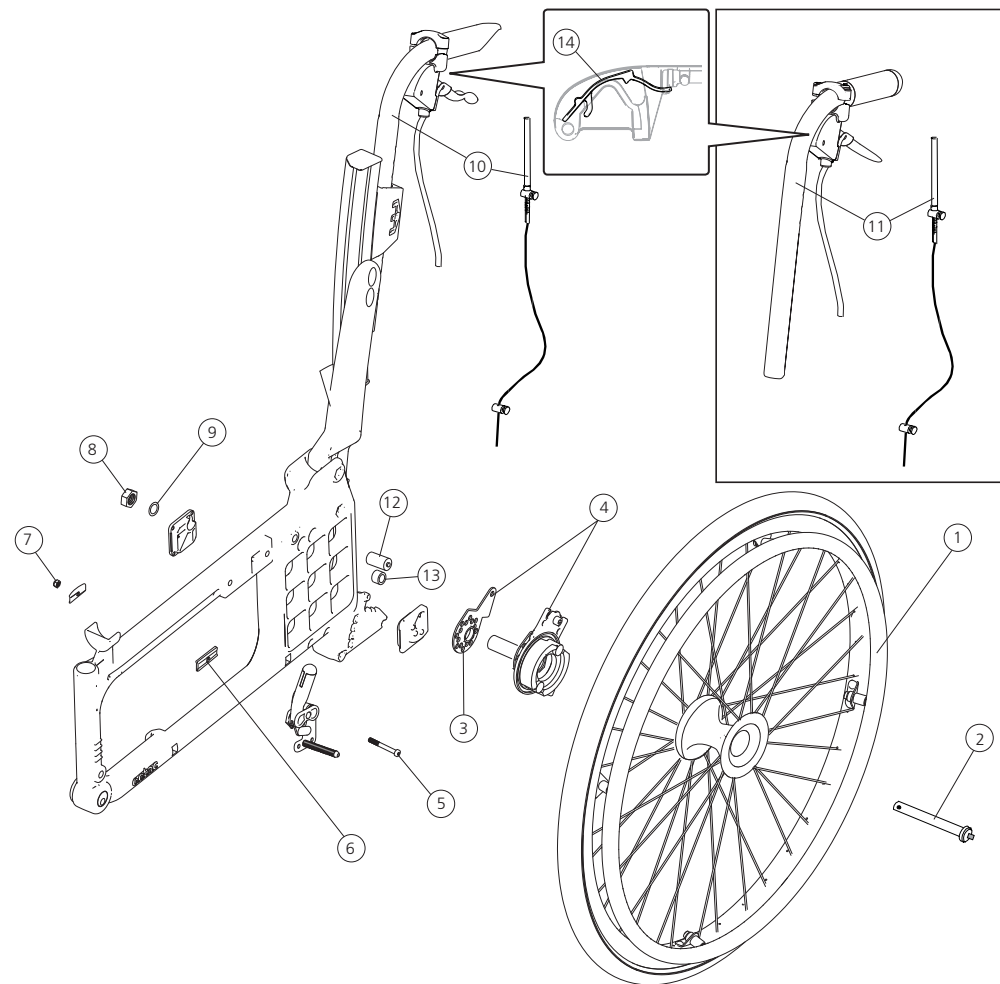

Rear wheel with attendant brake cont.


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	24" Rear wheel			
	Rear wheel 24" drum brake high pressure handrim aluminium	81349	●	●
	Rear wheel 24" drum brake solid handrim aluminium	81346	●	●
	Rear wheel 24" drum brake solid handrim plastic-coated	81347	●	●
	Rear wheel 24" drum brake solid handrim stainless	81348	●	●
	Rear wheel 24" drum brake high pressure handrim plastic-coated	81350	●	●
	Rear wheel 24" drum brake high pressure handrim stainless	81351	●	●
	Rear wheel 24" drum brake solid inner tube/stainless	27256	●	●
	Rear wheel 24" drum brake solid inner tube/cellular rubber	27702	●	●
	Rear wheel 24" drum brake solid inner tube/plastic coated	27703	●	●
	Rear wheel 24" drum brake solid inner tube wo handrim excl axle	27704	●	●
	Rear wheel 24" drum brake matt hub solid/wo handrim excl axle	28106	●	●
	Rear wheel 24" drum brake matt hub solid/plastic-coated	28107	●	●
	Rear wheel 24" drum brake matt hub solid/stainless	28108	●	●
	Rear wheel 24" drum brake matt hub high pr/plastic-coated	28109	●	●
	Rear wheel 24" drum brake matt hub high pressure/stainless	28110	●	●
	Rear wheel 24" db matt hub high pressure/wo handrim excl axle	28111	●	●
	26" Rear wheel			
	Rear wheel 26" DB solid sealed handrim stainless	28112	●	●
2-9	Attachment kit drum brake	27627	●	●
2-9, 11	Drum brake C5 without rearw	27890	●	
	Drum brake C5 XL without rearw	27891		●
2	Qr axle drum brake	81325	●	●
3	Carrier drum brake	82566	●	●
4	Drum brake Cross 5 right	27590	●	●
	Drum brake Cross 5 left	27591	●	●
5	Screw M6x55 MC6S	11351	●	●
6	Adapter brake 5 mm	51019	●	●
7	Locking nut M6 Zn8/P4 DIN 985-6	12035	●	●
8	Nut M16x1 NV 24 B8 fzb	12052	●	●
9	Schnorr washer 17x24x1.3 fzb	13084	●	●

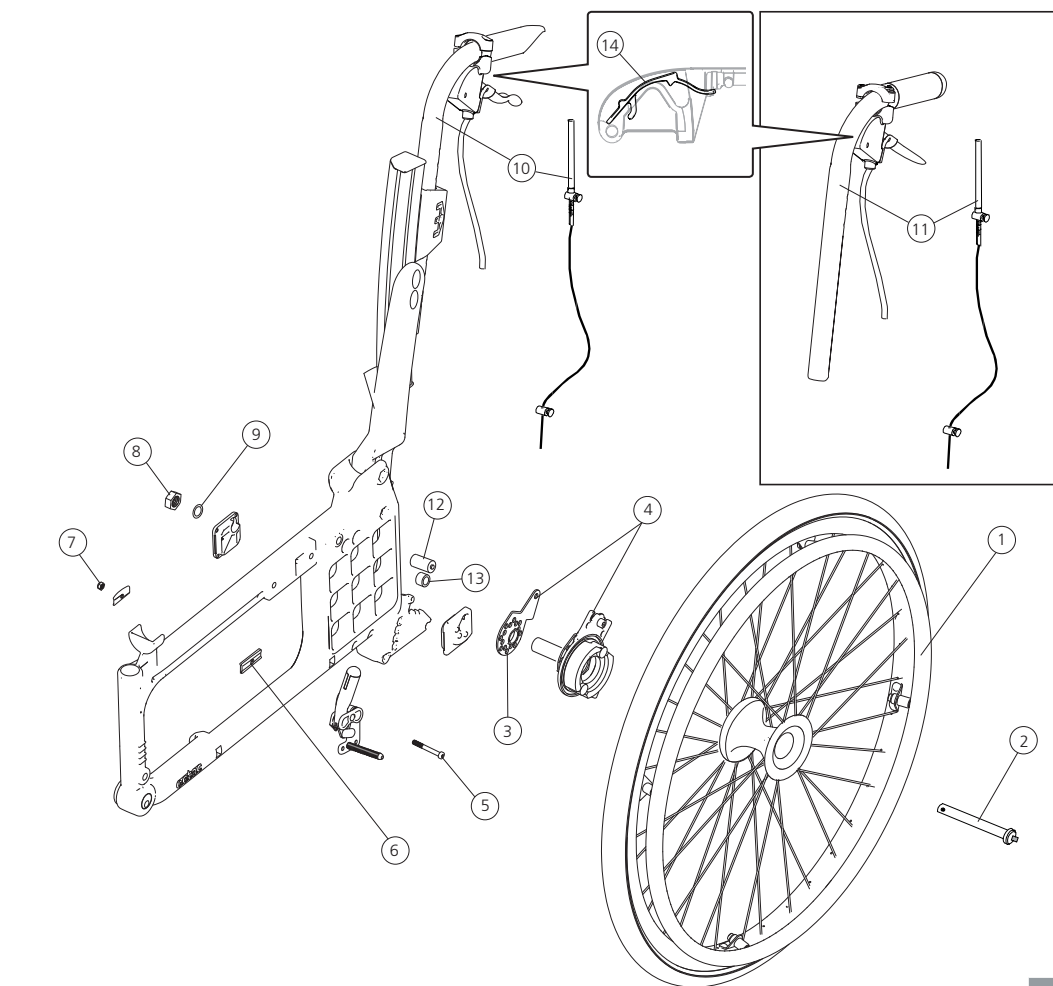

11	Push handle height adj Cross opening grip 100 mm	27625-01
	Push handle height adj Cross XL opening grip 100 mm	27626-01
12-13	Attachment kit drum brake low	27628
14	Wire lock brake handle 100 mm 10 pcs	82854
	Wire lock brake handle 75 mm 10 pcs	82855

Cross 5	Cross 5 XL
●	
	●
●	●
●	●


Attendant brake for fixed back support with fixed push handles

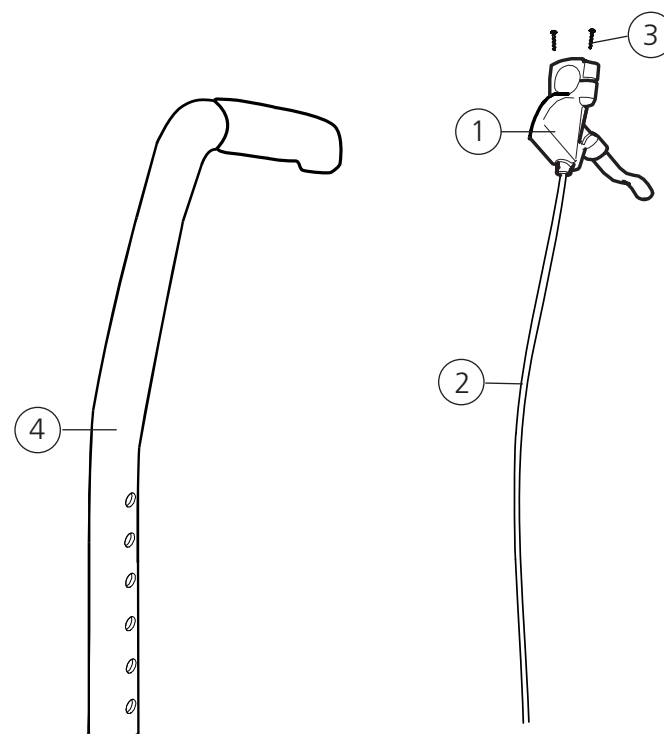

Pos.	Description	Item. No.
1	Brake handle drum brake	81804
2	Wire kit drum brake Cross Twin Tran	81805
3	Screw DG 5x20 WN 1552	11504
4	Back support tube with push handle drum brake	27345-01


Cross 5	Cross 5 XL
●	●
●	●
●	●
●	●

Single hand drive

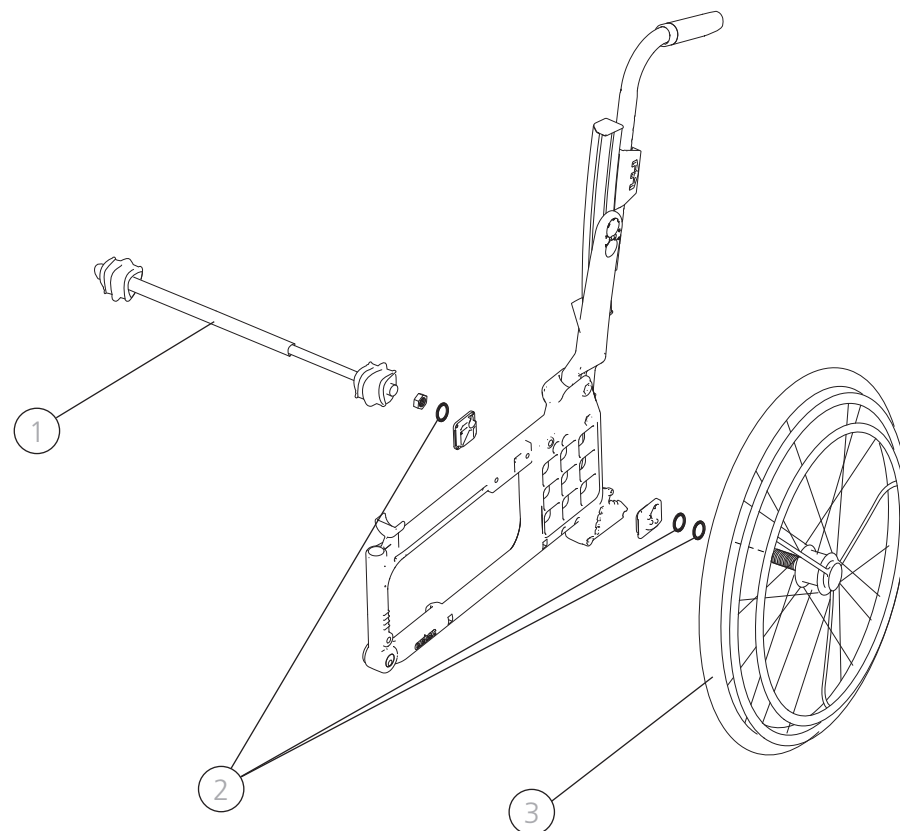


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Telescoping shaft single hand drive Cross 35-37.5 cm compl	27317	●	
	Telescoping shaft single hand drive compl Cross 40-42.5 cm/Next 35-37.5 cm	27318	●	
	Telescoping shaft single hand drive compl Cross 45-50 cm/Next 40-45 cm	27319	●	●
	Telescoping shaft single hand drive compl Cross 52.5-57.5 cm/Next 47.5-55 cm	27320		●
	Telescoping shaft single hand drive compl Cross 60 cm/Next 57.5 cm	27321		●
2-3	Single-hand drive 20"Cross/Cross XL compl	26534	●	●
	Single-hand drive 22"Cross/Cross XL compl	26533	●	●
	Single-hand drive 24"Cross/Cross XL compl	26522	●	●

Tyre and inner tube (C5)

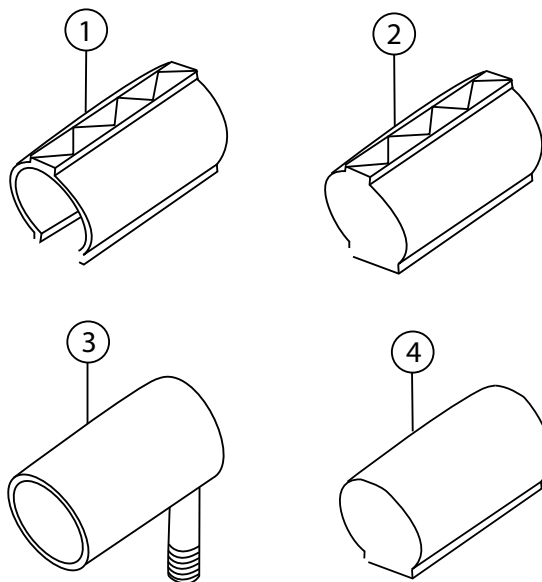


Pos.	Description	Item. No	Cross 5	Cross 5 XL
1	Tyre 22x1 pneumatic high pressure	81125	●	●
	Tyre 24x1 high pressure black	83329	●	●
	Tyre 24 Schwalbe Marathon Plus	83301	●	●
	Tyre 24 x1 high pressure for carbon	82907	●	
2	Tyre 20 solid	80787	●	●
	Tyre 22 solid grey	81384	●	●
	Tyre 24 solid grey	80768	●	●
3	Innertube 20 pneumatic	80132	●	●
	Innertube 22 pneumatic high pressure	80136	●	●
	Innertube 24 pneumatic high pressure	80560	●	●
	Tube 24 x1 high pressure for carbon	82908	●	
4	Innertube 20 massive	82706	●	●
	Innertube 22 massive	82705	●	●
	Innertube 24 massive	82704	●	●
5	Tyre flaps 24 for carbon	82909	●	
	Tyre flaps 24	97044	●	●

Tyre and inner tube (C6, Maxx)

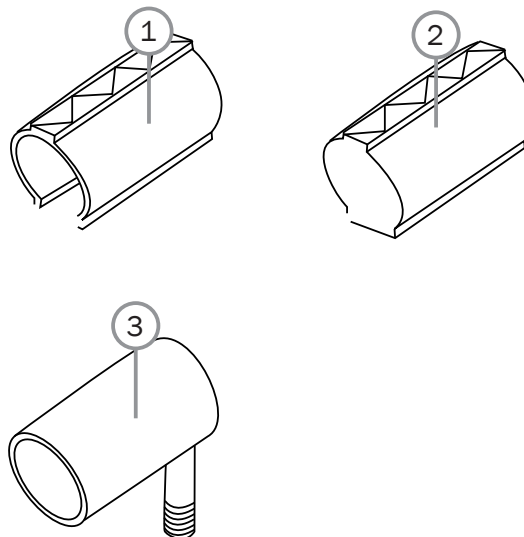

Pos.	Description	Art. No.	Cross 5	Cross 5 XL
1	Tyre 20x1 (25-451) Speedlite black	83897	●	●
	Tyre 22x1 (25-501) Speedlite black	83898	●	●
	Tyre 24x1 (25-540) Speedlite black	83899	●	●
	Tyre 25x1 (25-559) Trailblazer black	83900	●	●
2	Tyre 20x1 3/8 (37-451) solid black	83347	●	●
	Tyre 22x1 3/8 (37-501) solid black	83348	●	●
	Tyre 24x1 1/4 (32-540) solid black	83349	●	●
3	Inner tube 20x1 (25-451) high pressure	82199	●	●
	Inner tube 22x1 (25-501) high pressure	80136	●	●
	Inner tube 24x1 (25-540) high pressure	80560	●	●
	Inner tube 25x1 (25-559) high pressure	83904	●	●


Handrims (C5)

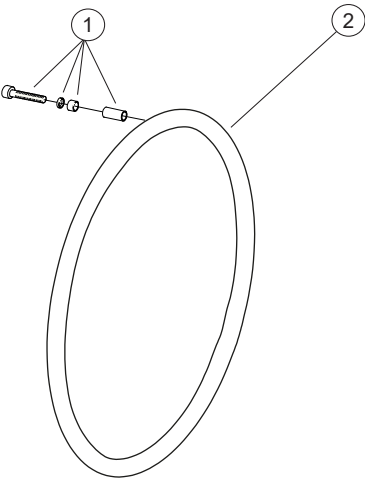

Pos.	Description	Item. No.	Cross	Cross 5 XL
1	Attachment kit handrim titanium/aluminium/cellular/stainless	24514	●	●
2	20" Rear wheel			
	Handrim D465 wheel 20" alu	81109	●	●
	Handrim D465 wheel 20" vinyl	81229	●	●
	Handrim D425 wheel 20" cellular, clip mounted	24442	●	●
	Handrim D465 wheel 20" stainless	21034	●	●
	Handrim D465 wheel 20" stainless sealed	83006	●	●
	Handrim D465 wheel 20" vinyl	83010	●	●
	22" Rear wheel			
	Handrim D500 wheel 22" alu	25636	●	●
	Handrim D500 wheel 22" stainless	24395	●	●
	Handrim D500 wheel 22" vinyl	80895	●	●
	Handrim D469 wheel 22" cellular, clip mounted	24551	●	●
	Handrim D500 wheel 22" stainless sealed	83007	●	●
	Handrim D500 wheel 22" vinyl	83011	●	●

2	24" Rear wheel			
	Handrim D540 wheel 24" alu	25181	●	●
	Handrim D540 wheel 24" stainless	24023	●	●
	Handrim D540 wheel 24" vinyl plastic-coated	82998	●	●
	Handrim D540 wheel 24" cellular clip mounted	80800	●	●
	Handrim D540 wheel 24" stainless sealed	83008	●	●
	Handrim D540 wheel 24" vinyl with sealing	83012	●	●
	26" Rear wheel			
	Handrim D585 wheel 26" alu	81508	●	●
	Handrim D585 wheel 26" stainless	81510	●	●
	Handrim D585 wheel 26" stainless	83009	●	●

Handrim C6, Maxx

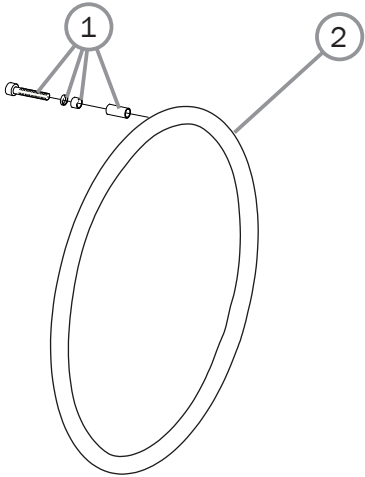


Pos.	Description	Art. No.		Cross	Cross 5 XL
1	Assembly kit handrim 6 ap (not plastic coated)	83892		●	●
2	20" Rear wheel				
	Handrim 20" al 6 ap	83890		●	●
	Handrim 20" stainless steel 6 ap	83891		●	●
	Handrim 20" plastic coated 6 ap	83339		●	●
	22" Rear wheel				
	Handrim 22 al 6 fp	83888		●	●
	Handrim 22 stainless steel 6 ap	83889		●	●
	Handrim 22 plastic coated 6 ap	83338		●	●

Pos.	Description	Art. No.		Cross	Cross 5 XL
2	24" Rear wheel				
	Handrim 24 al 6 ap	83886		●	●
	Handrim 24 stainless steel 6 ap	83887		●	●
	Handrim 24 plastic coated 6 ap	83335		●	●
	25" Rear wheel				
	Handrim 25 hard al 6 ap	83340		●	●
	Handrim 25 plastic coated 6 ap	83341		●	●

Castors, front forks 57 mm hub width


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Screw M 8x 1x40 MC6S 12.9 Zn8/P4	11305	●	●
2	Off center sleeve jagged	81994	●	●
3	Front fork housing Cross	80606	●	●
	Front fork housing extended	80899	●	●
	Front fork housing low Cross	83417	●	●
1, 3, 9-11, 13-19	Front fork housing Cross std cpl	28147	●	●
	Front fork housing extended cpl	25763	●	●
	Front fork housing low Cross cpl	28633	●	●
4	Screw M6x 75 MC6S 12.9 black chrome	11338	●	●
5	Castor			
6	Cover axle wheel	51284	●	●
	Cover axle wheel (50pcs)	27954	●	●
7	Locking nut M6 Zn8/P4	12035	●	●
4-7	Castor 4 100x30 solid hard	27870	●	●
	Castor 5 125x30 solid hard	27865	●	●
	Castor 5 125x30 solid hard 2 pcs	27865-02	●	●
	Castor 5 125x30 solid hard 10 pcs	27865-10	●	●
	Castor 6 150x30 solid soft	27863	●	●
	Castor 6 150x30 solid hard	27869	●	●
	Castor 6.5 165x30 solid hard	27866	●	●
	Castor 6.5 165x30 solid hard 2 pcs	27866-02	●	●
	Castor 6.5 165x30 solid hard 10 pcs	27866-10	●	●
	Castor 8 200x35 solid soft	27864	●	●
8	Front fork short 4" castor	26237-01	●	
	Front fork short excl axle	24554-01	●	●
	Front fork medium excl axle	24555-01	●	●
	Front fork long excl axle	24556-01	●	●
	Front fork short excl axle 4 mm black	24832-01	●	●
	Front fork medium excl axle 4 mm black	24833-01	●	●
			●	●
9	Locking nut 1/2 UNF low	12077	●	●
10	Washer BRB 13.5 Zn8/P4	13031	●	●
11	Washer 12.7x24x2.5	13032	●	●

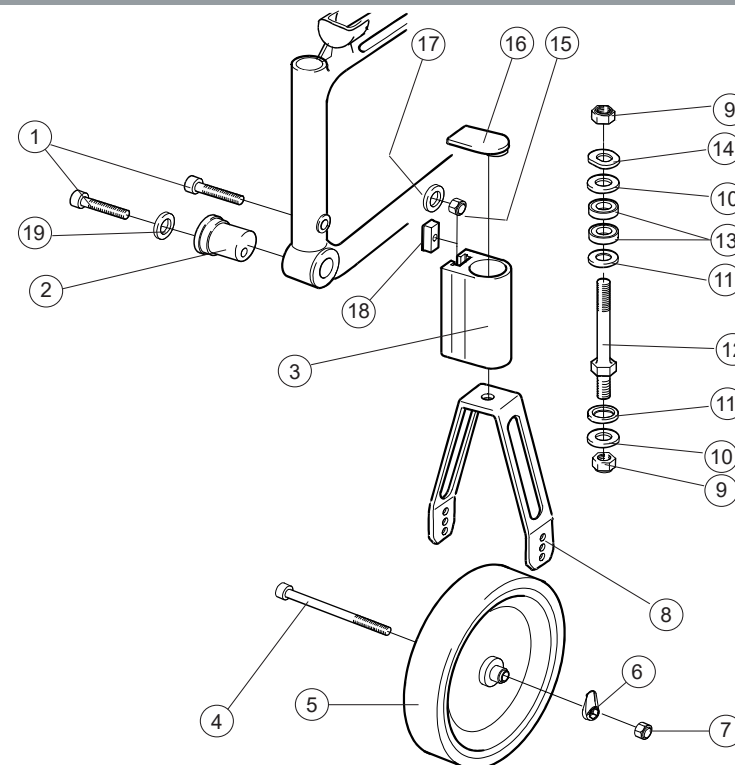


12	Axle front fork steel	83812	●	●
8-12	Front fork short 4" castor	26238	●	
	Front fork short steel axle	24677	●	●
	Front fork medium steel axle	24678	●	●
	Front fork long steel axle	24679	●	●
	Front fork short steel axle 4 mm	24834	●	●
	Front fork medium steel axle 4 mm	24835	●	●
13	Ball bearing 1/2 EE4 2RS	80244	●	●
14	Washer 13x24x1.2 DIN 137B zink plated	13040	●	●
15	Locking nut M 8x 1 DIN 985 zink plated	12051	●	●
16	Plug frontfork housing	51220	●	●
	Plug for extended frontfork housing	50429	●	●
17	Washer BRB 8.4x 16x 1.6 zink plated	13001	●	●
18	T-nut	82922	●	●
19	Washer	13092	●	●

Castors, front forks 45 mm hub width

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Screw M8x1x40 MC6S 12.9 Zn8/P4	11305	●	●
2	Washer	13092	●	●
3	Off center sleeve jagged	81994	●	●
4	Front fork housing H142	28968	●	●
5	Locking nut M 6 Zn8/P4	12035	●	●
6	Cover axle wheel PA 6	51284	●	●
	Cover axle wheel PA 6 (50 pcs)	28966	●	●
7	Screw M6x60 MC6S Zn8/P4	11408	●	●
8	Castor			
5-6, 8	Castor 4 100x34 cpl	28919	●	●
	Castor 5 125x34 cpl	28921	●	●
	Castor 6 150x34 cpl	28922	●	●
	Castor 8 200x38 cpl	28526	●	●
9	Front fork steel S w axle	24038	●	●
	Front fork steel S+25 w axle	28969	●	●
	Front fork alu XS w axle	28923	●	●
10	Locking nut 1/2 UNF low	12077	●	●
11	Washer BRB 13.5 Zn8/P4	13031	●	●
12	Washer 12.7x24x2.5	13032	●	●
13	Ball bearing 1/2 EE4 2RS	80244	●	●
14	Washer 13x24x1.2 DIN 137B zink plated	13040	●	●
15	Locking nut M 8x 1 DIN 985 zink plated	12051	●	●
16	Plug front fork housing	51220	●	●
	Plug for extended front fork housing	50429	●	●
17	Washer BRB 8.4x 16x 1.6 zink plated	13001	●	●
18	T-nut	82922	●	●

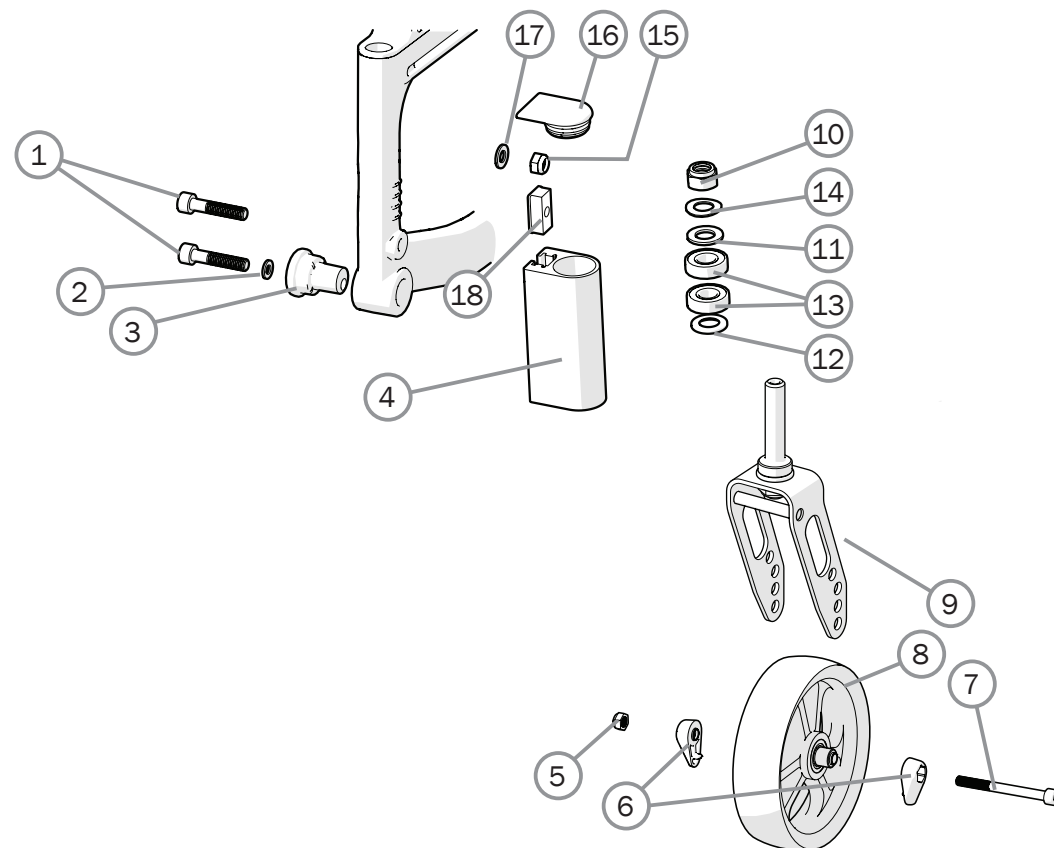


Widening kit

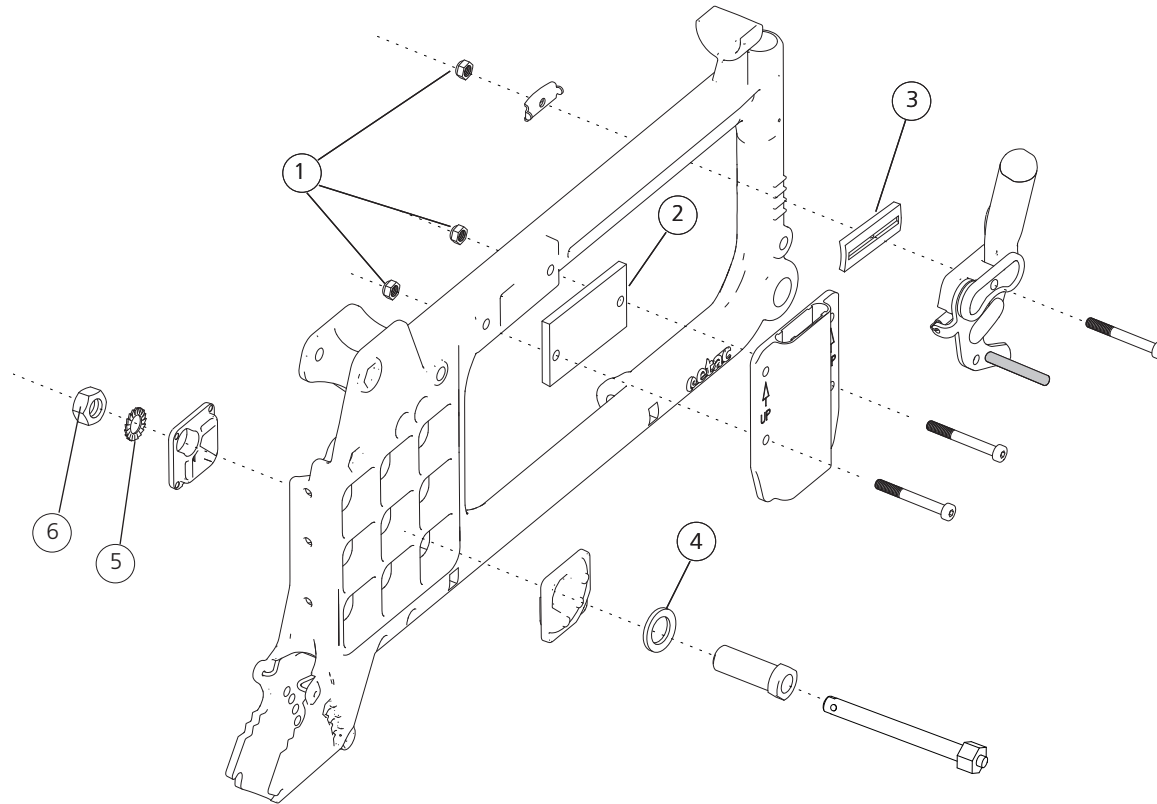

Pos.	Description	Item. No.
1-6	Width expansion kit 10 mm	27533
1	Locking nut M 6 black	12035
2	Spacer seat extension	51018
3	Adapter brake	51019
4	Spacer hub	20508
5	Schnorr washer 17x24x1.3	13084
6	Nut M16x1	12052

[illegible]

Seat upholstery/Seat extension back support 3A

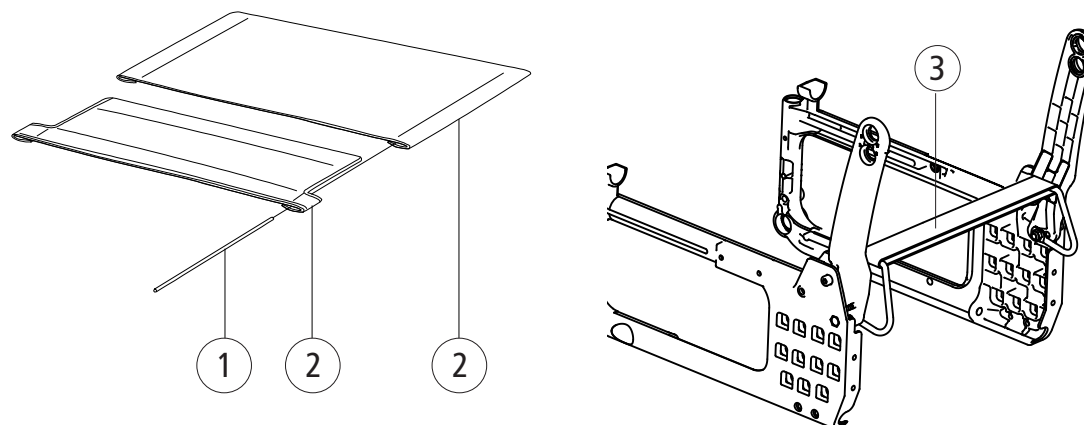

Cross 5
Cross 5 XL

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
	For Cross back 3A			
1	Tension rod L381 short upholstery seat	80617	●	●
	Tension rod L446 long upholstery seat	80618	●	●
1-2	Upholstery seat Basic short Basic/Cross 3A 35 cm (14) black	62718-60	●	
	Upholstery seat Basic short Basic/Cross 3A 37.5 cm (15) black	62719-60	●	
	Upholstery seat Basic short Basic/Cross 3A 40 cm (16) black	62720-60	●	
	Upholstery seat Basic short Basic/Cross 3A 42.5 cm (17) black	62721-60	●	
	Upholstery seat Basic short Basic/Cross 3A 45 cm (18) black	62722-60	●	
	Upholstery seat Basic short Basic/Cross 3A 47.5 cm (19)	62978-60	●	●
	Upholstery seat Basic short Basic/Cross 3A 50 cm (20) black	62979-60	●	●
	Upholstery seat Basic short Basic/Cross 3A 52.5 cm (21) black	62980-60		●
	Upholstery seat Basic short Basic/Cross 3A 55 cm (22) black	62981-60		●
	Upholstery seat Basic short Basic/Cross 3A 57.5 cm (23) black	62982-60		●
	Upholstery seat Basic short Basic/Cross 3A 60 cm (24) black	62983-60		●


Cross 5
Cross 5 XL

1-2	Upholstery seat Basic long Basic/Cross 3A 35 cm (14) black	62728-60	●	
	Upholstery seat Basic long Basic/Cross 3A 37.5 cm (15) black	62729-60	●	
	Upholstery seat Basic long Basic/Cross 3A 40 cm (16) black	62730-60	●	
	Upholstery seat Basic long Basic/Cross 3A 42.5 cm (17) black	62731-60	●	
	Upholstery seat Basic long Basic/Cross 3A 45 cm (18) black	62732-60	●	
	Upholstery seat Basic long Basic/Cross 3A 47.5 cm (19) black	62984-60	●	●
	Upholstery seat Basic long Basic/Cross 3A 50 cm (20) black	62985-60	●	●
	Upholstery seat Basic long Basic/Cross 3A 52.5 cm (21) black	62986-60		●
	Upholstery seat Basic long Basic/Cross 3A 55 cm (22) black	62987-60		●
3	Upholstery seat Basic long Basic/Cross 3A 57.5 cm (23) black	62988-60		●
	Upholstery seat Basic long Basic/Cross 3A 60 cm (24) black	62989-60		●
	Seat extender 35-40 cm	63291	●	
	Seat extender 42,5-50 cm	63292	●	●
	Seat extender 52,5-60 cm	63293	●	●


Solid seat

[illegible]

Pos.	Description	Item. No.
1	Seat hook back ECS	26414-01
2	Seat hook front ECS	26413-01
3	Washer BRB 5.3 zink plated	13017
4	Screw M5x 18 MC6S 8.8 zink plated	11424
5	Domed not M4 zink plated	12068
6	Nut M4 M6M zink plated	12067
7	Washer 4.3x9x0.8 zink plated	13066
8	Screw M4x12 M6S A2	11707
1-8	Solid seat ECS 35-37.5 cm (14-15) short compl	26720
	Solid seat ECS 40-42.5 cm (16-17) short compl	26721
	Solid seat ECS 45-47.5 cm (18-19) short compl	26722
	Solid seat ECS 50-52.5 cm (20-21) short compl	26723
	Solid seat ECS 55-57.5 cm (22-23) short compl	26724
	Solid seat ECS 60 cm (24) short compl	26660
	Solid seat ECS 35-37.5 cm (14-15) long compl	26725
	Solid seat ECS 40-42.5 cm (16-17) long compl	26726
	Solid seat ECS 45-47.5 cm (18-19) long compl	26727
	Solid seat ECS 50-52.5 cm (20-21) long compl	26728
	Solid seat ECS 55-57.5 cm (22-23) long compl	26729
	Solid seat ECS 60 cm (24) long compl	26662

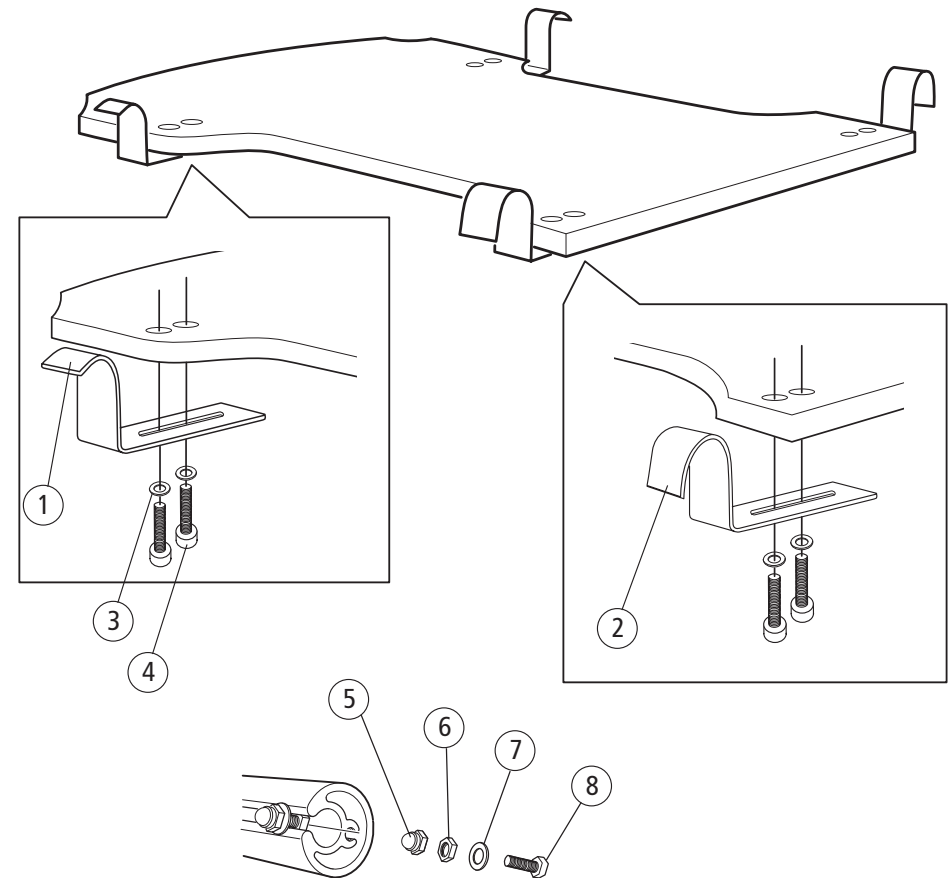


Back support upholstery back support model 2

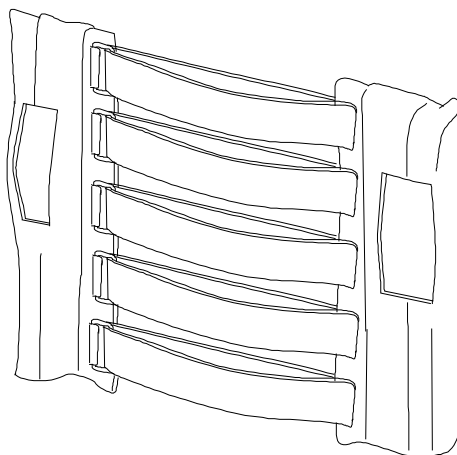


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Upholstery back 2C adj 35/37,5 cm	63317-60	●	
	Upholstery back 2C adj 40/42,5 cm	63318-60	●	
	Upholstery back 2C adj 45 cm	63319-60	●	
	Upholstery back 2C adj 47,5/50 cm	63320-60	●	●
	Upholstery back 2C adj 52,5/55 cm	63321-60	●	●
	Upholstery back 2C adj 57,5/60 cm	63322-60		●
	Upholstery back 2C adj high 35/37,5 cm	63327-60	●	
	Upholstery back 2C adj high 40/42,5 cm	63328-60	●	
	Upholstery back 2C adj high 45 cm	63329-60	●	
	Upholstery back 2C adj high 47,5/50 cm	63330-60	●	●

Backsupport upholstery, back support 3A

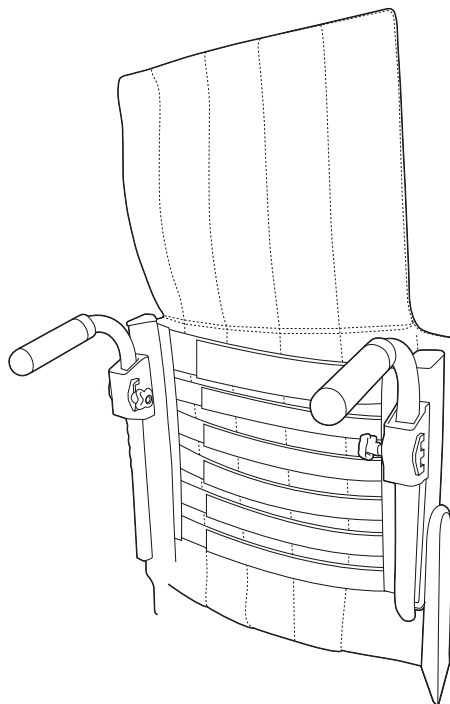


Pos.	Description	Item. No.	Cross	Cross 5 XL
1	Upholstery back Cross adj mod 3 rh adj 35-40 cm (14-16)	62953-60	●	
	Upholstery back Cross adj mod 3 rh jb 42.5-50 cm (17-20)	62954-60	●	●
	Upholstery back Cross adj mod 3 rh jb 52.5-60 cm (21-24)	62955-60		●
	Tension rod L296 Cross back mod 3	82461	●	●
	Upholstery back Cross 3A adj high 35-40 cm	63014-60	●	
	Upholstery back Cross 3A adj high 42.5-50 cm	63015-60	●	●
	Upholstery back Cross 3A adj high 52.5-60 cm	63016-60		●
	Tension rod L355 Cross 3A high back support	82586	●	●


Back support upholstery, back support 2B

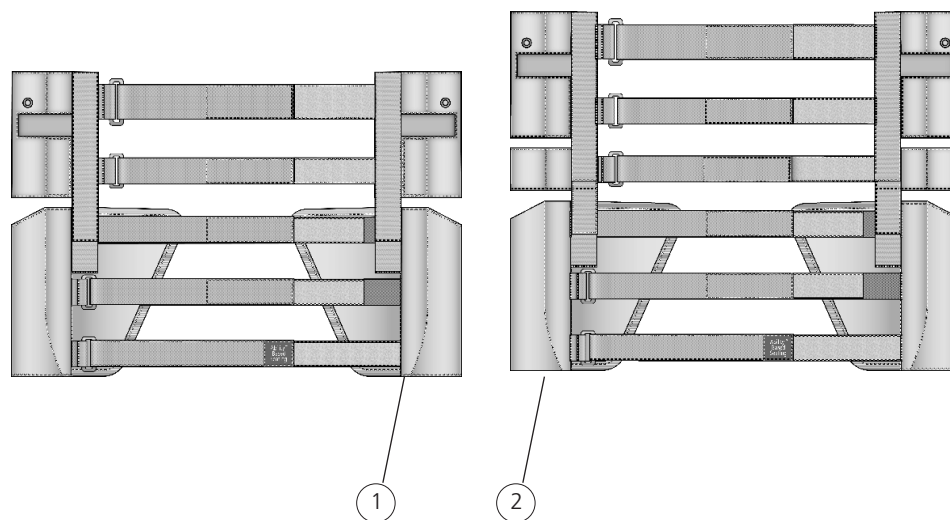

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Upholstery back B2B adj 35-40 cm	63257-60	●	
	Upholstery back B2B adj 42.5-50 cm	63258-60	●	
	Upholstery back B2B adj 52.5-60 cm	63259-60	●	
2	Upholstry back B2B high 35-40 cm	63260-60	●	
	Upholstry back B2B high 42.5-50 cm	63261-60	●	
	Upholstry back B2B high 52.5-60 cm	63262-60	●	
3	Extension part back 2B 35-40 cm mod 2	63273	●	
	Extension part back 2B 42,5-47,5 cm mod 2	63275	●	
	Extension part back 2B 50-55 cm mod 2	63276	●	
4	Pad size 20x15x1 cm	63272	●	
5	Insert widening pad	51218	●	
4-5	Widening pad	28637	●	

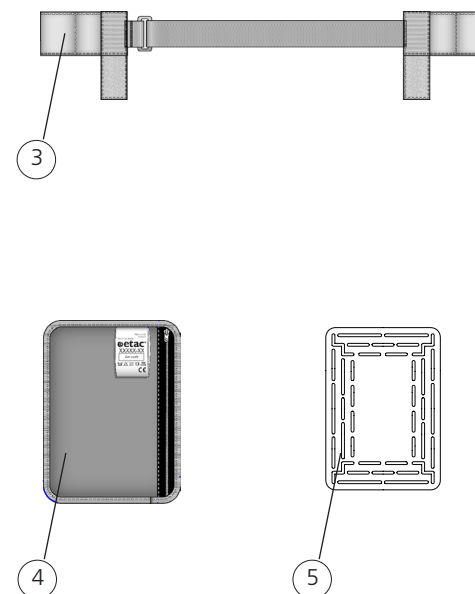


Back support upholstery, fixed back support

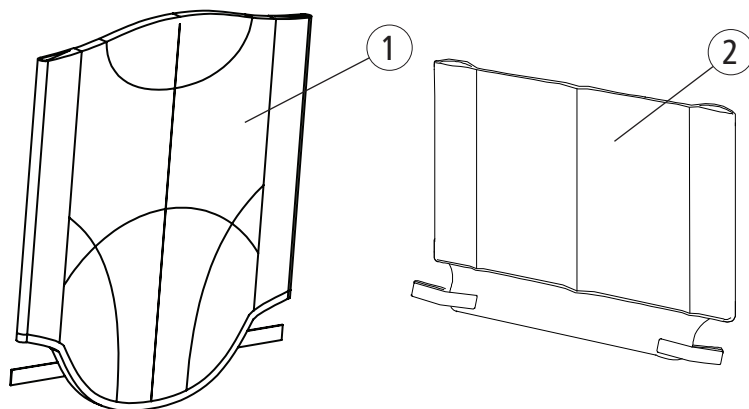

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Upholstery back M100 fixed 37.5 cm	62534-70	●	
	Upholstery back M100 fixed 40 cm	62535-70	●	
	Upholstery back M100 fixed 42,5 cm	62536-70	●	
	Upholstery back M100 fixed 45 cm	62537-70	●	
	Upholstery back M100 fixed 47,5 cm	62538-70	●	
	Upholstery back M100 fixed 50 cm	62539-70	●	
2	Upholstery back M100/Twin adj 35-37,5 cm	62364-70	●	
	Upholstery back M100/Twin adj 40-42,5 cm	62365-70	●	
	Upholstery back M100/Twin adj 45 cm	62366-70	●	
	Upholstery back M100/Twin adj 47,5-50 cm	62367-70	●	

Back support cover, back support model 2

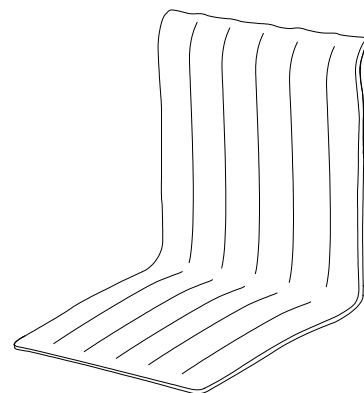


Pos.	Decription	Item. No.	Cross 5	Cross 5 XL
	Cover back 35 cm (14") dark grey	62135-60	●	
	Cover back 37.5 cm (15") dark grey	62387-60	●	
	Cover back 40 cm (16") dark grey	62136-60	●	
	Cover back 42,5 cm (17") dark grey	62388-60	●	
	Cover back 45 cm (18") dark grey	62137-60	●	
	Cover back 47.5 cm (19") dark grey	62389-60	●	●
	Cover back 50 cm (20") dark grey	62305-60	●	●
	Cover back 52.5 cm (21") dark grey	62432-60	●	●
	Cover back 55 cm (22") dark grey	62433-60	●	●
	Cover back 57,5 cm (23") dark grey	62434-60		●
	Cover back 60 cm (24") dark grey	62435-60		●
	Cover back high 35 cm (14") dark grey	62478-60	●	
	Cover back high 37,5 cm (15") dark grey	62479-60	●	
	Cover back high 40 cm (16") dark grey	62469-60	●	
	Cover back high 42,5 cm (17") dark grey	62470-60	●	
	Cover back high 45 cm (18") dark grey	62471-60	●	


	Cover back high 47,5 cm (19") dark grey	62472-60	●	●
	Cover back high 50 cm (20") dark grey	62473-60	●	●
	Cover back high 52.5 cm (21") dark grey	62480-60	●	●
	Cover back high 55 cm (22") dark grey	62481-60	●	●
	Cover back Cr/Ne high 57,5 cm (23")	62482-60		●
	Cover back Cr/Ne high 60 cm (24")	62483-60		●
	Cover back Cr/Ne comf sth/high dark grey 35 cm (14")	62895-60	●	
	Cover back Cr/Ne comf sth/high dark grey 37,5 cm (15")	62896-60	●	
	Cover back Cr/Ne comf sth/high dark grey 40 cm (16")	62897-60	●	
	Cover back Cr/Ne comf sth/high dark grey 42,5 cm (17")	62898-60	●	
	Cover back Cr/Ne comf sth/high dark grey 45cm (18")	62899-60	●	
	Cover back Cr/Ne comf sth/high dark grey 47.5 cm (19")	62900-60	●	●
	Cover back Cr/Ne comf sth/high dark grey 50 cm (20")	62901-60	●	●
	Cover back Cr/Ne Komf std/high 52.5 cm (21") dark grey	62902-60	●	●
	Cover back Cr/Ne Komf std/high 55 cm (22") dark grey	62903-60	●	●
	Cover back Cr/Ne Comf sth/high 57,5 cm (23")	62904-60		●
	Cover back Cr/Ne Comf sth/high 60 cm (24")	62905-60		●
	Adapter seat upholstery loop-loop	28484	●	●

(widened backrest +5 cm)

Back support cover, back support 3A

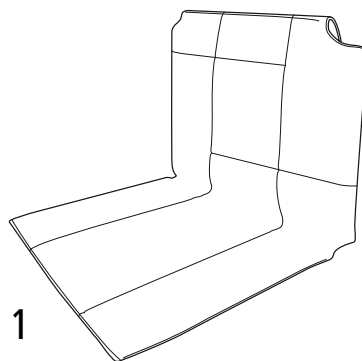

1

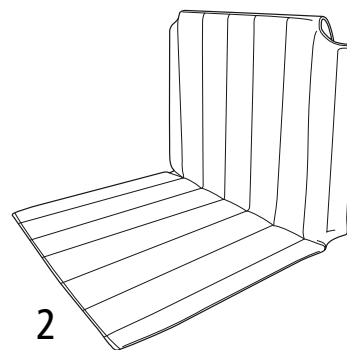

2



Cross 5	Cross 5 XL
●	
●	
●	
●	
●	
●	●
●	●
	●
	●
	●
	●

Pos.	Decription	Item. No.
1	Cover back Cross mod 3 35 cm (14) dark grey	62941-60
	Cover back Cross mod 3 37.5 cm (15) dark grey	62942-60
	Cover back Cross mod 3 40 cm (16) dark grey	62943-60
	Cover back Cross mod 3 42.5 cm (17) dark grey	62944-60
	Cover back Cross mod 3 45 cm (18) dark grey	62945-60
	Cover back Cross mod 3 47.5 cm (19) dark grey	62946-60
	Cover back Cross mod 3 50 cm (20) dark grey	62947-60
	Cover back Cross mod 3 52.5 cm (21) dark grey	62948-60
	Cover back Cross mod 3 55 cm (22) dark grey	62949-60
	Cover back Cross mod 3 57.5 cm (23) dark grey	62950-60
	Cover back Cross mod 3 60 cm (24) dark grey	62951-60

Cross 5	Cross 5 XL
●	
●	
●	
●	
●	
●	●
●	●
	●
	●
	●
	●

2	Cover back Cross Comfort mod 3 35 cm (14) dark grey	62956-60
	Cover back Cross Comfort mod 3 37.5 cm (15) dark grey	62957-60
	Cover back Cross Comfort mod 3 40 cm (16) dark grey	62958-60
	Cover back Cross Comfort mod 3 42.5 cm (17) dark grey	62959-60
	Cover back Cross Comfort mod 3 45 cm (18) dark grey	62960-60
	Cover back Cross Comfort mod 3 47.5 cm (19) dark grey	62961-60
	Cover back Cross Comfort mod 3 50 cm (20) dark grey	62962-60
	Cover back Cross Comfort mod 3 52.5 cm (21) dark grey	62963-60
	Cover back Cross Comfort mod 3 55 cm (22) dark grey	62964-60
	Cover back Cross Comfort mod 3 57.5 cm (23) dark grey	62965-60
	Cover back Cross Comfort mod 3 60 cm (24) dark grey	62966-60

Back support cover, back support 3A cont.

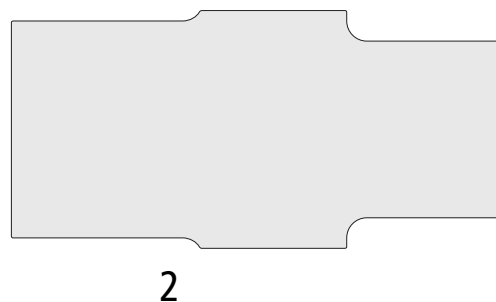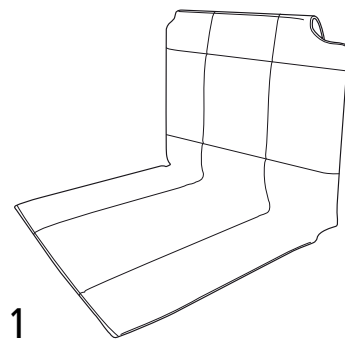


Cross 5
Cross 5 XL

Pos.	Decription	Item. No.	Cross 5	Cross 5 XL
1	Cover back Cross mod 3 Tidy 35 cm (14)	63087-63	●	
	Cover back Cross mod 3 Tidy 37,5 cm (15)	63088-63	●	
	Cover back Cross mod 3 Tidy 40 cm (16)	63089-63	●	
	Cover back Cross mod 3 Tidy 42,5 cm (17)	63090-63	●	
	Cover back Cross mod 3 Tidy 45 cm (18)	63091-63	●	
	Cover back Cross mod 3 Tidy 47,5 cm (19)	63092-63	●	●
	Cover back Cross mod 3 Tidy 50 cm (20)	63093-63	●	●
	Cover back Cross mod 3 Tidy 52,5 cm (21)	63094-63		●
	Cover back Cross mod 3 Tidy 55 cm (22)	63095-63		●
	Cover back Cross mod 3 Tidy 57,5 cm (23)	63096-63		●
	Cover back Cross mod 3 Tidy 60 cm (24)	63097-63		●


Cross 5
Cross 5 XL

2	Foam cover upholstery Tidy mod 3 35 cm (14)	82877	●	
	Foam cover upholstery Tidy mod 3 37.5 cm (15)	82878	●	
	Foam cover upholstery Tidy mod 3 40 cm (16)	82879	●	
	Foam cover upholstery Tidy mod 3 42.5 cm (17)	82880	●	
	Foam cover upholstery Tidy mod 3 45 cm (18)	82881	●	
	Foam cover upholstery Tidy mod 3 47.5 cm (19)	82882	●	●
	Foam cover upholstery Tidy mod 3 50 cm (20)	82883	●	●
	Foam cover upholstery Tidy mod 3 52.5 cm (21)	82884		●
	Foam cover upholstery Tidy mod 3 55 cm (22)	82885		●
	Foam cover upholstery Tidy mod 3 57.5 cm (23)	82886		●
	Foam cover upholstery Tidy mod 3 60 cm (24)	82887		●

Back support cover 3A cont.

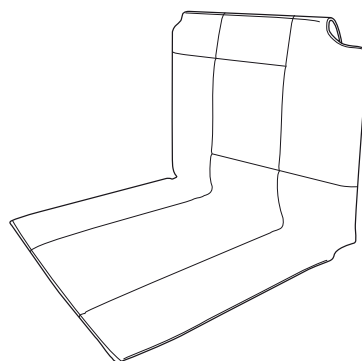

1

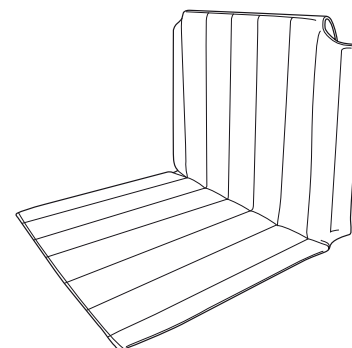

2


	Cross 5	Cross 5 XL
	●	
	●	
	●	
	●	
	●	
	●	●
	●	●
		●
		●
		●
		●

Pos.	Description	Item. No.
1	Cover back Cross 3A high 35 cm dark grey	63020-60
	Cover back Cross 3A high 37.5 cm dark grey	63021-60
	Cover back Cross 3A high 40 cm dark grey	63022-60
	Cover back Cross 3A high 42,5 cm dark grey	63023-60
	Cover back Cross 3A high 45 cm dark grey	63024-60
	Cover back Cross 3A high 47,5 cm dark grey	63025-60
	Cover back Cross 3A high 50 cm dark grey	63026-60
	Cover back Cross 3A high 52,5 cm dark grey	63027-60
	Cover back Cross 3A high 55 cm dark grey	63028-60
	Cover back Cross 3A high 57,5 cm dark grey	63029-60
	Cover back Cross 3A high 60 cm dark grey	63030-60

	Cross 5	Cross 5 XL
	●	
	●	
	●	
	●	
	●	
	●	●
	●	●
		●
		●
		●
		●

2	Cover back Cross 3A comfort high 35 cm dark grey	63033-60
	Cover back Cross 3A comfort high 37,5 cm dark grey	63034-60
	Cover back Cross 3A comfort high 40 cm dark grey	63035-60
	Cover back Cross 3A comfort high 42,5 cm dark grey	63036-60
	Cover back Cross 3A comfort high 45 cm dark grey	63037-60
	Cover back Cross 3A comfort high 47,5 cm dark grey	63038-60
	Cover back Cross 3A comfort high 50 cm dark grey	63039-60
	Cover back Cross 3A comfort high 52,5 cm dark grey	63040-60
	Cover back Cross 3A comfort high 55 cm dark grey	63041-60
	Cover back Cross 3A comfort high 57,5 cm dark grey	63042-60
	Cover back Cross 3A comfort high 60 cm dark grey	63043-60


Back support cover, back support 2B

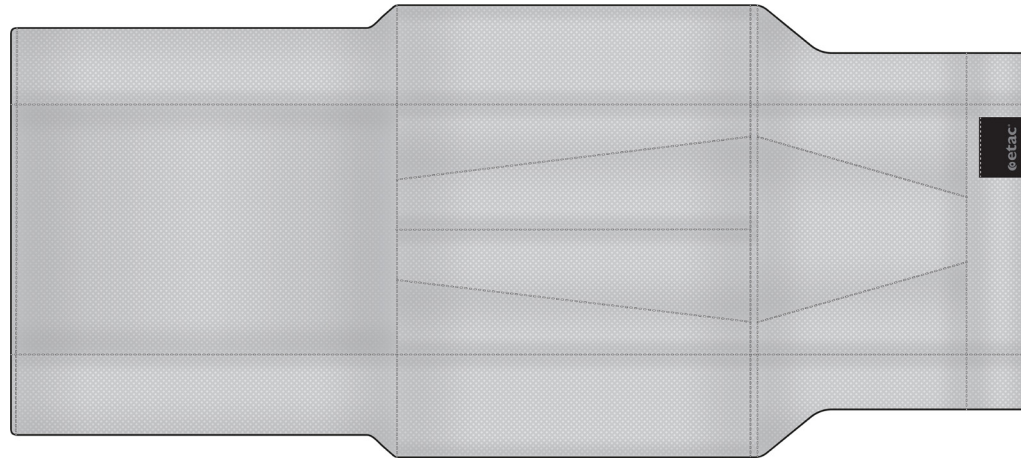

Pos.	Description	Item. No.		
			Cross 5	Cross 5 XL
1	Cover back 3A/B2B 35 cm Black spacer	63204-68	●	
	Cover back 3A/B2B 37.5 cm Black spacer	63205-68	●	
	Cover back 3A/B2B 40 cm Black spacer	63206-68	●	
	Cover back 3A/B2B 42.5 cm Black spacer	63207-68	●	
	Cover back 3A/B2B 45 cm Black spacer	63208-68	●	
	Cover back 3A/B2B 47.5 cm Black spacer	63209-68	●	
	Cover back 3A/B2B 50 cm Black spacer	63210-68	●	
	Cover back 3A/B2B 52.5 cm Black spacer	63211-68	●	
	Cover back 3A/B2B 55 cm Black spacer	63212-68	●	
	Cover back 3A/B2B 57.5 cm Black spacer	63213-68	●	
	Cover back 3A/B2B 60 cm Black spacer	63214-68	●	


Arm support detachable pad

Pos.	Description	Item. no.	Cross 5	Cross 5 XL
1-6	Arm support detachable pad short right	28780-1	●	●
	Arm support detachable pad short left	28780-2	●	●
	Arm support detachable pad long right	28781-1	●	●
	Arm support detachable pad long left	28781-2	●	●
	Arm support detachable pad short right with finger screw	28782-1	●	●
	Arm support detachable pad short left with finger screw	28782-2	●	●
	Arm support detachable pad long right with finger screw	28783-1	●	●
	Arm support detachable pad long left with finger screw	28783-2	●	●
1	Arm support detachable pad short without pad	28684	●	●
	Arm support detachable pad long right without pad	28685-1	●	●
	Arm support detachable pad long left without pad	28685-2	●	●
2	Stop bar for arm support	80661	●	●
3	Screw B 8x19 RXS/A2 Ph st 4.2	11064	●	●
4	Arm support pad M100 short right	82294-1	●	●
	Arm support pad M100 short left	82294-2	●	●
	Arm support pad M100 long right	82295-1	●	●
	Arm support pad M100 long left	82295-2	●	●
5	Sideguard right	50316	●	●
	Sideguard left	50317	●	●
6	Screw M5x14 MF6S 10.9 black Esloc	11237	●	●
7	Finger Screw	82402	●	●

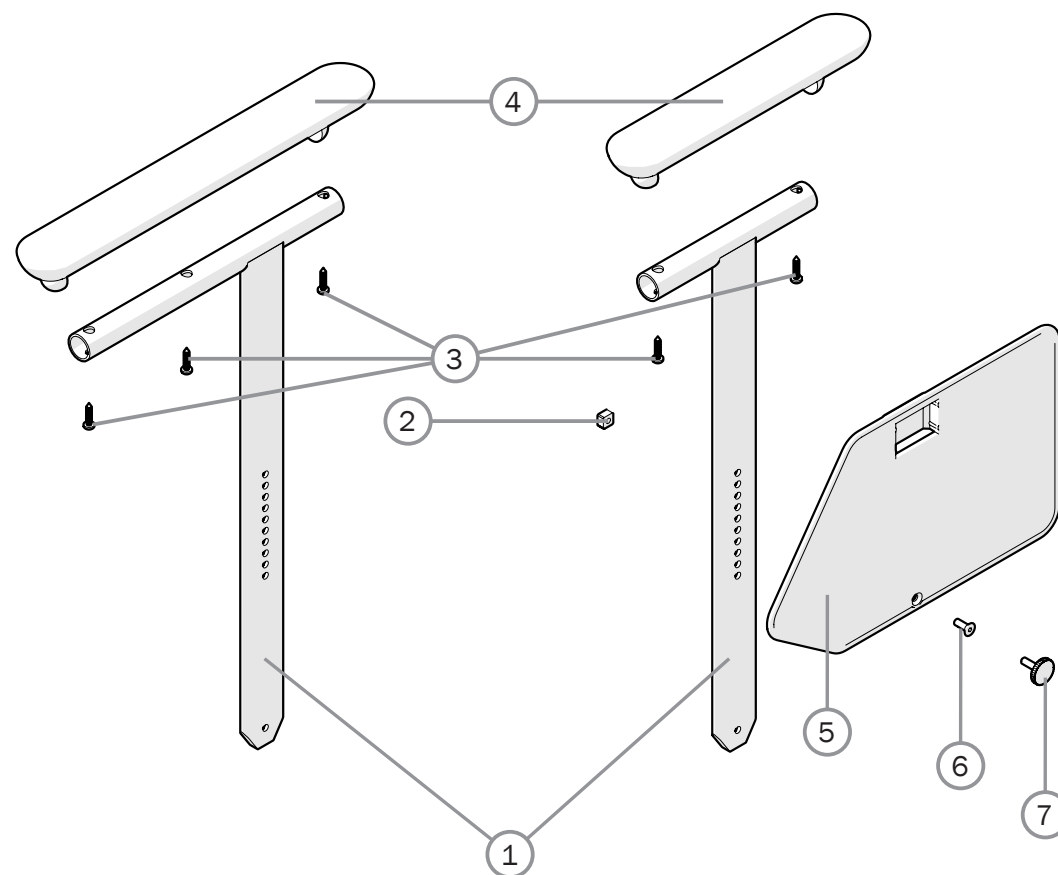


Arm support with solid arm support plate

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Cover arm support short dark grey	62282-60	●	●
	Cover arm support long dark grey	62283-60	●	●
	Cover arm support Gel short compl dark grey	26240-60	●	●
	Cover arm support Gel long compl dark grey	26241-60	●	●
	Cover arm support short Dartex	62282-63	●	●
	Cover arm support long Dartex	62283-63	●	●
	Cover arm support Gel short compl Dartex	26240-63	●	●
	Cover arm support Gel long compl Dartex	26241-63	●	●
2-5	Arm support Cross short right L 25 cm	25172-1	●	●
	Arm support Cross short left L 25 cm	25172-2	●	●
	Arm support Cross long right L 38 cm	25173-1	●	●
	Arm support Cross long left L 38 cm	25173-2	●	●
3	Screw M 5x 14 MF6S 10.9 black	11237	●	●
4	Sideguard Cross/M100 nylon right	50316	●	●
	Sideguard Cross/M100 nylon left	50317	●	●
5	Stop bar for arm support	80661	●	●

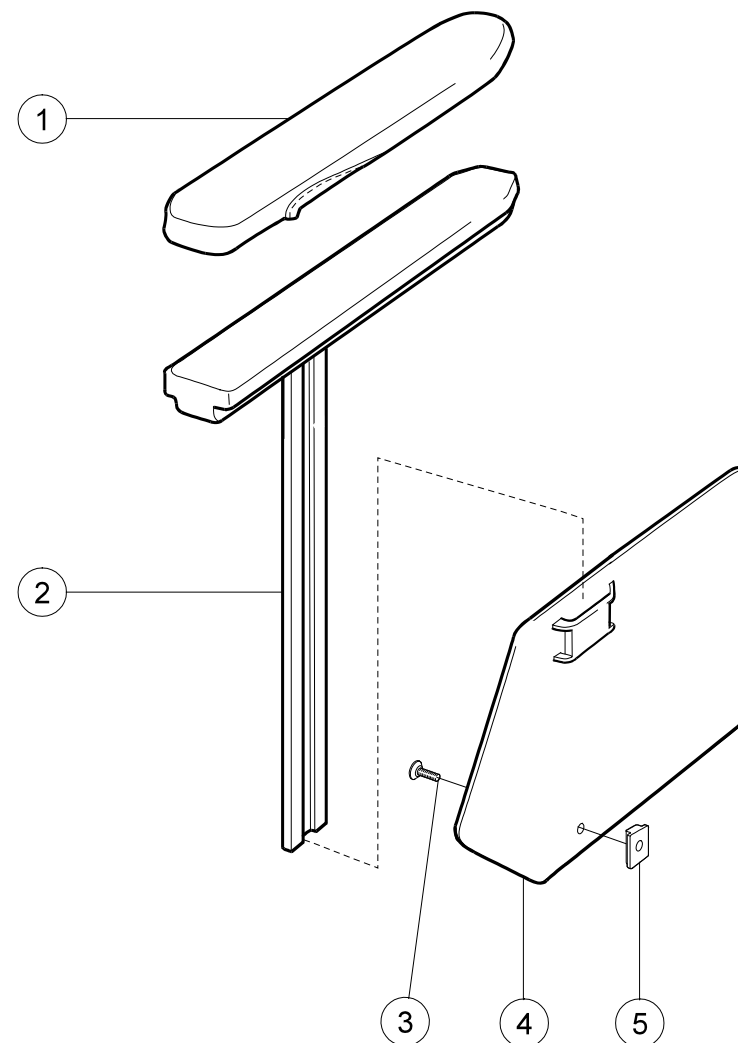


Sideguard

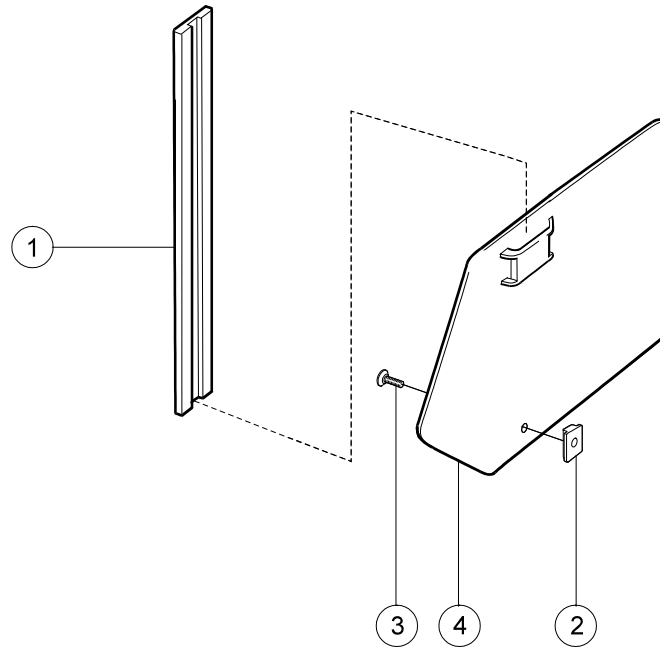


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Sideguard bracket Cross 6005 T6 SS4107-06	80631	●	●
2	Stop bar for arm support	80661	●	●
3	Screw M 5x 14 MF6S 10.9 black	11237	●	●
4	Sideguard Cross/M100 nylon right	50316	●	●
	Sideguard Cross/M100 nylon left	50317	●	●
1-4	Sideguard Cross right black	25189-1	●	●
	Sideguard Cross left black	25189-2	●	●

Mudguard

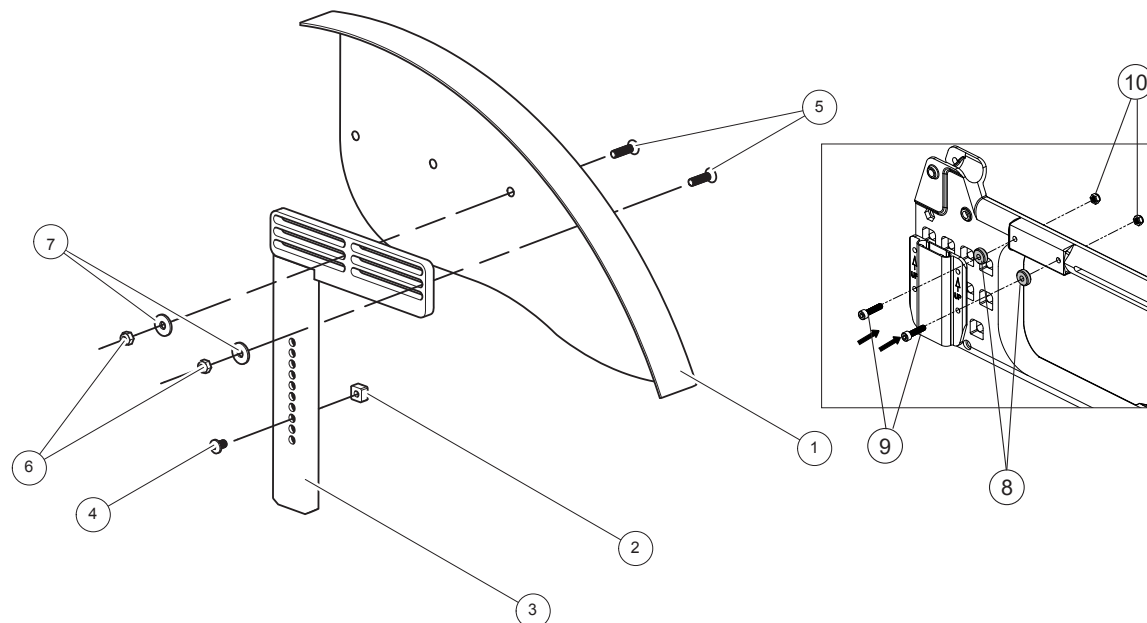

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1-7	Mudguard black Cross/Elite ri with adj attachment	27397-1	●	●
	Mudguard black Cross/Elite le with adj attachment	27397-2	●	●
1	Mudguard black right	27414-1	●	●
	Mudguard black left	27414-2	●	●
2	Stop bar for arm support	80661	●	●
3	Attachment mudguard	27396-01	●	●
4	Screw M 5x 8 K6S black	11304	●	●
5	Screw M 5x 20 MF6S black	11571	●	●
6	Locking nut M 5	12036	●	●
7	Washer BRB 5.3x15x1.2 black	13004	●	●
8	Spacer arm support	20231	●	●
9	Screw M 6x 25 MC6S 12.9 black lock	11317	●	●
10	Locking nut M 6 black (See part Side frame)	12035	●	●

Thigh support

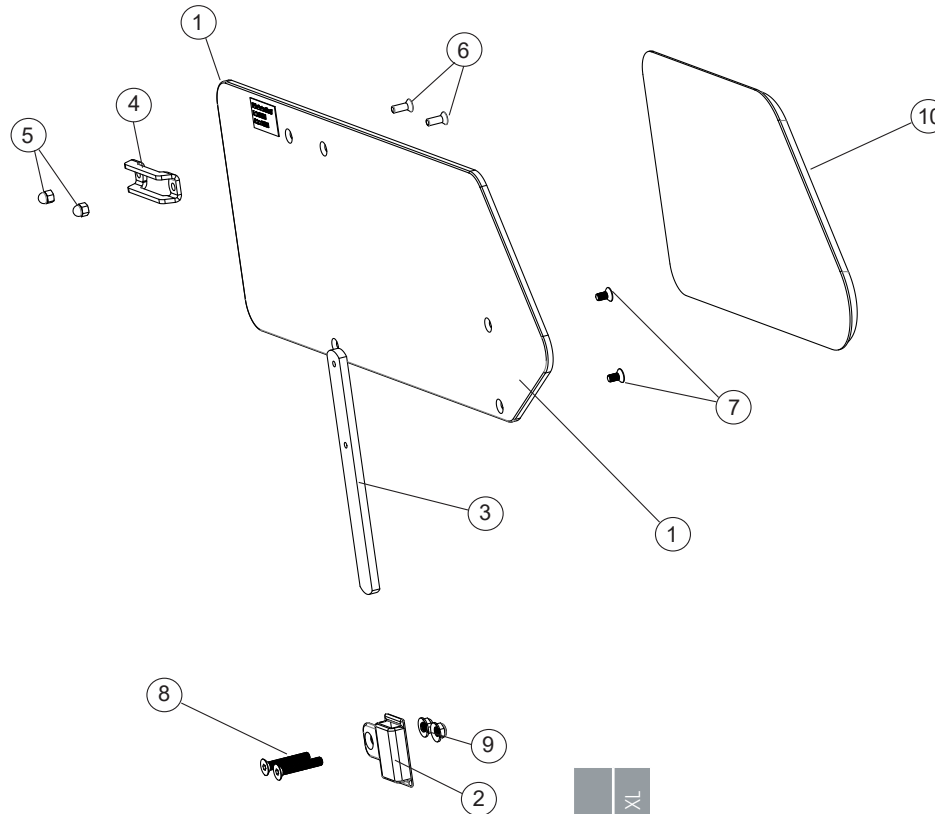


Cross 5
Cross 5 XL

Pos.	Description	Item. No.
1-10	Thigh support short right compl	27391-1
	Thigh support short left compl	27391-2
	Thigh support long right compl	27392-1
	Thigh support long left compl	27392-2
1	Thigh support short	82523
	Thigh support long	82524
2	Attachment thigh support	82489
3	Attachment plate thigh support	82490

4	Attachment plate	82408	●	●
	Locking nut M 5 domed zink plated	12168		
6	Screw M 5x 14 MF6S 10.9 black	11237	●	●
7	Screw M 5x 10 MF6S 8.8 zink plated	11689	●	●
8	Screw MF6S M6 x 30 fzb Esloc	11639	●	●
9	Locking nut M 6	12169	●	●
10	Soft pad thigh support right compl	82488-1	●	●
	Soft pad thigh support left compl	82488-2	●	●

Cross 5
Cross 5 XL

Lateral support, back support 3A

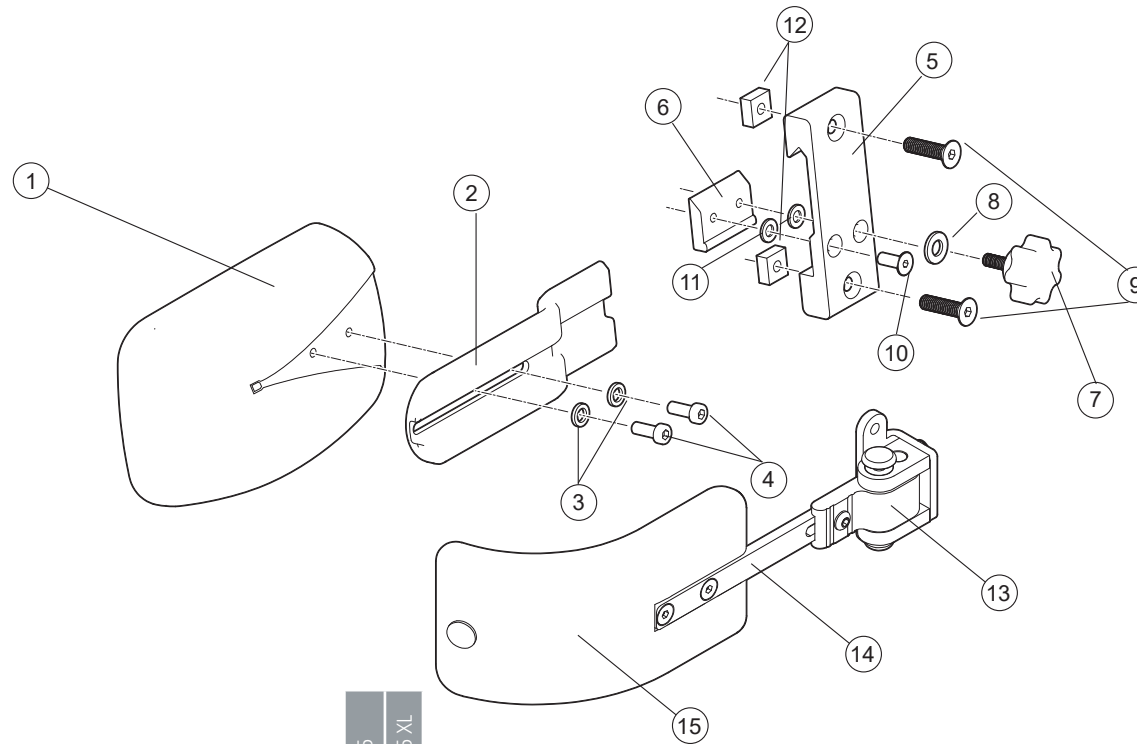
[illegible]

11	Washer 5.3x 10x 0.5m DIN 137A zink plated	13026
12	Nut M 6 FK4M	12166
13	Attachment trunk support swing away R compl	27947-1
	Attachment trunk support swing away L compl	27947-2
14	Bar trunk support swing away L straight 12	27948
15	Cushion trunk support medium bent black H 6.5 L 17	27949
	Cushion trunk support large bent black H 9 cm L 19	27950

						Cross 5
						Cross 5 XL


Pos.	Description	Item. No.
1-12	Trunk support Cross back mod 3 incl attachment	27395
1	Trunk supp pad curved small Dartex Cross back mod 3	82491
2	Bracket trunk support	82507
3	Spring washer 6.1x9.9x1.6 DIN 7980	13014
4	Screw M 5x 12 K6S 10.9 zink plated	11208
5	Attachment arm support Elite/Sting without screws	80403
6	Locking plate arm support	80292
7	Knob M 5x15 leg support amputee Cross	80807
8	Washer BRB 5.3x15x1.2 black	13004
9	Screw M 6x 25 MF6S black esloc	11430
10	Screw M 5x 14 MF6S 10.9 black	11237

Lateral support, back support 3A cont.

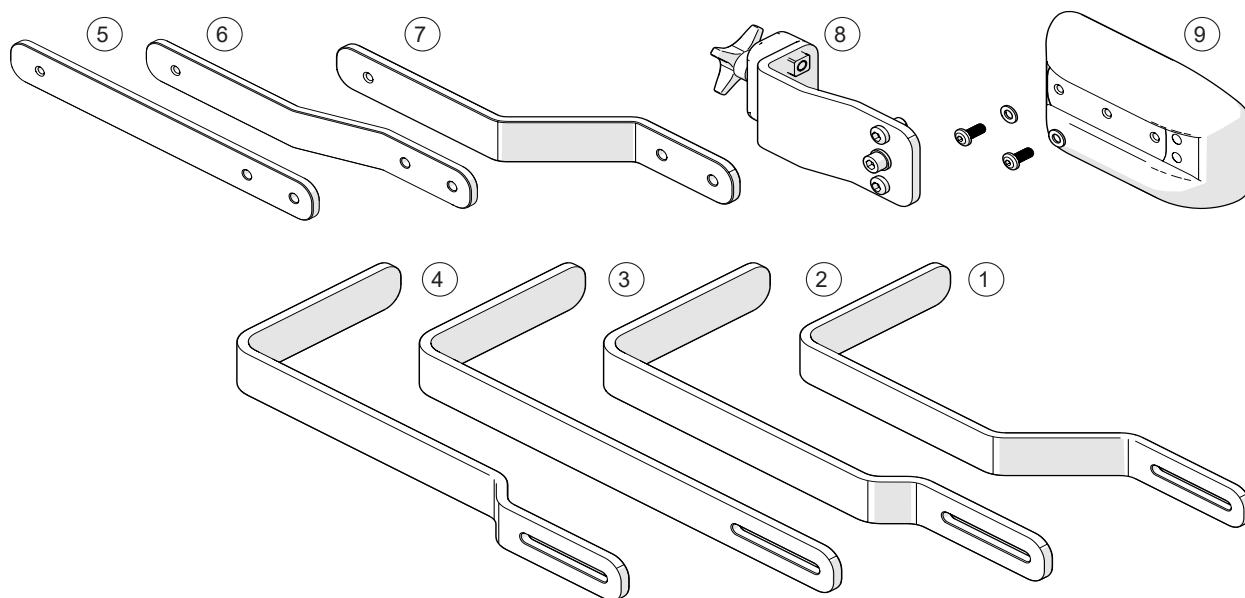


Pos.	Description	Item. No.		
			Cross 5	Cross 5 XL
1	Trunk support bar -45 mm	83002	●	●
2	Trunk support bar -15 mm	83017	●	●
3	Trunk support bar 0 mm	83018	●	●
4	Trunk support bar +15 mm	83019	●	●
5	Trunk support bar ssb -0 mm	83046	●	●
6	Trunk support bar ssb -15 mm	83045	●	●
7	Trunk support bar ssb -45 mm	83001	●	●
8	Trunk support attachment 3A compl	28146	●	●
9	Trunk support pad straight	83005	●	●

Tray

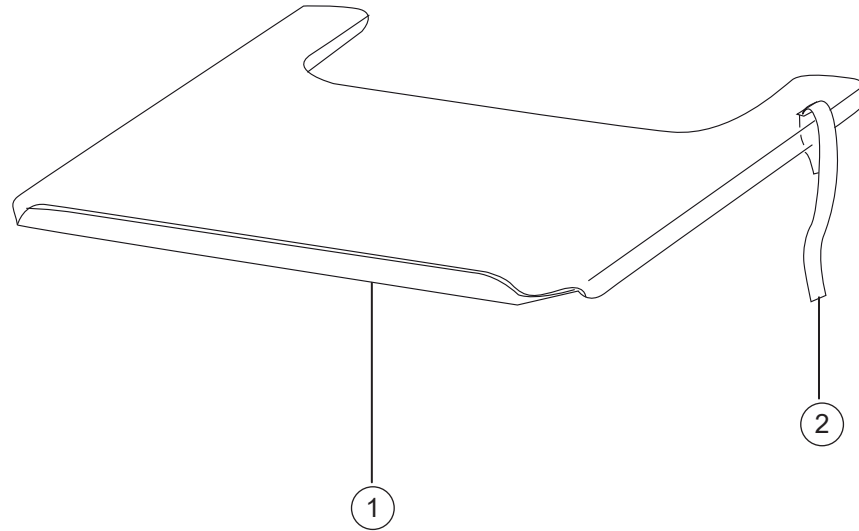


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
	Strap for tray 27.5-50 cm (11-20)	62103	●	●
	Tray 45 cm compl for wheelchair 35 cm (14)	24417	●	
	Tray 47.5 cm compl for wheelchair 37.5 cm (15)	25187	●	
	Tray 50 cm compl for wheelchair 40 cm (16)	24418	●	
	Tray 52.5 cm compl for wheelchair 42.5 cm (17)	25188	●	
	Tray 55 cm compl for wheelchair 45 cm (18)	24419	●	
	Tray 57.5 cm compl for wheelchair 47.5 cm (19)	25731	●	●
	Tray 60 cm compl for wheelchair 50 cm (20)	24601	●	●
	Tray 62.5 cm for 52.5 cm chair compl	26020		●
	Tray 65 cm for 55 cm chair compl	26021		●
	Tray 67.5 cm for 57.5 cm chair compl	26022		●
	Tray 70 cm for 60 cm chair compl	26907		●

Tray hemi

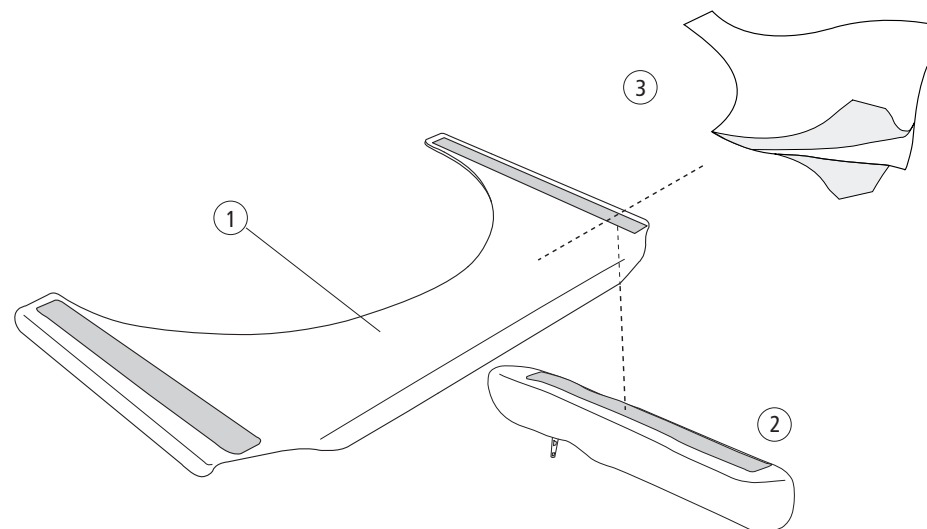


Pos.	Description	Item. No.	Cross 5	Cross 5 XL
2	Cover arm support table hemi short with Velcro	81631	●	●
	Cover arm support table hemi long with Velcro	81632	●	●
3	Anti glide table hemi 35-50 cm	81633	●	●
1-2	Tray hemi 45 cm for wheelchair 35 cm (14) with short arm support cover	26556	●	
	Tray hemi 47.5 cm for wheelchair 37.5 cm (15) with short arm support cover	26557	●	
	Tray hemi 50 cm for wheelchair 40 cm (16) with short arm support cover	26558	●	
	Tray hemi 52.5 cm for wheelchair 42.5 cm (17) with short arm support cover	26559	●	
	Tray hemi 55 cm for wheelchair 45 cm (18) with short arm support cover	26560	●	
	Tray hemi 57.5 cm for wheelchair 47.5 cm (19) with short arm support cover	26561	●	●
	Tray hemi 60 cm for wheelchair 50 cm (20) with short arm support cover	26562	●	●
	Tray hemi 45 cm for wheelchair 35 cm (14) with long arm support cover	26563	●	
	Tray hemi 47.5 cm for wheelchair 37.5 cm (15) with long arm support cover	26564	●	
	Tray hemi 50 cm for wheelchair 40 cm (16) with long arm support cover	26565	●	
	Tray hemi 52.5 cm for wheelchair 42.5 cm (17) with long arm support cover	26566	●	
	Tray hemi 55 cm for wheelchair 45 cm (18) with long arm support cover	26567	●	
	Tray hemi 57.5 cm for wheelchair 47.5 cm (19) with long arm support cover	26568	●	●
	Tray hemi 60 cm for wheelchair 50 cm (20) with long arm support cover	26569	●	●

Tray hemi/communication

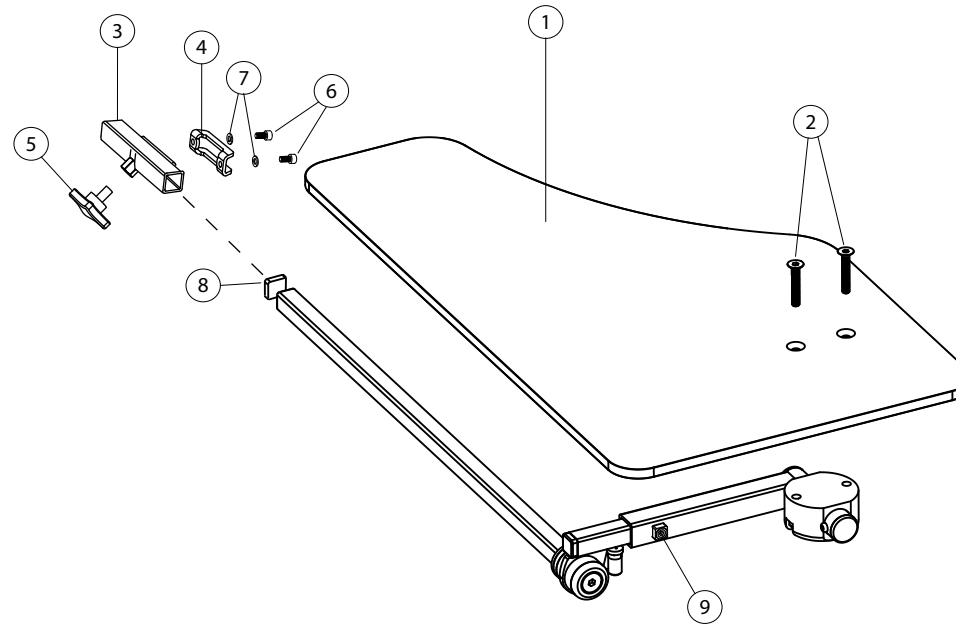


Pos.	Description	Art. nr.	Cross 5	Cross 5 XL
1-9	Tray Hemi/Communicatin half right	27352-1	●	●
	Tray Hemi/Communicatin half left	27352-2	●	●
1	Plate Tray hemi	82420	●	●
2	Screw M 6x 35 MF6S 8.8 black	11287	●	●
3	Hold arm support	27356	●	●
4	Attachment plate arm support	82408	●	●
5	Locking wheel M8x14mm L	400118	●	●
6	Screw M 5x 8 MC6S black	11414	●	●
7	Spring washer 5,3 x 10 x 0,5	13026	●	●
8	End stop IFR 1515-1,5s	50972	●	●
9	Stop screw M6 x 10	11614	●	●

Anti tip

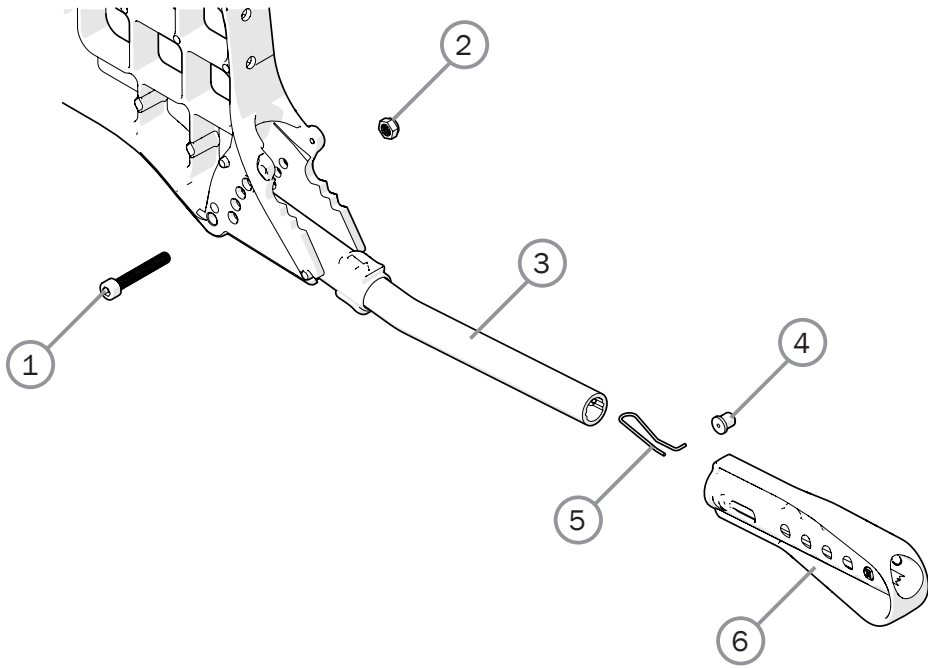

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1-6	Anti-tip multi Cross 5/6 Prio cc 59 cpl	28457	●	●
1	Screw M6x 40 MC6S 10.9 zink plated	11229	●	●
2	Locking nut M6	12017	●	●
3	Anti-tip tube		●	●
4	Pin		●	●
5	Spring		●	●
6	Anti-tip extension multi	51201	●	●

Anti tip mod 2 Cross 5

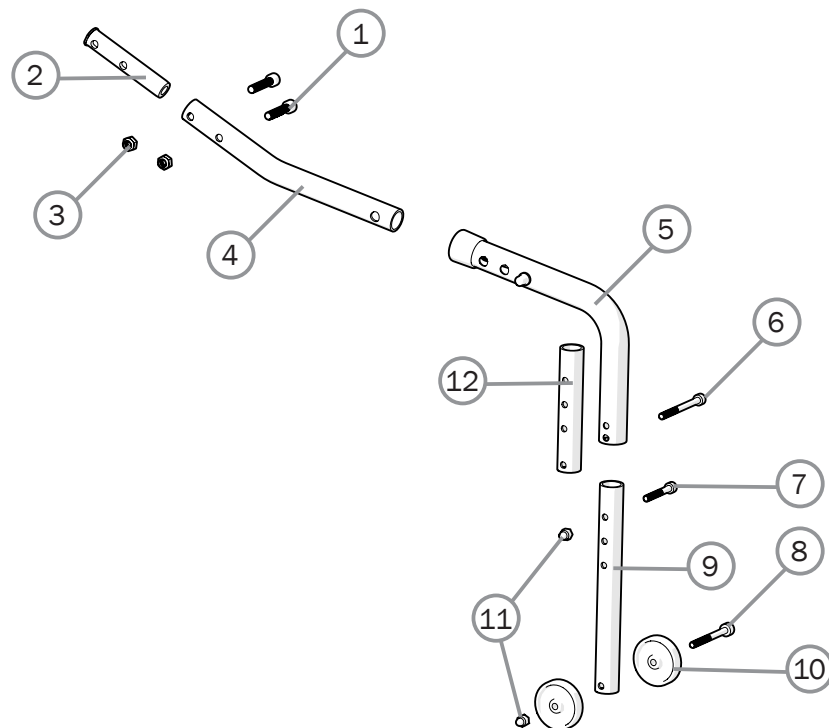


Cross 5						

7	Screw M5x30 MC6S	11346
8	Screw M5x50 MC6S 8.8	11687
9	Extension anti-tip	28467
10	Wheel 40/5,2/12 anti-tip	80727
11	Locking nut M5 domed	12168
12	Extension short	25376

	Cross 5	Cross 5 X1
	●	
	●	
	●	
	●	
	●	


Pos.	Description	Item. no.
1-12	Anti-tip mod 2 std	28470
1	Screw M6x30 MC6S	11019
2	Support sleeve anti-tip mod 2	20530
3	Locking nut M6 DIN 985-6	12017
4	Holder anti-tip mod 2	28468
5	Tube anti-tip	25374
6	Screw M5x55 MC6S 8.8	11686

Anti tip mod 2 low Cross 5

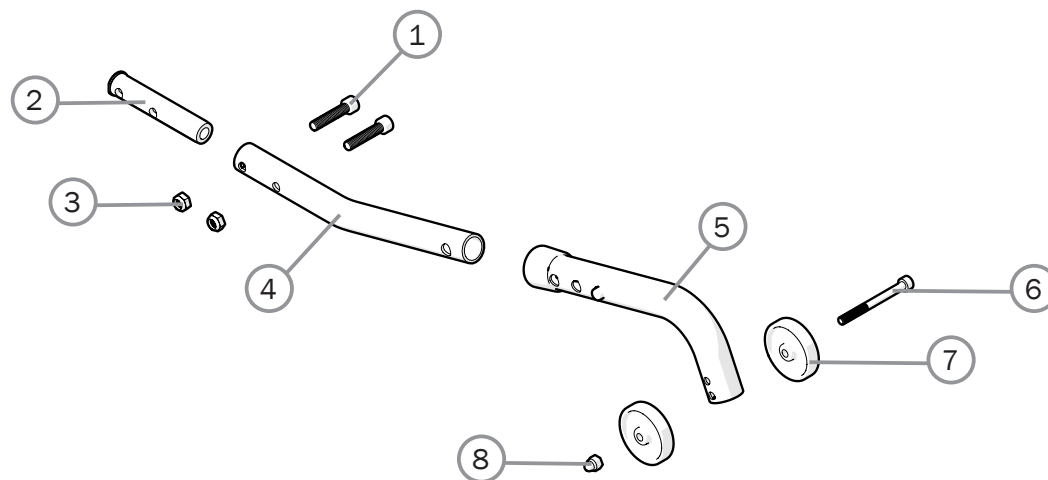


Pos.	Description	Item. no.
1-8	Anti tip mod 2 low	28471
1	Screw M6x30 MC6S	11019
2	Support sleeve anti-tip mod 2	20530
3	Locking nut M6 DIN 985-6	12017
4	Holder anti-tip mod 2	28468
5	Tube anti-tip long	28466
6	Screw M5x55 MC6S 8.8	11686
7	Wheel 40/5,2/12 anti-tip	80727
8	Locking nut M5 domed	12168

Spoke guard

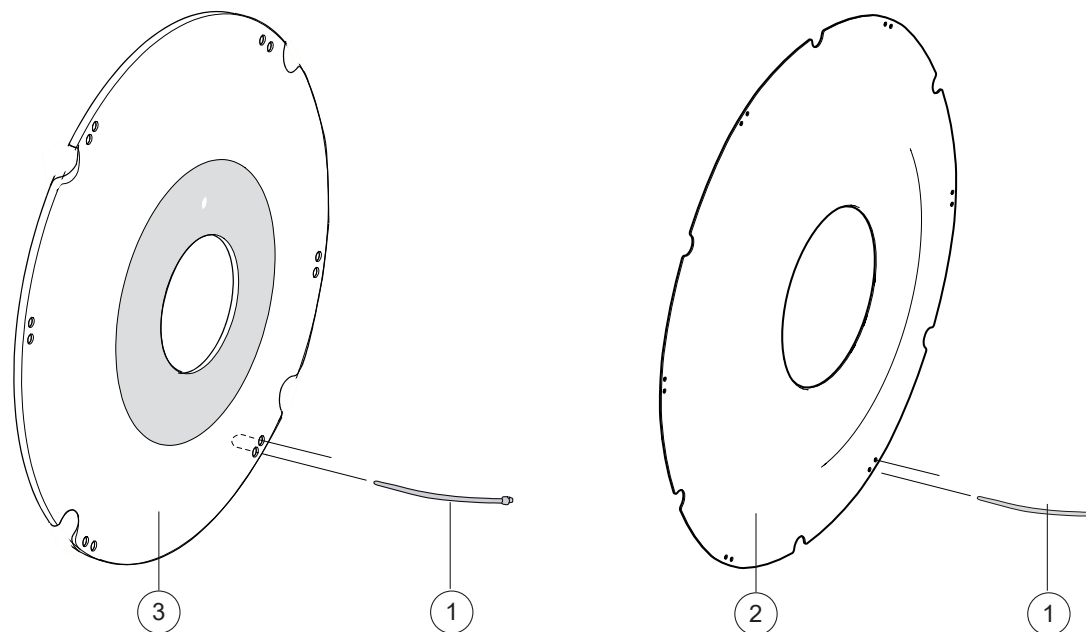

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1	Attachment kit spokeguard	26276	●	●
1-2	Spoke guard 20" cpl C6/Maxx	28934	●	●
	Spoke guard 22" cpl C6/Maxx	28935	●	●
	Spoke guard 24" cpl C6/Maxx	28936	●	●
1, 3	Spoke guard 20" with logo Etac grey	25502-28	●	●
	Spoke guard 20" with logo Etac yellow	25502-29	●	●
	Spoke guard 20" AB with logo Etac grey	28816-28	●	●
	Spoke guard 22" with logo Etac grey	25503-28	●	●
	Spoke guard 22" with logo Etac yellow	25503-29	●	●
	Spoke guard 22" AB with logo Etac grey	28817-28	●	●
	Spoke guard 24" with logo Etac grey	25504-28	●	●
	Spoke guard 24" with logo Etac yellow	25504-29	●	●
	Spoke guard 24" AB with logo Etac grey	28818-28	●	●


Tetra quickrelease adapter

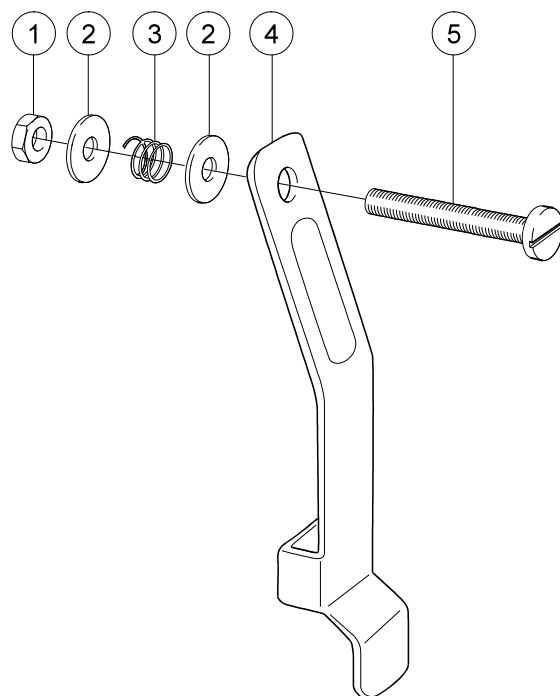

Pos.	Description	Item. No.
1	Locking nut M 5 DIN 985-6 zink plated	12003
2	Washer 6.4x 14x 1.55 alu	13010
3	Spring 1.5 inv 6 L15 zink plated	80019
4	Tongue tetra knob black	24040-01
5	Screw M 5x 35 zink plated	11001
1-5	Tetra quick release adapt black	24051-01


Cross 5		●	●	●	●	●
Cross 5 X1		●	●	●	●	●

Cane holder

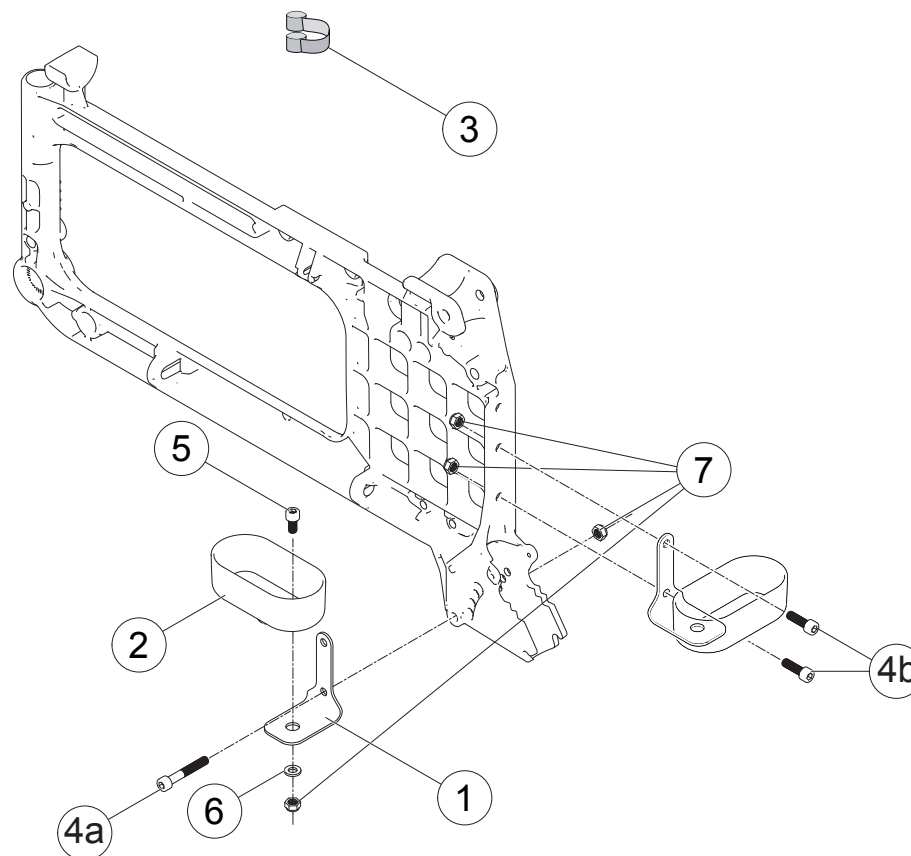

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1-7	Cane holder universal compl Cross/Next/Twin/Transit	27327	●	●
1	Bracket cane holder	82390	●	●
2	Cane holder M100	50866	●	●
3	Elastic ribbon Cross black	80621	●	●
4a	Screw M 6x 40 MC6S zink plated	11229	●	●
4b	Screw M 6x 14 MC6S black	11420	●	●
5	Screw M6 x 12 MC6S 8.8 zink plated	11577	●	●
6	Washer 6.4x14x1.5 DIN 134 black	13058	●	●
7	Locking nut M 6 zink plated	12017	●	●

Head support

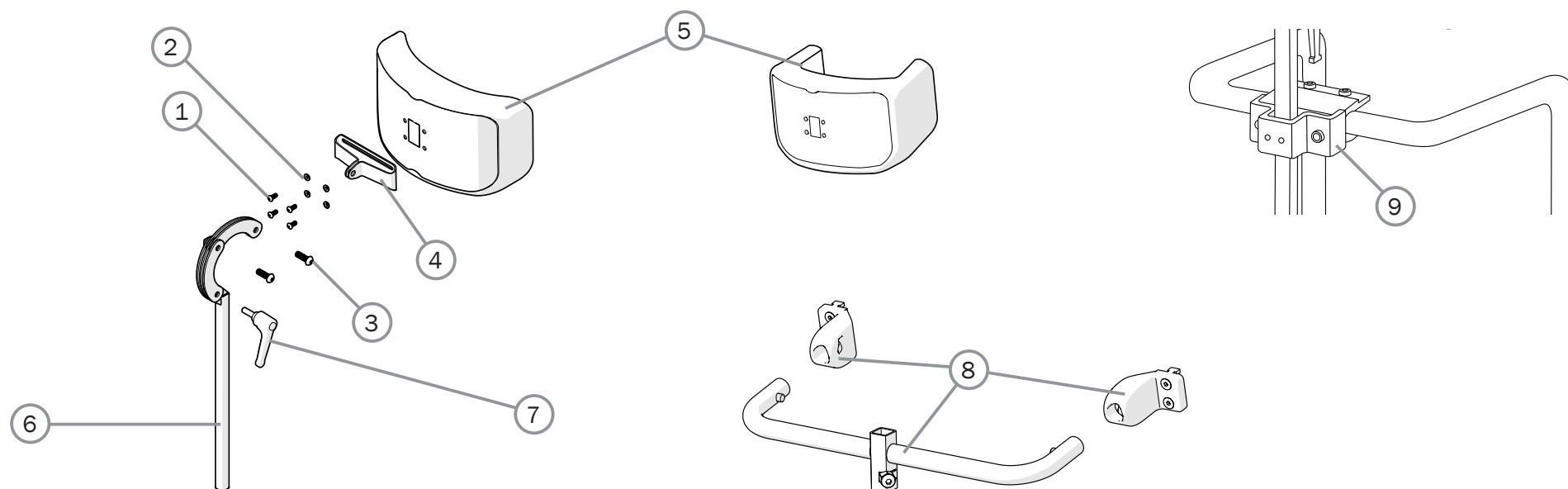

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
1-7	Head support mod 4 hygiene	28534-65	●	●
	Head support mod 4 dark grey	28534-60	●	●
	Head support with side support grey hygiene	28536-65	●	●
1	Screw M 5x 12 K6S 10.9 fzb	11208	●	●
2	Joint bushing	50285	●	●
3	Screw M8x25	11706	●	●
4	Attachment head support	83288	●	●
5	Cover for head support plush	63177-60	●	●
	Cover for head support hygiene	63177-65	●	●
	Cover for head support with side support	63185-65	●	●
6	Tube hold neckrest L 350	28533	●	●
7	Handle head support	80684	●	●

Pos.	Description	Item. No.	Cross 5	Cross 5 XL
8	Head support attachment 3A 35 cm	28125	●	
	Head support attachment 3A 37,5 cm	28126	●	
	Head support attachment 3A 40 cm	28127	●	
	Head support attachment 3A 42,5 cm	28128	●	
	Head support attachment 3A 45 cm	28129	●	
	Head support attachment 3A 47,5 cm	28130	●	
	Head support attachment 3A 50 cm	28131	●	
	Head support attachment 3A 52,5 cm	28132	●	
	Head support attachment 3A 55 cm	28133	●	●
	Head support attachment 3A 57,5 cm	28134		●
	Head support attachment 3A 60 cm	28135		●
9	Head support bracket	25795	●	●

Drip stand

Pos.	Description	Item. no.		
			Cross 5	Cross 5 XL
1-7	IV stand compl	28323	●	●
1	IV stand	28338	●	●
2-7	Attachment IV stand	28339	●	●
2	Knob		●	●
3	Attachment		●	●
4	Plate		●	●
5	Nut		●	●
6	Screw		●	●
7	Washer		●	●

