


Etac Prio 3A

Spareparts list english



1a17831.16 2026-02-12

Applies to Etac Prio with serial no beginning with 38D or
serial no from S/N 1003310170001

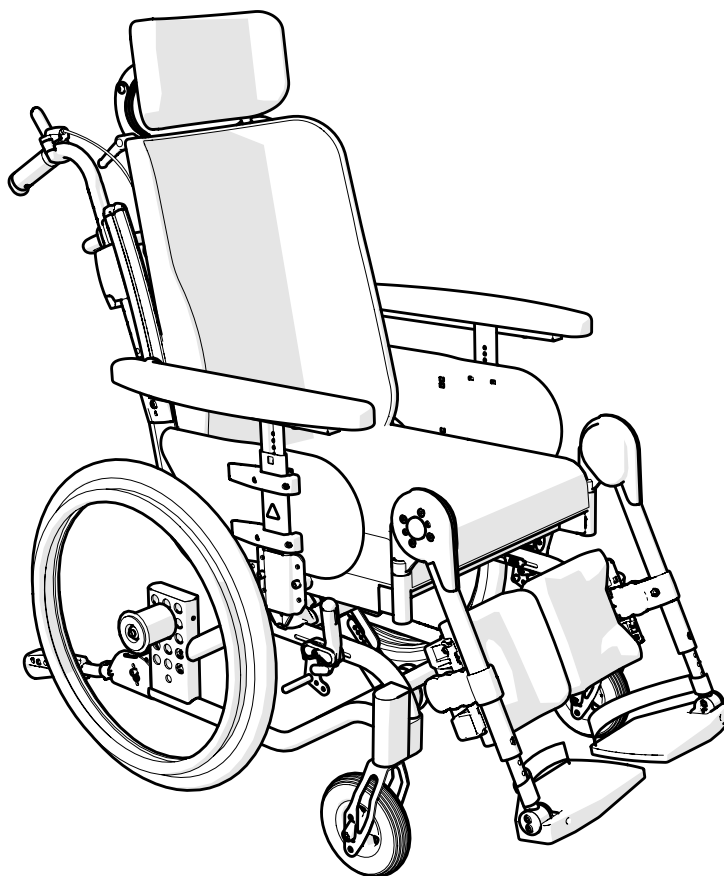

Index


Index	Page
Frame	3
Seat frame	4
Gas pistons, comfort lock	5
Electric recline function	6-7
Back support frame	8-9
Push handles	10
Bow handle	11
Bow handle angle adjustable	12
Leg support	13-14
Amputee leg support	15
Foot support	16-17
Foot support one piece	18
Footbox	19
User brake	20-21
Rear wheel	22-25
Attendant brake	26-27
Tyre and innertube	28-29
Handrim	30-31
Castor	32-33
Widening kit	34
Solid seat	35
Arm support	36-37
Arm support Hemi	38
Trunk support	39-42
Leg Abduction	43
Tray	44
Attachment harness	45
IV stand	46
Basket	47
Positioning belt	48
Anti tip	49
Spoke guard	50
Head support	51
Seat cushion	52-53
Seat cushion - Cover	54
Back support cushion	55
Back support cover	56
Extension Back support upholstery	57
Back support upholstery	58
Back angle control	59
User operated self-tilt	60

Explanation for list of points

● Applies to model

Prio equipped with 16" rear wheels	Prio
Prio equipped with attendant brakes	Prio AB
Prio equipped with handrim rear wheels	Prio 16"

Frame

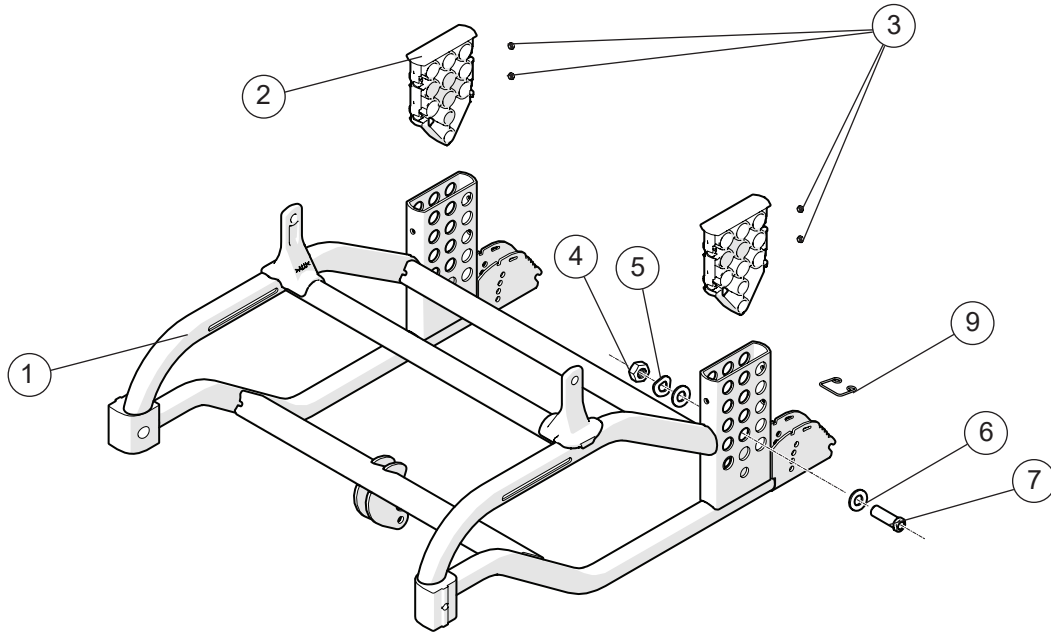


Pos.	Description	Item no.
1	Frame Prio	
2	Cover rear wheel attachm Prio	
3	Square nut high M6	
4	Nut M16x 1 NV 24 B8	12052
5	Locking washer AZ 5/8	13039
6	BRB 17x30x3 Washer	13117
7	Axle sleeve D12	20537
	Axle sleeve for axle D12,7x110	83893
8	Touch up paint	80517-01
9	Spring anti-tip	83142

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	
●	●	
●	●	
●	●	
●	●	●
●	●	●


Seat frame

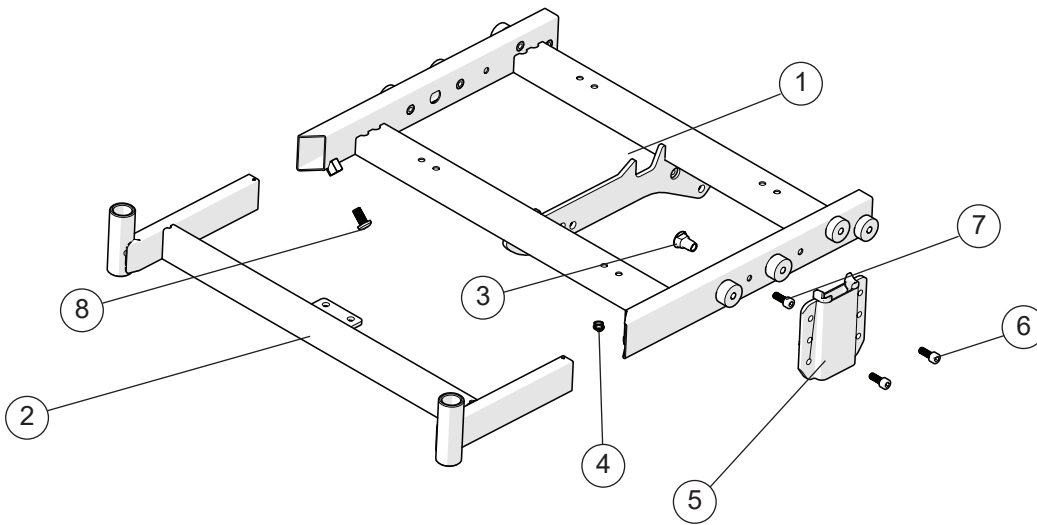

Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Pos.	Description	Item no.
1	Seat frame rear Prio	
2	Seat frame front Prio	
3	Joint nut seat frame Prio	12117
4	Wire attachment	50893
5	Attachment armrest Prio	28514
6	Screw M 6x 16 MC6S 12.9	11253
7	Screw M 6x 14 MC6LS SW5 Zn8/P4	11700
8	Screw M8x16 K6S A2	11014

Gas pistons, comfort lock

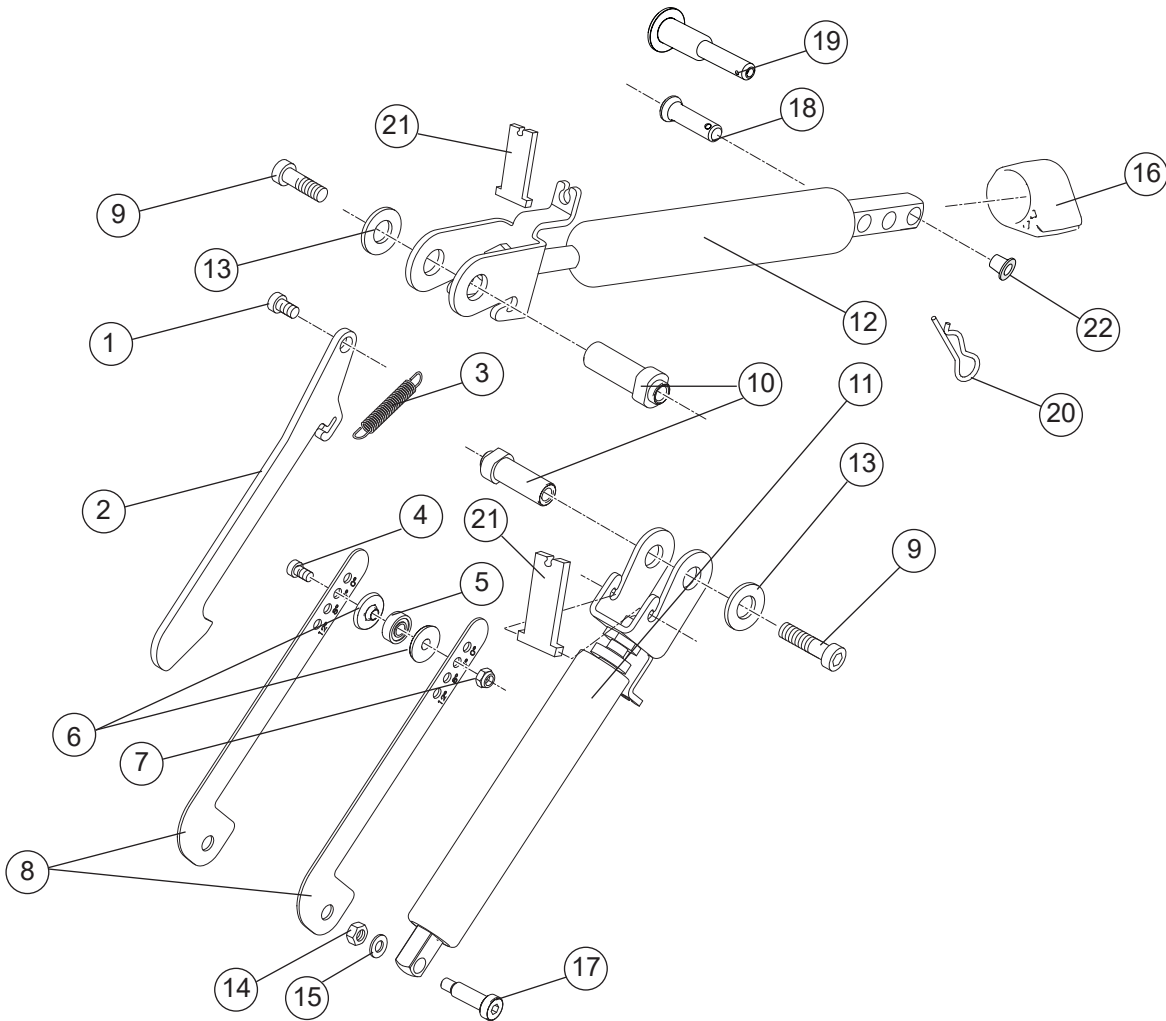

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Screw M 6x 16 MC6S 12.9	11253	●	●	●
2	Comfort lock	82667	●	●	●
3	Spring	82690	●	●	●
4	Screw M5x16	11424	●	●	●
5	Bearings Dy15 Di6 t5 (NOMO 696	82660	●	●	●
6	Bearing bushing comfort lock K	20509	●	●	●
7	Locking nut M 5	12003	●	●	●
8	Adjustment plate comfort lock	82691	●	●	●
9	Screw M8x20 K6S 10.9 zink plated	11704	●	●	●
10	Joint sleeve key socket gas pi	20511	●	●	●
11	Gas spring cpl	24581	●	●	●
12	Gas spring 3A cpl	24582	●	●	●
13	Washer 25/12/1.5 black pa	51169	●	●	●
14	Locking nut M 6	12017	●	●	●
15	Washer BRB 6x3x10x1.5 fzb	13080	●	●	●
16	Pull mechanism active backrest plastic Prio	51042	●	●	●
17	Screw 12.9 8xM6x21.5 Prio	20513	●	●	●
18	Bolt D8 L23.3	83003	●	●	●
19	Locking pin QR	83159	●	●	●
20	Locking pin R t2	83037	●	●	●
21	Lock clapper	82697	●	●	●
22	Cover plug gas spring Prio 3A	51238	●	●	●

Actuator, electric recline function

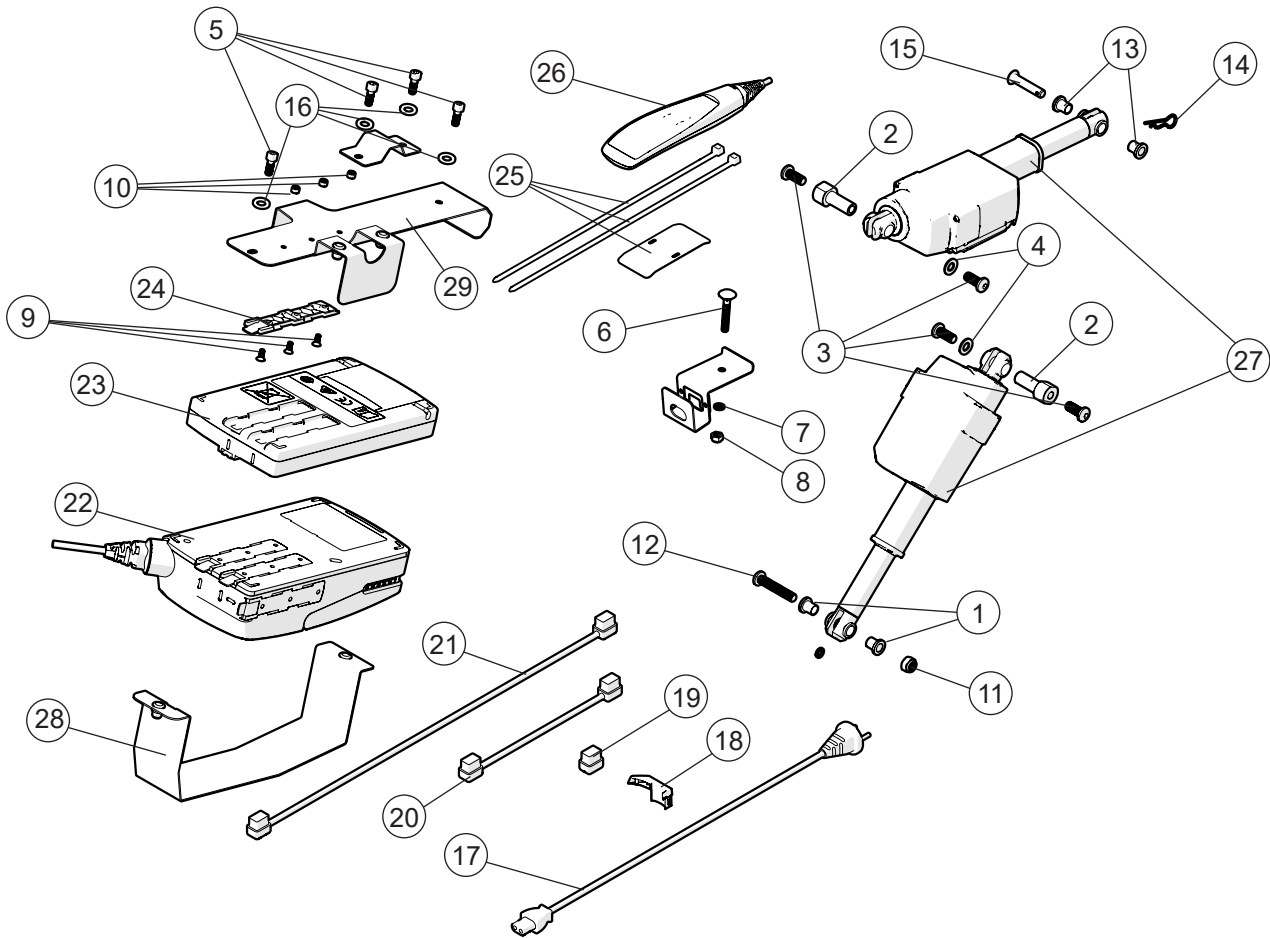

Pos.	Description	Item. no.
1	Spacer electric tilt Prio	
2	Spacer M8 electric tilt Prio	
3	Screw M 8x 20 fzb	
4	Cir clip RSM 5 stainless	
5	Screw M 6x 16 MC6S 12.9	
6	Screw M 6x 40 MVBF fzb	
7	Washer BRB 6x3x10x1.5 fzb	
8	Locking nut M 6 fzb	
9	Screw M 4x 10 MF6S Fzb	
10	Locking nut M 4 fzb	
11	Locking nut M 8	
12	Screw M 8x 45 MK6S Fzb	
13	Spacer POM electric tilt Prio	
14	Locking pin R t2	
15	Bolt electric tilt Prio	
16	Washer BRB 6.5x12x1.5	
1-16	Screw kit	28370
17	Power plug power tilt Prio EU	83173
	Plug Great Britain	1140106

Prio


Prio AB

Prio 16"

Actuator, electric recline function cont.


Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
18	Cablelock actuator electric tilt P	83170	●	●	●
19	Plug controlbox electric tilt Prio	51155	●	●	●
20	Battery cabel electric tilt Prio	83167	●	●	●
21	Cabel actuator electric tilt Prio	83164	●	●	●
22	Control box electric tilt Prio	83166	●	●	●
23	Battery electric tilt Prio	83837	●	●	●
24	Bracket electric tilt Prio	83171	●	●	●
25	Bracket handcontrol Prio	28577	●	●	●
26	Handcontroll electric tilt Prio	83169	●	●	●
27	Actuator electric tilt Prio	83165	●	●	●
28	Attachment lower, electric tilt, mounted		●	●	●
29	Attachment upper, electric tilt, mounted		●	●	●
1-29	Electric tilt Prio compl kit	28898	●	●	●
19, 28, 29	Attachment electric tilt BA22 cpl	24841	●	●	●

Extension back 3A

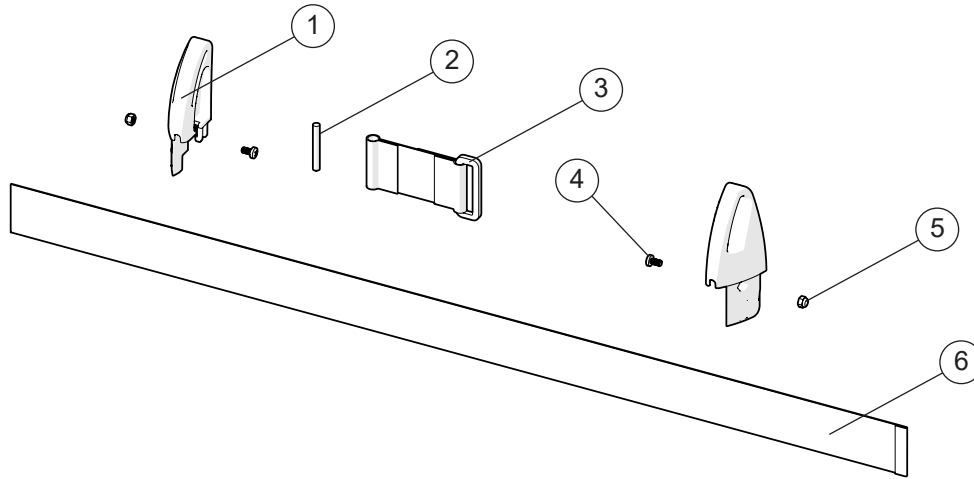

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-6	Extension back 3A 35-40 cpl	28658	●	●	●
	Extension back 3A 42,5-50 cpl	28659	●	●	●
	Extension back 3A 52,5-60 cpl	28660	●	●	●
1	Extension back R	51223-1	●	●	●
	Extension back L	51223-2	●	●	●
2	Tension rod	51222	●	●	●
3	Loop extension back 3A cpl	63294	●	●	●
4	Screw M4x8 MRX black 4.8 pos	11441	●	●	●
5	Locking nut M4 zink plated	12026	●	●	●
6	Seat depth extension 35-40 cm	63291	●	●	●
	Seat depth extension 42,5-50 cm	63292	●	●	●
	Seat depth extension 52,5-60 cm	63293	●	●	●


Push handles

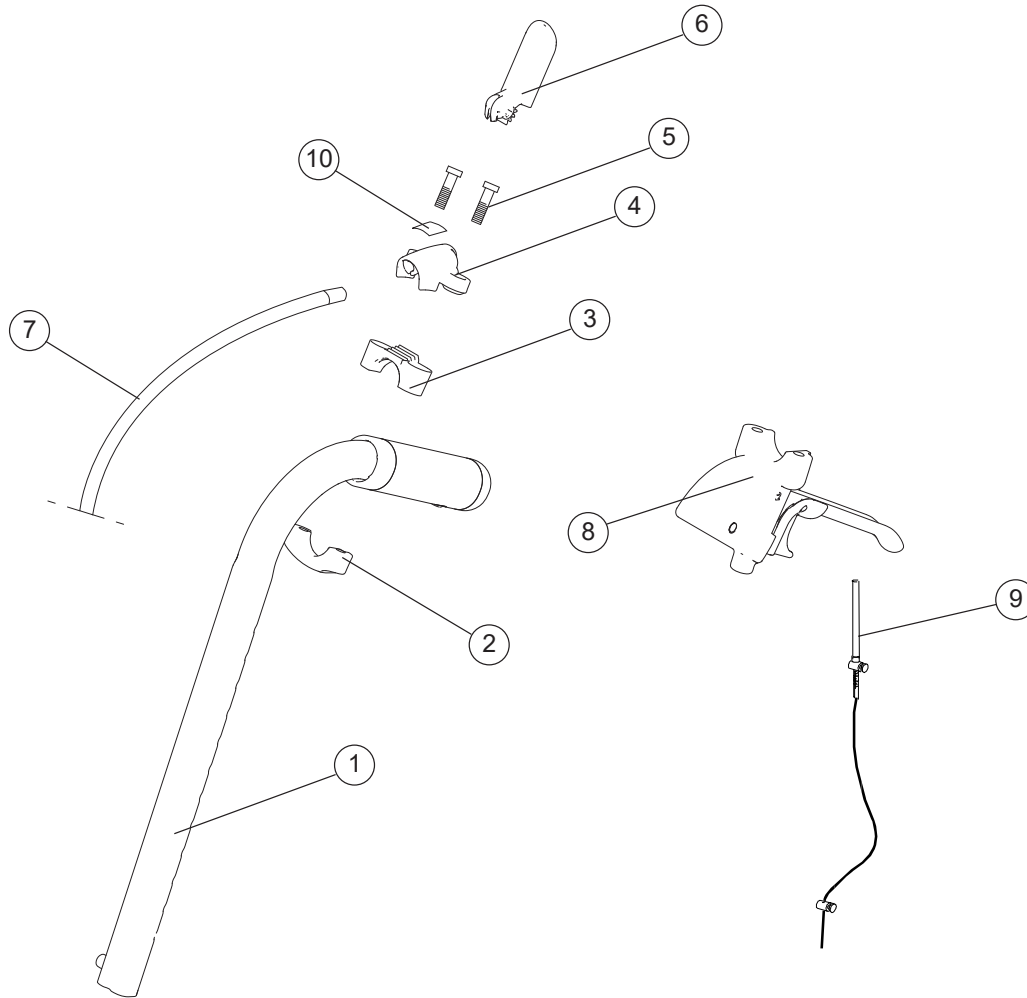

Pos.	Description	Item no.
1-7	Push handle seat angle adj	28220-01
	Push handle back tilt controls	28219-01
1-9	Push handle seat tilt brake controls	28222-01
	Push handle back tilt brake controls	28221-01
1	Push handle	27588-01
2	Attachment drum brake Prio	51050
3	Lower attachment wire control	51034
4	Upper attachment wire control	51032
5	Screw DG50 x 30 WN	11655
6	Control Prio	51033
7	Wire back support angle	83303
	Wire seat tilt	83302
8	Brake handle Prio compl (see section "Drum brake")	
9	Wire kit drum brake (see section "Drum brake")	
10	Label tilt controls	71373
	Label back support angle controls	71374

Prio
Prio AB
Prio 16"

Bow handle

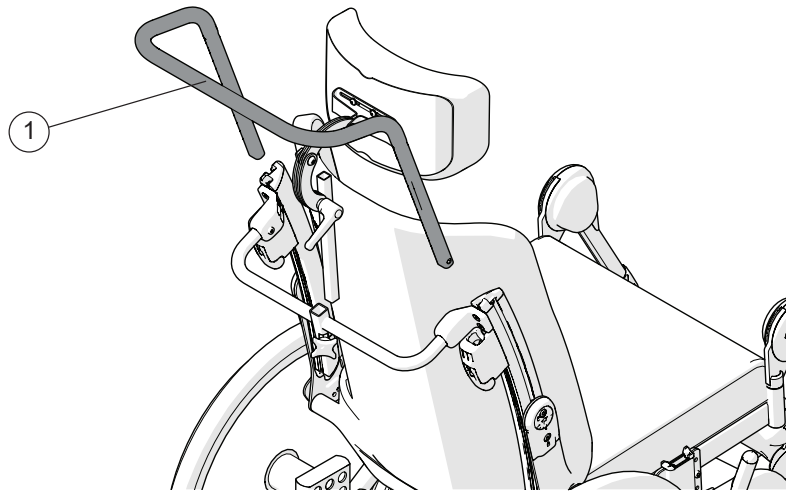


Prio
Prio AB
Prio 16"

Pos.	Description	Item no.
1	Bow handle 40 cm height adj compl	25580-01
	Bow handle 45 cm height adj compl	25582-01
	Bow handle 50 cm height adj compl	26064-01
	Bow handle 55 cm height adj compl	26069-01


●	●	●
●	●	●
●	●	●
●	●	●

Bow handle angle adjustable

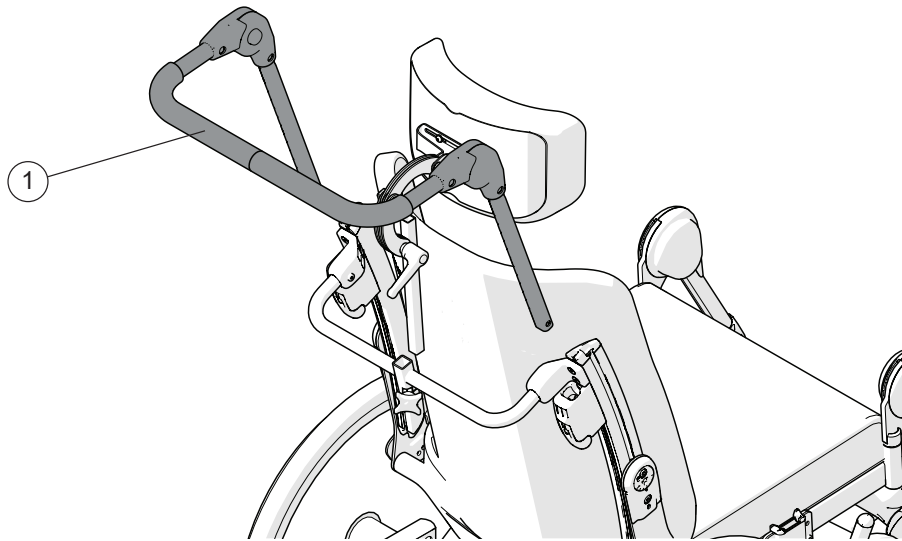

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Bow handle angle adj 40 cm	27163	●	●	●
	Bow handle angle adj 45 cm	27165	●	●	●
	Bow handle angle adj 50 cm	27167	●	●	●

Leg support elevating

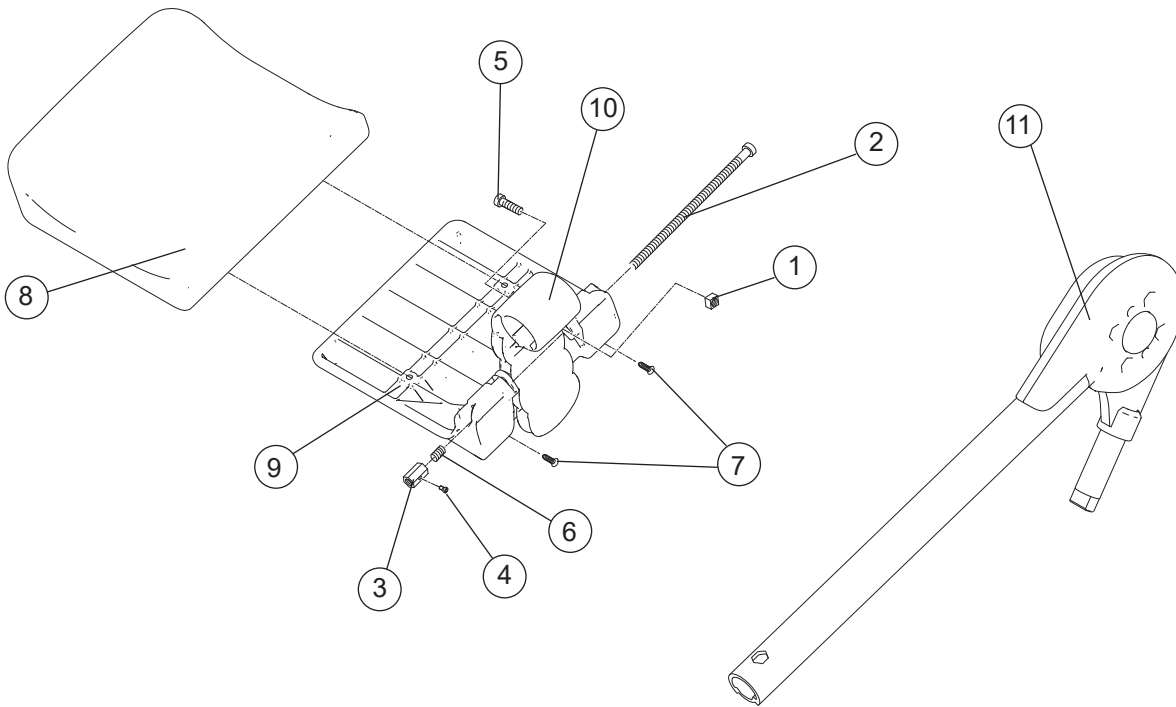


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Nut attachment calf support	20517	●	●	●
2	Screw MC6S M6 x120 10.9		●	●	●
3	Nut M6 calf support		●	●	●
4	Screw M 4x 10 MC6S 8.8		●	●	●
5	Screw M 6x 20 MC6S 8.8		●	●	●
6	Spring		●	●	●
7	Screw B 8x19 RXS/A2 ph st 4.2	11064	●	●	●
8	(Pad calf support Prio 40 grey hygiene)		●	●	●
9	(Calf support)		●	●	●
10	(Attachment calf support)		●	●	●
1-10	Calf support cpl 40 cm grey hygiene	27733-65	●	●	●
	Calf support cpl 45 cm grey hygiene	27734-65	●	●	●
	Calf support cpl 50 cm grey hygiene	27735-65	●	●	●
	Calf support cpl 55 cm grey hygiene	27736-65	●	●	●

Leg support elevating


			Prio	Prio AB	Prio 16"
Pos.	Description	Item no.			
11	Leg support elevating right	27730-01-1	●	●	●
	Leg support elevating left	27730-01-2	●	●	●
1-11	Leg support elevating Prio 40 cm green hygiene complete right	27857-64-1	●	●	●
	Leg support elevating Prio 40 cm green hygiene complete left	27857-64-2	●	●	●
	Leg support elevating Prio 40 cm grey hygiene complete right	27857-65-1	●	●	●
	Leg support elevating Prio 40 cm grey hygiene complete left	27857-65-2	●	●	●
	Leg support elevating Prio 45 cm green hygiene complete right	27858-64-1	●	●	●
	Leg support elevating Prio 45 cm green hygiene complete left	27858-64-2	●	●	●
	Leg support elevating Prio 45 cm grey hygiene complete right	27858-65-1	●	●	●
	Leg support elevating Prio 45 cm grey hygiene complete left	27858-65-2	●	●	●

Foot support

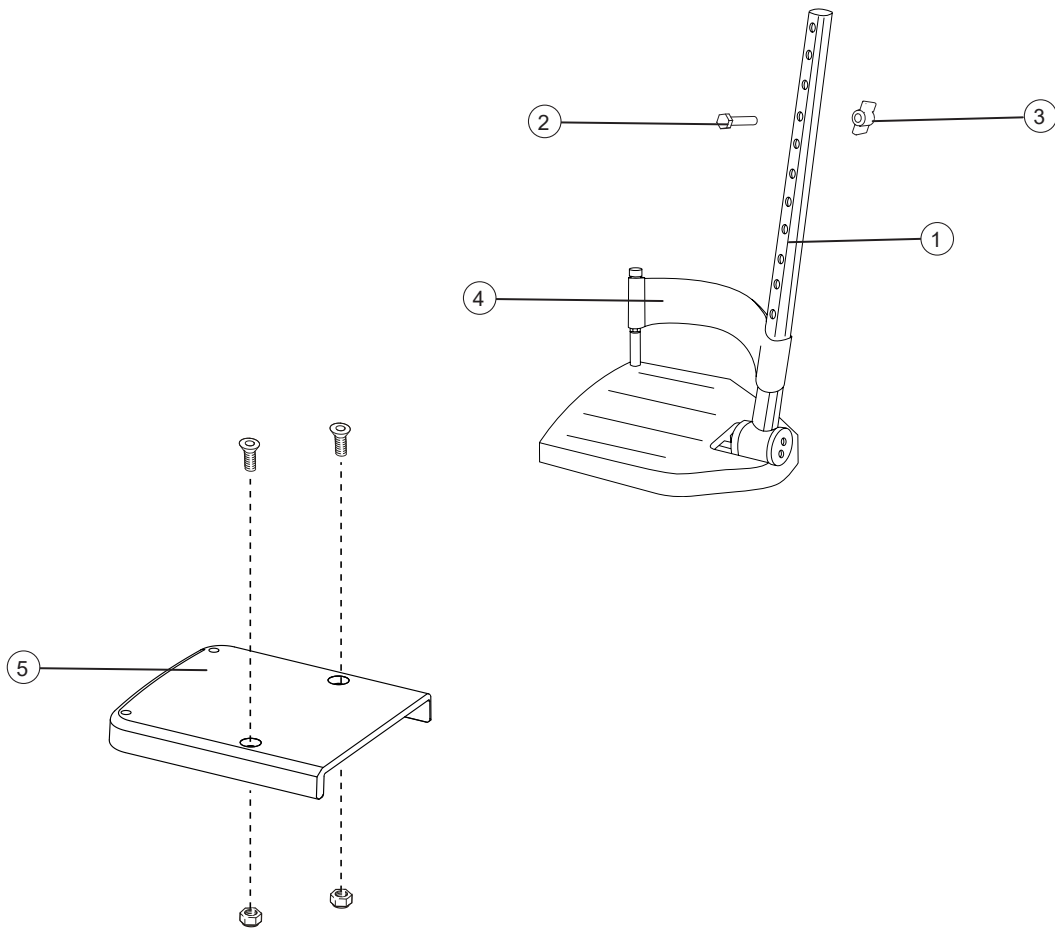

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-3	Foot support depth 13 cm 40 cm right long attachment tube	25650-1	●	●	●
	Foot support depth 13 cm 40 cm left long attachment tube	25650-2	●	●	●
	Foot support depth 13 cm 45 cm right long attachment tube	25737-1	●	●	●
	Foot support depth 13 cm 45 cm left long attachment tube	25737-2	●	●	●
	Foot support depth 13 cm 50 cm right long attachment tube	25652-1	●	●	●
	Foot support depth 13 cm 50 cm left long attachment tube	25652-2	●	●	●
	Foot support depth 13 cm 40 cm right medium attachment tube	25644-1	●	●	●
	Foot support depth 13 cm 40 cm left medium attachment tube	25644-2	●	●	●
	Foot support depth 13 cm 45 cm right medium attachment tube	25736-1	●	●	●
	Foot support depth 13 cm 45 cm left medium attachment tube	25736-2	●	●	●
	Foot support depth 13 cm 50 cm right medium attachment tube	25646-1	●	●	●
	Foot support depth 13 cm 50 cm left medium attachment tube	25646-2	●	●	●
2	Screw M6x 30 M6S	11555	●	●	●
3	Wing nut 26-M 6	80847	●	●	●
4	Heel strap compl	24458	●	●	●
5	Foot support widened 52,5-60 cm compl	28443	●	●	●

Foot support cont.

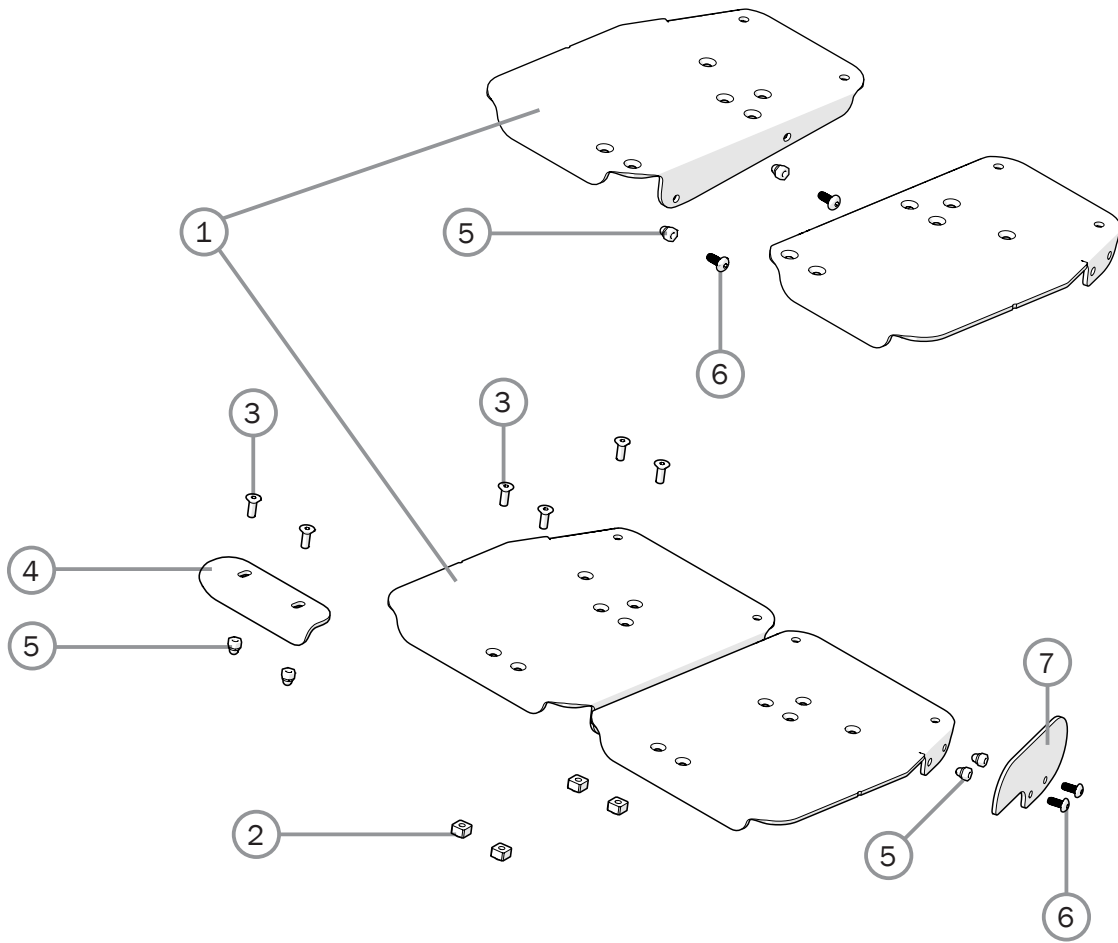


Pos.	Description	Item no.			
1-3	Extender footplate M right	28775-1	●	●	●
	Extender footplate M left	28775-2	●	●	●
	Extender footplate L right	28776-1	●	●	●
	Extender footplate L left	28776-2	●	●	●
1	Extender footplate M right		●	●	●
	Extender footplate M left		●	●	●
	Extender footplate L right		●	●	●
	Extender footplate L left		●	●	●
2	Stop bar	80661	●	●	●
3	Screw M5x14 MF6S Zn8/P esloc	11237	●	●	●
3-6	Guide one-piece footplate cpl	28773	●	●	●
4	Guide one-piece footplate		●	●	●
5	Locking nut M 5 domed zink plated	12168	●	●	●
6	Screw M5x12 K6S 10.9 zink plated	11208	●	●	●
7	Side protection footplate	83420	●	●	●
5-7	Side protection footplate cpl H35 mm	28772	●	●	●

Prio

Prio AB

Prio 16"

Foot support one piece

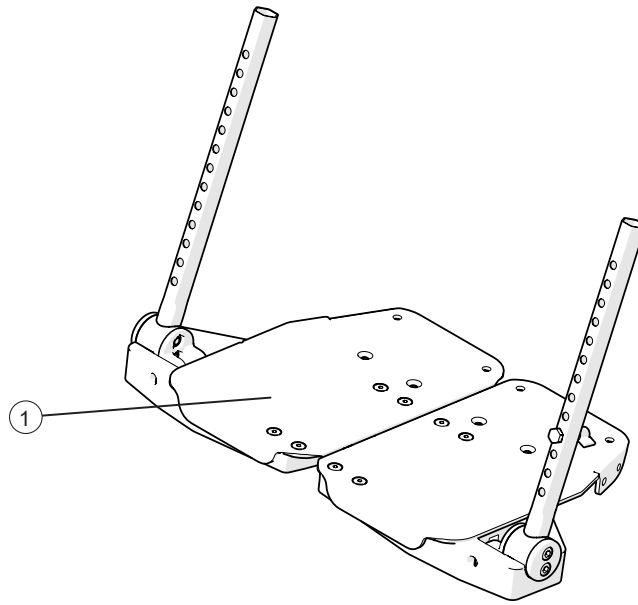


Pos.	Description	Item no.
1	Foot support one piece=40 cpl	28662
	Foot support one piece=45 cpl	28663
	Foot support one piece=50 cpl	28664
	Foot support one piece=55 cpl	28665

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●


Footbox

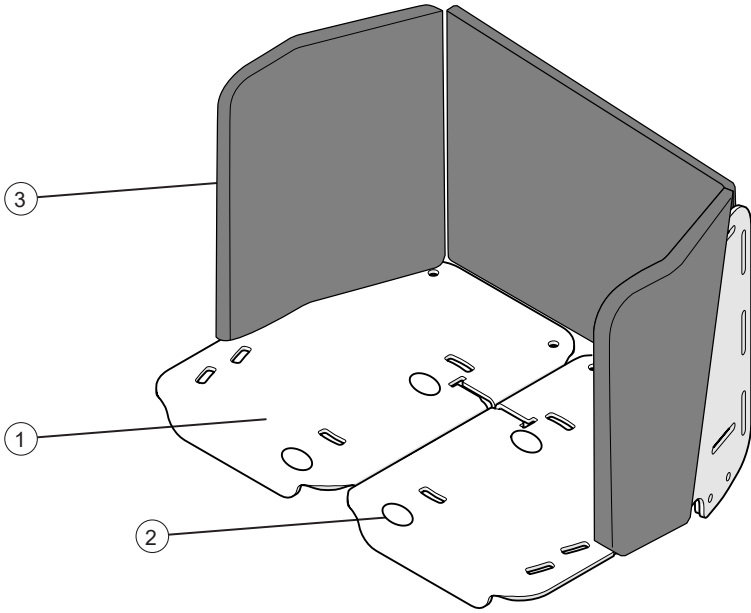

Prio
Prio AB
Prio 16"


Pos.	Description	Item. no.			
1-3	Footbox W=40-45 cm	28642	●	●	●
	Footbox W=50-55 cm	28643	●	●	●
1	Footbox W=40-45 cm		●	●	●
	Footbox W=50-55 cm		●	●	●
2	Screw kit		●	●	●
3	Padding footbox W=40-45 cm	28778	●	●	●
	Padding footbox W=50-55 cm	28779	●	●	●

User brake

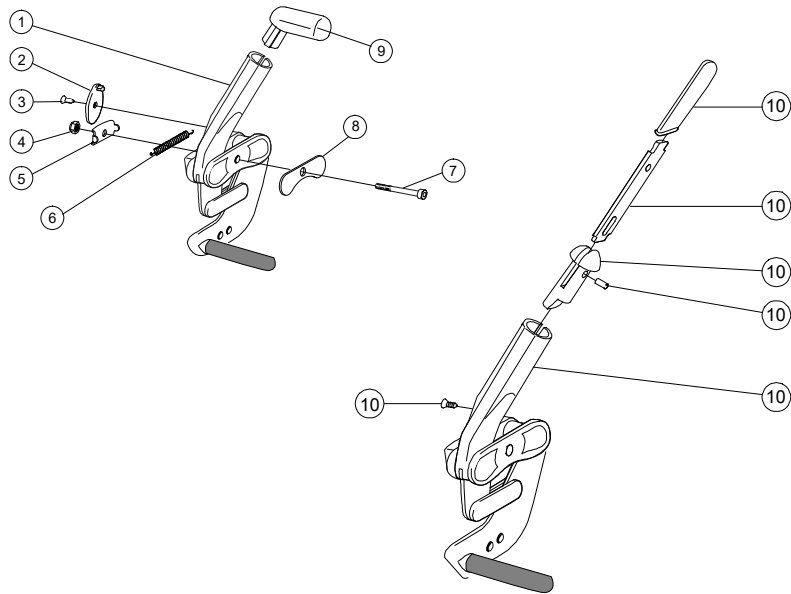

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-9	User brake angle red handle R Prio	27951-1	●	●	
	User brake angle red handle L Prio	27951-2	●	●	
	User brake angle black handle R Prio	27894-1	●	●	
	User brake angle black handle L Prio	27894-2	●	●	
2	Stop detail brake Cross mod new	50373	●	●	
3	Screw B 6 3.5x 9.5 FXS black	11384	●	●	
4	Locking nut M 6 zink plated	12017	●	●	
5	Washer inner brake mod 3	82564	●	●	
6	Spring for brake Cross	80796	●	●	
7	Screw M6x 60 MC6S Zn8/P4	11408	●	●	
8	Washer outer brake mod 3	82575	●	●	
9	Plug brake Cross red	51012	●	●	
	Plug brake Cross black	50395	●	●	
10	Brake lever extension Cross compl	25515	●	●	
1-8,10	User brake extension handle R Prio	27895-1	●	●	
	User brake extension handle L Prio	27895-2	●	●	


Brake adapter

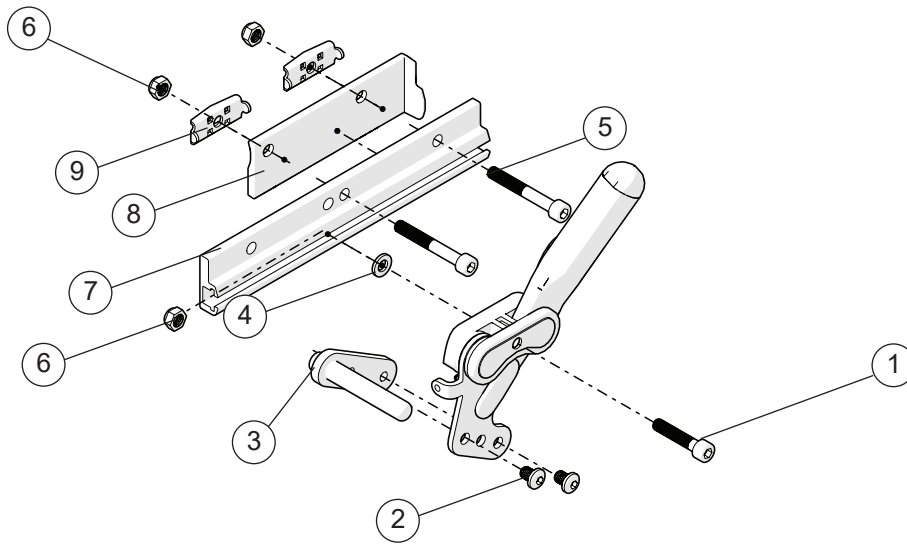

Prio
Prio AB
Prio 16"


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1-9	Adapter brake Prio 16/20	28654	●	●	●
1	Screw M6x45 MC6S 12.9 black		●	●	
2	Screw K65 M6x8 10.9 black		●	●	
3	Adapter brake arm Prio 20-22		●	●	
2-3	Adapter brake Prio 22	28403	●	●	
4	Washer 6.4x16x1,6		●	●	
5	Screw M6x 30 MC6S		●	●	
6	Locking nut M6 DIN 985-6		●	●	
7	Attachment brake		●	●	
8	Spacer brake adapter right Prio	51148-1	●	●	
	Spacer brake adapter left Prio	51148-2	●	●	
9	Washer inner brake mod 3	82564	●	●	

Rear wheel 20" (C5)

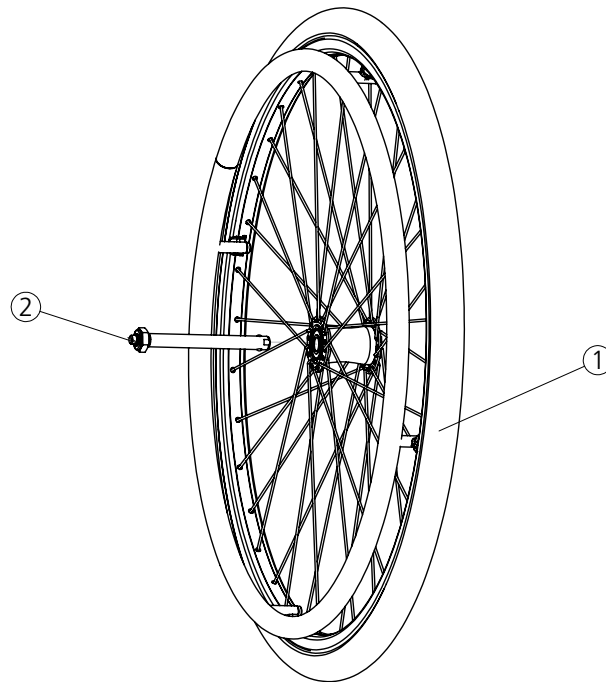


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Rear wheel 20 solid/alu steel hub M100	82486	●	●	
	Rear wheel 20 C5 solid alu handrim aluminium excl axle	27542	●	●	
	Rear wheel 20 C5 solid cell handrim cellular rubber excl axle	27543	●	●	
	Rear wheel 20 C5 solid stainle handrim stainless excl axle	27544	●	●	
2	Axle QR D12 rear wheel Cross Prio Crissy	82483	●	●	
	Axle QR D12 Cross	82581	●	●	

Rear wheel 22" (C5)

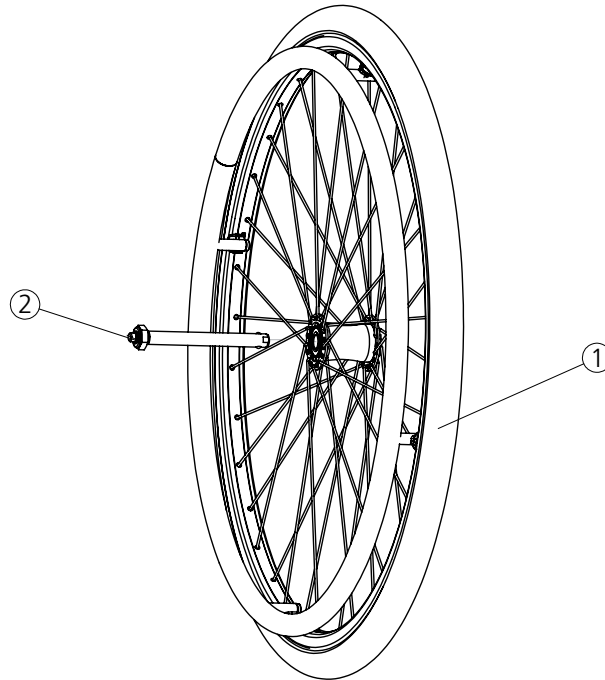

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Rear wheel 22 solid/alu steel hub M100	82485	●	●	
	Rear wheel 22" C5 high pressure handrim cellular rubber excl axle	27548	●	●	
	Rear wheel 22" C5 high pressure/plastic coated excl axle	27549	●	●	
	Rear wheel 22" C5 high pressure/stainless steel excl axle	27550	●	●	
	Rear wheel 22" C5 high pressure handrim cellular rubber excl axle	27552	●	●	
	Rear wheel 22" C5 high pressure/plastic coated excl axle	27553	●	●	
	Rear wheel 22" C5 high pressure/stainless steel excl axle	27554	●	●	
	Rear wheel 22 C5 solid ss handrim stainless excl axle	27555	●	●	
2	Axle QR D12 rear wheel Cross Prio Crissy	82483	●	●	
	Axle QR D12 Cross	82581	●	●	

Rear wheel 24" (C5)

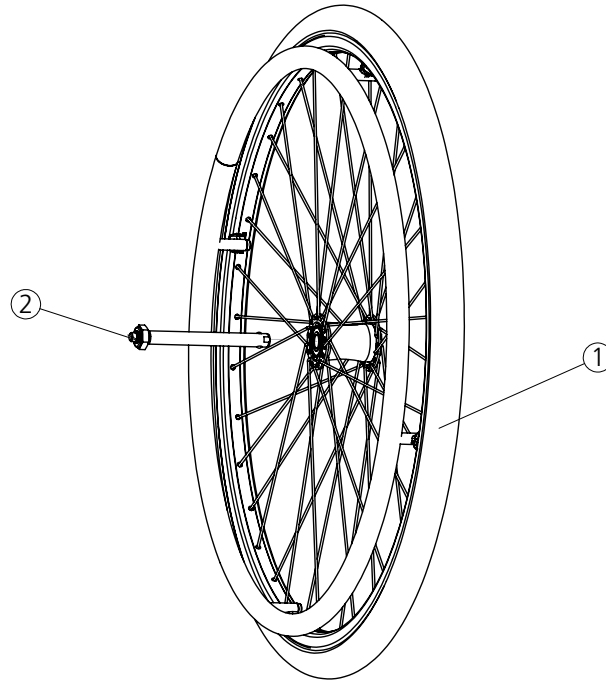


Pos.	Description	Item no.			
			Prio	Prio AB	Prio 16"
1	Rear wheel 24 HP cellular cellular rubber black excl axle	28862	●	●	
	Rear wheel 24 HP plastic plastic coated black excl axle	28863	●	●	
	Rear wheel 24 HP stainless steel stainless steel black excl axle	28864	●	●	
	Rear wheel 24 C5 solid without handrim excl axle	27563	●	●	
	Rear wheel 24 C5 solid/alu handrim aluminium excl axle	27564	●	●	
	Rear wheel 24 C5 solid cellula handrim cellular rubber excl axle	27565	●	●	
	Rear wheel 24 C5 solid plastic handrim plastic coated excl axle	27566	●	●	
2	Axle QR D12 rear wheel Cross Prio Crissy	82483	●	●	
	Axle QR D12 Cross	82581	●	●	

Rear wheel 20", 22", 24" (C6)

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	20" Rear wheel				
	Rear wheel 20 C6 hp al Speedlite excl axle	83908	●	●	
	Rear wheel 20 C6 hp ss Speedlite excl axle	83909	●	●	
	Rear wheel 20 C6 hp plastic Speedlite excl axle	83884	●	●	
	Rear wheel 20 C6 solid al excl axle	83885	●	●	
	Rear wheel 20 C6 solid ss excl axle	83910	●	●	
	Rear wheel 20 C6 solid plastic excl axle	83911	●	●	
	22" Rear wheel				
	Rear wheel 22 C6 hp al Speedlite excl axle	83880	●	●	
	Rear wheel 22 C6 hp ss Speedlite excl axle	83906	●	●	
	Rear wheel 22 C6 hp plastic Speedlite excl axle	83881	●	●	
	Rear wheel 22 C6 solid al excl axle	83882	●	●	
	Rear wheel 22 C6 solid ss excl axle	83907	●	●	
	Rear wheel 22 C6 solid plastic excl axle	83883	●	●	
	Rear wheel 22 Maxx hp hard al Speedlite excl axle	28956	●	●	
	Rear wheel 22 Maxx hp plastic Speedlite excl axle	28957	●	●	
	Rear wheel 22 Maxx hp ti Speedlite excl axle	24087	●	●	
	Rear wheel 22 Maxx hp hard al Trailblazer R excl axle	24052-1	●	●	
	Rear wheel 22 Maxx hp hard al Trailblazer L excl axle	24052-2	●	●	
	Rear wheel 22 Maxx hp plastic Trailblazer R excl axle	24053-1	●	●	
	Rear wheel 22 Maxx hp plastic Trailblazer L excl axle	24053-2	●	●	
	Rear wheel 22 Maxx hp ti Trailblazer R exc axle	24115-1	●	●	
	Rear wheel 22 Maxx hp ti Trailblazer L exc axle	24115-2	●	●	
	24" Rear wheel				
	Rear wheel 24 C6 hp al Speedlite excl axle	83873	●	●	
	Rear wheel 24 C6 hp ss Speedlite excl axle	83874	●	●	
	Rear wheel 24 C6 hp plastic Speedlite excl axle	83875	●	●	
	Rear wheel 24 C6 solid al excl axle	83876	●	●	
	Rear wheel 24 C6 solid ss excl axle	83877	●	●	
	Rear wheel 24 C6 solid plastic excl axle	83878	●	●	
	Rear wheel 24 Maxx hp hard al Speedlite excl axle	28950	●	●	
	Rear wheel 24 Maxx hp plastic Speedlite excl axle	28951	●	●	
	Rear wheel 24 Maxx hp ti Speedlite excl axle	24088	●	●	
	Rear wheel 24 Maxx hp hard al Trailblazer R excl axle	28952-1	●	●	
	Rear wheel 24 Maxx hp hard al Trailblazer L excl axle	28952-2	●	●	
	Rear wheel 24 Maxx hp plastic Trailblazer R excl axle	28953-1	●	●	
	Rear wheel 24 Maxx hp plastic Trailblazer L excl axle	28953-2	●	●	
	Rear wheel 24 Maxx hp ti Trailblazer R excl axle	24116-1	●	●	
	Rear wheel 24 Maxx hp ti Trailblazer L excl axle	24116-2	●	●	
	Rear wheel 24 Maxx solid hard al excl axle	24126	●	●	
Rear wheel 24 Maxx solid ti excl axle	24127	●	●		
Rear wheel 24 Maxx solid plastic excl axle	24128	●	●		
2	Axle				
	QR-axle D12.7x110 Cross Crissy Prio	83872	●	●	

Attendant brake

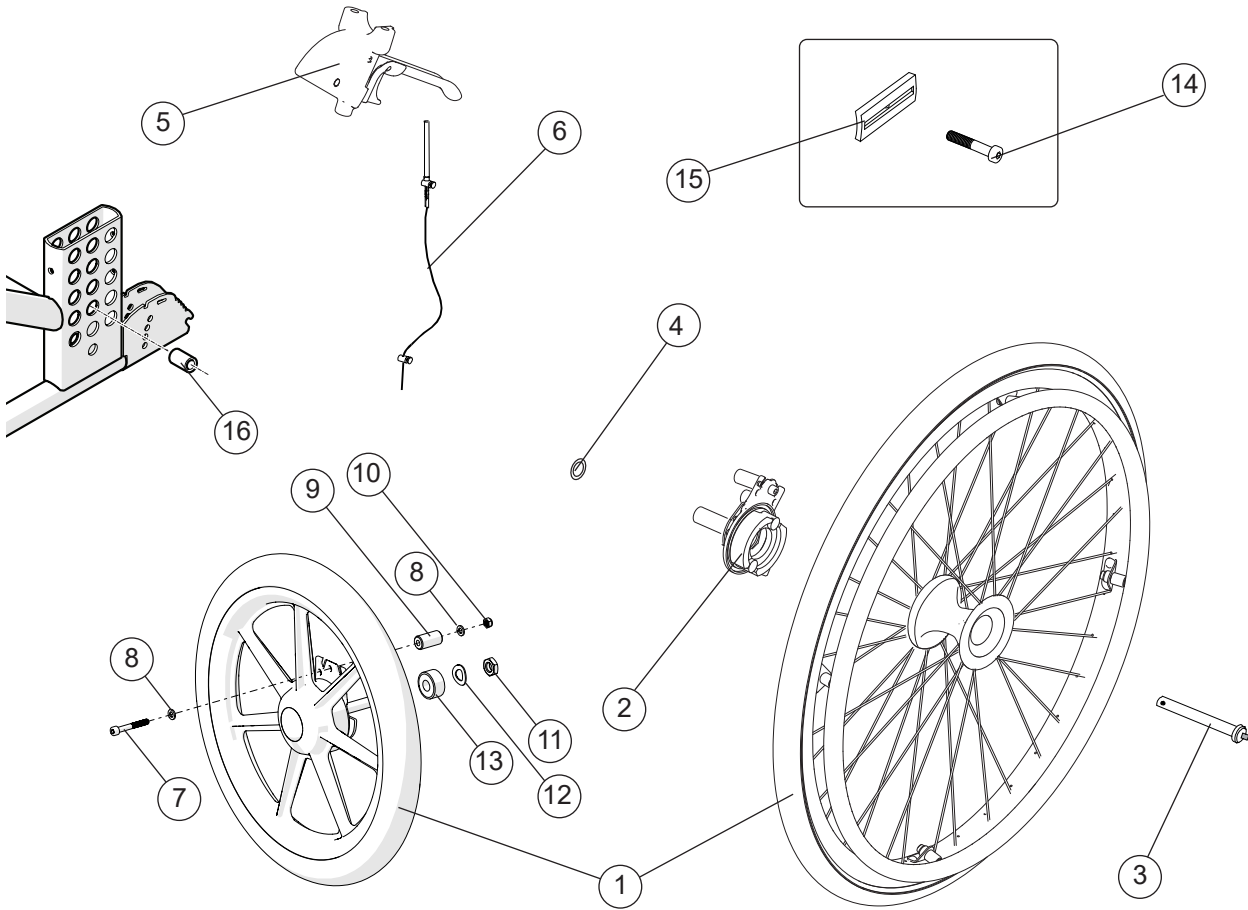


Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Rear wheel 20 DB solid al excl axle	81337B	●	●	
	Rear wheel 20 DB solid excl handrim and axle	83090B	●	●	
	Rear wheel 22 DB solid al excl axle	81339B	●	●	
	Rear wheel 22 DB solid handrim stainless	81341	●	●	
	Rear wheel 22 DB solid plastic excl axle	81340B	●	●	
	Rear wheel 22 DB hp al excl axle	81342B	●	●	
	Rear wheel 22 DB hp ss excl axle	81344B	●	●	
	Rear wheel 22 DB hp plastic excl axle	81343B	●	●	
	Rear wheel 24 drum brake solid handrim aluminium	81346	●	●	
	Rear wheel 24 DB solid ss excl axle	81348B	●	●	
	Rear wheel 24 DB solid plastic excl axle	81347B	●	●	
	Rear wheel 24 DB hp ss excl axle	28624B	●	●	
	Rear wheel 24 DB hp al excl axle	28625B	●	●	
	Rear wheel 24 DB hp plastic excl axle	28626B	●	●	
	Rear wheel 16 solid right black	83386			●
	Rear wheel 16 solid left black	83387			●

Attendant brake cont.

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
2	Brake shoe kit drum brake R Prio	82822	●	●	
	Brake shoe kit drumbrake L Prio	82823	●	●	
3	Qr axle drumbrake	81325	●	●	
4	Schnorr washer 17x24x1.3	13084	●	●	
5	Brake handle Prio compl	82585	●	●	●
6	Wire kit drum brake Prio	82856	●	●	●
7	Screw M6x 45 MC6S 12.9 black				●
8	Washer brb 11.8x6.5x1.5				●
9	Spacer 6.5x15.9x30				●
10	Nut M6 Zn8/P4 DIN 985-6				●
11	Nut M12 FZB DIN 439B low	12076			●
12	Spring washer 13x24x1.2 DIN 137B fzb				●
13	Spacer 12x24x10				●
14	Screw M6x65 MC6S 12.9 fzb	11236	●	●	
15	Adapter brake 5	51019	●	●	
2-6 14-15	Attachment kit drum brake Prio 20/22/24	28423	●	●	
5-13	Assembly kit 16"	28459			●
	Push handles, see section "Push handles"				
16	Adapter sleeve 16" Prio (2 pcs and assembly instruction) when mounting in pos A and B	28414			●


Tyre and innertube (C5)

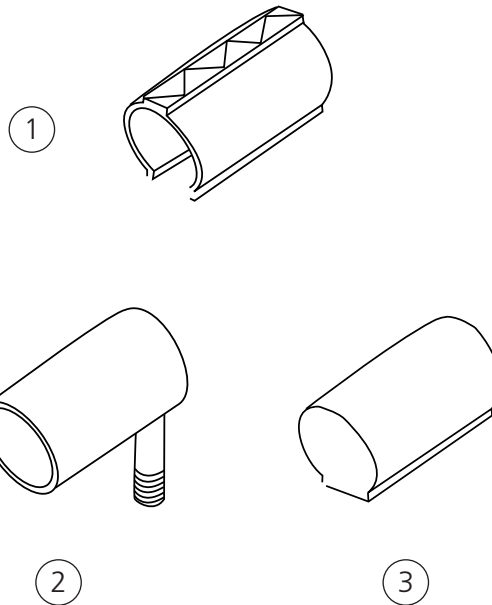

Pos.	Description	Item no.			
1	Tyre 20" solid	80787	●	●	
	Tyre 22" solid	81384	●	●	
	Tyre 22"x1" pneumatic high pressure	81125	●	●	
	Tyre 24" solid	80768	●	●	
	Tyre 24" high pressure black	83329	●	●	
	Tyre 24" Schwalbe Marathon plus	83301	●	●	
2	Innertube 20" pneumatic	80132	●	●	
	Innertube 22" pneumatic high pressure	80136	●	●	
	Innertube 24" pneumatic high pressure	80560	●	●	
3	Innertube 20" solidive	82706	●	●	
	Innertube 22" solidive	82705	●	●	
	Innertube 24" solidive	82704	●	●	


Prio
 Prio AB
 Prio 16"

Tyre and innertube (C6)

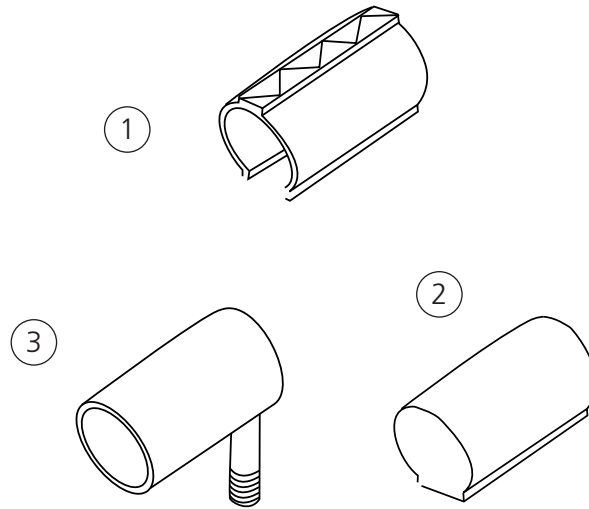

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Tyre 20x1 (25-451) Speedlite black	83897	●	●	
	Tyre 22x1 (25-501) Speedlite black	83898	●	●	
	Tyre 24x1 (25-540) Speedlite black	83899	●	●	
2	Tyre 20x1 3/8 (37-451) solid black	83347	●	●	
	Tyre 22x1 3/8 (37-501) solid black	83348	●	●	
	Tyre 24x1 1/4 (32-540) solid black	83349	●	●	
3	Innertube 20x1 (25-451) high pressure	82199	●	●	
	Innertube 22x1 (25-501) high pressure	80136	●	●	
	Innertube 24x1 (25-540) high pressure	80560	●	●	
	Innertube 25x1 (25-559) high pressure	83904	●	●	

Handrims C5

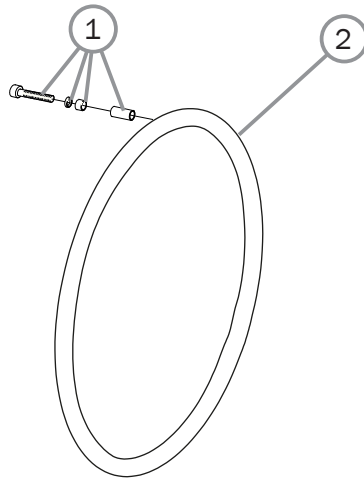


Prio

Prio AB

Prio 16"


Pos.	Description	Item no.			
1	Assembly kit handrim 4m plas. Plastic handrim	24514	●	●	
	Assembly kit handrim 4m	28113	●	●	
2	20" Rear wheel				
	Handrim D465 wheel 20" alu	81109	●	●	
	Handrim D465 wheel 20" vinyl	81229	●	●	
	Handrim D425 wheel 20" cellular, clip mounted	24442	●	●	
	Handrim D465 wheel 20" stainless	21034	●	●	
	Handrim D465 wheel 20" stainless	83006	●	●	
	Handrim D465 wheel 20" vinyl	83010	●	●	
	22" Rear wheel				
	Handrim D500 wheel 22" alu	25636	●	●	
	Handrim D500 wheel 22" stainless	24395	●	●	
	Handrim D500 wheel 22" vinyl	80895	●	●	
	Handrim D469 wheel 22" cellular, clip mounted	24551	●	●	
	Handrim D500 wheel 22" stainless	83007	●	●	
	Handrim D500 wheel 22" vinyl	83011	●	●	
	24" Rear wheel				
	Handrim D540 wheel 24" alu	25181	●	●	
	Handrim D540 wheel 24" stainless	24023	●	●	
	Handrim D540 wheel 24" vinyl	82998	●	●	
	Handrim D540 wheel 24" cellular clip mounted	80800	●	●	
	Handrim D540 wheel 24" stainless	83008	●	●	
Handrim D540 wheel 24" vinyl	83012	●	●		

Handrims (C6)

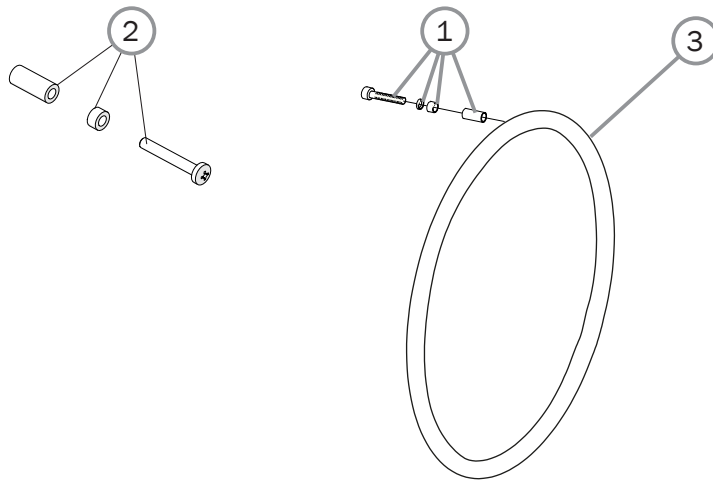

Pos.	Description	Item no.	Prio	Prio AB	Prio 16"
1	Assembly kit handrim 6 ap (not plastic coated)	83892	●	●	
2	Assembly kit handrim 6m plastic handrim C6 Maxx	24081	●	●	
3	20" Rear wheel				
	Handrim 20" al 6 ap	83890	●	●	
	Handrim 20" stainl 6 ap	83891	●	●	
	Handrim 20" plastic coated 6 ap	83339	●	●	
	22" Rear wheel		●	●	
	Handrim 22 al 6 ap	83888	●	●	
	Handrim 22 stainl 6 ap	83889	●	●	
	Handrim 22 plastic coated 6 ap	83338			
	24" Rear wheel		●	●	
	Handrim 24 al 6 ap	83886	●	●	
	Handrim 24 stainl 6 ap	83887	●	●	
	Handrim 24 plastic coated 6 ap	83335	●	●	

Widening kit

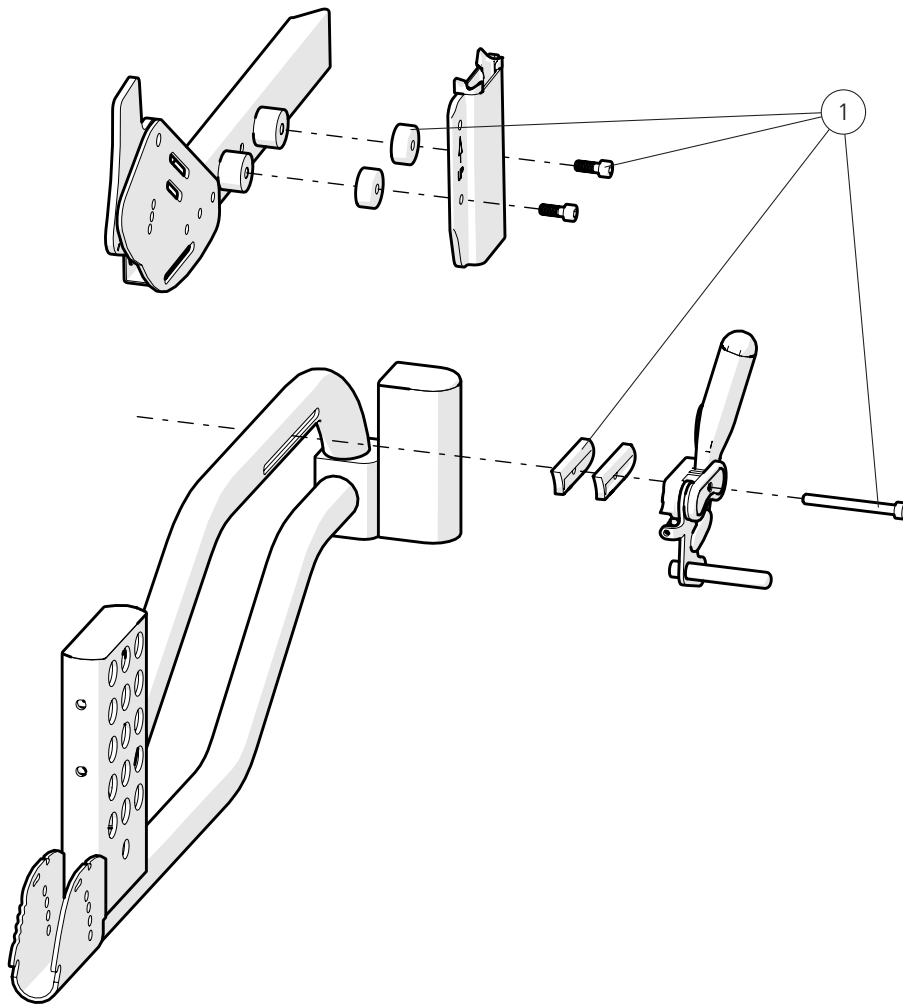

Pos.	Description	Item no.
1	Widening kit 25 mm	27892


Prio

Prio AB

Prio 16"



Arm support

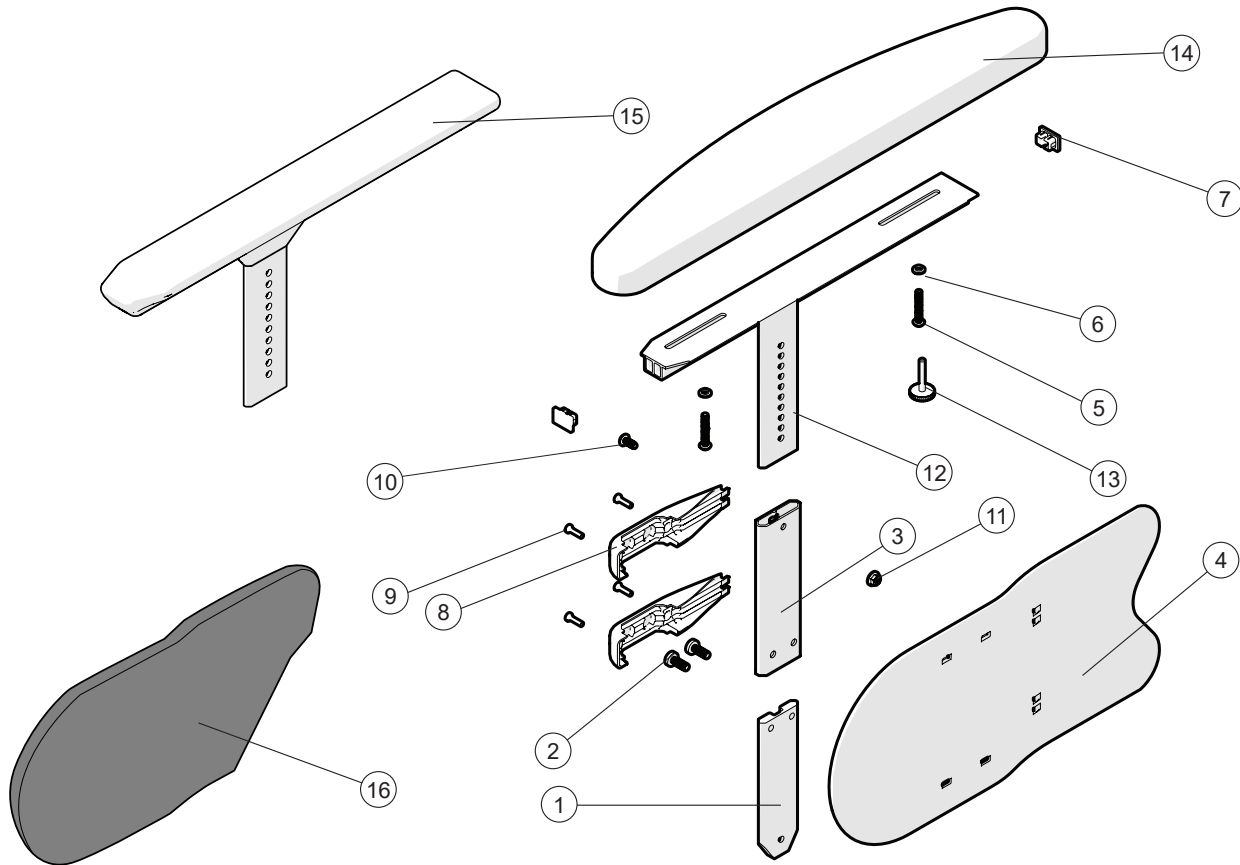


Pos.	Description	Item no.			
1-14	Arm support telescope L49 ri	28827-1	●	●	●
	Arm support telescope L49 le	28827-2	●	●	●
	Arm support telescope L49 ri fingerscrew	28828-1	●	●	●
	Arm support telescope L49 le fingerscrew	28828-2	●	●	●
1-13, 15	Arm support telescope L38 ri	28826-1	●	●	●
	Arm support telescope L38 le	28826-2	●	●	●
1-13	Arm support telescope without cushion ri	28832-1	●	●	●
	Arm support telescope without cushion le	28832-2	●	●	●
1	Profile		●	●	●
2	Screw		●	●	●
3	Profile sleeve		●	●	●
4	Side guard	51256	●	●	●
5	Screw M5x30	11446	●	●	●
6	Joint bushing	50285	●	●	●
7	End cap	51195	●	●	●
8	Clamp side guard	51208	●	●	●
9	Screw K40x14	11419	●	●	●
10	Screw M6x12 MC6S 8.8 zink plated	11022	●	●	●
11	Locking nut M6 zink plated DIN 985-6	12017	●	●	●
12	Arm support profile ri		●	●	●
	Arm support profile le		●	●	●
13	Fingerscrew	83803	●	●	●
14	Arm support cushion	83266	●	●	●
15	Arm support solid plate ri		●	●	●
	Arm support solid plate le		●	●	●
16	Padding side guard ri	63271-1	●	●	●
	Padding side guard le	63271-2	●	●	●

Prio

Prio AB

Prio 16"

Arm support

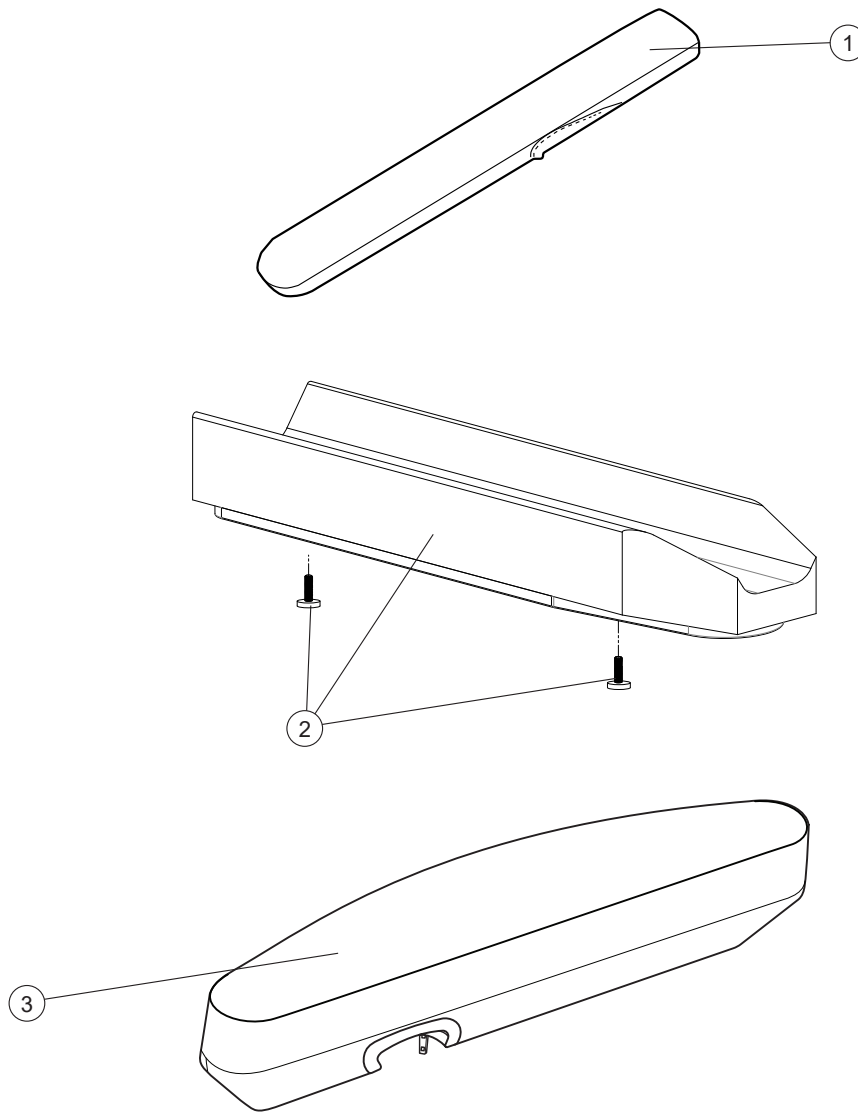


Pos.	Description	Item no.
1	Cover arm support long	62283-60
	Cover arm support gel long	26241-60
	Cover arm support long Dartex	62283-63
	Cover arm support gel long Dartex	26241-63
2	Arm support pad soft wide incl screw	28640
3	Arm support pad Prio ri	63193-60-1
	Arm support pad Prio le	63193-60-2

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Arm support Hemi

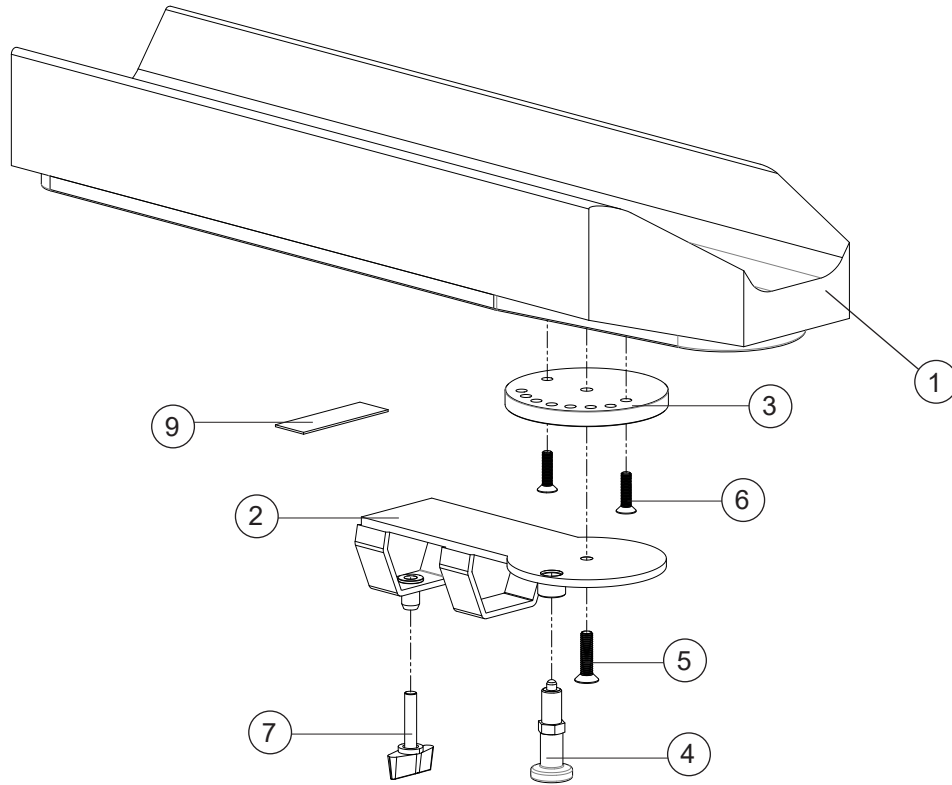


Pos.	Description	Item. no.
1-7	Arm support pad hemi right	25770-60-1
	Arm support pad hemi left	25770-60-2
1	Arm support pad hemi	62910-60
2	Attachment arm support hemi right	27265-1
	Attachment arm support hemi left	27265-2
3	Turntable right	27280-1
	Turntable left	27280-2
4	Adjusting bolt M10x1	82338
5	Screw M6x25 MF6S svkr esloc	11430
6	Screw M5x20 MF6S black chrome	11526
7	Handle M6x30	82033

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Trunk support

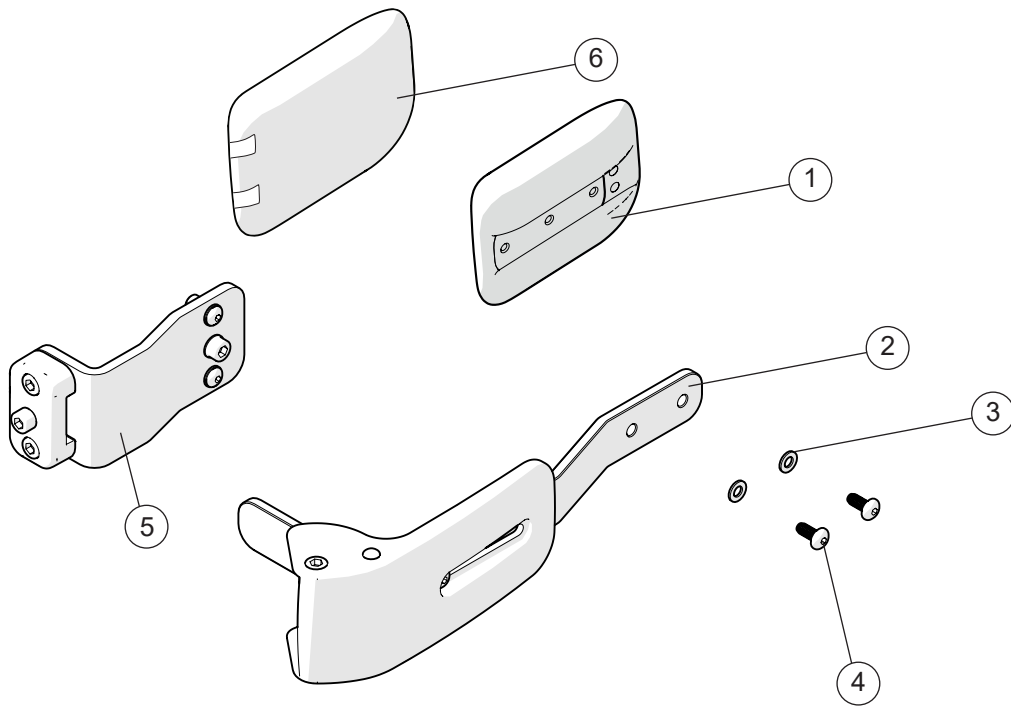

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-5	Trunk support Prio incl attachment right	28181-1	●	●	●
	Trunk support Prio incl attachment left	28181-2	●	●	●
	Trunk support Prio incl attachment right -15 mm	28180-1	●	●	●
	Trunk support Prio incl attachment left -15 mm	28180-2	●	●	●
	Trunk support Prio incl attachment right -45 mm	28141-1	●	●	●
	Trunk support Prio incl attachment left -45 mm	28141-1	●	●	●
1	Trunk support pad	83005	●	●	●
2	Trunk support arm		●	●	●
3	Washer 5.3x10x1 BRB DIN 125	13017	●	●	●
4	Screw M 5x 12 K6S 10.9	11208	●	●	●
5	Trunk support attachment 3A compl	28146	●	●	●
6	Cover trunk support	28647	●	●	●


See page 36 for different bars

Trunk support

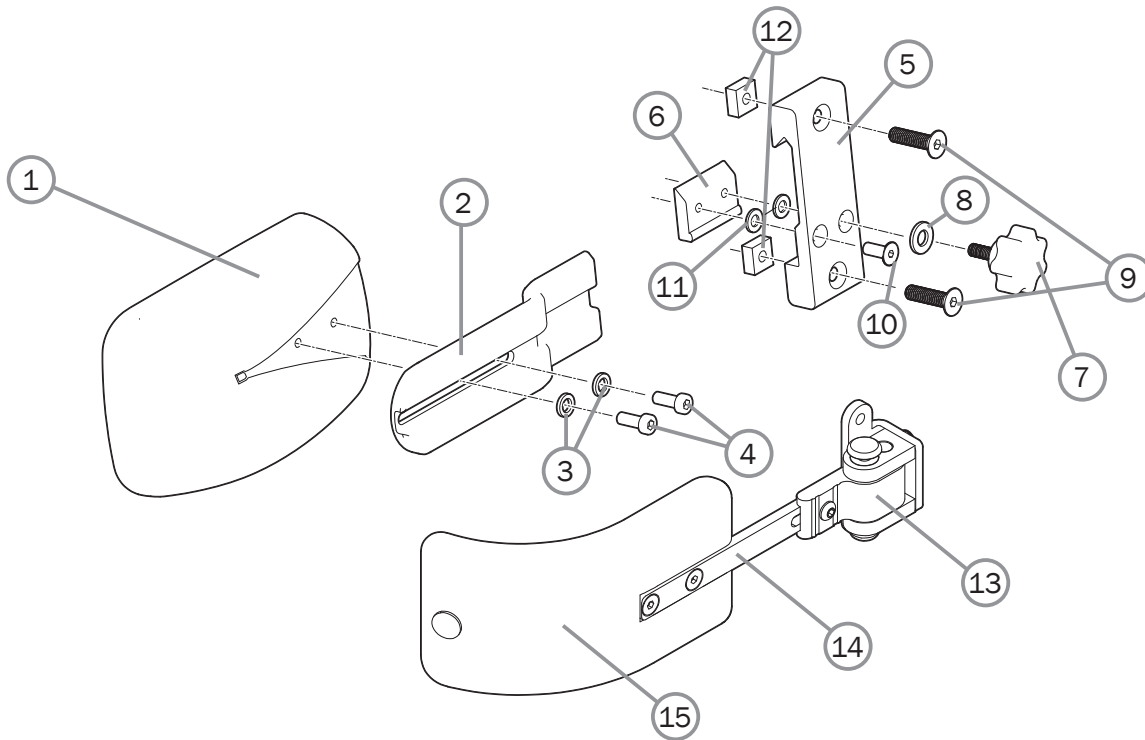

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-12	Trunk support Cross back mod 3 incl attachment	27395	●	●	●
1	Trunk supp pad curved small Dartex Cross back mod 3	82491	●	●	●
2	Bracket trunk support	82507	●	●	●
3	Spring washer 6.1x9.9x1.6 DIN 7980	13014	●	●	●
4	Screw M 5x 12 K6S 10.9 zink plated	11208	●	●	●
5	Attachment arm support Elite/Sting without screws	80403	●	●	●
6	Locking plate arm support	80292	●	●	●
7	Knob M 5x15 leg support amputee Cross	80807	●	●	●
8	Washer BRB 5.3x15x1.2 black	13004	●	●	●
9	Screw M 6x 25 MF6S black esloc	11430	●	●	●
10	Screw M 5x 14 MF6S 10.9 black	11237	●	●	●
11	Washer 5.3x 10x 0.5m DIN 137A zink plated	13026	●	●	●
12	Nut M 6 FK4M	12166	●	●	●
13	Attachment trunk support swing away R compl	27947-1	●	●	●
	Attachment trunk support swing away L compl	27947-2	●	●	●
14	Bar trunk support swing away L straight 12	27948	●	●	●
15	Cushion trunk support medium bent black H 6.5 L 17	27949	●	●	●
	Cushion trunk support large bent black H 9 cm L 19	27950	●	●	●

Trunk support

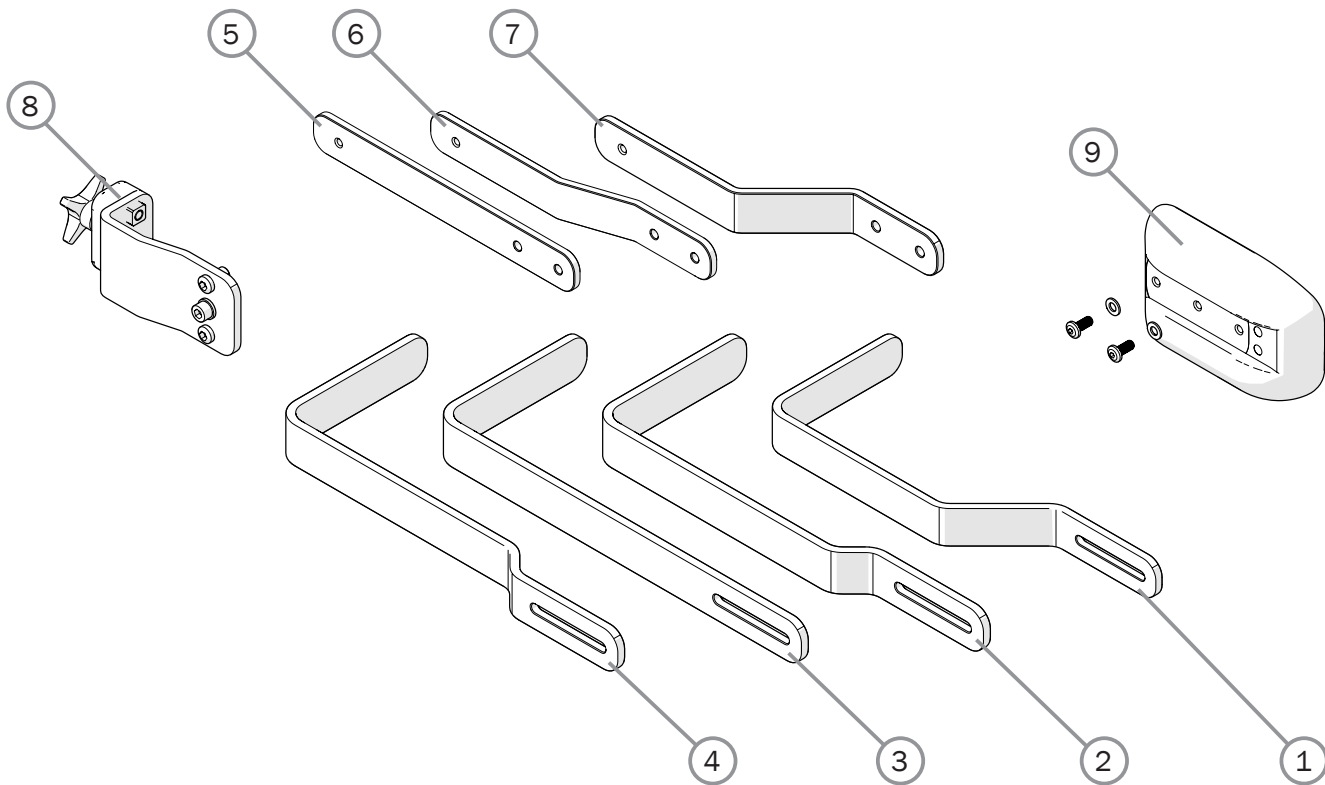


Pos.	Description	Item. no.
1	Trunk support bar -45 mm	83002
2	Trunk support bar -15 mm	83017
3	Trunk support bar 0 mm	83018
4	Trunk support bar +15 mm	83019
5	Trunk support bar ssb -0 mm	83046
6	Trunk support bar ssb -15 mm	83045
7	Trunk support bar ssb -45 mm	83001
8	Trunk support attachment 3A compl	28146
9	Trunk support pad straight compl	28276

Prio
Prio AB
Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Trunk support attachment

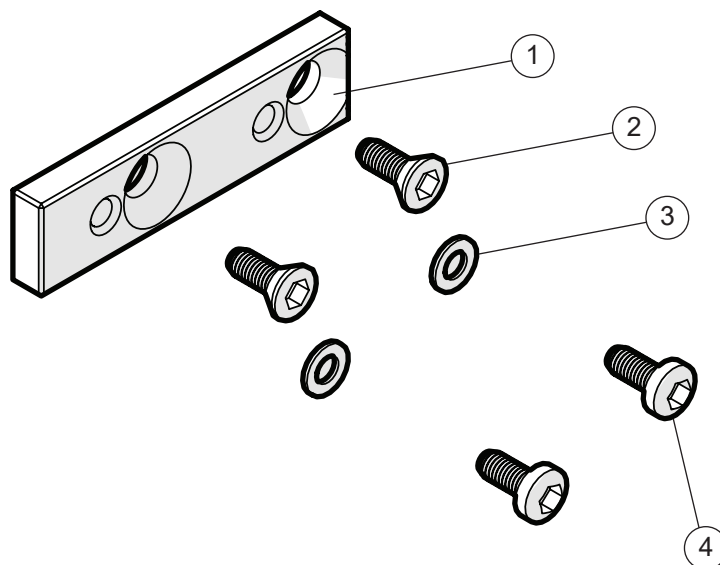

Pos.	Description	Item. no.
1-4	Adapterkit trunk support cushion R82	28240
1	Trunk support adapter	83160
2	Screw	11592
3	Washer 5.3x10x1 BRB fzb DIN 125	13017
4	Screw M 5x 12 K6S 10.9	11208

Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Leg abduction

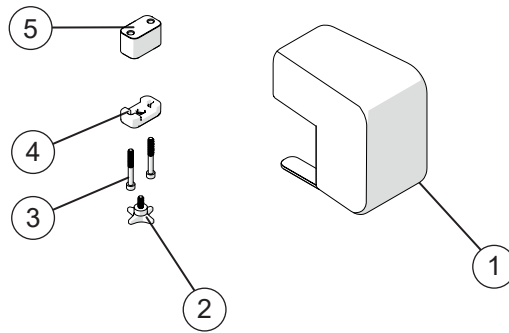

Pos.	Description	Item. no.
1-5	Leg abduction compl	28022
1	Leg abduction	28194
2	Knob 40 M6x17	80297
3	Screw	11037
4	Bracket	28305-01
5	Spacer	83042


Prio

Prio AB

Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Tray

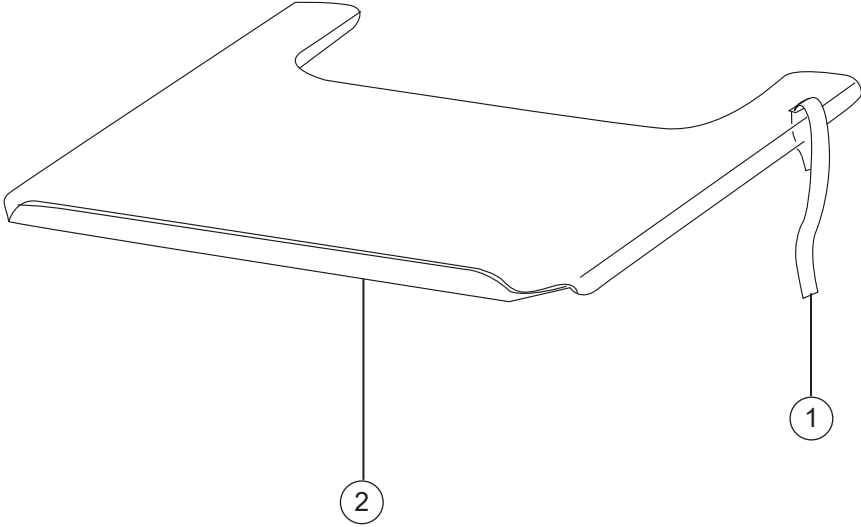

	Prio	Prio AB	Prio 16"
	●	●	●
	●	●	●
	●	●	●
	●	●	●

Pos.	Description	Item no.
1	Attachment strap tray	82921
1-2	Tray Prio 40 cm table width 57.5 cm	27971
	Tray Prio 45 cm table width 62.5 cm	27972
	Tray Prio 50 cm table width 67.5 cm	27973
	Tray Prio 55 cm table width 72.5 cm	27974

Attachment harness

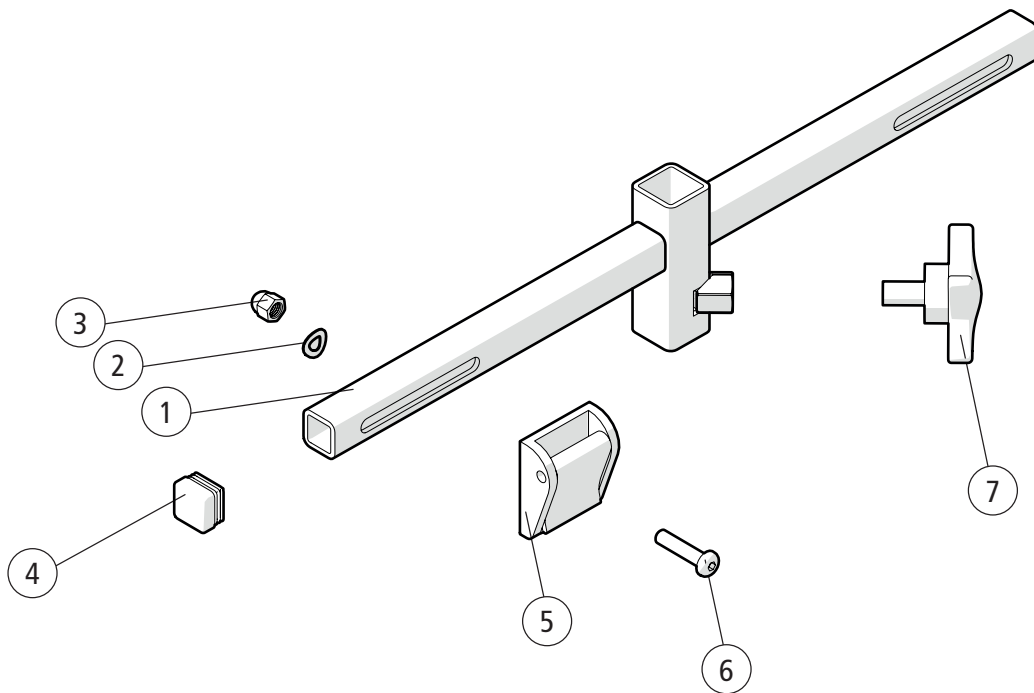

Pos.	Description	Item. no.
1-7	Attachment harness cpl	28186
1	Attachment harness	
2	Washer D10x5.3x0.5	13017
3	Domed nut M5	12167
4	End plug	50972
5	Fix lock 25 mm	310445025
6	Screw M 5x 22 MK6S fzb	11075
7	Knob	400118


Prio

Prio AB

Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Drip stand

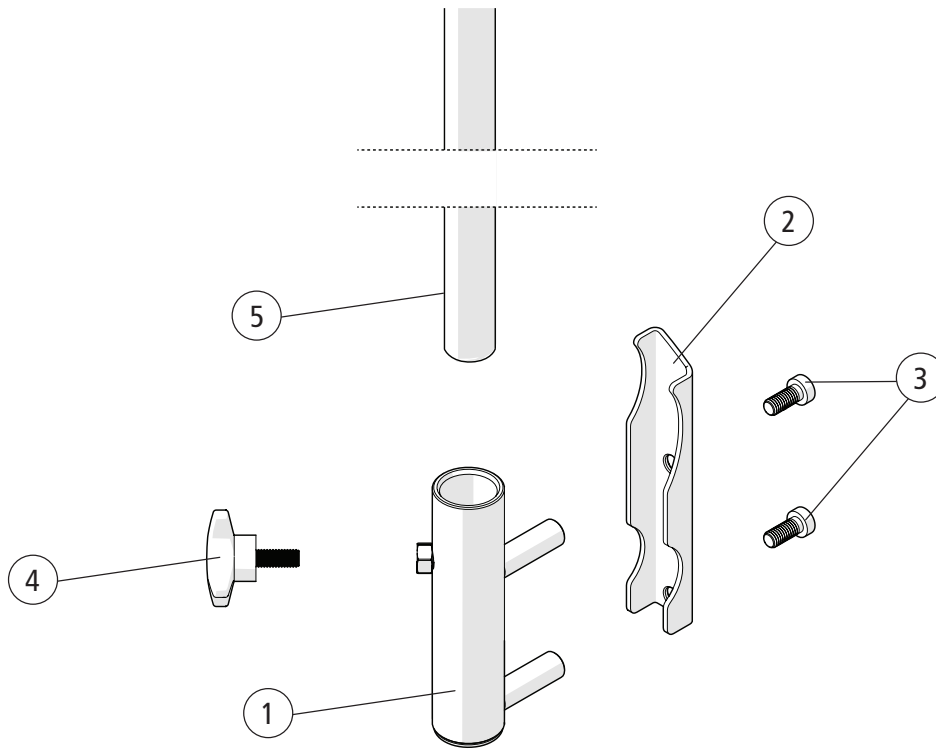

Pos.	Description	Item. no.
1-5	Drip stand cpl	28341
1-4	Attachment	28318
1	Drip stand welded Prio	
2	Bracket	
3	Screw M 6x 14 MC6LS SW5 Zn8/P4	11539
4	Knob	80297
5	Hook drip stand	83053



Prio

Prio AB

Prio 16"


<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>

Basket

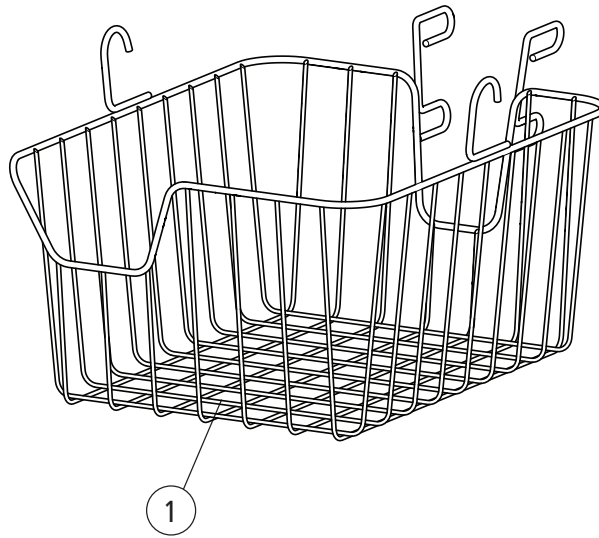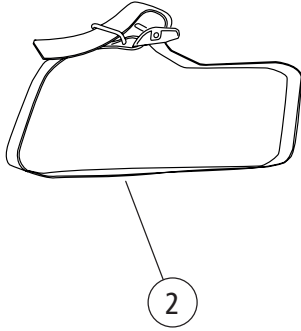

Pos.	Description	Item. no.	Prio	Prio AB	Prio 16"
1-2	Basket compl Prio	28316	●	●	●
1	Basket		●	●	●
2	Strap	80459	●	●	●

Positioning belt

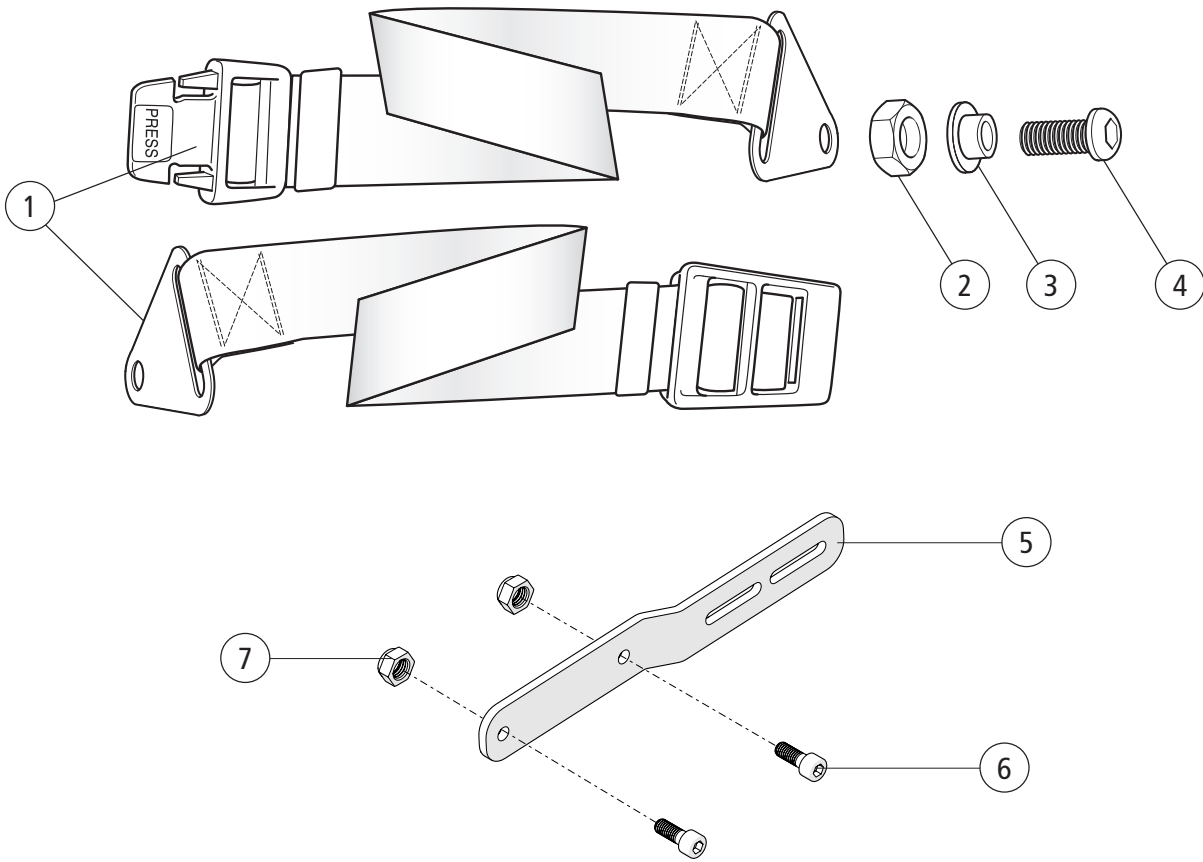

Pos.	Description	Item no.
1-4	Positioning belt lenght 100 mm	27904
	Positioning belt lenght 128 mm	28544
2	Locking nut M 6	
3	Spacer positioning belt	
4	Screw M 6x 14 K65	
5-7	Attachment positioning belt	28641
6	Screw M6x16 MC6S 12.9	11253
7	Locking nut M6	12035


Prio

PrioAB

Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Anti tip

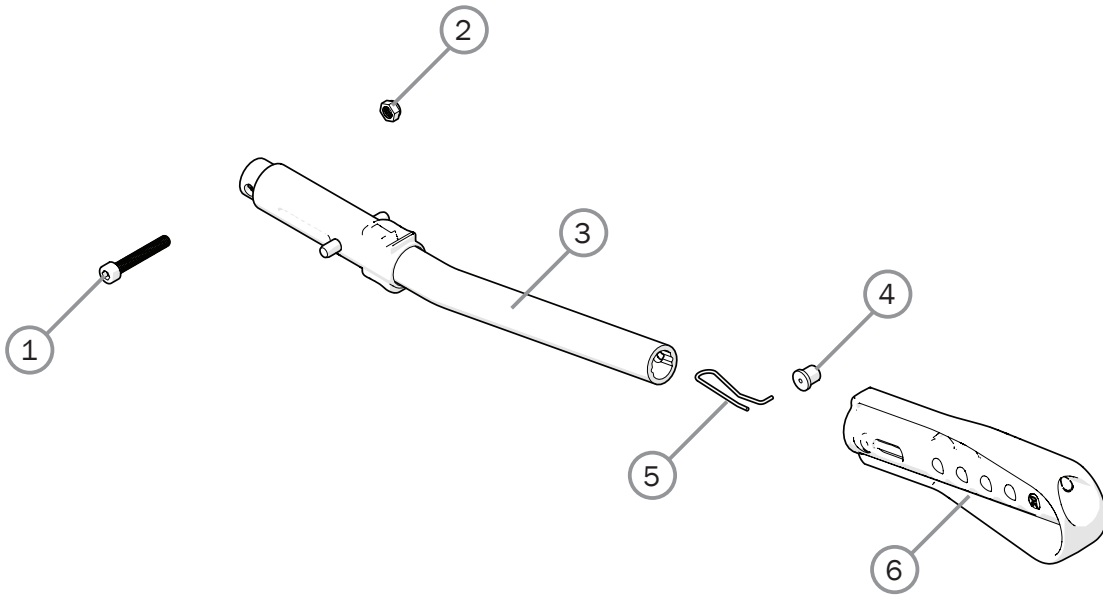

Pos.	Description	Item no.
1-6	Anti-tip multi Cross 5/6 Prio cc 59 cpl	28457
1	Screw M6x 40 MC6S 10.9 zink plated	11229
2	Locking nut M6	12017
3	Anti-tip tube	
4	Pin	
5	Spring	
6	Anti-tip extension multi	51201

Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●


Spoke guard

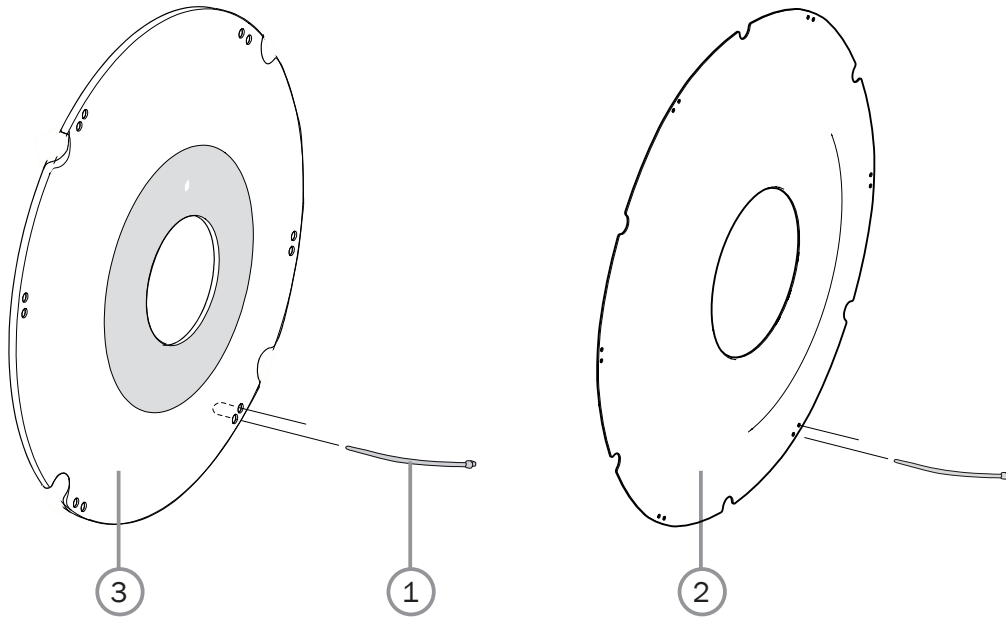

Prio
 Prio AB
 Prio 16"

Pos.	Description	Item no.			
1	Attachment kit spokeguard	26276	●	●	
1-2	Spoke guard 20" cpl C6/Maxx	28934	●	●	
	Spoke guard 22" cpl C6/Maxx	28935	●	●	
	Spoke guard 24" cpl C6/Maxx	28936	●	●	
1, 3	Spoke guard 20" with logo Etac grey	25502-28	●	●	
	Spoke guard 20" AB with logo Etac grey	28816-28	●	●	
	Spoke guard 22" with logo Etac grey	25503-28	●	●	
	Spoke guard 22" with logo Etac yellow	25503-29	●	●	
	Spoke guard 22" AB with logo Etac grey	28817-28	●	●	
	Spoke guard 24" with logo Etac grey	25504-28	●	●	
	Spoke guard 24" with logo Etac yellow	25504-29	●	●	
	Spoke guard 24" AB with logo Etac grey	28818-28	●	●	


Head support

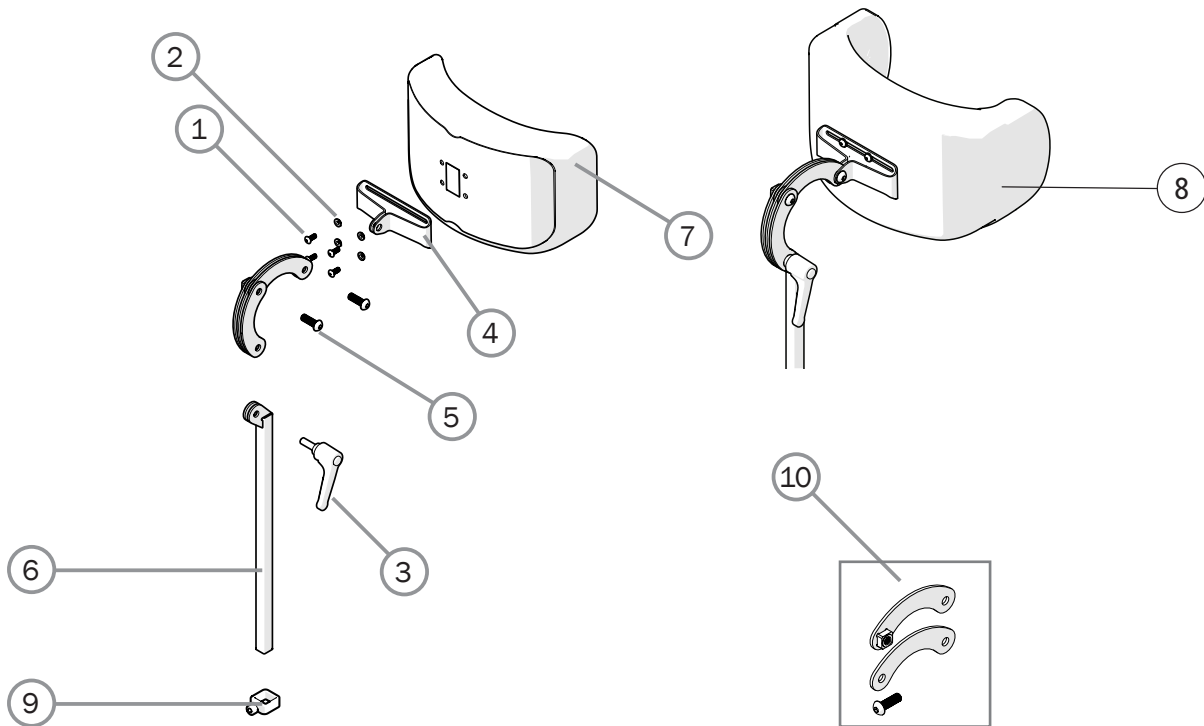

Prio
Prio AB
Prio 16"

Pos.	Description	Item no.			
1	Screw M5x16 MRX	11208	●	●	●
2	Joint bushing	50285	●	●	●
3	Handle M8x25 mm black	80684	●	●	●
4	Attachment head support	83288	●	●	●
5	Screw M8x25	11706	●	●	●
6	Tube Hold Neckrest L 350	28533	●	●	●
1-6	Head support angle adj plush	28534-60	●	●	●
	Head support angle adj hygiene	28534-65	●	●	●
	Head support with side support compl grey hygiene	28536-65	●	●	●
7	Cover head support trad plush	63177-60	●	●	●
	Cover head support trad hygiene	63177-65	●	●	●
8	Cover head support with side support	63185-65	●	●	●
9	Memory function head support	28564	●	●	●
10	Extension head support	29012	●	●	●


Seat cushion

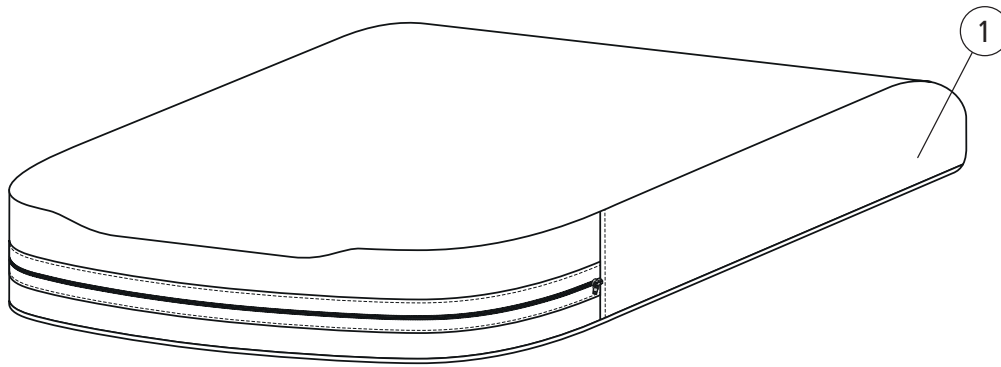

Pos.	Description	Item no.			
1	Seat cushion Prio 40 cm short STD plush	27751-60	●	●	●
	Seat cushion Prio 40 cm short STD hygiene grey	27751-65	●	●	●
	Seat cushion Prio 45 cm short STD plush	27752-60	●	●	●
	Seat cushion Prio 45 cm short STD hygiene grey	27752-65	●	●	●
	Seat cushion Prio 50 cm short STD plush	27753-60	●	●	●
	Seat cushion Prio 50 cm short STD hygiene grey	27753-65	●	●	●
	Seat cushion Prio 55 cm short STD plush	27754-60	●	●	●
	Seat cushion Prio 55 cm short STD hygiene grey	27754-65	●	●	●
	Seat cushion Prio 40 cm short comfort plush	27755-60	●	●	●
	Seat cushion Prio 40 cm short comfort hygiene grey	27755-65	●	●	●
	Seat cushion Prio 45 cm short comfort plush	27756-60	●	●	●
	Seat cushion Prio 45 cm short comfort hygiene grey	27756-65	●	●	●
	Seat cushion Prio 50 cm short comfort plush	27757-60	●	●	●
	Seat cushion Prio 50 cm short comfort hygiene grey	27757-65	●	●	●
	Seat cushion Prio 55 cm short comfort plush	27758-60	●	●	●
	Seat cushion Prio 55 cm short comfort hygiene grey	27758-65	●	●	●
	Seat cushion Prio short hygiene std Tidy 40 cm	28055-65	●	●	●
	Seat cushion Prio short hygiene std Tidy 45 cm	28056-65	●	●	●
	Seat cushion Prio short hygiene std Tidy 50 cm	28057-65	●	●	●
	Seat cushion Prio short hygiene std Tidy 55 cm	28058-65	●	●	●

Prio	Prio AB	Prio 16"
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Seat cushion

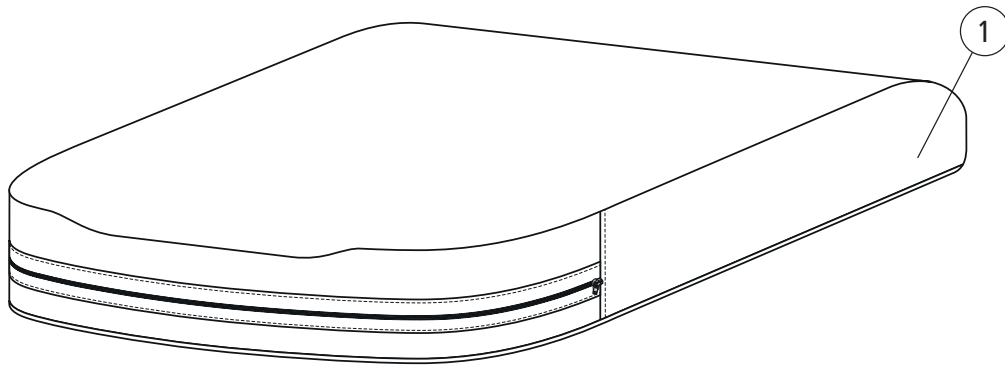


Pos.	Description	Item no.			
	Seat cushion Prio 40 cm long STD plush	27759-60	●	●	●
	Seat cushion Prio 40 cm long STD hygiene grey	27759-65	●	●	●
	Seat cushion Prio 45 cm long STD plush	27760-60	●	●	●
	Seat cushion Prio 45 cm long STD hygiene grey	27760-65	●	●	●
	Seat cushion Prio 50 cm long STD plush	27761-60	●	●	●
	Seat cushion Prio 50 cm long STD hygiene grey	27761-65	●	●	●
	Seat cushion Prio 55 cm long STD plush	27762-60	●	●	●
	Seat cushion Prio 55 cm long STD hygiene grey	27762-65	●	●	●
	Seat cushion Prio 40 cm long comfort plush	27763-60	●	●	●
	Seat cushion Prio 40 cm long comfort hygiene grey	27763-65	●	●	●
	Seat cushion Prio 45 cm long comfort plush	27764-60	●	●	●
	Seat cushion Prio 45 cm long comfort hygiene grey	27764-65	●	●	●
	Seat cushion Prio 50 cm long comfort plush	27765-60	●	●	●
	Seat cushion Prio 50 cm long comfort hygiene grey	27765-65	●	●	●
	Seat cushion Prio 55 cm long comfort plush	27766-60	●	●	●
	Seat cushion Prio 55 cm long comfort hygiene grey	27766-65	●	●	●
	Seat cushion Prio long hygiene std Tidy 40 cm	28059-65	●	●	●
	Seat cushion Prio long hygiene std Tidy 45 cm	28060-65	●	●	●
	Seat cushion Prio long hygiene std Tidy 50 cm	28061-65	●	●	●
	Seat cushion Prio long hygiene std Tidy 55 cm	28062-65	●	●	●

Prio

Prio AB

Prio 16"


Seat cushion - Cover

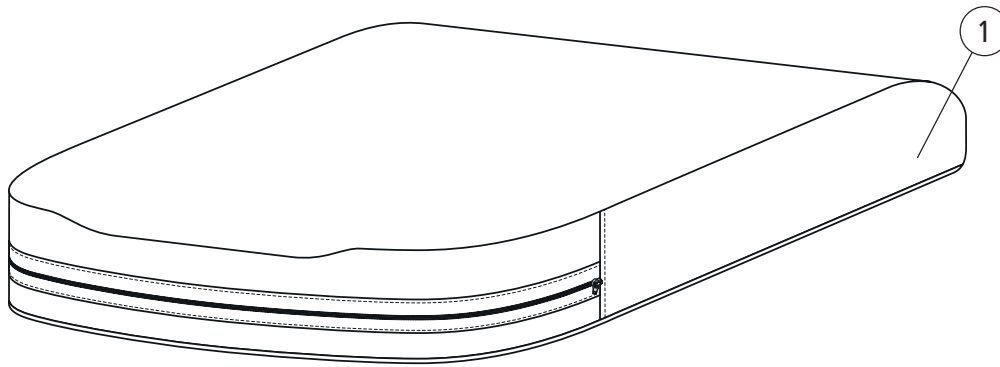

Pos.	Description	Item no.			
1	Cover seat cushion 40 (16) short plush Prio	63060-60	●	●	●
	Cover seat cushion 40 (16) short hygiene grey Prio	63060-65	●	●	●
	Cover seat cushion 45 (18) short plush Prio	63061-60	●	●	●
	Cover seat cushion 45 (18) short hygiene grey Prio	63061-65	●	●	●
	Cover seat cushion 50 (20) short plush Prio	63062-60	●	●	●
	Cover seat cushion 50 (20) short hygiene grey Prio	63062-65	●	●	●
	Cover seat cushion 55 (22) short plush Prio	63063-60	●	●	●
	Cover seat cushion 55 (22) short hygiene grey Prio	63063-65	●	●	●
	Cover seat cushion 40 (16) long plush Prio	63064-60	●	●	●
	Cover seat cushion 40 (16) long hygiene grey Prio	63064-65	●	●	●
	Cover seat cushion 45 (18) long plush Prio	63065-60	●	●	●
	Cover seat cushion 45 (18) long hygiene grey Prio	63065-65	●	●	●
	Cover seat cushion 50 (20) long plush Prio	63066-60	●	●	●
	Cover seat cushion 50 (20) long hygiene grey Prio	63066-65	●	●	●
	Cover seat cushion 55 (22) long plush Prio	63067-60	●	●	●
	Cover seat cushion 55 (22) long hygiene grey Prio	63067-65	●	●	●
	Cover seat 40 short Tidy	63159-65	●	●	●
	Cover seat 45 short Tidy	63160-65	●	●	●
	Cover seat 50 short Tidy	63161-65	●	●	●
	Cover seat 55 short Tidy	63162-65	●	●	●
Cover seat 40 long Tidy	63163-65	●	●	●	
Cover seat 45 long Tidy	63164-65	●	●	●	
Cover seat 50 long Tidy	63165-65	●	●	●	
Cover seat 55 long Tidy	63166-65	●	●	●	


Prio
Prio AB
Prio 16"

Back support cushion

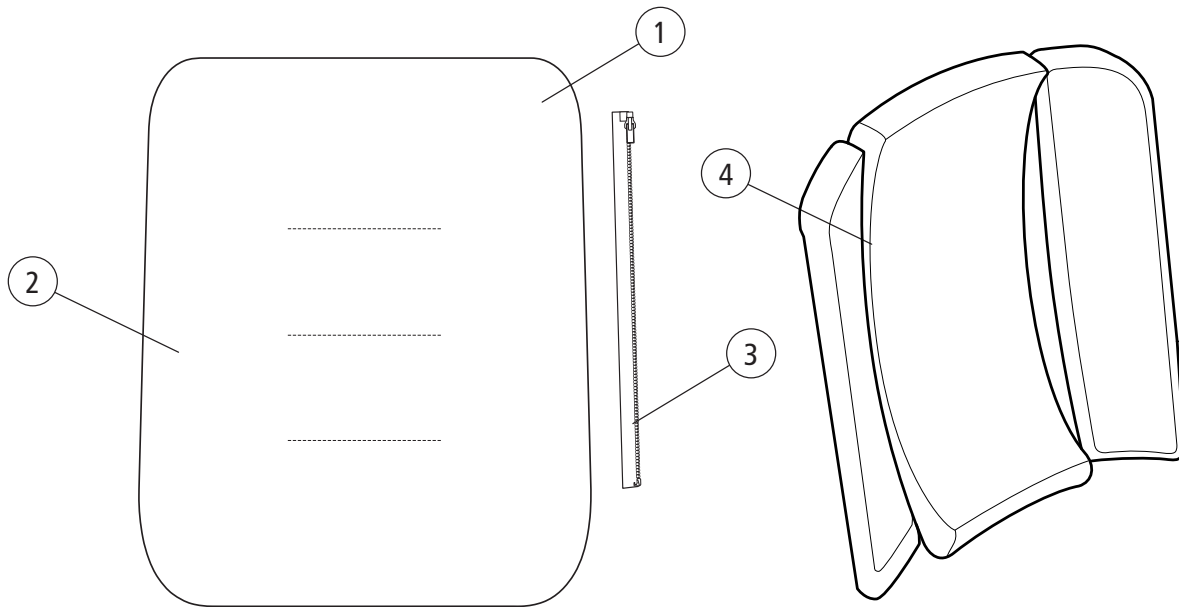

Pos.	Description	Item no.			
			Prio	Prio AB	Prio 16"
1	Back support cushion 40 cm plush Prio excl zipper	28410-60	●	●	●
	Back support cushion 40 cm hygiene Prio grey excl zipper	28410-65	●	●	●
	Back support cushion 45 cm plush Prio excl zipper	28411-60	●	●	●
	Back support cushion 45 cm hygiene Prio grey excl zipper	28411-65	●	●	●
	Back support cushion 50 cm plush Prio excl zipper	28412-60	●	●	●
	Back support cushion 50 cm hygiene Prio grey excl zipper	28412-65	●	●	●
	Back support cushion 55 cm plush Prio excl zipper	28413-60	●	●	●
	Back support cushion 55 cm hygiene Prio grey excl zipper	28413-65	●	●	●
2	Cover back support plush Prio 40 cm grey excl zipper	63230-60	●	●	●
	Cover back support hygiene Prio 40 cm grey excl zipper	63230-65	●	●	●
	Cover back support plush Prio 45 cm grey excl zipper	63231-60	●	●	●
	Cover back support hygiene Prio 45 cm grey excl zipper	63231-65	●	●	●
	Cover back support plush Prio 50 cm grey excl zipper	63232-60	●	●	●
	Cover back support hygiene Prio 50 cm grey excl zipper	63232-65	●	●	●
	Cover back support plush Prio 55 cm grey excl zipper	63233-60	●	●	●
	Cover back support hygiene Prio 55 cm grey excl zipper	63233-65	●	●	●
3	Velcro attachment back Prio	83188	●	●	●
4	Back support cushion B40 3D plush	63343-60	●	●	●
	Back support cushion B45 3D plush	63344-60	●	●	●
	Back support cushion B50 3D plush	63345-60	●	●	●
	Back support cushion B55 3D plush	63346-60	●	●	●

Back support cover

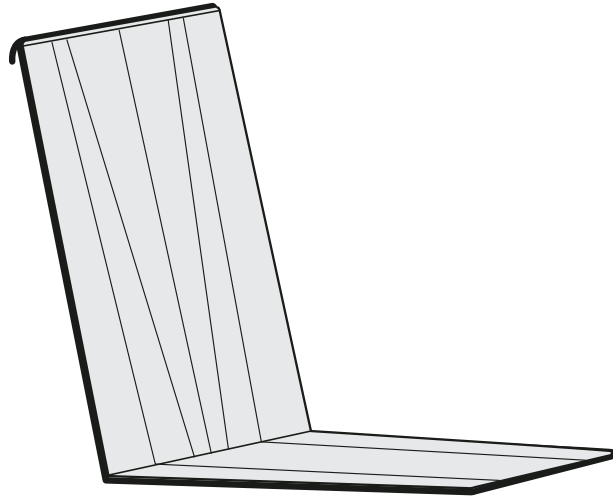

Pos.	Description	Item no.
1	Cover back support W40 plush	63339-60
	Cover back support W45 plush	63340-60
	Cover back support W50 plush	63341-60
	Cover back support W55 plush	63342-60


Prio

Prio AB

Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●

Extension back support upholstery

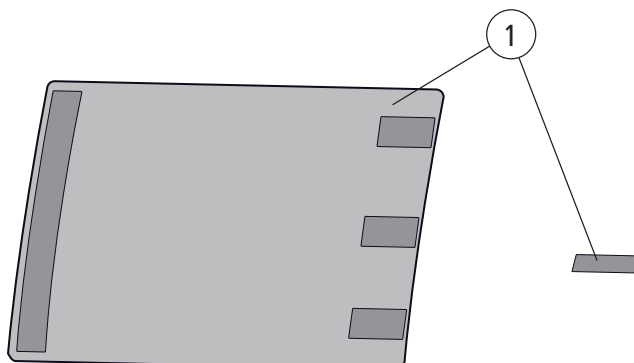

Pos.	Description	Item. No.
1	Extension back support upholstery	28657



Prio

Prio AB

Prio 16"



Back support upholstery

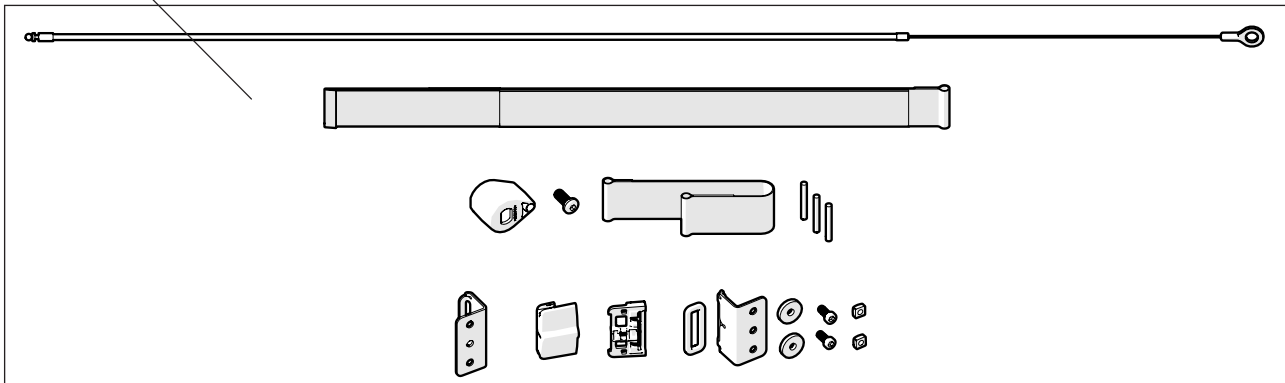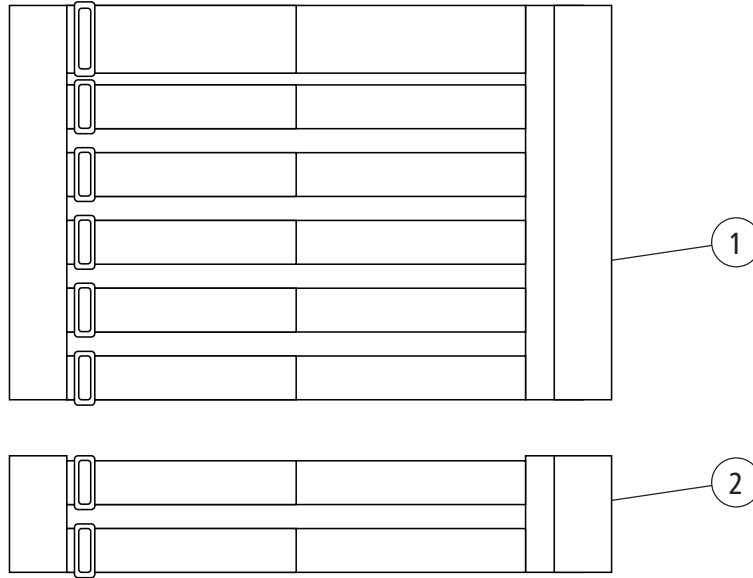

Pos.	Description	Item no.
1	Upholstery back 3A rb 40-45 cm	62954-60
	Upholstery back 3A rb 50-55 cm	62955-60
2	Upholstery back 3A rb lower 40-45 cm	63337-60
	Upholstery back 3A rb lower 50-55 cm	63338-60
3	Dynamic pelvic support Prio 40 cm	28704
	Dynamic pelvic support Prio 45 cm	28705
	Dynamic pelvic support Prio 50/55 cm	28999


Prio

Prio AB

Prio 16"

●	●	●
●	●	●
●	●	●
●	●	●
●	●	●

Back angle control

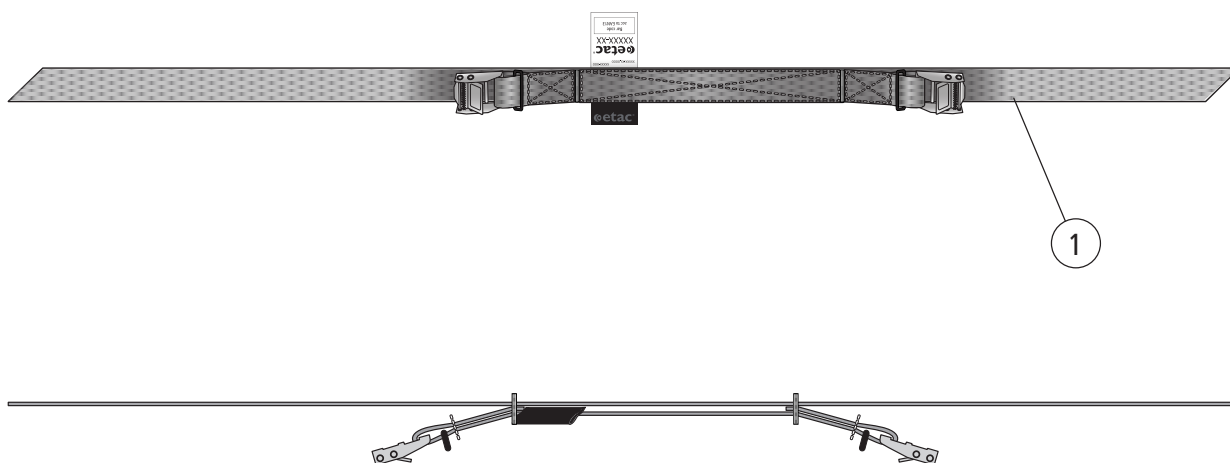

Pos.	Description	Item no.
1	Back angle control cpl	28573


Prio

Prio AB

Prio 1G"



User operated self-tilt

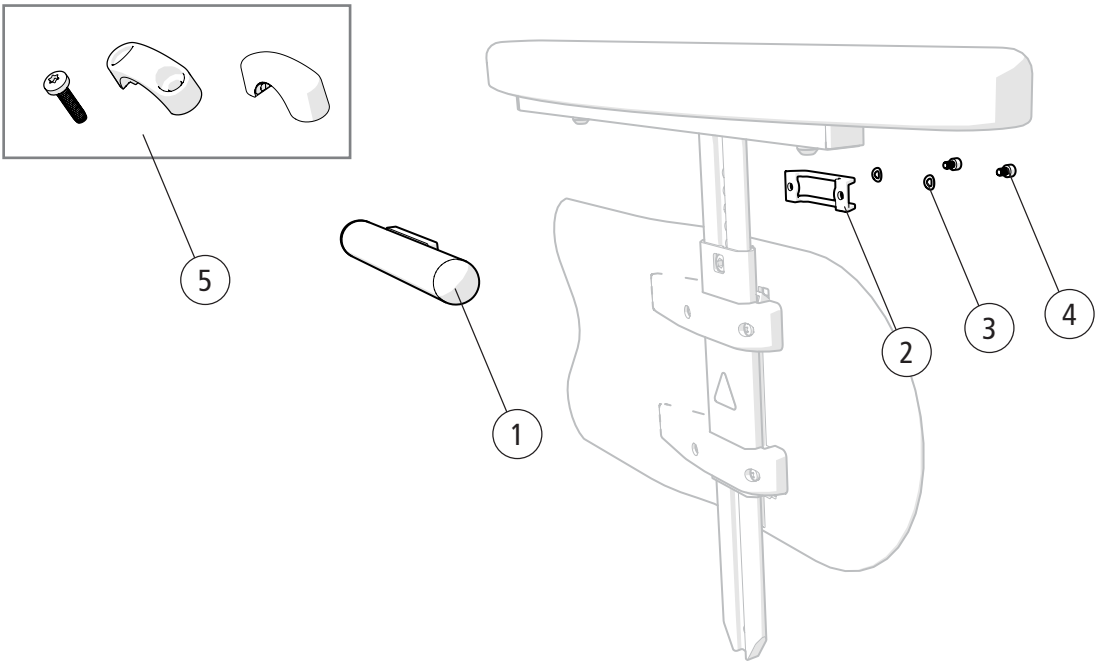

Pos.	Description	Item no.
1-4	User operated self-tilt	28770
5	Clamp self-tilt	28178

Prio

Prio AB

Prio 16"

