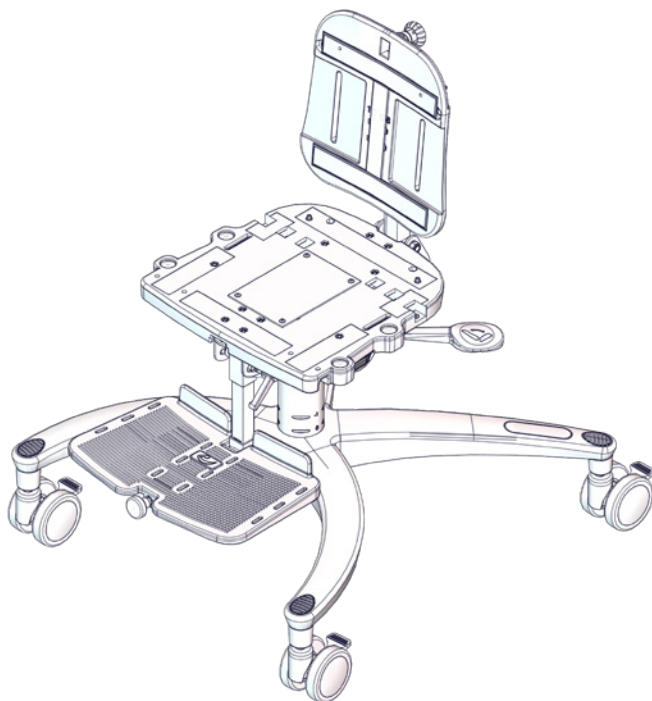
# User manual

## Wombat Solo


**R82**<sup>®</sup>  
by Etac


M1160 - 2025-04-08 - Rev. 8.0

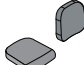
[www.etac.com](http://www.etac.com)

	4		34-38
	5-7		39-41
	8	ENGLISH	42-45
	9-10	DEUTSCH	46-50
	11-14	NEDERLANDS	51-55
	15	DANSK	56-59
	16-17	NORSK	60-63
	18	SVENSKA	64-67
	19-20	SUOMI	68-71
	21-22	ESPAÑOL	72-76
	23	FRANÇAIS	77-81
	24-26	ITALIANO	82-85
	27-28	PORTUGUÊS	86-90
	29	PORTUGUÊS DO BRASIL	91-94
	30	РОССИЯ	95-99
	31	POLSKI	100-104
	31	ČESKY	105-108
mm (inch)	32-33	中國	109-112
kg (lb)		日本	113-117
		ΕΛΛΗΝΙΚΗ	118-122
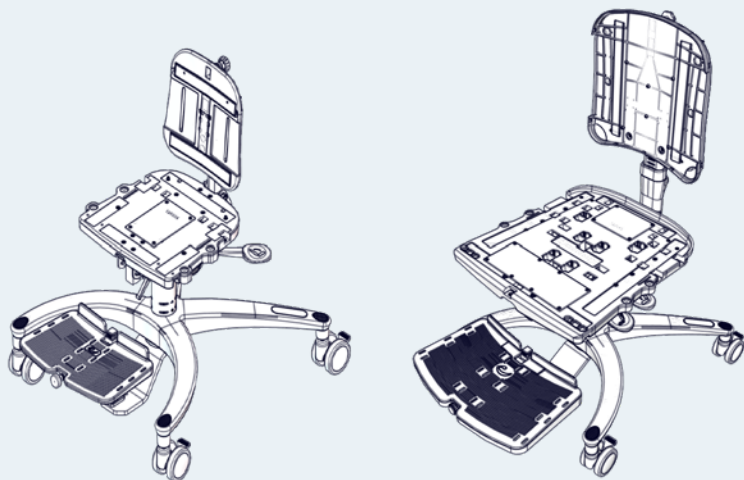
### WOMBAT SOLO 1+2

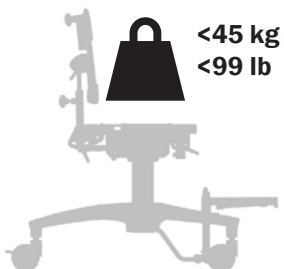
### WOMBAT SOLO 3


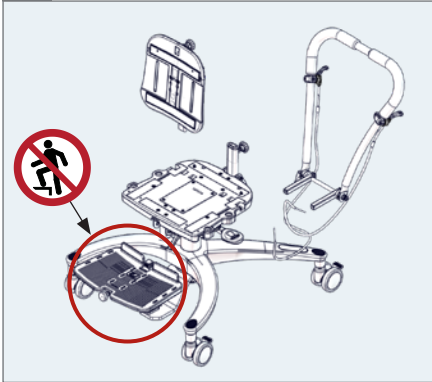
Size 1 & 2

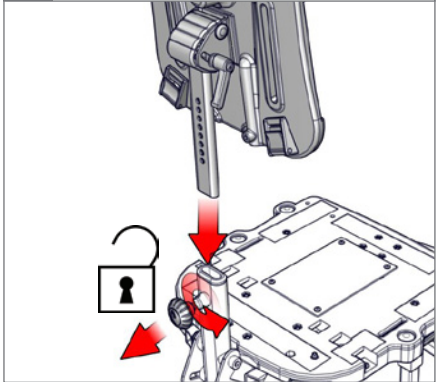
1.

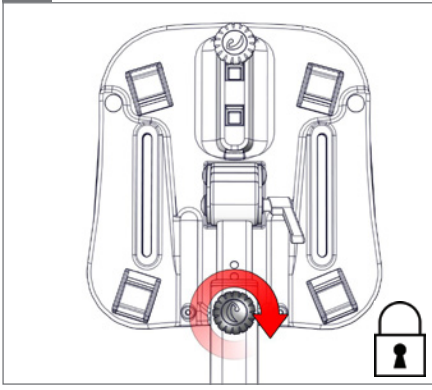2.

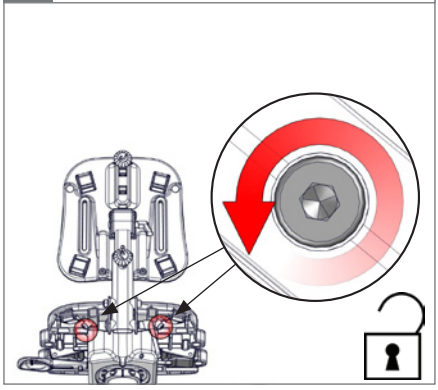
3.


4.

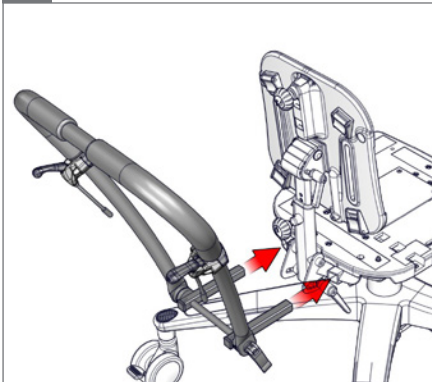○ 5 mm












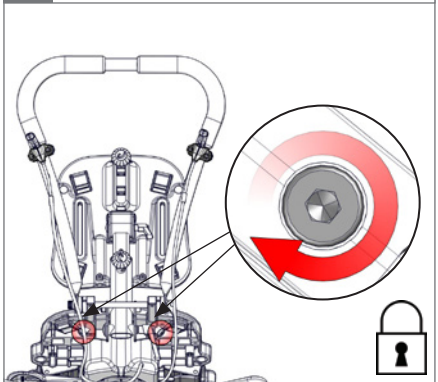
5.










Size 3

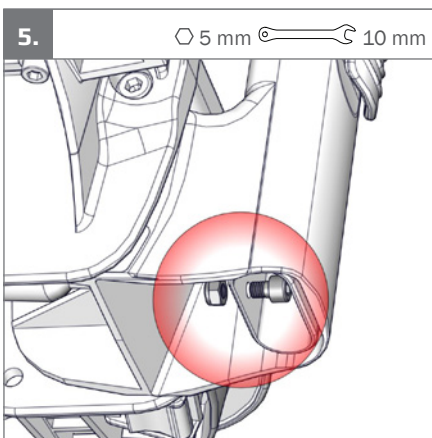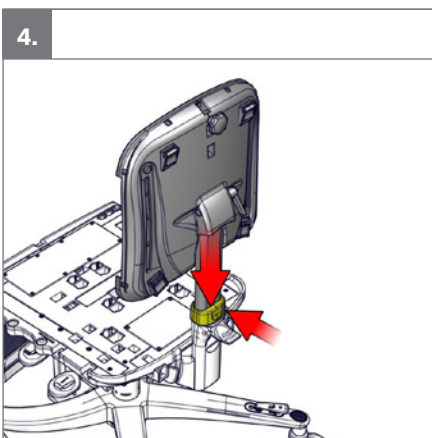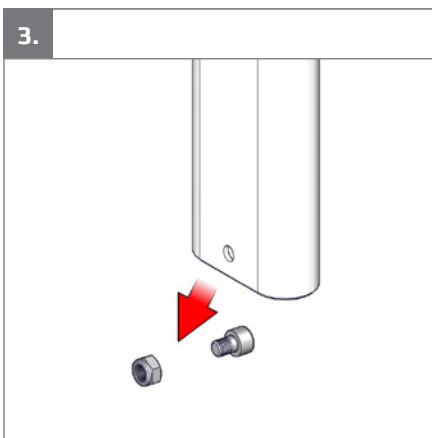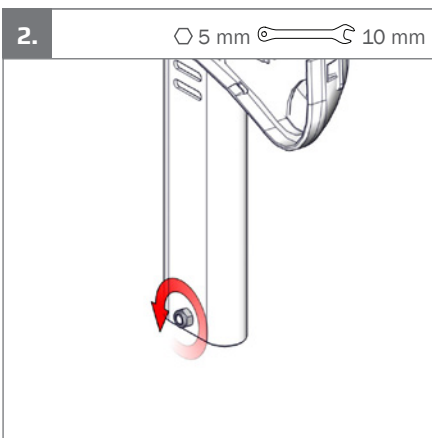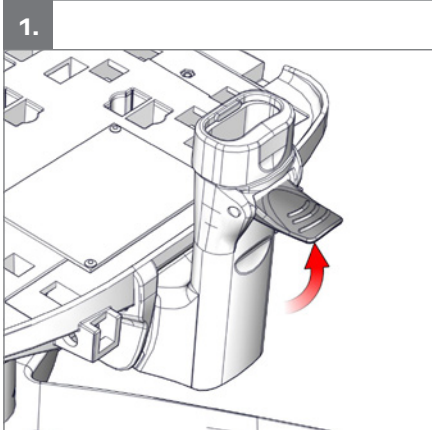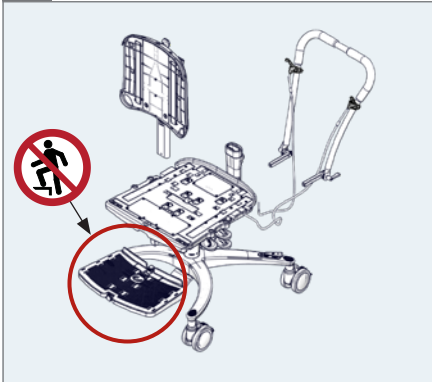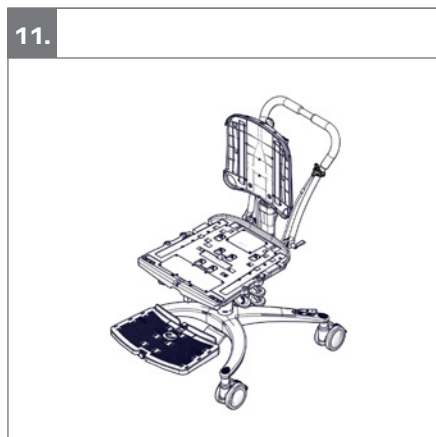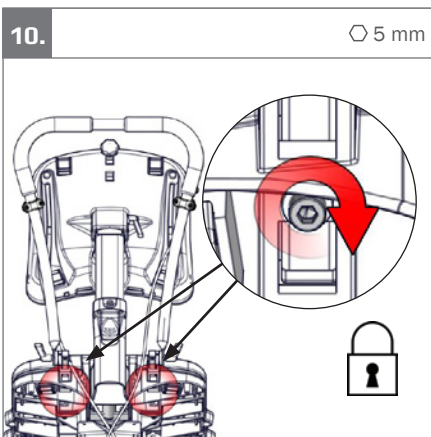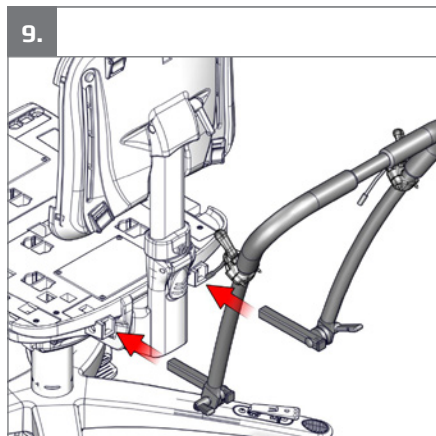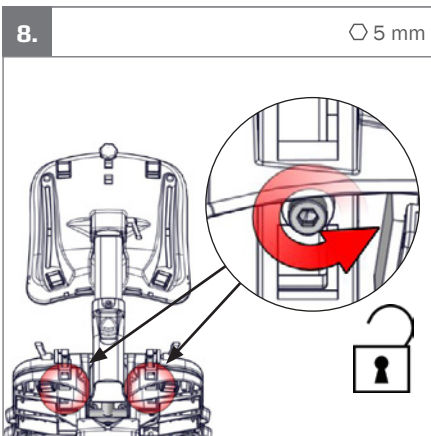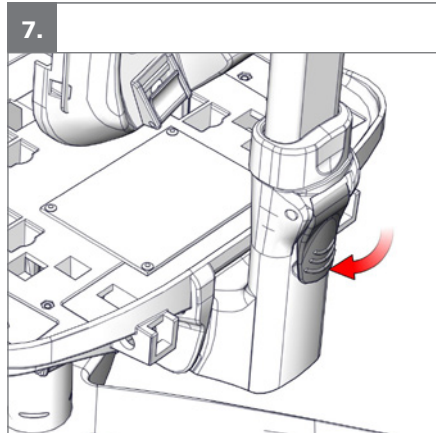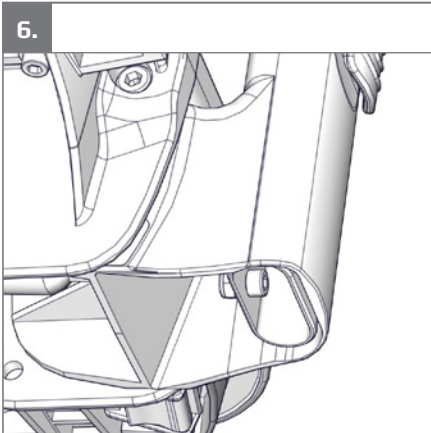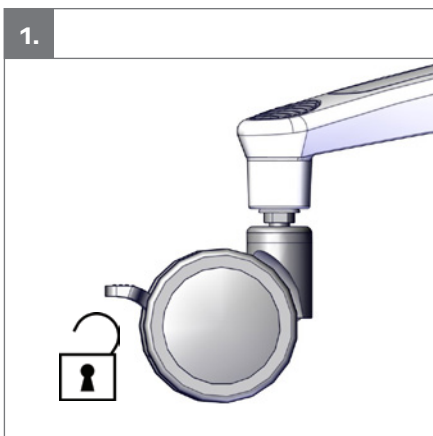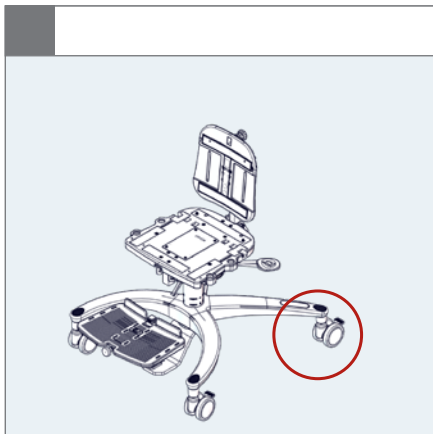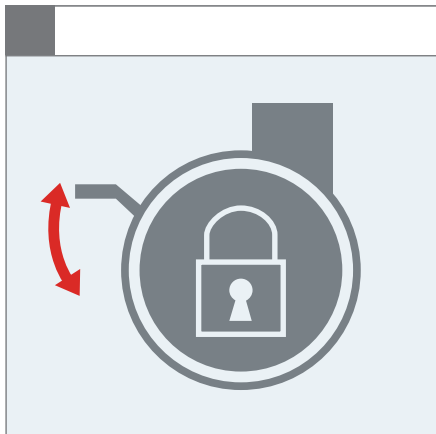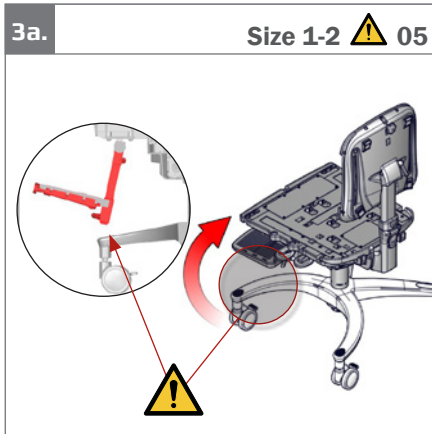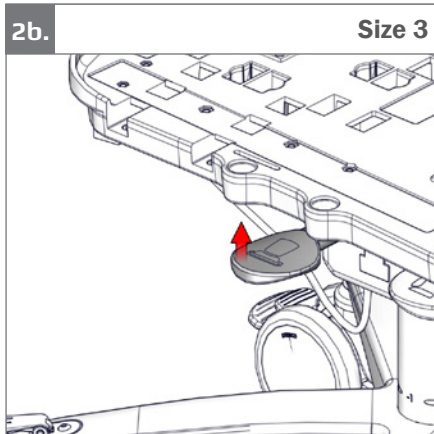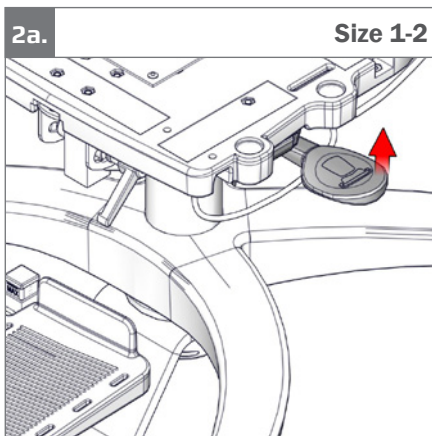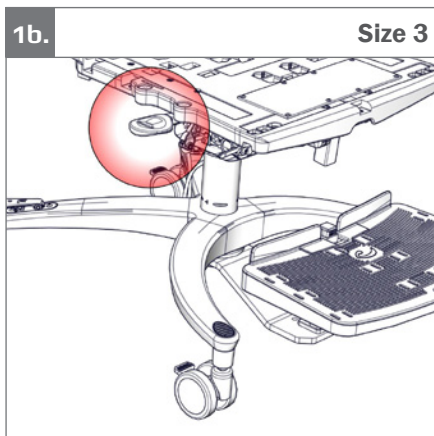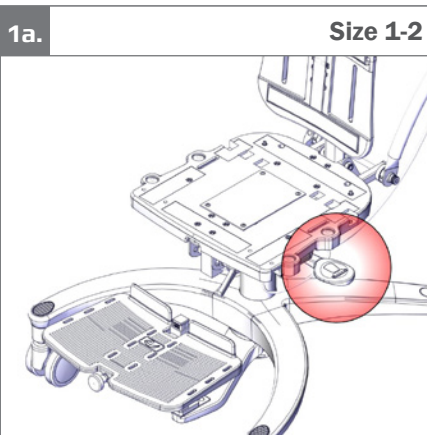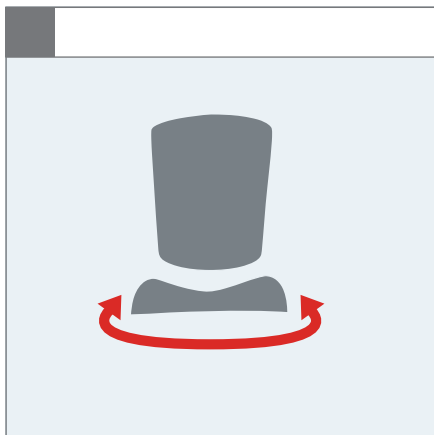
3b. **Size 3** ⚠️ 05

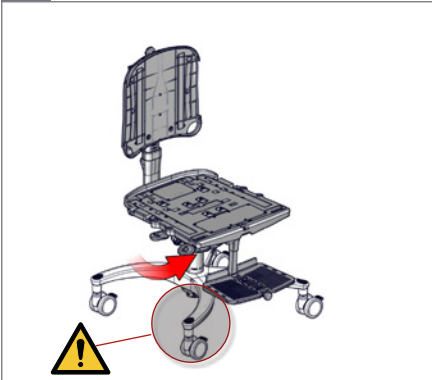
4a.


4b.








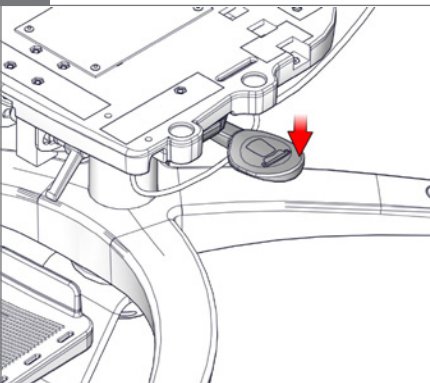
5a. **Size 1-2**

5b. **Size 3**

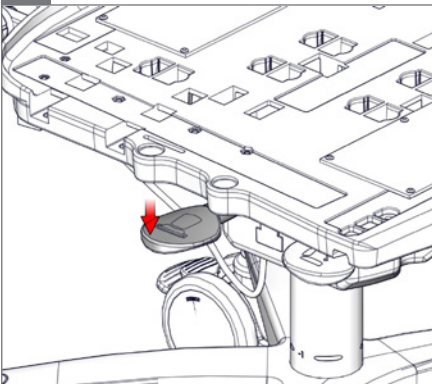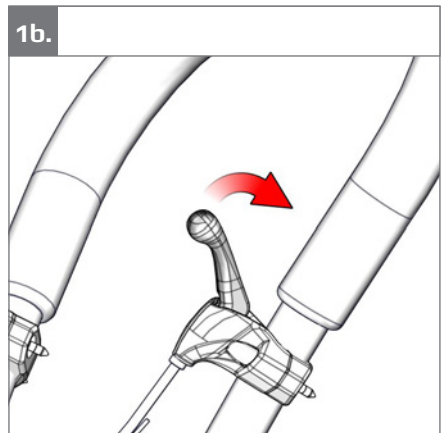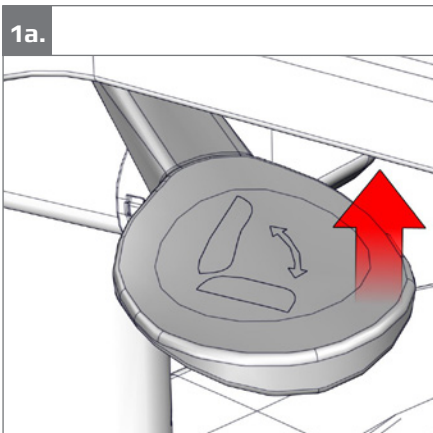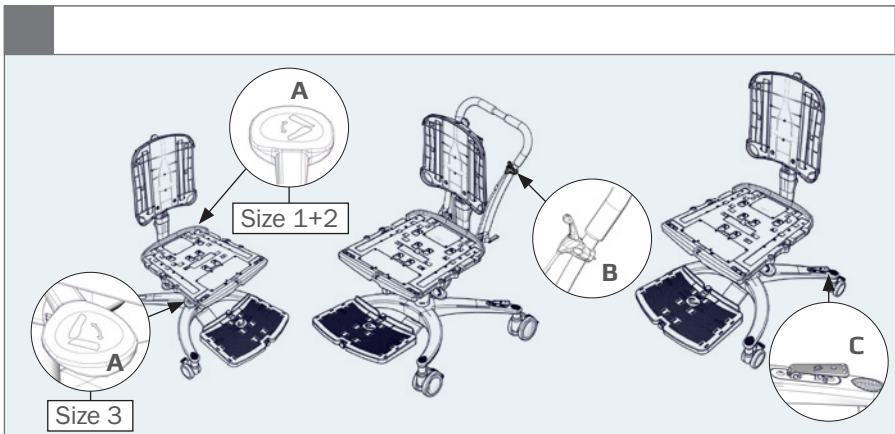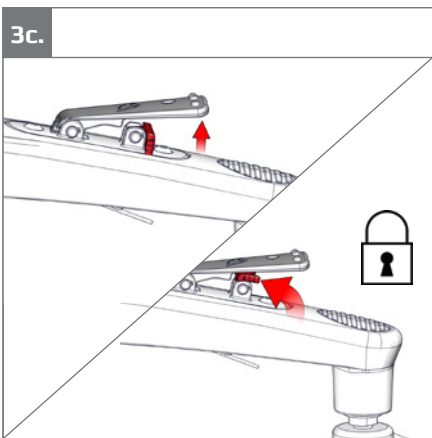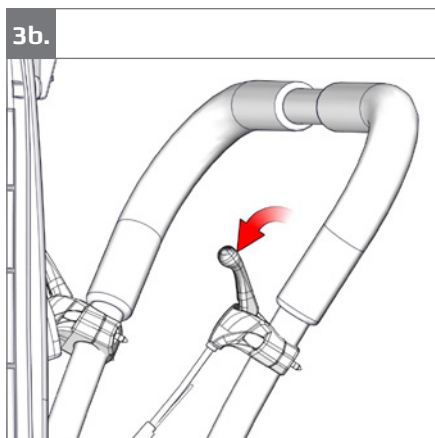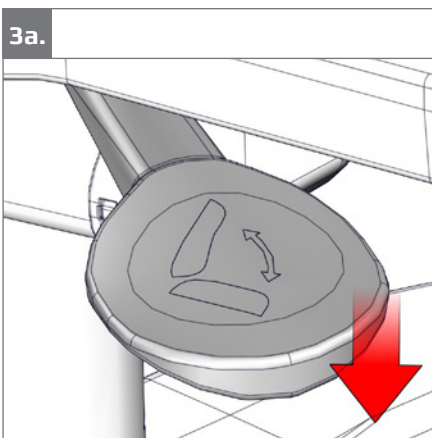
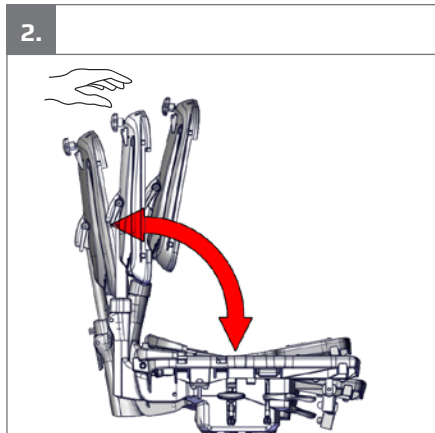
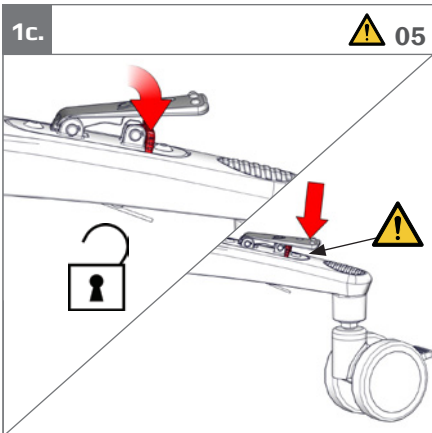
### Size 1-2

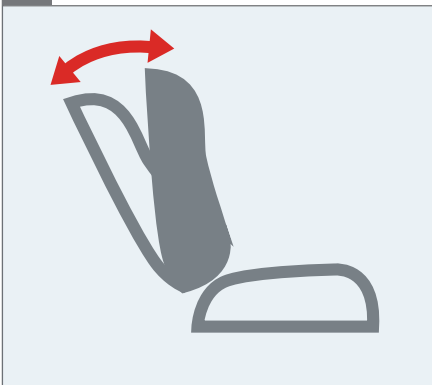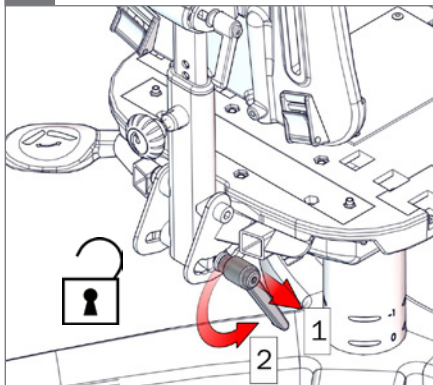
1.

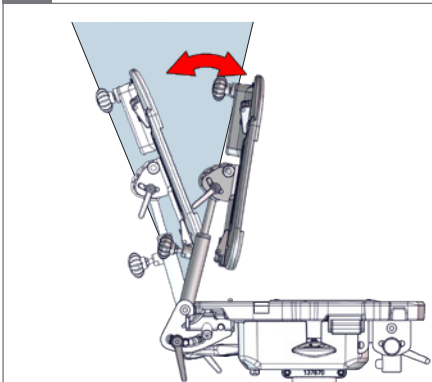
2.

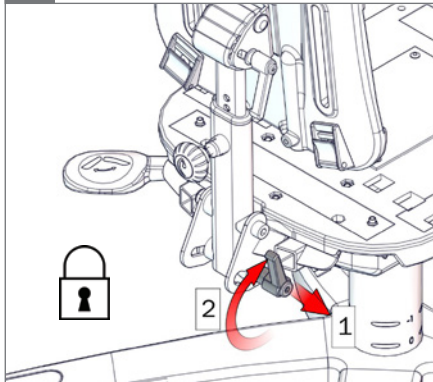
3.

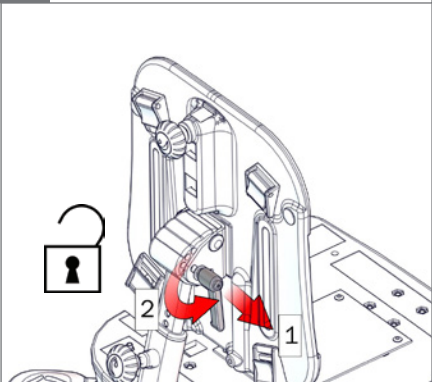
4.

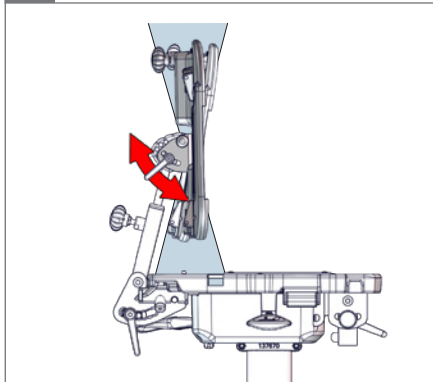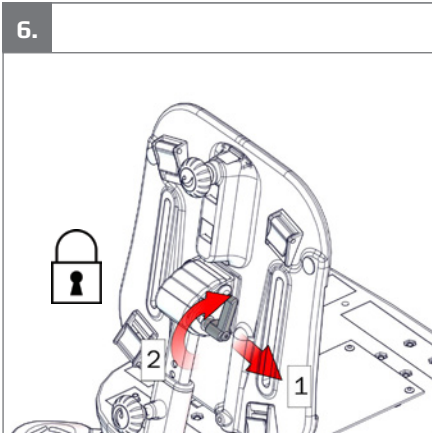
5.



Size 3

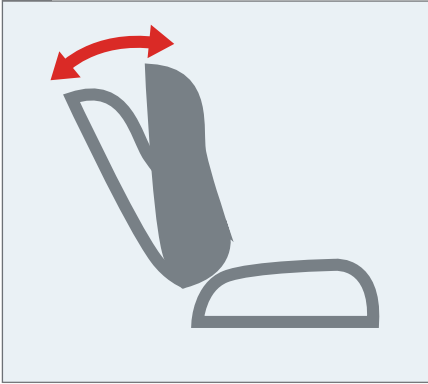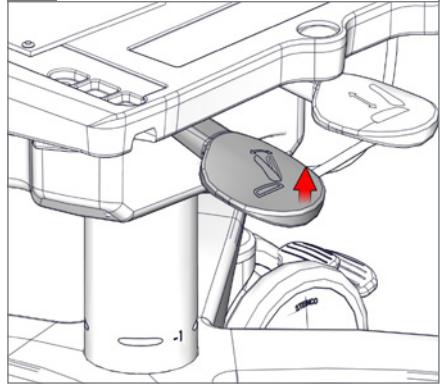
1.

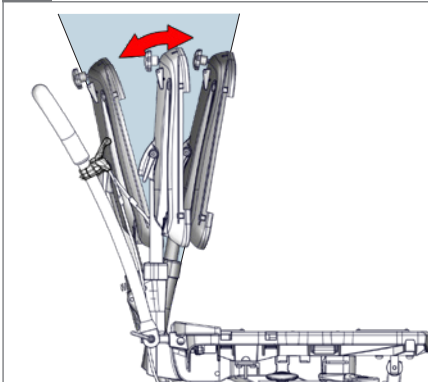
2.

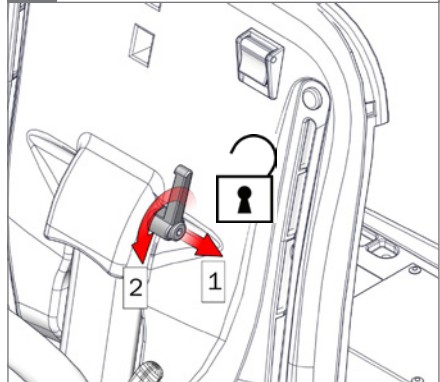
3.




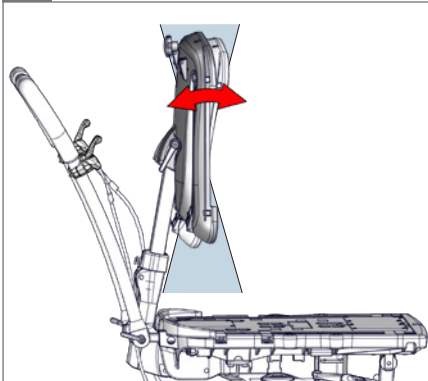
4.

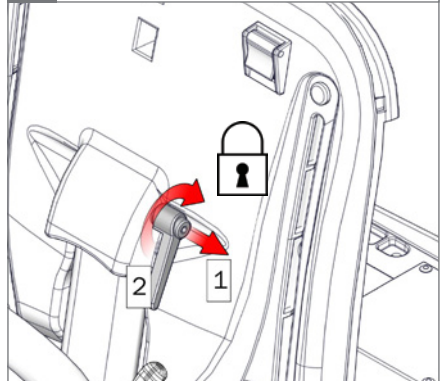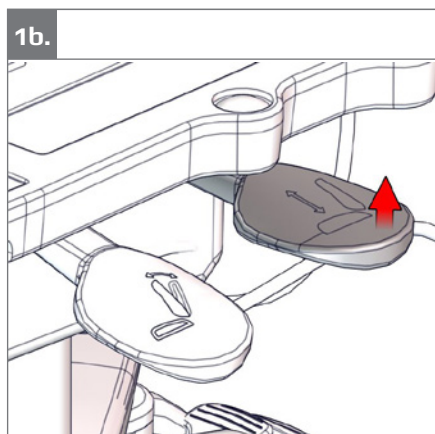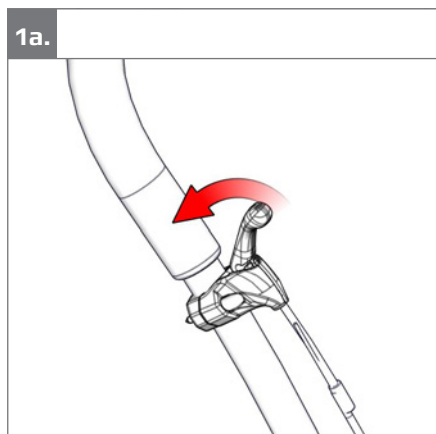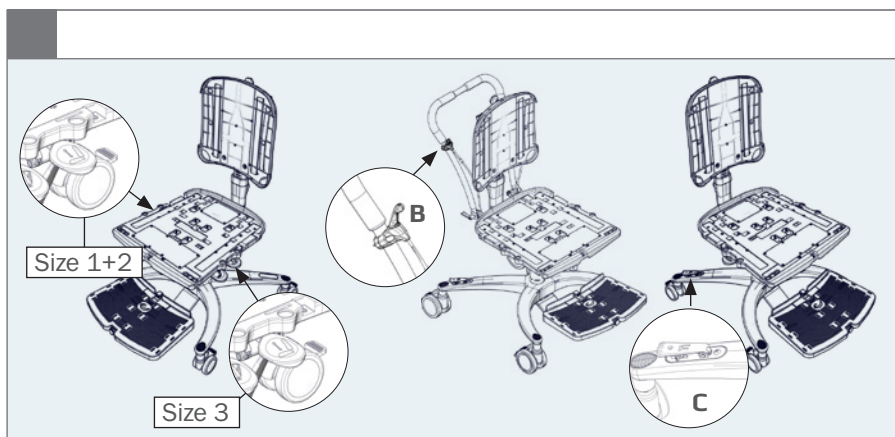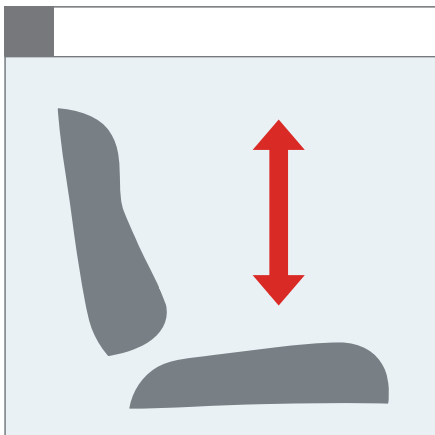
5.



1c.

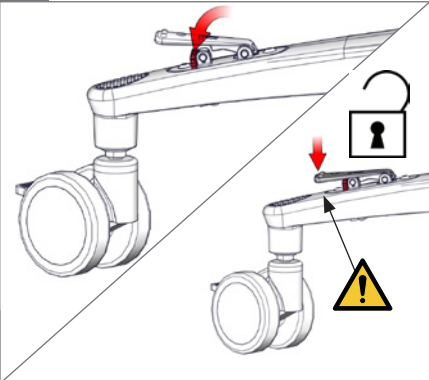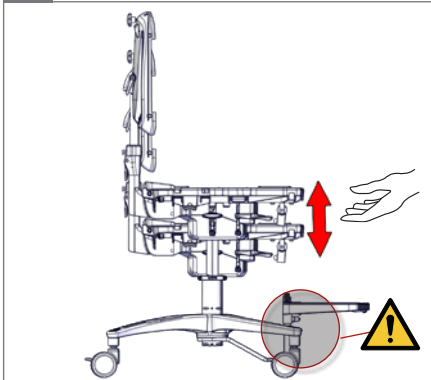
2.


3a.










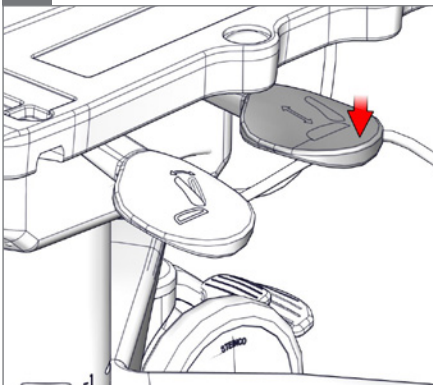
3b.







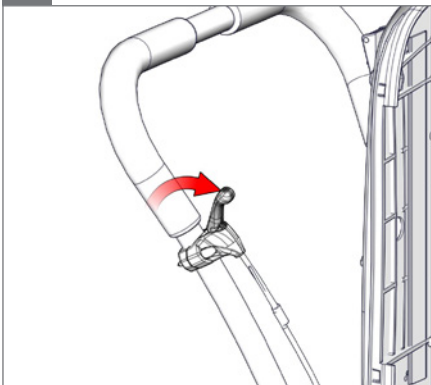
3c.

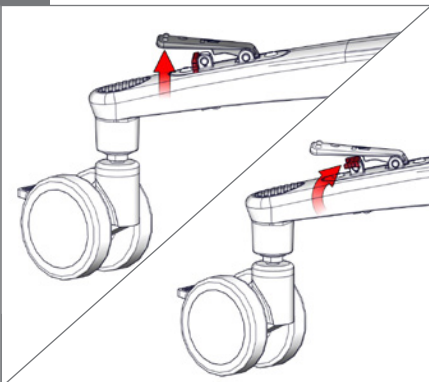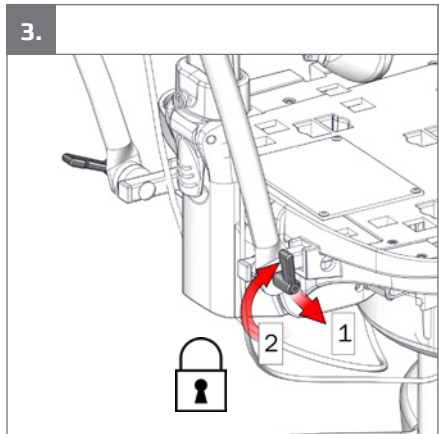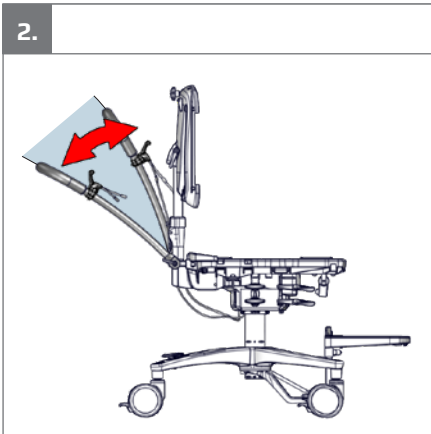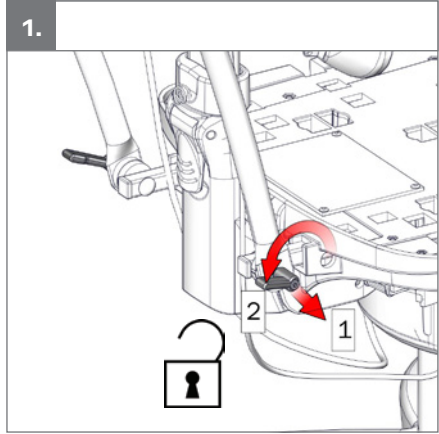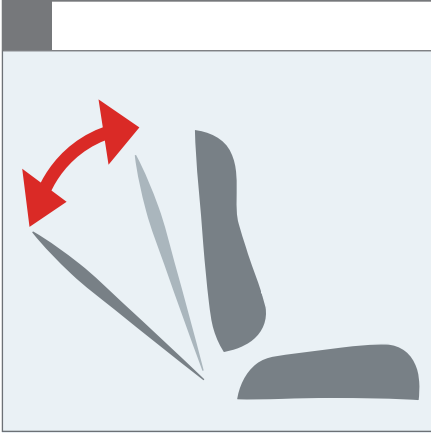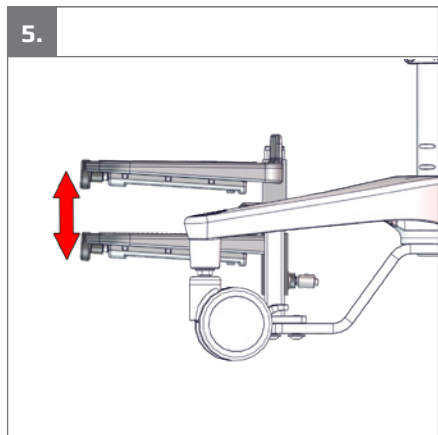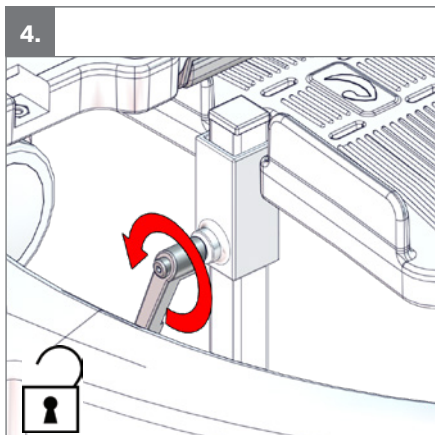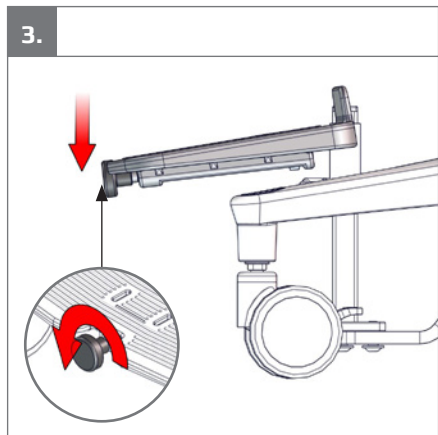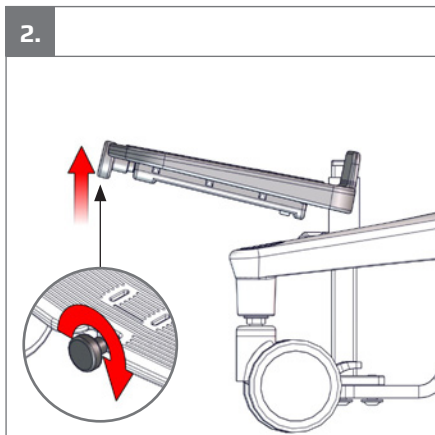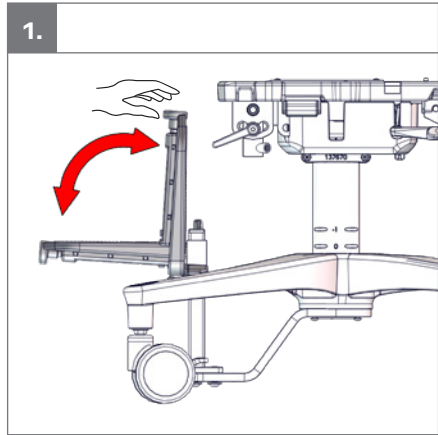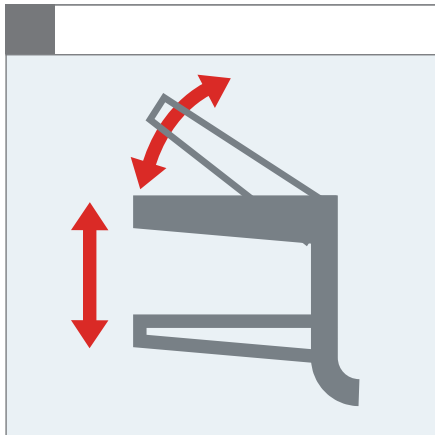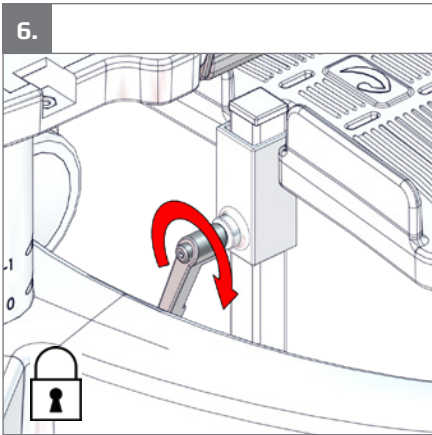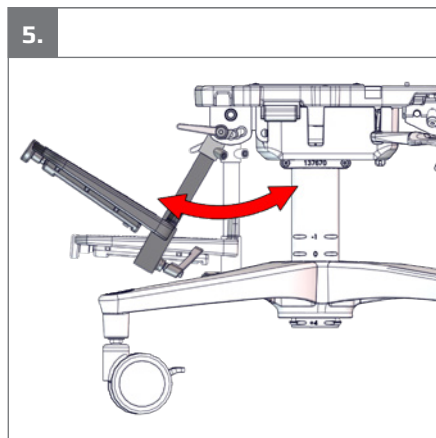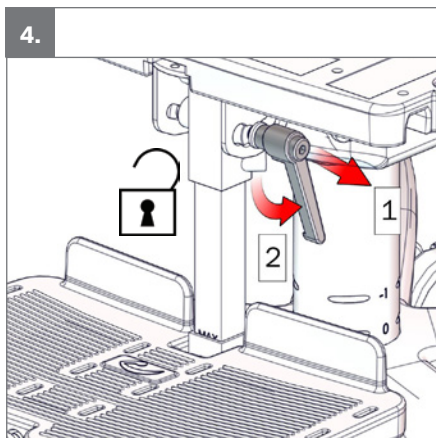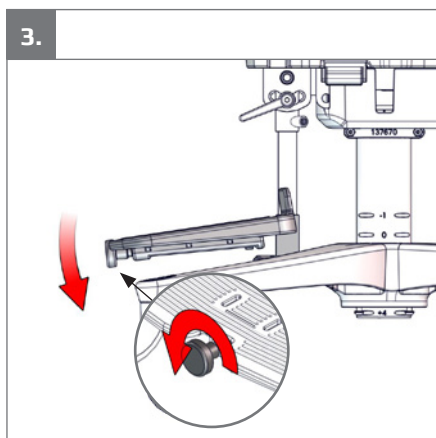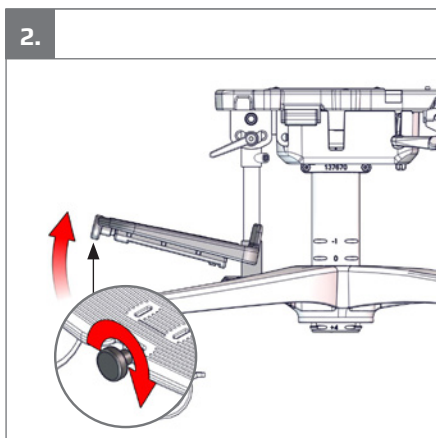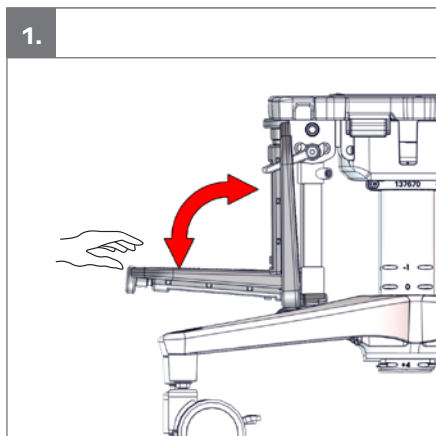
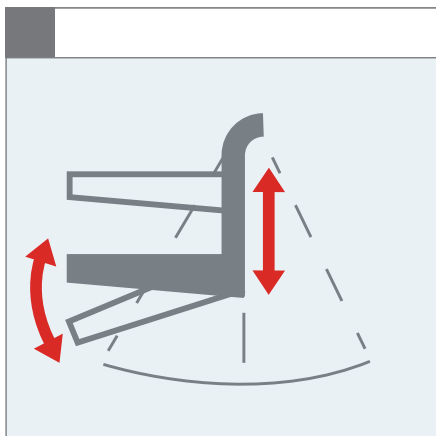
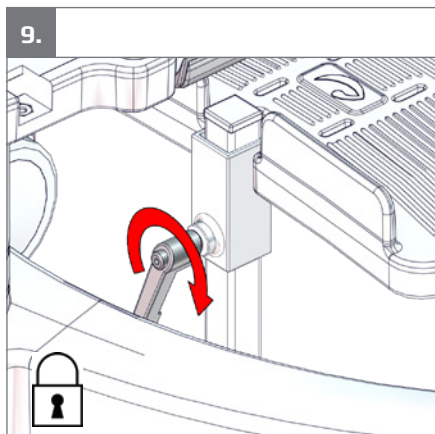
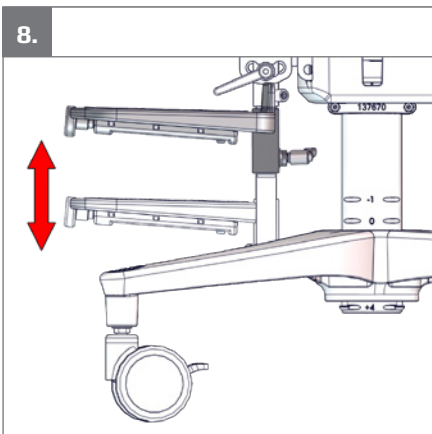
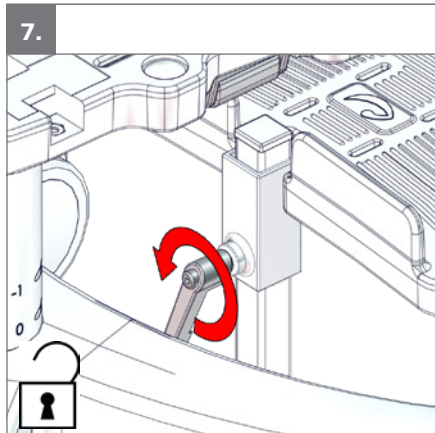
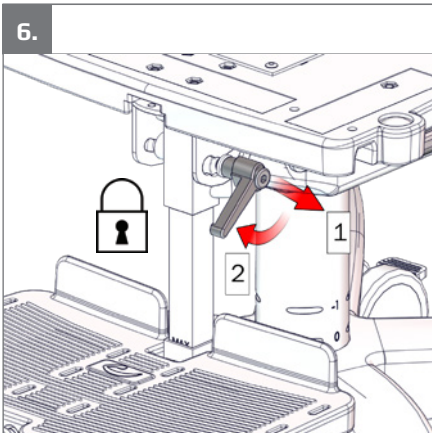
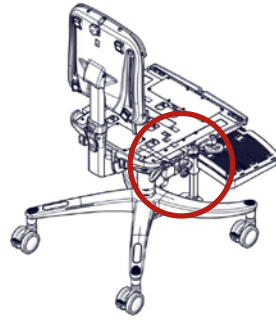
1.


2a.

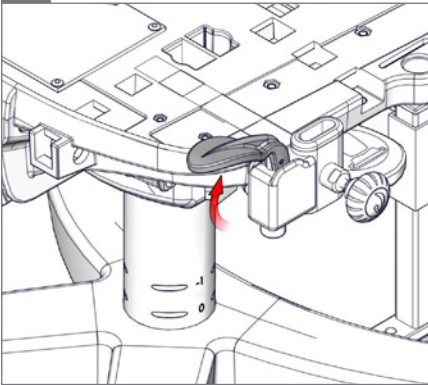
2b.

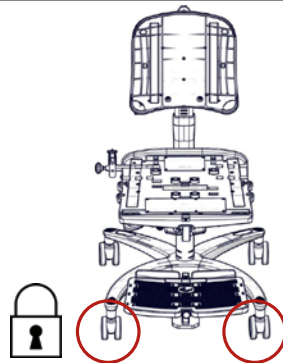
3a.


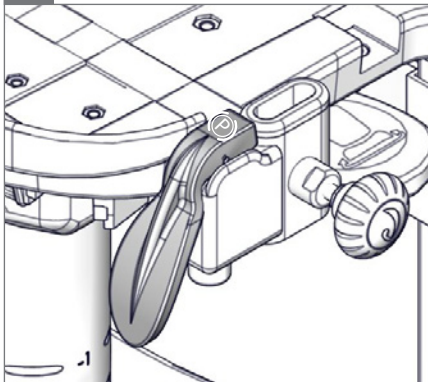
3b.

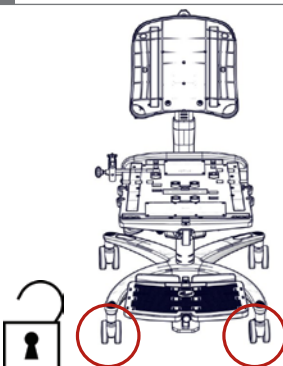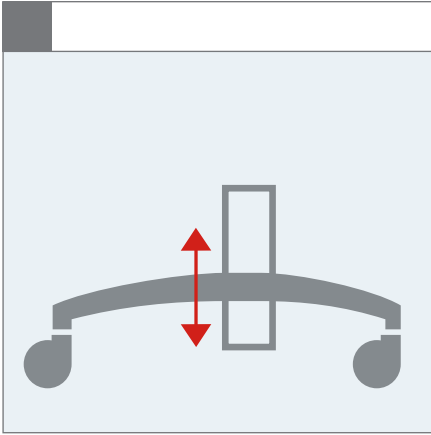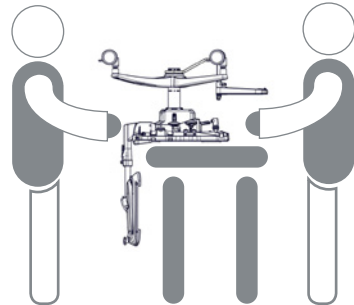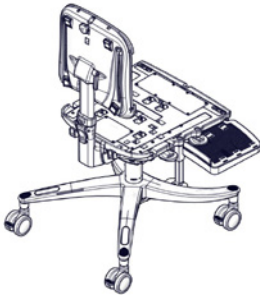
1.


2.

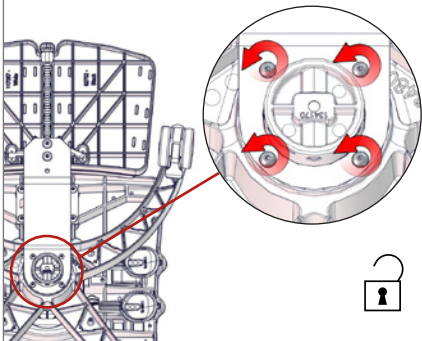○ 3 mm


3.

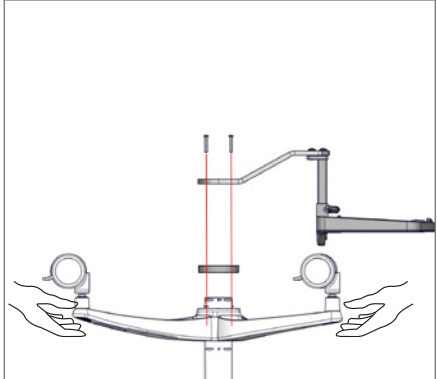○ 3 mm


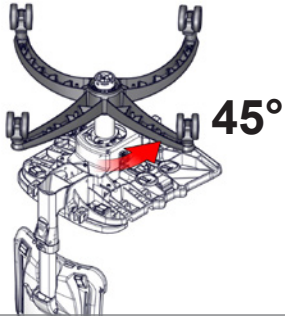
4.

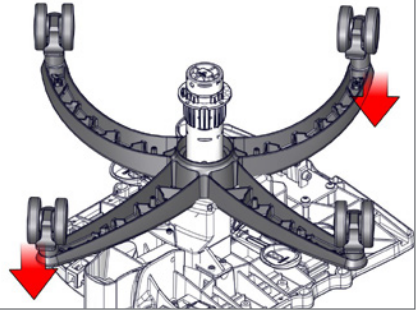
5.

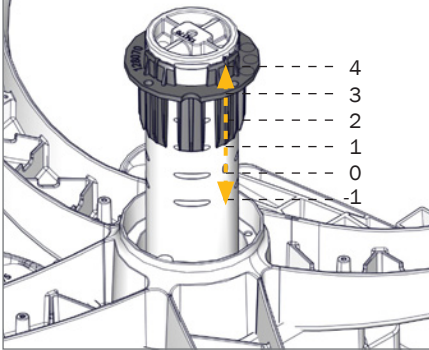
6.

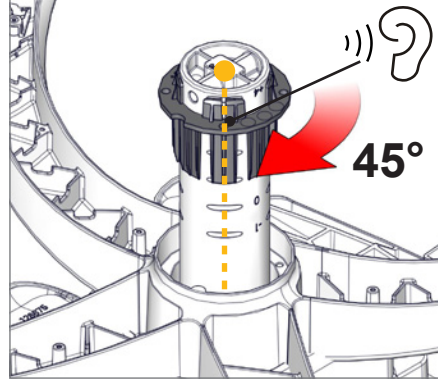7.

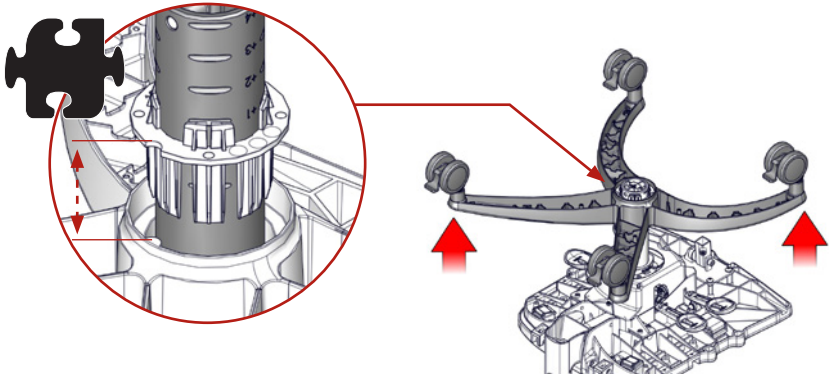
8.

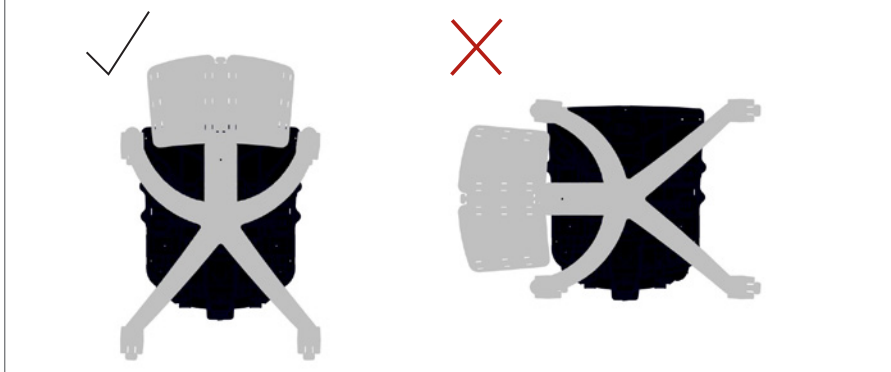
9.


10.

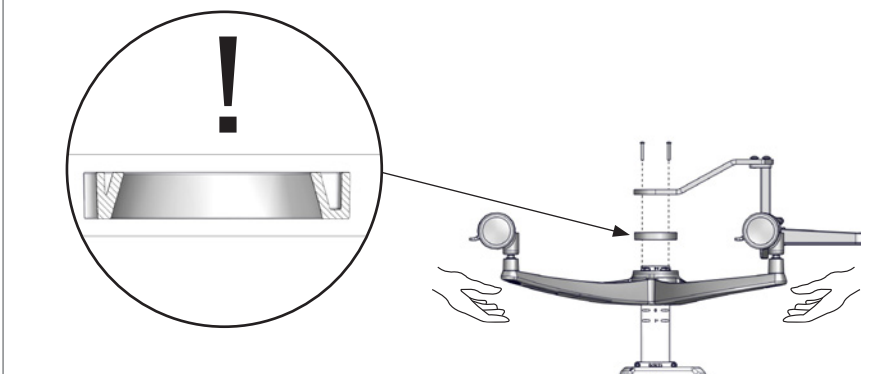⊕ 3 mm







11.

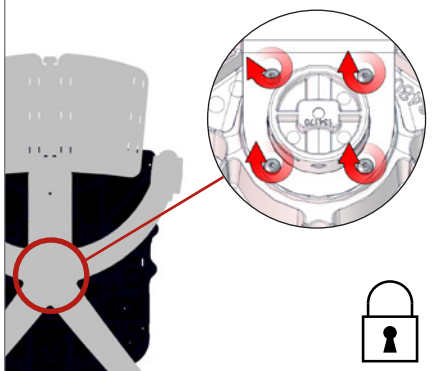⊕ 3 mm

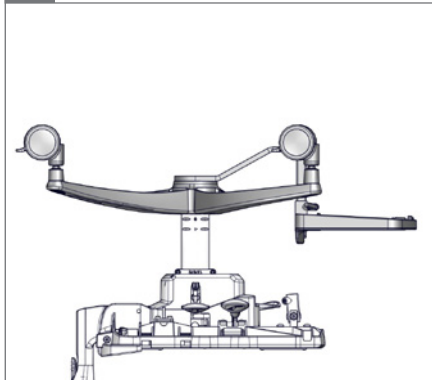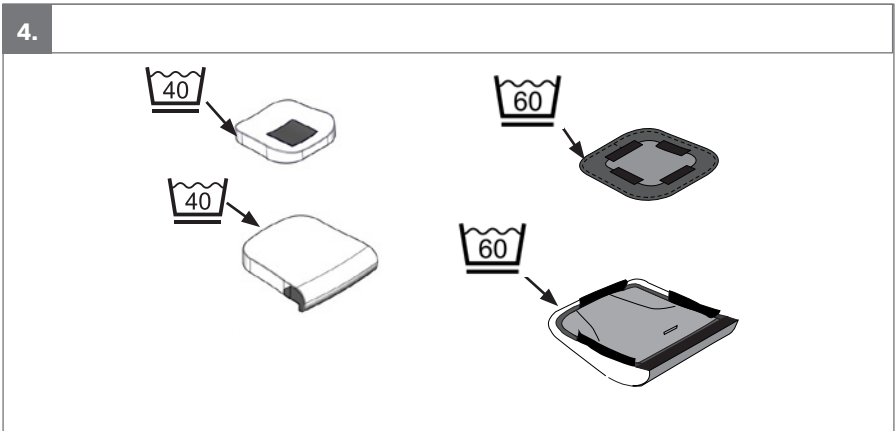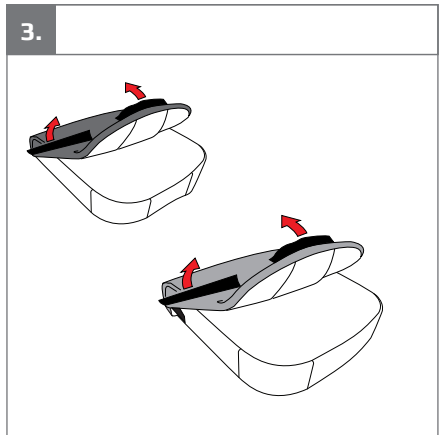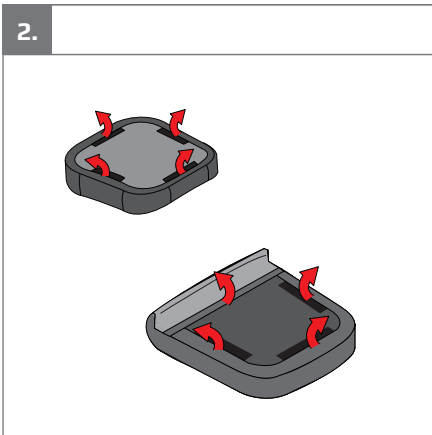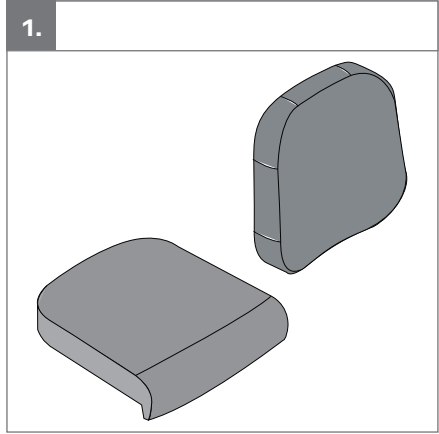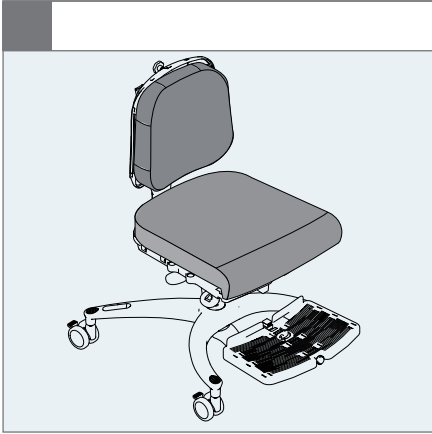
12.


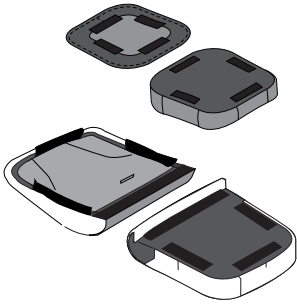
5.

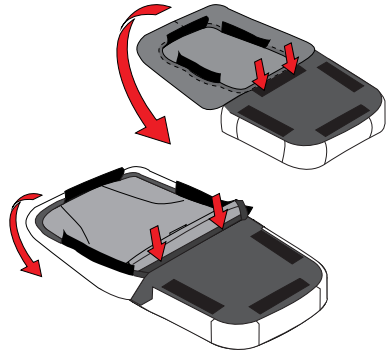
6.

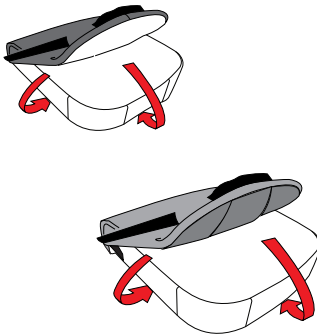
7.

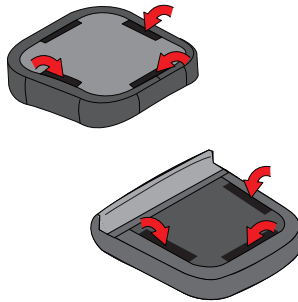
8.


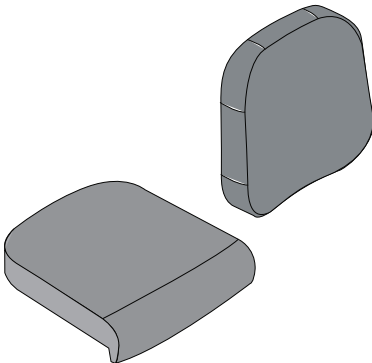
9.

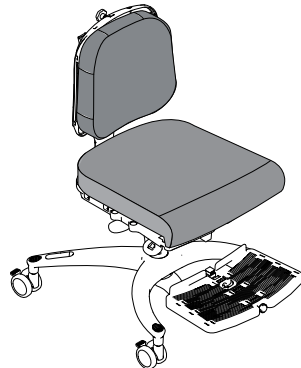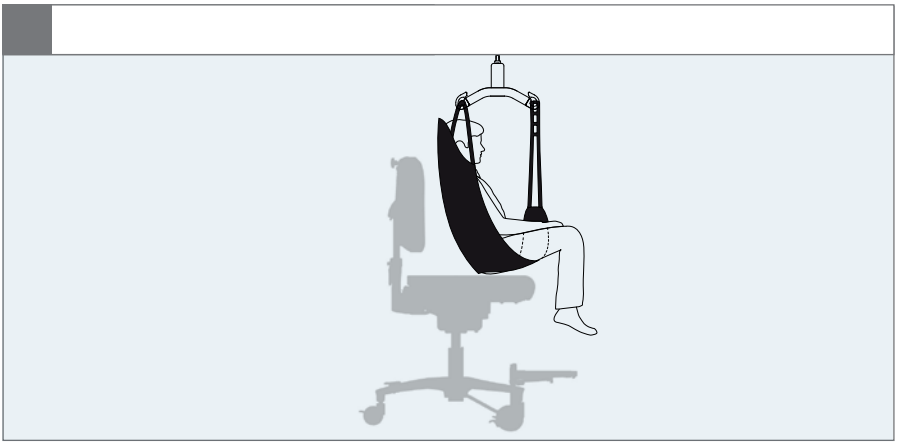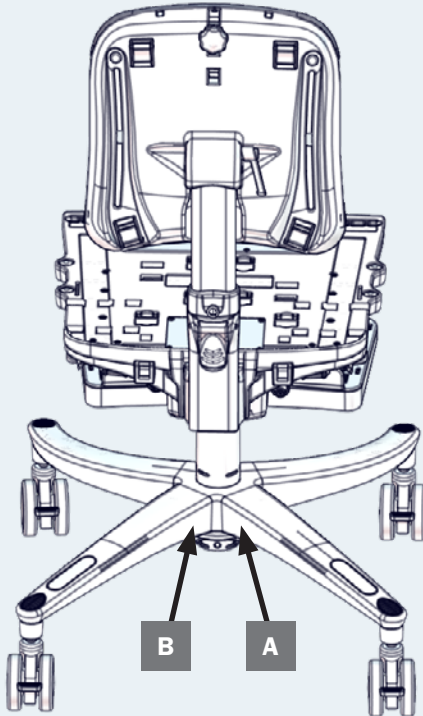10.

**A**

**R82**<sup>®</sup>  
 by Etac

R82 A/S, Parallevej 3  
 DK-8751 Gedved  
 etac.com  
 Made in xxxxxxx

MD

CE

UK

YYYY-MM-DD

**B**

Posting group  
 Product name, size x

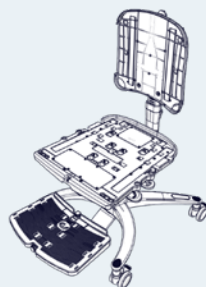
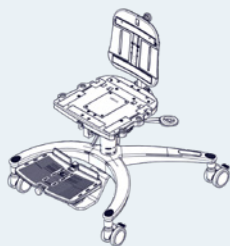
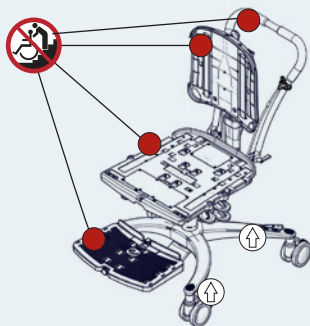
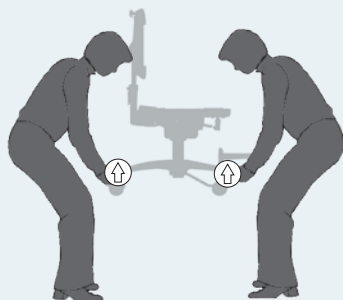
DMR ver.: xxxxx

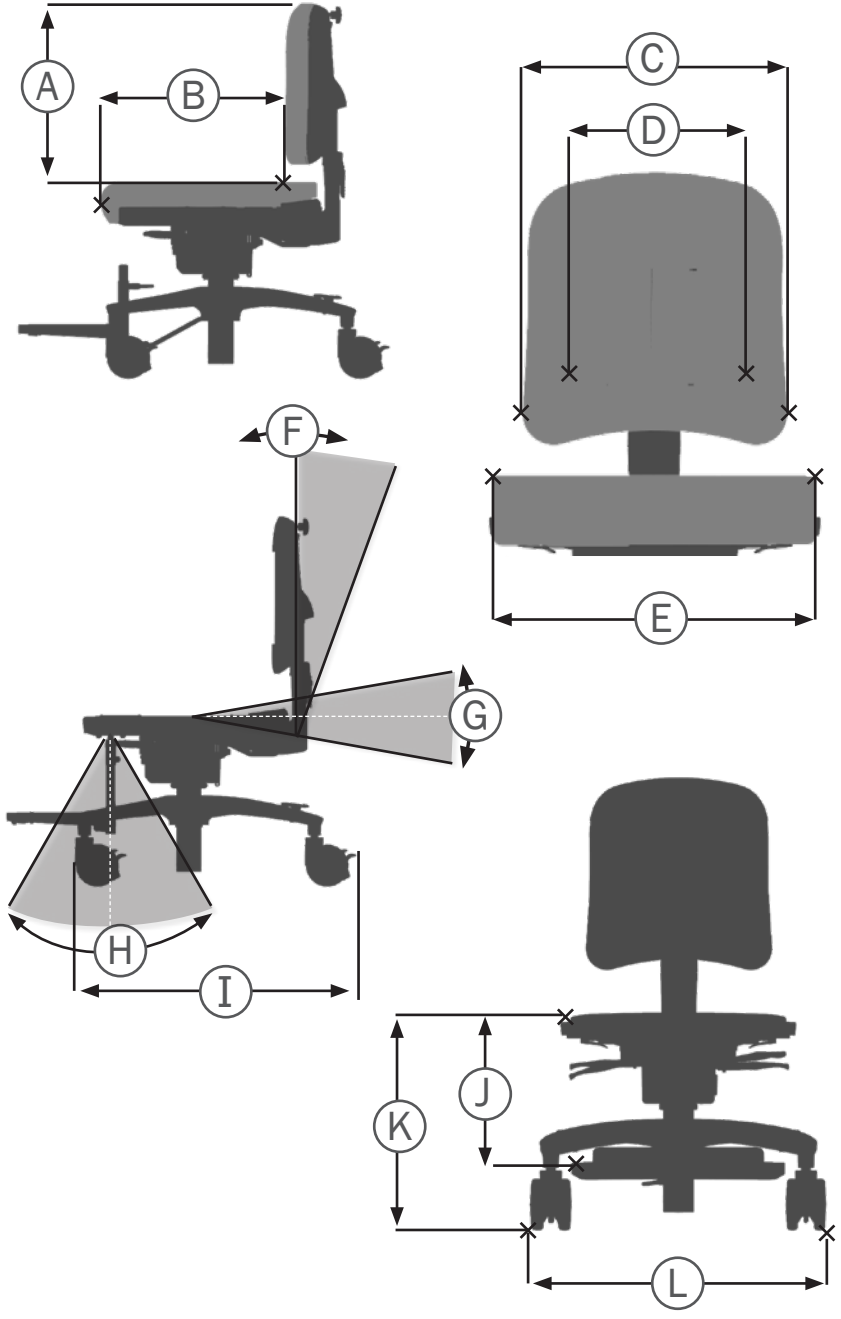
SR xxxxxxxxxxxxx

REP xxxxxxxxxxxxxxxxxxxxx

▲ Max. user mass: xxx kg  
 ▲ Max. rated load: xxx kg  
 ▲ Mass device: xxx kg

UEN (XX)XXXXXXXXXXXXXX(XX)XXXXXX(XX)XXXXXX

mm  
(inch)  
kg  
(lb)

mm (inch) kg (lb)	 1	 2	 3
 Backrest height (A)	280-340 mm (11-13½")	310-380 mm (12¼-15")	390-500 mm (15¼-19¾")
 Backrest height (A), long back	360-430 mm (14¼-17")	395-460 mm (15½-18")	460-550 mm (18-21¾")
 Effective seat depth (B)	180-310 mm (7-12¼")	250-380 mm (9¾-15")	320-500 mm (12½-19¾")
 Back width (C)	250 mm (9¾")	300 mm (11¾")	350 mm (13¾")
 Effective back width, contoured cushion (D)	170 mm (6¾")	230 mm (9")	270 mm (10½")
 Effective seat width (E)	300 mm (11¾")	370 mm (14½")	420 mm (16½")
 Backrest angle (F)	0° - 28°	0° - 28°	0° - 33°
 Seat plane angle (G)	(-10°) - (15°)	(-10°) - (15°)	(-10°) - (10°)
 Leg to seat surface angle (H)	(-30°) - (30°)	(-30°) - (30°)	(-30°) - (30°)
 Length (I)	645 mm (25½")	645 mm (25½")	645 mm (25½")
 Footrest to seat distance, seat mounted (J) **	85-185 mm (3¼-7¾")	85-285 mm (3¼-11¼")	85-375 mm (3¼-14¾")
 Footrest to seat distance frame mounted (J) **	95-320 mm (3¾-12½")	95-320 mm (3¾-12½")	245-395 mm (9¾-15½")
 Seat surface height at front edge (K)*	310-430 mm (12¼-17")	310-430 mm (12¼-17")	375-491 mm (14¾-19¼")
 Seat surface height at front edge with cushion (K)	375-495 mm (14¾-19½")	375-495 mm (14¾-19½")	440-556 mm (17¼-22")
 Overall width (L)	560 mm (22")	560 mm (22")	560 mm (22")
 Max user mass	45 kg (99.2 lb)	45 kg (99.2 lb)	70 kg (154.3 lb)
 Mass device	12.5 kg (27.5 lb)	13 kg (28.7 lb)	16 kg (35.3 lb)
 Max rated load	59.5 kg (131.1 lb)	60 kg (132.2 lb)	78 kg (171.9 lb)

\* Measured without cushions. There are 6 positions for height adjustment in the center column. Each increment adds 22 mm to measure starting from position -1.

\*\* Add 65 mm to measure with cushion.

A-E Measured with cushion

I Add 15 mm to measure when 100 mm castors are mounted.

L Add 10 mm to measure when 100 mm castors are mounted.

K Add 25 mm to measure when 100 mm castors are mounted.

**EN:** Warning: This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger  
 01 = Entrapment Hazards  
 02 = Tilting Hazards  
 03 = Transportation in motor vehicles  
 05 = Squeezing Hazard

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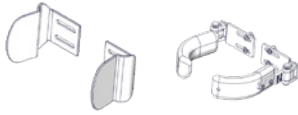
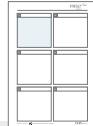
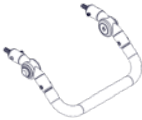
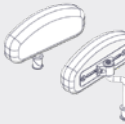
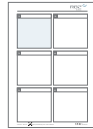
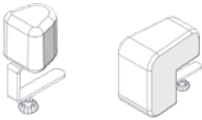
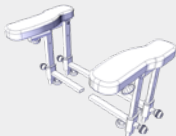
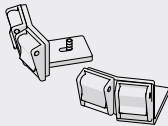
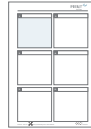
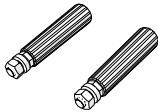
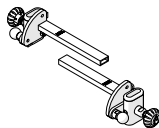
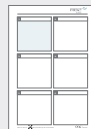
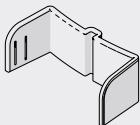
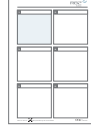
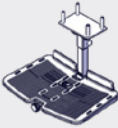
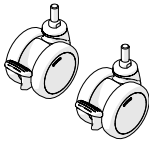
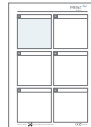
9996098149

## General

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User manual before use and save it for future reference.

### Intended purpose

The Wombat Solo is a chair for children with disabilities.

### Contra indications

There are no known contraindications.

Caution! When used for:

- Children with limitation in their ability to maintain the position of their head, use the product with a headrest.
- Children with limitation in their ability to maintain a sitting position, use the product with accessories (e.g. side supports or vest).
- Children who experience severe or complete impairments in involuntary movement functions of the trunk, consider a modular seating system with a dynamic back.

### Application areas

The product is for indoor use

### Enclosed tools:

- 3+4+5 mm Allen key

## CE Declaration of Conformity

This product conforms to the requirements of the Medical Devices Regulation (2017/745).

The CE mark must be removed when:

- Rebuilding the product
- Using the product outside the intended purpose and declaration of conformity
- Using other than original R82 spare parts and fittings

### Combinations

When combining the device with other products, the combination must not change the intended purpose of the products or modify the products in such a way that compliance with the applicable requirements may be affected. The person / company responsible for the combination of the products must ensure that the applicable requirements are fulfilled.

Applicable requirements examples:

- Material safety (biocompatibility, flammability), mechanical safety (stability, squeezing hazards, inter connection), electrical safety

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

[www.etac.com/support](http://www.etac.com/support)

### Presales information and instructions

Presales information and instructions are available at [www.etac.com](http://www.etac.com), under your product, in the documents section.

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions. Any adverse event in relation to the device must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

### Caregiver

- Read the "User manual" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- This product contains small parts which could present a choking hazard if removed from their intended placing
- Users who have difficulties reading the user guide can have someone read the user guide aloud to understand how to use the product.
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible
- Firmly grab hold and support the back of the chair before adjusting it to prevent a sudden recline.
- Transportation of the product whilst the user is in it is not allowed. It is allowed to use the product with the user safely strapped in it for transfers over short distances. Do not cross any obstacles during transfer.
- The handles underneath the seat plate can be operated by the user.
- Settings and adjustments to the product or accessories are only to be made by persons who have read the user guide.
- Ensure that moving parts does not squeeze or trap body parts when adjusting them
- Ensure that body parts are not trapped in openings for mounting of supports/ accessories.
- Be aware of any MAX mark indications before adjustment

### Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive

skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

### User

- Never carry the product with a user seated in it
- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

### Product

- Wombat Solo comes with multiple adjustment options. When adjusting the product, take care that body parts are not trapped in holes and openings intended for adjustment.
- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Lock the castors before placing the user in the product
- Ensure the stability of the product is maintained before placing the user in it
- For independent transfer ensure the castors are locked sufficiently.
- When the footplate is close to the floor and the castors, the castors may be unable to swivel
- The footplate is for support only! The foot support does not have a weight bearing function
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- This device is tested for resistance to ignition at level non-ignition in accordance with applicable standards.
- The product might lose its flame resistant

---

characteristics if using other than original  
R82: Cushioning

- This product is not to be occupied in a motor vehicle. The user must be positioned in a vehicle seat and the product must be stored in a luggage compartment
- The user must not occupy the product while it is being transported into or out of a vehicle by a lift

## Service & maintenance information

Service and maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to service and maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to service and maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the device. Contact the local dealer for assistance, if needed, in setting up, using or how to perform service and maintenance of the device.

### Lifetime

The lifetime of this device in normal use is 8 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

### Reuse

This device and its accessories/components are suitable for reuse.

### Refurbishment for reuse

Before reuse or changing the user, follow the service checklist, maintenance and washing instructions and carry out regular inspections.

### Service interval

The frequency of inspection may be altered depending upon use and wear. It is recommended that the device is inspected yearly, every time the device is re-issued for use and after a long-term storage (more than 4 months). The inspection should be carried out by a person who understands the use of the device.

### Service checklist:

Check and adjust the following as per their intended use:

- Seat plane adjustment. The seat plane is adjustable and stays in position after adjustment.
- Frame is adjustable in height
- Adjustment options for the foot rest, it is possible to lock the angle and fasten the telescopic tubes in position
- Seat back is adjustable and can be locked in position
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Repair or replace any damaged or worn out parts.

### Maintenance

Before every use it is recommended to:

Wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before use.

Carry out a visual inspection, for any damaged

or worn-out parts.

### Washing

#### Main device

This device can be washed at 60°C using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the device. Unmount all accessories and wash them separately.

Tilt/rotate the device after washing to facilitate water drainage from within the device.

#### Fabric and covers

The cover can be washed at max. 60°C using a mild detergent in a washing machine.

Separate foam and cover before wash.

#### Handwash

This device and its accessories can also be washed by hand. Use warm water and a mild detergent/ soap without chlorine and let all parts dry before use.

### Disinfection

The device can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the device, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

### Materials

- Aluminium
- Plastic
- Steel
- Stainless steel
- Rubber
- Wood

### Surface treatment

The following surface treatment processes were used to protect against corrosion:

- Lacquered surfaces = Polyester powder coating or ED-coating
- Non-lacquered aluminium parts = Anodized coating
- Non-lacquered steel surfaces = Galvanized

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**Etac Ltd.**
**Gelbart AG**