
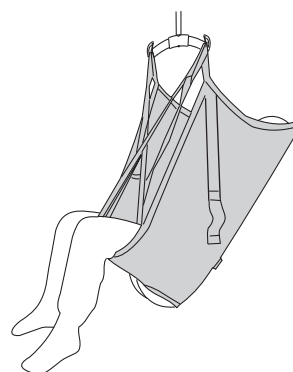


## Instructions for Use

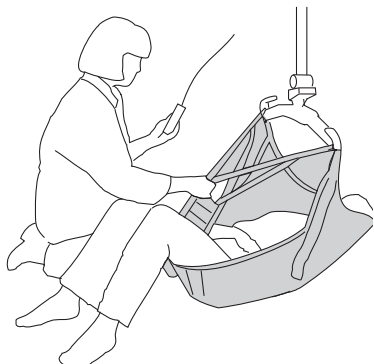

## Product Description

### Intended Use

**Liko Original HighBack Sling** is a basic model which is designed to adapt to the patient without individual adjustments and is one of our most used slings.

It provides for a slightly semi-reclined sitting posture and excellent support for the entire body, which is good for patients with reduced torso stability. A properly fitted and carefully applied sling will give the patient an excellent sense of security, as well as a high degree of comfort.

HighBack Slings of all variants and models are recommended for most common transfers, for instance:

- Transfers between bed and wheel chair.
- Transfer to and from toilet and bath-tub.
- Transfer for lifting to and from the floor.

The High Back Slings of all models are intended for use in following environments: Health care, Intensive care, Emergency ward, Rehabilitation, Habilitation environment and home healthcare environment.

### Sizes

The Original HighBack Sling is available in different sizes. It is important to choose the correct size in order to achieve the highest level of comfort and safety. A sling which is too large increases the risk of the patient sliding out of it, while one which is too small can cut into the groin and cause discomfort.

For larger sizes, a combination with a Twin bar 670 or a SlingGuard 670 Twin can be a good choice.

### Fabrics

The Original HighBack Sling is available in different fabrics. Since the Original HighBack Sling is not usually left under

the patient after a completed transfer, we recommend the polyester design. This fabric is durable and has low friction, making the sling easy to apply and easy to remove.

The Original HighBack Sling is also available in net polyester. This fabric is recommended when the sling is left in place after the transfer or when an airier alternative is desired.

For bathing and showering, a plastic-coated net design is available, which is water permeable and can easily be wiped. This material dries quickly and does not chill the patient in the same way as a damp, absorbent fabric would.

The sling's lift straps are always made of durable polyester.

### Leg Supports

The Original HighBack Sling is available with leg supports in different designs. Most common are reinforced leg supports. The reinforcement provides high comfort, distributes the pressure and prevents the sling from creasing under the thighs.

One design has simple fabric without reinforcement in the leg support, making the sling more pliable and easy to care for.

### Suitable lifts

The Original HighBack Sling can be used along with all of **Liko** overhead and mobile lifts. You can choose between **LikoGuard** overhead lift, **Likorall** overhead lift, and **Multirall** overhead lift or **Golvo** mobile lift, **Viking** mobile lift, **Uno** mobile lift, and **Liko M220/M230** mobile lift.

*In this document, the person being lifted is referred to as the patient and the person helping is referred to as the caregiver.*

### IMPORTANT!

Lifting and transferring a patient always involves a certain level of risk. Read the instructions for use for both the patient lift and lifting accessories before use. It is important to completely understand the contents of the instructions for use. The equipment should only be used by a trained caregiver. To be trained, a caregiver needs to read and understand the Instructions for Use. Ensure that the lifting accessories are suitable for the lift used. Exercise care and caution during use. As a caregiver, you are always responsible for the patient's safety. You must be aware of the patient's ability to make it through the lifting situation. If something is unclear, contact the manufacturer or supplier.

# Table of Contents

Intended Use .....	1
Safety Information .....	3
Symbol Description .....	4
Care and Inspection.....	5
Definitions .....	5
How to Attach a Sling to Different Sling Bars.....	6
How to Attach a Sling to a Sling Bar with 4 Hooks.....	7
Fitting Advice.....	8
How to Select Sling Size.....	10
Dimensions.....	11
Lifting from a Sitting Position .....	12
Lifting to a Sitting Position .....	13
Lifting from the Bed.....	14
Lifting to the Bed .....	15
Lifting from the Floor.....	16
Lifting to the Floor.....	16
Optional Components .....	17
Overview <b>Liko</b> Original Highback Sling .....	18
Recommended Combinations .....	18

## Safety Information

### **⚠ Before lifting, keep the following points in mind:**

- A responsible person within your organization should decide on a case-by-case basis whether one or more caregivers are needed.
- Ensure that the sling selected meets the patient's needs with reference to model, size, fabric and design.
- To protect patient safety and hygiene, launder the sling between patients.
- Plan the lifting operation so that it can be done as safely and smoothly as possible.
- Check that the lifting accessory hangs vertically and can move freely.
- Although the Liko sling bars are equipped with latches, special caution must be exercised. Before the patient is lifted from the underlying surface, but when the straps are fully extended, make sure the straps are correctly connected to the sling bar hooks.
- Make sure the patient is sitting securely in the sling before transferring to another location.
- Never lift a patient higher off the underlying surface than is needed to complete the lifting and transfer procedure.
- Make sure that the wheels on the wheelchair, bed, gurney, etc., are locked during the lifting/transfer operation.
- Always work ergonomically. Use the patient's ability to actively participate in the operations.
- It is recommended to remove the sling after the transfer. If for some reason it is preferable to leave the sling after transfer, it is recommended to use a sling in net polyester.

**⚠ Incorrect attachment of sling to slingbar may cause severe injury to the patient.**

**⚠ Never leave a patient unattended during a lifting situation!**

**⚠ Manually turning the patient in bed may cause injuries to the caregiver.**

**⚠ Make sure that the patient and caregiver stay clear of pinch points and moving parts during a lifting event. Injury could occur.**

**⚠ Evaluate patients for entrapment risk and monitor patients appropriately. Make sure that the patient's head and limbs are not in or between sling loops during lifting event. Failure to do either of these could cause serious injury or death.**

**⚠ Pinching or minor injuries can be caused by usage of a worn sling, if a sling is placed uneven on the patient or if the sling is not removed carefully.**

**⚠ Do not operate, store, or transport sling in the vicinity of pets, pests, or unattended children!**

**⚠ Never leave children unattended in the vicinity of the sling!**

**⚠ Do not store the sling where it will be exposed to direct sunlight, or to heat sources, such as a radiator, fireplace, or stove/oven!**

**⚠ The device has not been tested for use in the Magnetic Resonance Imaging (MRI) setting. Transfer the patient to a compatible surface to do an MRI, and remove the sling from the MRI room.**

**CE** Medical Device Class I Product

**PATENT [www.hillrom.com/patents](http://www.hillrom.com/patents)**

**May be covered by one or more patents. See above Internet address.**

The Hillrom companies are the proprietors of European, US, and other patents and pending patent applications.

### **Design and Quality by Liko in Sweden**

Liko's products undergo continuous development, which is why we reserve the right to make product changes without prior notice. Contact your Hillrom representative for advice and information about product upgrades.

### **Notice to Users and/or Patients in EU**

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

# Symbol Description

These symbols may be found in this document and/or on the product.

Symbol	Description	Symbol	Description
	Warning; this situation requires extra care and attention. This symbol is a yellow triangle with a black border, and a black exclamation mark at the center.		
	Recyclable		
	Read instructions for use before use. This symbol is a blue circle with a white image of a person reading a book.		
	Read instructions for use before use		<b>Caution!</b> consult instructions for use.
	CE-mark. European Union medical device conformity mark, Class I Medical Device.		
	Swiss authorized representative		
	Legal manufacturer		Date of manufacture
	Product Identifier		Medical device
	Serial Number		Unique device identification
	Do not dry clean		Line drying in shade
	Do not iron		Do not bleach
	Tumble drying		Do not wash
	Do not tumble dry		
	Washing Symbol is always in combination with recommended temperature in °C and °F.		
	Latex Free		
	Periodic inspection		
	Do not use product (Only on Solo products, and only visible after wash when use is prohibited)		Patient name
	Single patient use (only on Solo products)		
	Direction upwards and symbol on outside of equipment.		Symbol on outside of equipment.
	GS1 Data Matrix Barcode that may contain following information. (01) Global Trade Item Number (11) Production Date (21) Serial Number		

## Care and Inspection

Check the sling before each use. Check the following points with regard to wear and damage:

- Fabric
- Straps
- Seams
- Suspension loops

### ⚠ Do not use damaged lifting accessories.

If anything is unclear, please contact the manufacturer or supplier.

**If the sling is used for bath or shower:** After use in bath and shower situations, the sling should be placed so that it dries as soon as possible. Do not place the sling in direct sunlight.

**Material:** The Original HighBack Sling is available in polyester, net polyester, and plastic-coated net polyester. Refer to "Overview LIKO Original HighBack Sling, Mod 200, 210" on page 18.

**Washing and Disinfection Instructions:** Please see the sling's product label and the attached appendix "Care and Maintenance of Liko Slings".

### Periodic Inspection

The product must be inspected at least once every 6 months. More frequent inspections might be required if the products are used or washed more often than normal. See Liko's protocol "Periodic Inspection Liko Textile Products". For protocols please contact your Hillrom representative.

### Expected Lifetime

The product has an expected lifetime of 5 years during normal use. Expected lifetime varies depending on fabric, use frequency, washing procedure, and weight carried. Hillrom recommends the product to be used only during its expected lifetime.


### ♻ Recycling Instructions


The entire product, including laths, padding material, etc. should be sorted as Combustible waste.

Hillrom evaluates and provides guidance to its users on the safe handling and disposal of its devices to aid in the prevention of injury, including, but not limited to: cuts, punctures of the skin, abrasions, and any required cleaning and disinfection of the medical device after use and prior to its disposal. Customers should adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories.







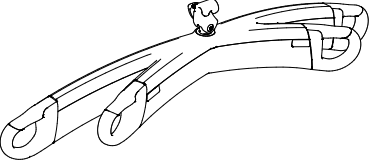
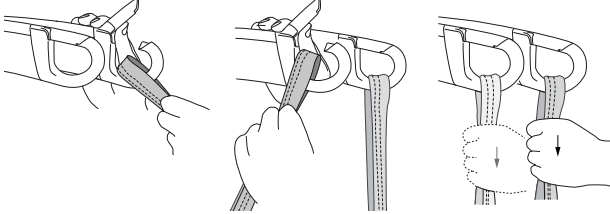
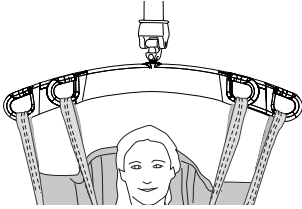
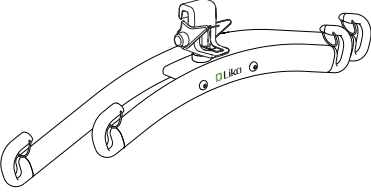
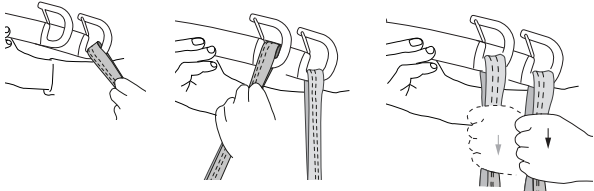
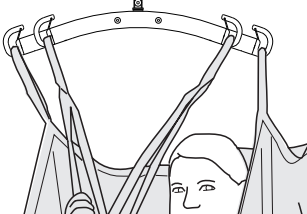
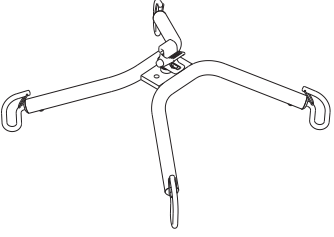
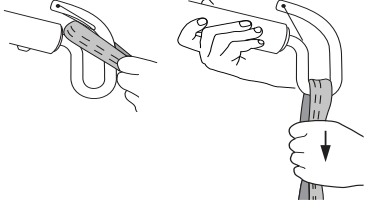
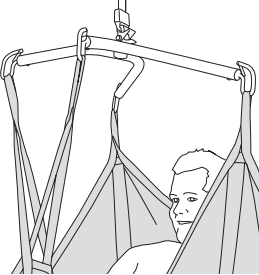
If in doubt, the user of the device shall first contact Hillrom Technical Support for guidance on safe disposal protocols

## Definitions

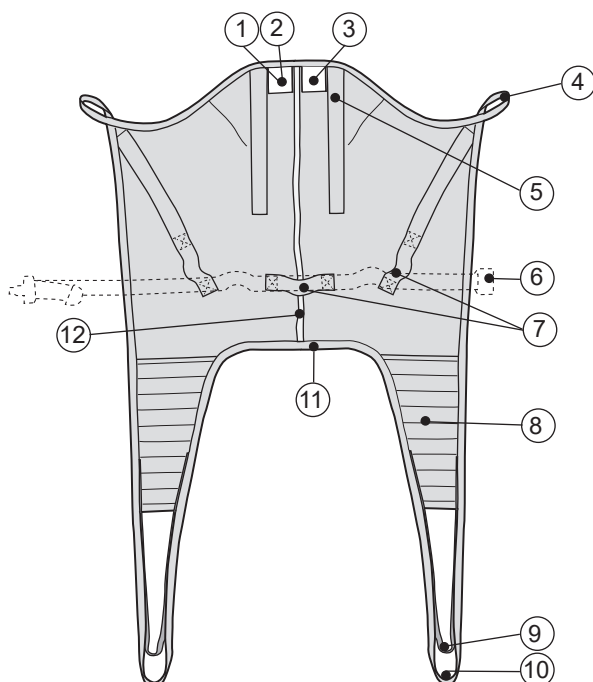


1. Product label, including:  
Manufacturing year and month (  )  
Serial number ( **SN** )
2. Label: Individual marking (Belongs to)
3. Label: Periodic Inspection
4. Upper strap loop
5. Head support with laths
6. Belt (size XS, S)
7. Handle
8. Leg support
9. Shortening position
10. Leg support loop
11. Lower edge
12. Center back strap with Liko size color  
(XS = Red, S = Orange, MS = Grey, M = Yellow, L = Blue,  
XL = Black, XXL = White)






## How to Attach a Sling to Different Sling Bars

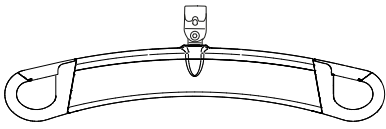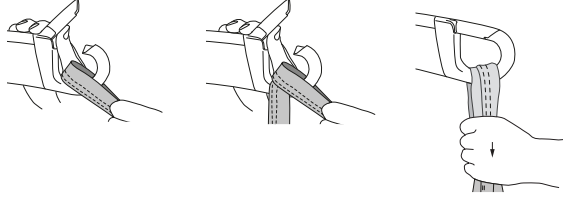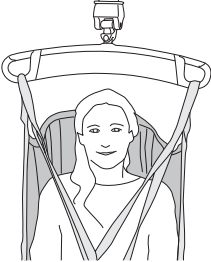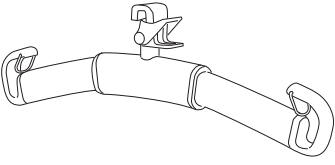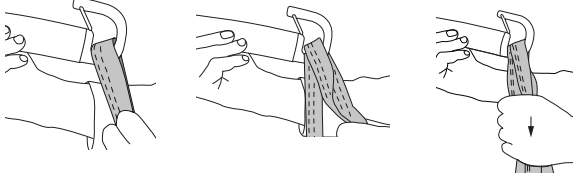Sling bar	Sling loops to sling bar hooks	Correct
 <p data-bbox="204 497 347 533">SlingGuard</p>		
 <p data-bbox="215 788 336 824">Universal</p>		
 <p data-bbox="172 1070 384 1106">SlingGuard Twin</p>		
 <p data-bbox="178 1370 375 1406">Universal Twin</p>		
 <p data-bbox="215 1675 336 1711">Cross-bar</p>		

### Wrong!

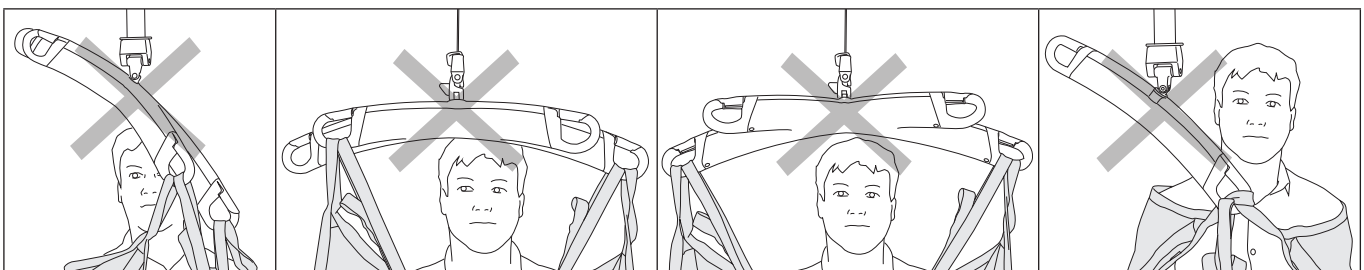


# How to Attach an Original HighBack Sling to a Sling Bar with 4 Hooks (Twin Bars and Cross Bars)

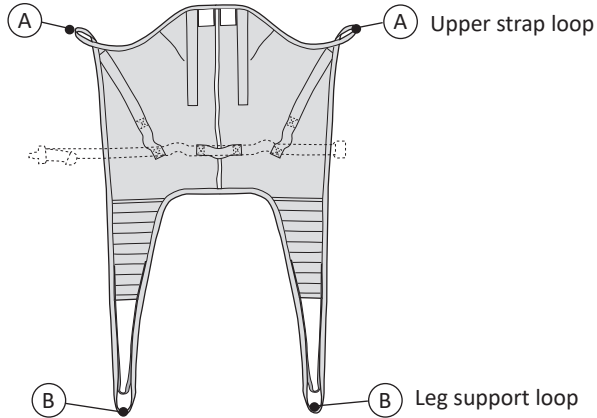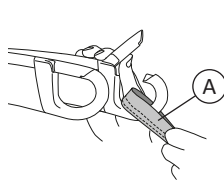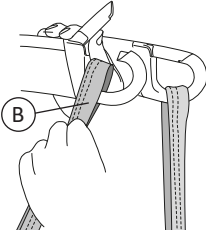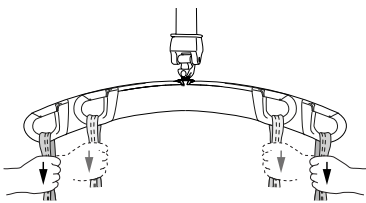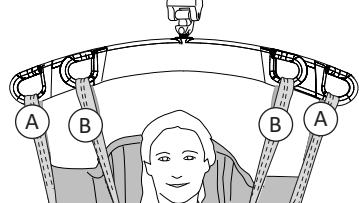 <p>A Upper strap loop</p> <p>B Leg support loop</p>	<p>1.</p>  
<p>2.</p> 	<p>3.</p>  <p>The illustrations 1, 2, and 3 shows a twin bar but this also applies to a Cross-bar and sling bars with Side Bars.</p>

# Fitting Advice

## Different Sitting Positions

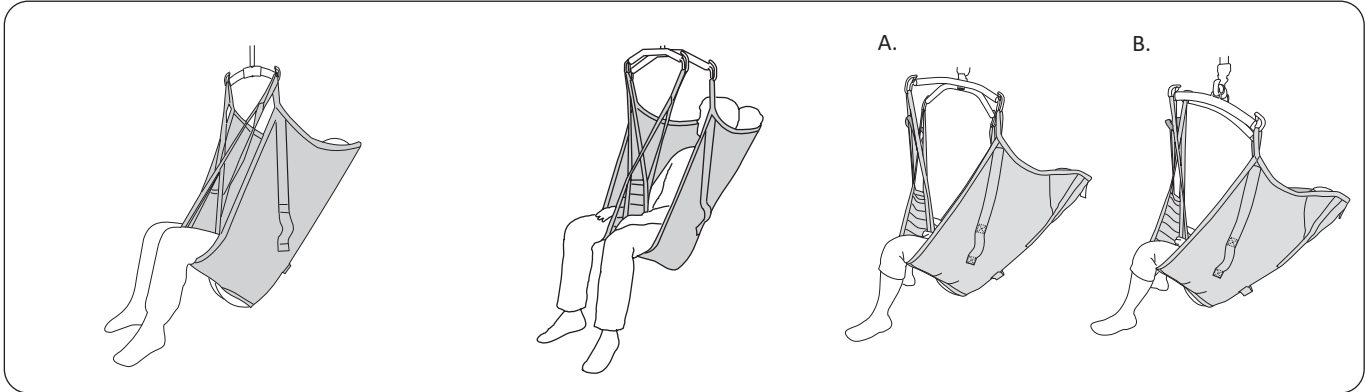


**Liko Original HighBack Sling in two-point suspension** provides for a slightly reclined sitting posture with excellent head support and approx. 90 degrees angle in the hip joint.

**Liko Original HighBack Sling** in combination with a twin bar 670 or a SlingGuard 670 Twin provides larger width at the shoulders than at the leg supports, which is suitable if the patient is large or sensitive to pressure to the shoulders.

- A. In combination with **Liko Cross-bar** a slightly more reclined posture and a larger angle is obtained. Complete with a small pillow behind the head.
- B. In combination with **SideBars** a slightly more reclined posture and a larger width at the shoulders than at the leg supports is provided, which is suitable if the patient is sensitive to pressure to the shoulders.

## Different Ways of Connecting Leg Supports

Before a connection alternative is chosen, an individual assessment should be performed to anticipate possible risks. The following are general advice since the same principles are valid for several sling models.

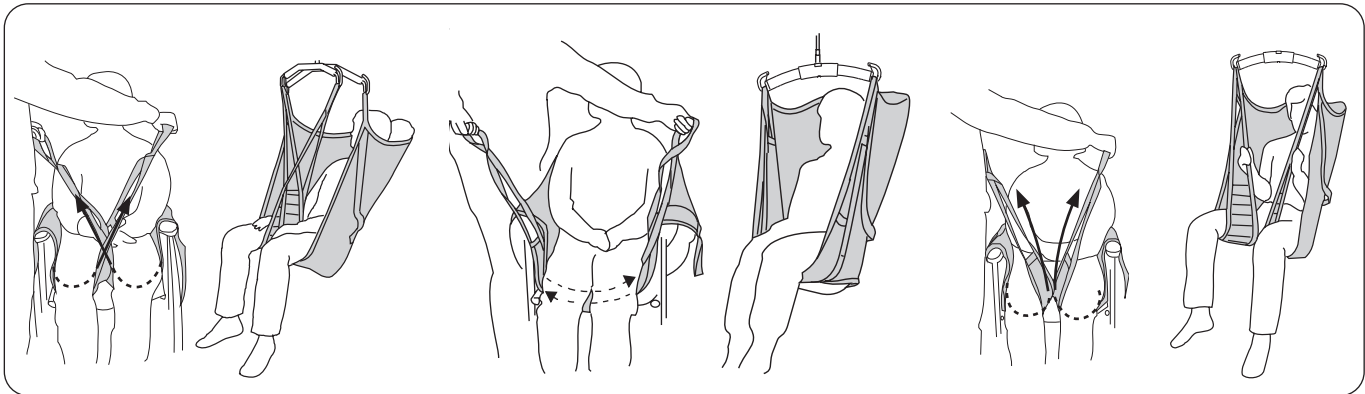

**Crossed-over leg supports.**  
The most common way of applying the sling. One leg support loop is passed through the other before the loops are connected to the sling bar.


**Overlapped under both thighs.**  
This can be appropriate, for example, when lifting a patient with amputated legs. **Tip:** In order to avoid the patient's sitting posture being too reclined, Extension Loops can be attached to the leg support loops.

**Each leg support connected separately to the sling bar.**  
This can be appropriate when lifting patients with particularly sensitive genitals.

**⚠ Increased risk of sliding out.**

## The Patient does not Sit Well in the Sling – Why is This?

The following are general advice since the same principles are valid for several sling models.

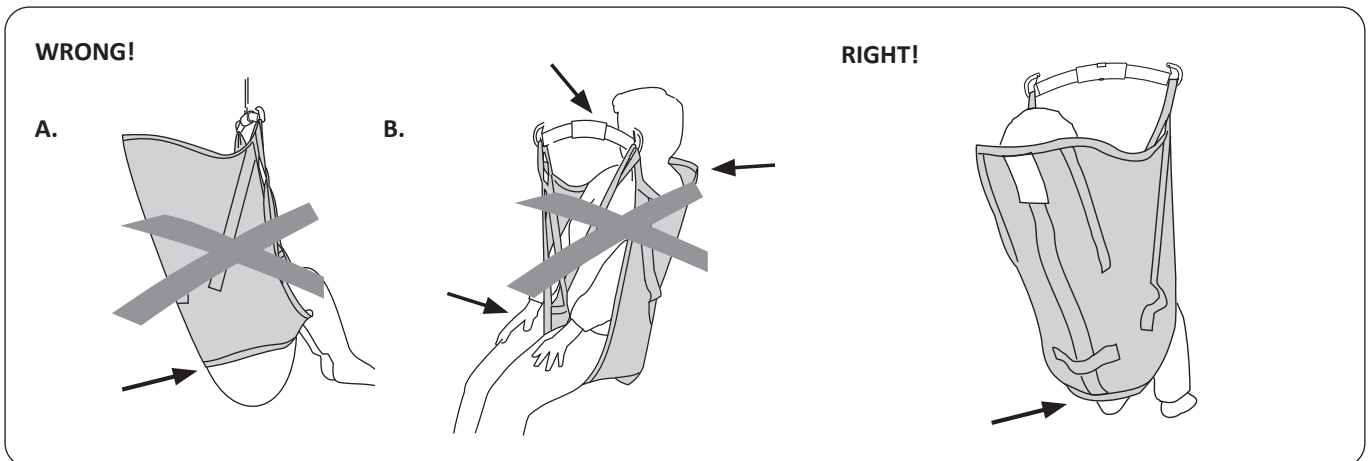


**A. Risk of sliding out.** The buttocks hang down. The sling may be too large. It can also be due to the fact that the sling has not been pulled down the patient's back sufficiently.

Before lifting, check that the lower edge is stretched and that the fabric reaches properly around the legs.

**B. A sling that is too small** can cause the leg supports to cut into the groin, causing discomfort, and the sling bar to be too close to the face and/or the back to get insufficient support.

The lower edge of the sling is at the same level as the coccyx. The leg supports are properly positioned under the legs.

## Shortening Position

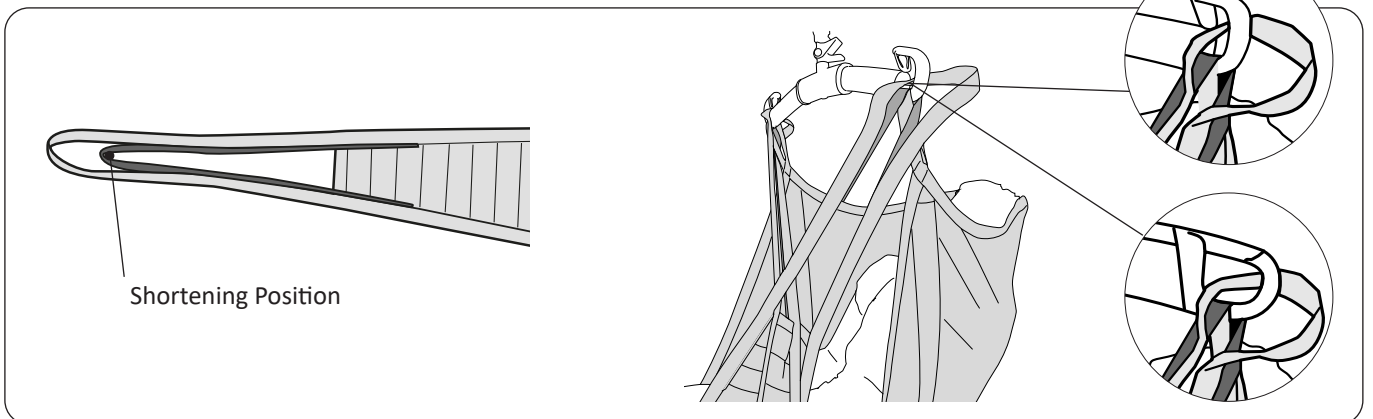

Our sling models are carefully designed to ensure that the patient always sits with pressure well distributed between legs and back section for good support during transfers. However, in certain cases it can be preferable to use the inner setting, the "shortening position", on the leg support loop to change the patient's sitting position.

In cases where the shortening position is used, we recommend that you also connect the outer leg support loop to the sling bar hook in order to have better visual control of the loops.

## How to Select Sling Size


**⚠️ Only use a sling that is the correct size for the patient. Correct size must be assessed by a trained caregiver. Patient entrapment or asphyxiation could occur.**

The hip width measurement on the patient and on the sling gives guidance when choosing the size of the sling. Compare the measurement of the patient (see instruction for measuring) and the measurement of the sling (see table or instruction for measuring). The weight of the patient can also be used as a guidance, see “Overview LIKO Original HighBack Sling, Mod 200, 210” on page 18.

### Choosing the correct size:

Choose a sling with a hip width measurement that is within +/- 8 cm (3 in) of the patient’s hip width measurement.

### Measure the hip width of the patient as follows:

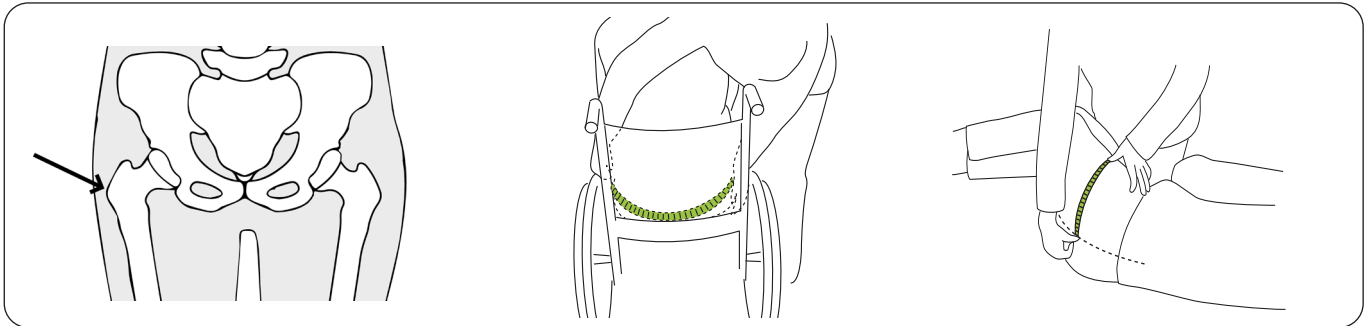


**Trochanter:** The trochanter is the part of the femur that can be felt at the patient’s hip. Apply gentle pressure to find the trochanter for measurement.

**Alternative A.** With the patient leaned forward, measure the hip width at the widest place i.e. from one trochanter to the other trochanter.

**Alternative B.** With the patient on their side, measure from the trochanter to the midline/backbone and multiply by 2.

### Reference page 9 “Dimensions” for the hip width of the sling as follows:

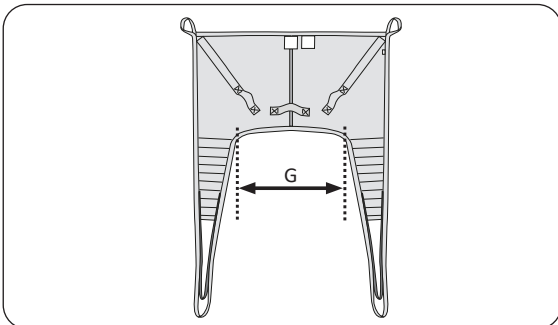


### Hip width measurement:

Sling Size	Hip Width
<b>XS</b>	25 (10)
<b>S</b>	32 (13)
<b>MS</b>	39 (15)
<b>M</b>	50 (20)
<b>L*</b>	64 (25)
<b>XL*</b>	74 (29)
<b>XXL*</b>	103 (41)

\* has a fold in the lower edge.  
All measurements are in cm (in).

## Dimensions

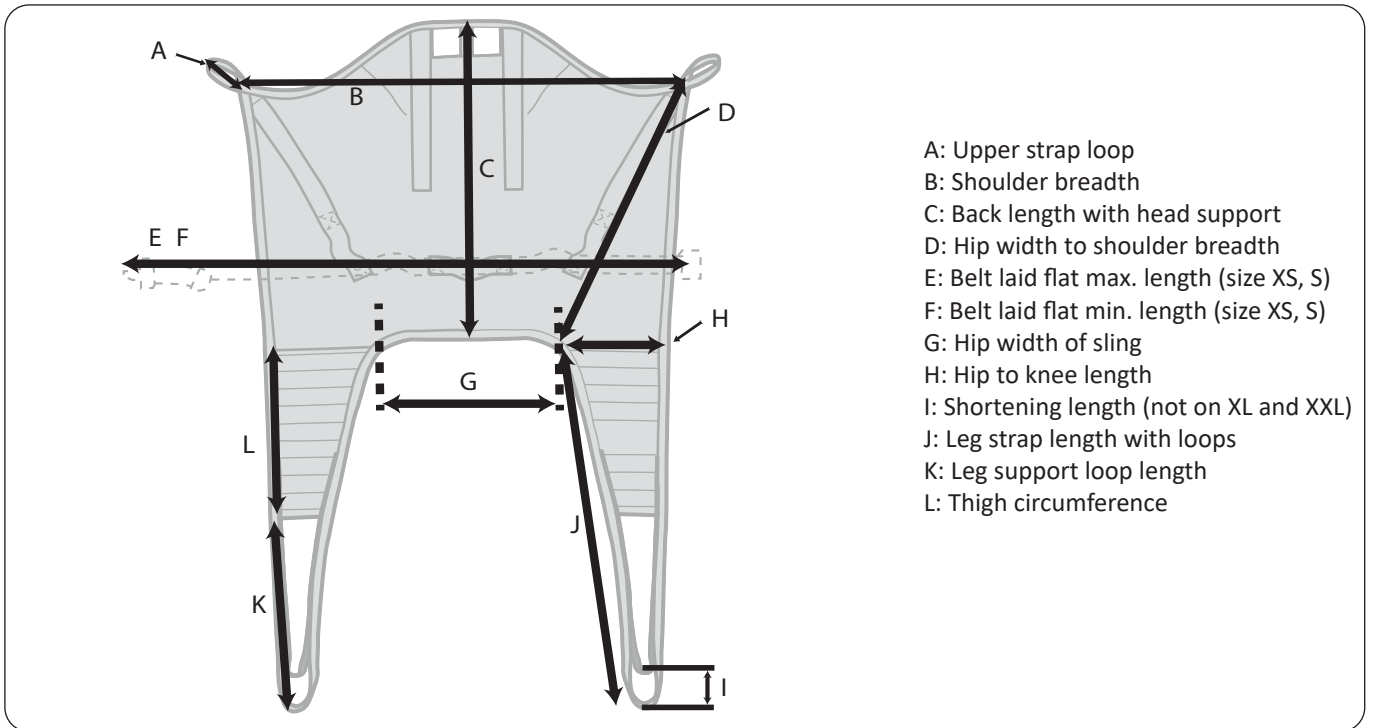

### Size Measurements in Centimeters

SIZE	A	B	C	D	E	F	G	H	I	J	K	L	Maximum Load (kg)	Prod. No.
XS	26	76	66	58	99	75	25	20	10	86	60	26	200	35200**3
S	21	90	77	67	111	86	32	23	10	93	60	32	200	35200**4
MS	15	104	92	80	N/A	N/A	39	29	10	100	55	45	200	35210**5
M	15	120	91	80	N/A	N/A	50	31	10	102	55	46	200	35200**5
L	15	119	100	87	N/A	N/A	64	32	10	107	60	48	200	35200**6
XL	15	113	100	87	N/A	N/A	74	31	N/A	106	55	53	500	35200**7
XXL	15	124	102	89	N/A	N/A	103	32	N/A	114	55	59	500	35200418


### Size Measurements in Inches

SIZE	A	B	C	D	E	F	G	H	I	J	K	L	Maximum Load (lbs)	Prod. No.
XS	10	30	26	23	39	30	10	8	4	34	24	10	440	35200**3
S	9	35	30	26	44	34	13	9	4	37	24	13	440	35200**4
MS	6	40	36	32	N/A	N/A	15	11	4	39	22	18	440	35210**5
M	6	47	36	31	N/A	N/A	20	12	4	40	22	18	440	35200**5
L	6	47	39	34	N/A	N/A	25	13	4	42	24	19	440	35200**6
XL	6	45	39	34	N/A	N/A	29	12	N/A	42	22	21	1100	35200**7
XXL	6	49	40	35	N/A	N/A	41	13	N/A	45	22	23	1100	35200418

**NOTE:** The \*\* in the product number represent material options. Refer to "Overview LIKO Original HighBack Sling, Mod 200, 210" on page 18


## Lifting from a Sitting Position

⚠ Two caregivers should be used to place the sling in accordance with ergonomic principles. The patient is usually tilted forward during the application, and it is important that one caregiver stands in front of the patient to prevent them from falling forward.

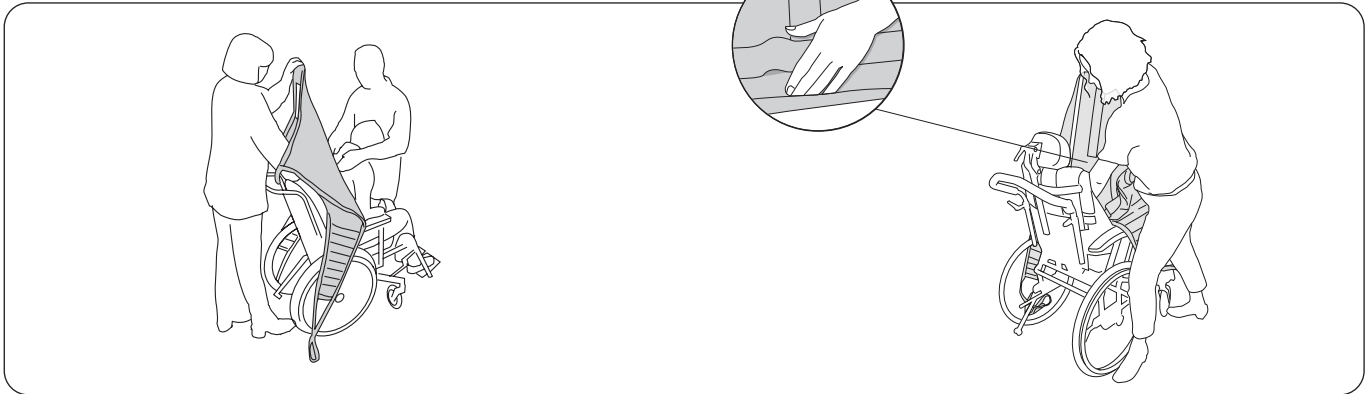


Place the sling behind the patient's back with the product label outwards. Fold up the lower edge of the sling around your finger tips to facilitate guiding the sling into place. Sliding sheets can also be used for application of the sling.

Using the palm of your hand, push the lower edge of the sling down to the level of the patient's coccyx -**this is very important**. **Tip:** Correct positioning is facilitated if the patient leans/is leaned forward slightly.

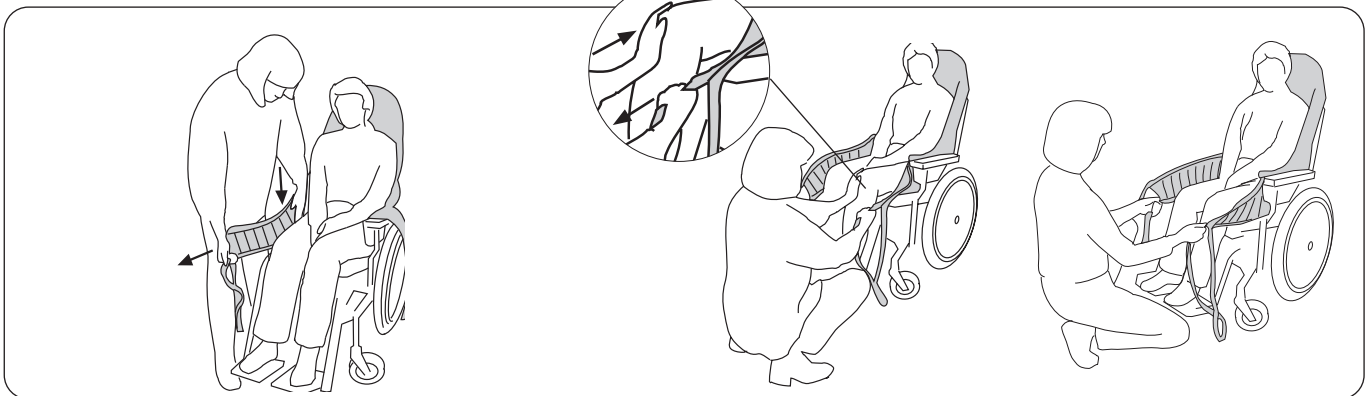


Pull the sling's leg supports forward along the outside of the patient's thighs. Place the palm of your hand between the patient's body and the sling and push the leg support's lower edge down towards the seat. Simultaneously, pull the leg support forward with the other hand to stretch it.

Pull the leg supports forward to smooth out any creases in the back. Check that both leg supports protrude the same distance. **Tip:** A gentle hold under the knee-cap makes it easier to pull the leg supports forward.

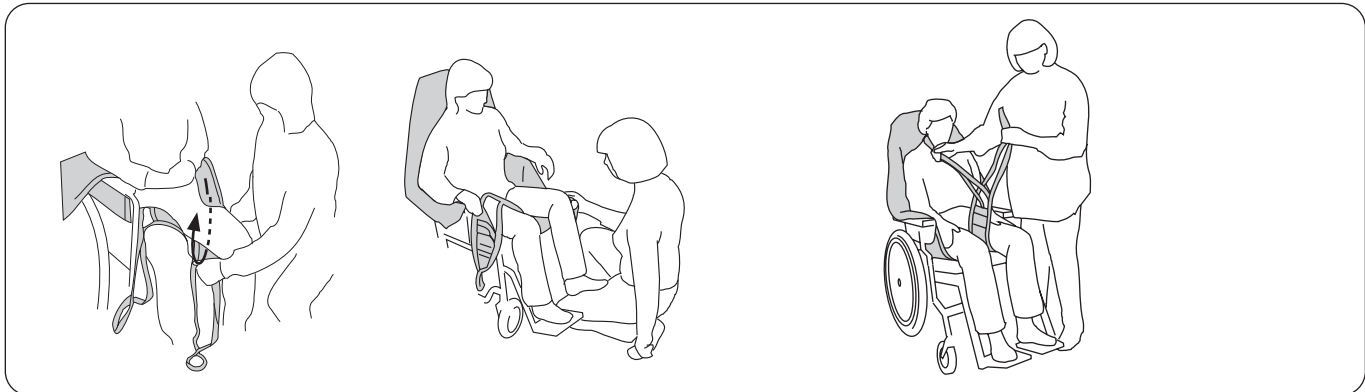

Insert one leg support under each thigh. Make sure the fabric lies flat and that it reaches properly around the leg. **Tip:** The application of the leg supports is facilitated if the patient's legs are slightly raised from the seat. This can be achieved by placing the patient's feet on the foot-rests or on your own leg, as illustrated.

Prepare the connection of the leg supports. The most common way of applying the sling is with crossed leg supports. For alternative methods, see "Different Ways of Connecting Leg Supports" on page 8.

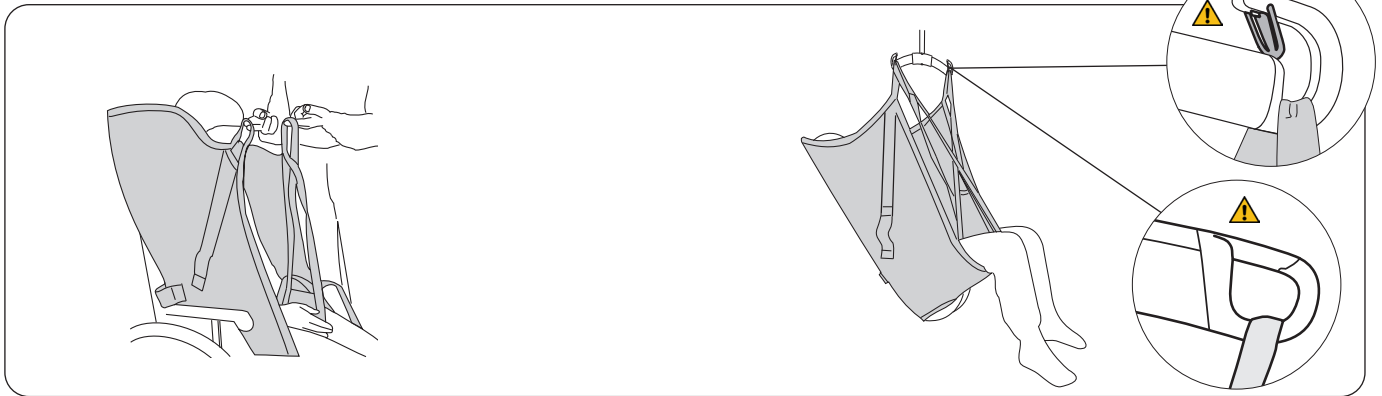


**Tip:** Check that all suspension loops have the same length, i.e., are at an equal height when they are stretched prior to lifting. Adjust the position of the sling if they are not at an equal height.

First connect the upper strap loops to the sling bar hooks, then the leg support loops. See page 6-7 for instructions on how to attach the sling loops to different sling bars. Lift the patient.

## Lifting to a Sitting Position

When placing a patient in a wheelchair or chair, or on a toilet, it is important that the patient gets far enough back to sit well. This can be accomplished in several ways.

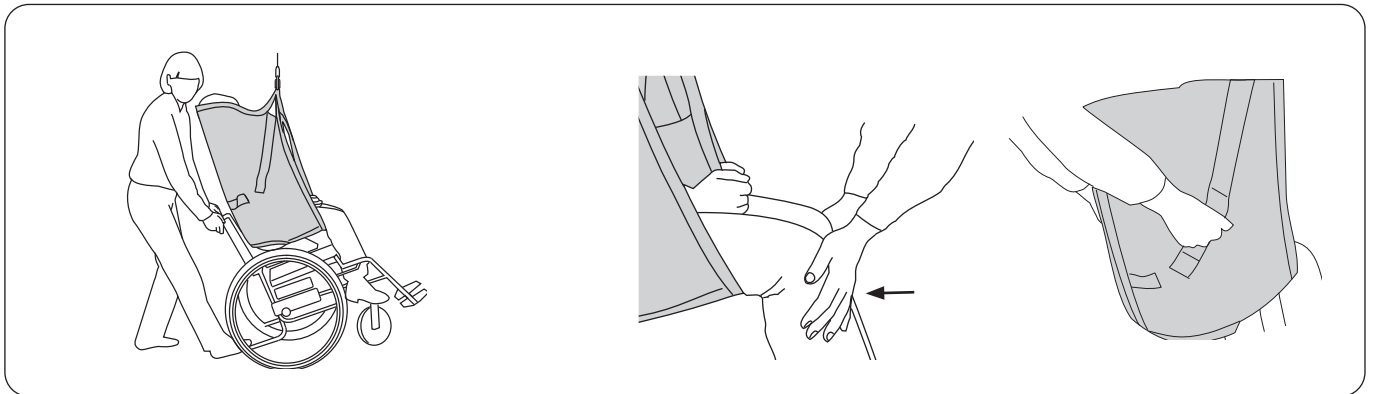

*When placing a patient in a wheelchair:* Lean the chair backwards during the lowering of the patient. Let the buttocks slide down the backrest during the last part. Control the angle of the wheelchair with your foot on the tipping support.

*When placing a patient in a chair or on a toilet (Alternative method for wheelchair):* Apply gentle pressure under the patient's knees during the lowering sequence, so that the buttocks are guided towards the backrest of the chair. If the sling has handles, these can be used to guide.

**⚠ Do not use the handles to lift! Excessive force on the handles can cause the sling to tear.**

## Removing the Sling in a Chair/Wheelchair/Toilet

It is recommended to remove the sling after the transfer. If it is preferable to leave the sling in the chair after transfer, it is recommended to use a sling in net polyester. Do not leave the sling on when you transfer a patient to a toilet.



Carefully remove the leg supports. The most gentle way of doing this is by pulling out the leg support loops under the fabric part.


Remove the sling by carefully pulling the leg support up.

## Lifting from the Bed

The sling can be applied in different ways depending on the status of the patient. Either it is applied with the patient lying in bed or sitting in bed. If an advanced pressure-relieving mattress is used, it can to advantage be inflated maximally to facilitate the application of the sling.


Remember your own work posture and the patient's comfort. Use the bed's functions for raising and lowering. Good work posture is obtained when the mattress is level with the caregiver's knuckles when the arms are held completely down. Depending on the burden on the caregiver, an evaluation of how many caregivers are needed for the operation must be made on a case to case basis.

### Applying the Sling when Lying in Bed

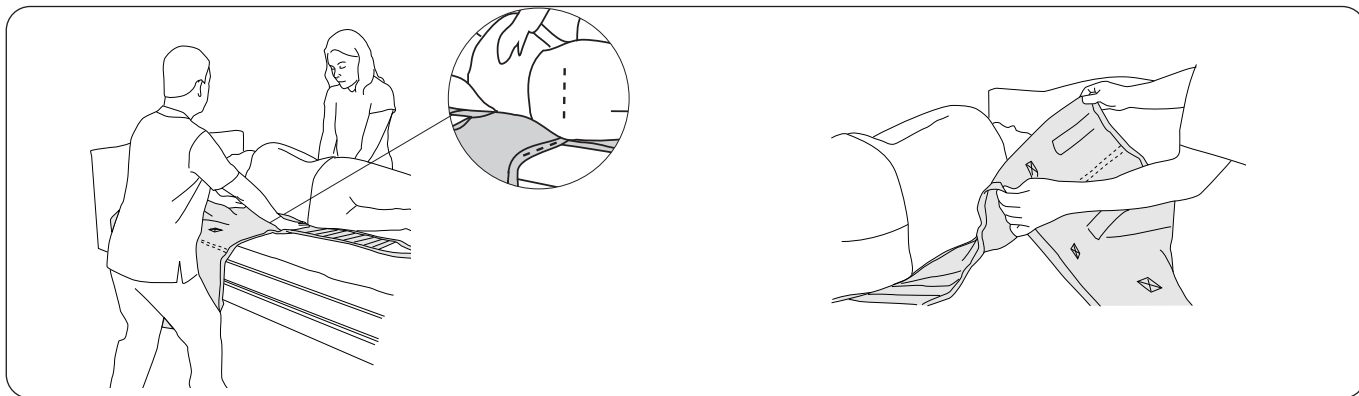

Turn the patient to the side. This operation is facilitated if the patient himself can contribute by gripping the bed rail on the side to which he is to be turned. Place the edge of the sling towards the back of the patient with the lower edge **level with the coccyx**.

Fold the sling so that its "center back strap" corresponds to the patient's spine. Check that the fold covers the sling's edge which is in towards the back of the patient in order to reduce the friction. Turn back the patient gently and pull out the sling on the other side. With this application technique, the patient needs to be turned once only.

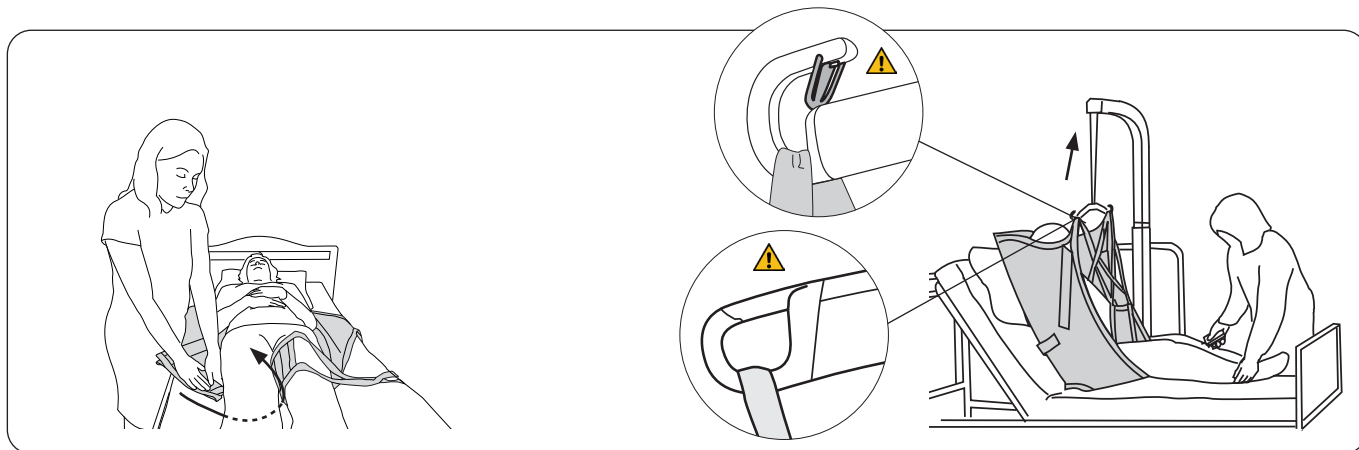

Insert the leg supports under each leg. This is best done by pressing the leg support down against the mattress while inserting it under the knee. Make sure the fabric lies flat and that it reaches properly around the leg. The leg supports can be connected in different ways, see "Different Ways of Connecting Leg Supports" on page 8.

Raise the head end of the bed. First connect the upper strap loops to the hooks of the sling bar, then the leg support loops. See page 6-7 for instructions on how to attach the sling loops to different sling bars. Lift the patient.

### Applying the Sling when Sitting in Bed

Raise the head end of the bed behind the patient. Lay the sling on the bed with the product label facing down towards the mattress. Slide the sling down behind the patient's back so that the lower edge is level with the coccyx. **Note!** The application is facilitated if the patient leans/is leaned forward.


Sliding sheets can also be used for application of the sling.

## Lifting to the Bed

We recommend removing the sling after transfer. If for some reason it is preferable to leave the sling in the bed after transfer, we recommend using a sling made of net polyester.


Remember your own work posture and the patient's comfort. Use the bed's functions for raising and lowering. Good work posture is obtained when the mattress is level with the caregiver's knuckles when the arms are held completely down. Depending on the burden on the caregiver, an evaluation of how many caregivers are needed for the operation must be made on a case to case basis.

### Removing the Sling when the Patient is Lying in Bed

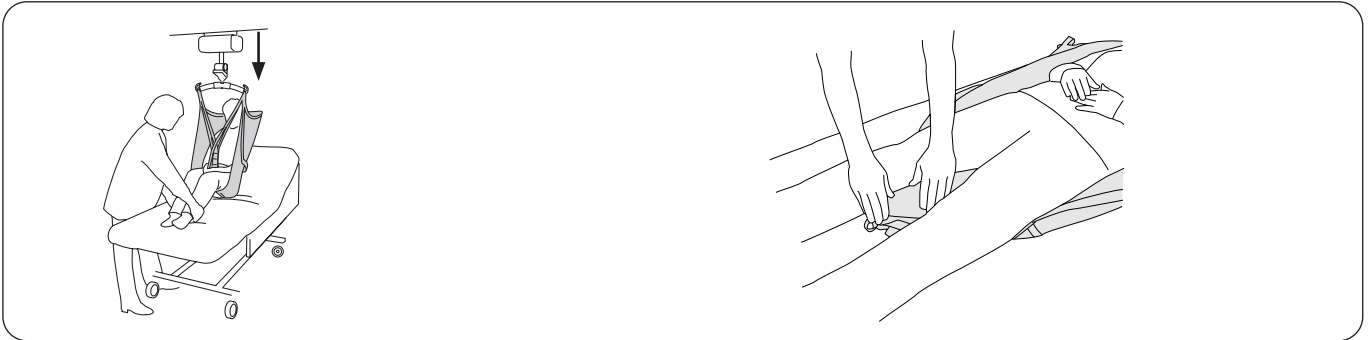

Position the patient above the bed. Raise the head end of the bed for the patient's comfort. Lower the patient onto the bed. Lower the head end.


Place the leg support loops under the leg supports between the patient's legs.

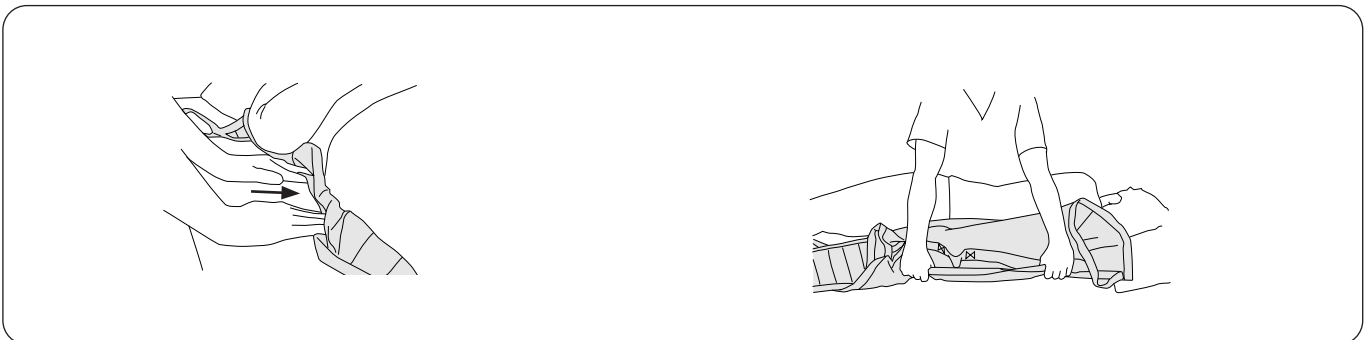

Grasp the edge of the sling. Push in the sling under itself. Press the sling against the mattress and in under the patient.

Carefully turn the patient onto the side. Grasp the sling from below and remove it carefully.

### Removing the Sling when the Patient is Sitting in Bed

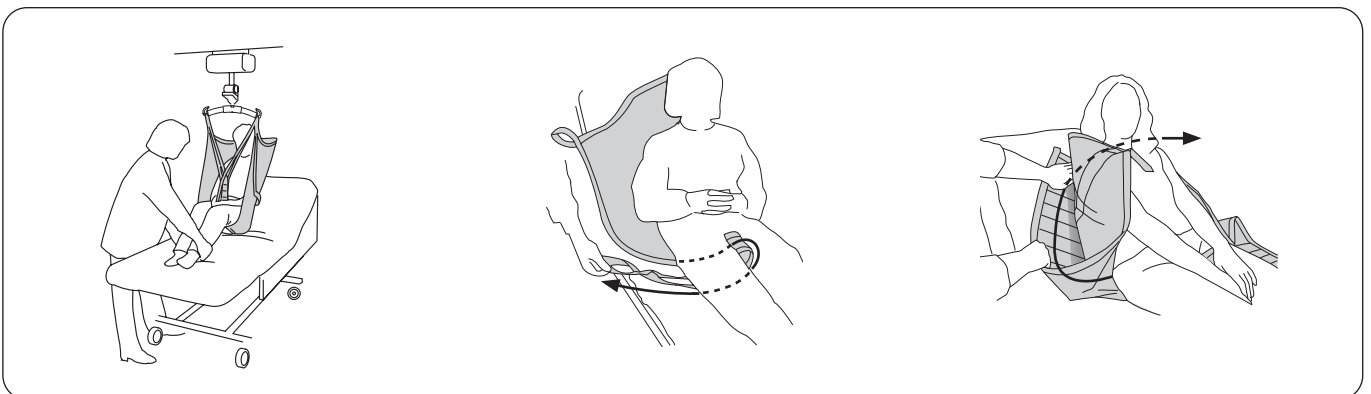

Position the patient above the bed. Raise the head end of the bed for the patient's comfort. Lower the patient onto the bed.


Remove the leg supports by pulling out the leg support loops under the fabric part of the leg support. **Tip:** This is facilitated if the patient's legs are bent.

Remove the sling by pulling one of the leg supports behind the sling and over to the opposite side.

## Lifting from the Floor


Lifting from the floor with Liko's **Golvo** mobile lift is described below. The same technique is suitable also for lifting with Liko's other mobile and overhead lifts. Read the Instructions for use for the lift being used.

**Tip:** A pillow under the patient's head could be comfortable.

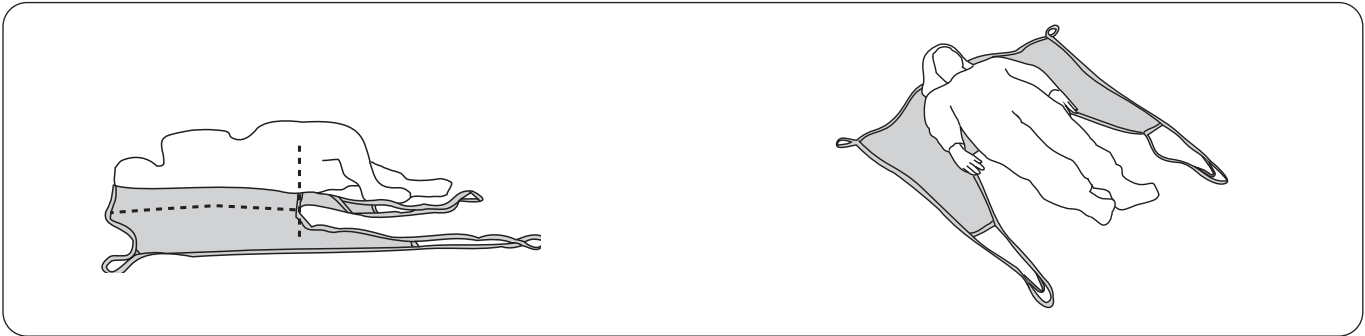

Carefully turn the patient onto the side and place the sling's edge under the patient with the lower edge **level with the coccyx**. Fold the sling so that its centre back strap corresponds to the patient's spine when the patient is turned back again gently. Carefully pull the sling out from the other side.

Check that the sling lies symmetrically under the patient; adjust the position if necessary. Place the leg supports under the legs.

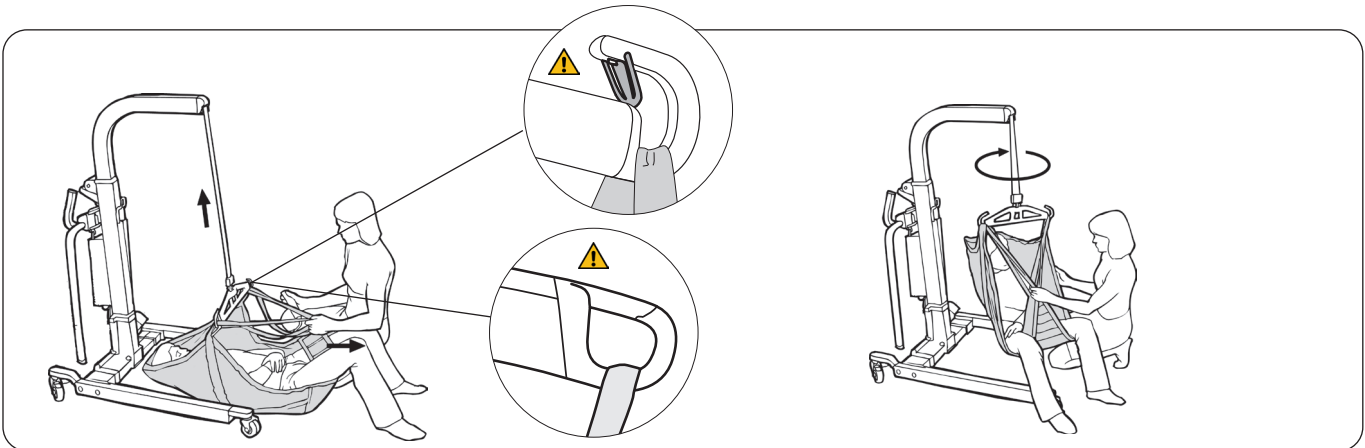


Smooth out the leg supports under the patient's legs. Keep the leg supports away from the groin to prevent discomfort to the patient. This is facilitated if the patient bends the legs. See page 6-7 for instructions on how to attach the sling loops to different sling bars.

Lift the patient from the floor. The patient can be rotated slightly to avoid their head getting too close to the mast.

**⚠ The mobile lift should be locked when lifting with the patient's head against the lift.**

## Lifting to the Floor

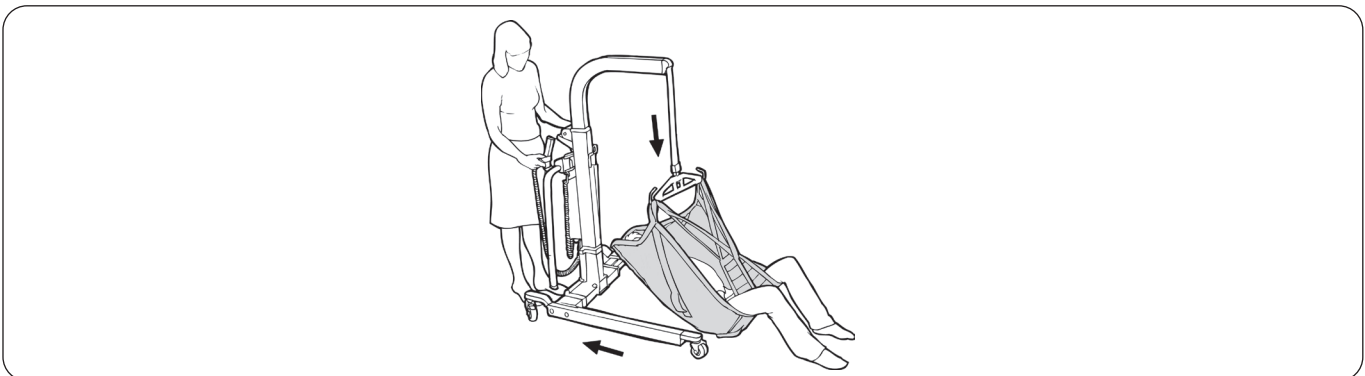


When lifting to the floor, the wheels of the lift should not be locked. When the patient touches the floor, the lift should be pulled backwards to give space for the head. **Tip:** If lifting is to a thinner gym mat, the lift can be pushed in under this mat, so that the mat acts as a soft protection between the lift and the patient. If the sling is to be removed, see "Removing the Sling when the Patient is Lying in Bed", page 10.

## Optional Components


Extension Loop, green, 12 cm (5 in)  
Extension Loop, green, 22 cm (9 in)  
Extension Loop, grey, 12 cm (5 in)  
Extension Loop, grey, 22 cm (9 in)

Prod. No. 3691102  
Prod. No. 3691103  
Prod. No. 3691302  
Prod. No. 3691303

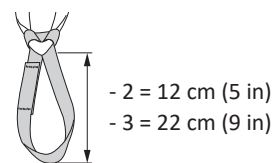

Padded Case for Leg Support

Prod. No. 36660502  
Prod. No. 36660503  
Prod. No. 36660504

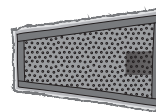


# Overview LIKO Original HighBack Sling, Mod 200, 210

Product	Prod. No.	Size		Patient's Weight <sup>1</sup>	Max. Load <sup>2</sup>
Original HighBack Sling	35200 -- 3	XS	Extra Small	15-20 kg (33-44 lbs)	200 kg (440 lbs)
Original HighBack Sling	35200 -- 4	S	Small (child)	20-30 kg (44-66 lbs)	200 kg (440 lbs)
Original HighBack Sling	35210 -- 5	MS	Medium Slim (slim adult)	30-50 kg (66-110 lbs)	200 kg (440 lbs)
Original HighBack Sling	35200 -- 5	M	Medium	40-80 kg (88-176 lbs)	200 kg (440 lbs)
Original HighBack Sling	35200 -- 6	L	Large	70-120 kg (154-264 lbs)	200 kg (440 lbs)
Original HighBack Sling	35200 -- 7	XL	Extra Large	110-250 kg (242-550 lbs)	500 kg (1,100 lbs)
Original HighBack Sling	35200 -- 8	XXL	Extra Extra Large	200-500 kg (440-1100 lbs)	500 kg (1,100 lbs)

<sup>1</sup> The indicated patient weight is a guideline only – there may be deviations.

<sup>2</sup> To maintain maximum load, lift/accessories intended for the same load or more are required.

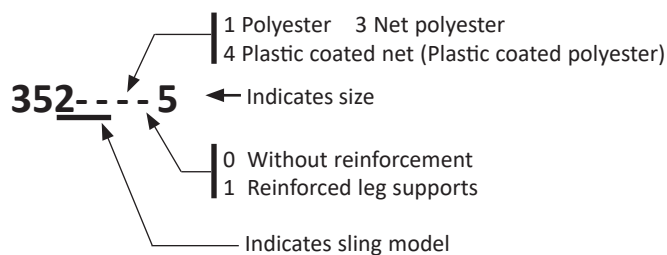

## Recommended Combinations

**Recommended combinations of the Liko Original HighBack Sling applied with crossed leg supports and Liko's sling bars:**

	Mini 220	Universal SlingBar 350	Universal SlingBar 450	Standard 450	SlingGuard 450	Universal Slingbar 600	SlingGuard 670 Twin	Universal Twin bar 670	Sling Cross-bar 450	Sling Cross-bar 670	Univ. SideBars + Universal 350	SideBars + Standard 450	Univ. SideBars + Universal 450
Original Highback Sling, XS	✓	✓	✓	✓	✓		✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, S	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, MS	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, M	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, L	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, XL*		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Original Highback Sling, XXL*			✓		✓	✓	✓	✓		✓		✓	✓

\* XL & XXL Slings to be used in combination with Ultra Lift System, Refer to 7\*\*111103 Instructions for Use.

While this chart serves as guidance, it is important to acknowledge that various circumstances may render Sling- Sling bar combinations inappropriate. Approved combinations in the matrix must still be assessed and validated by a trained health care professional.

**Explanation:** ✓ = Recommended

### Other Combinations

⚠ Combinations of accessories/products other than those recommended by Liko can result in risks for the safety of the patient. Injury could occur.

Golvo, Hillrom, Liko, LikoGuard, Likorall and Viking are trademarks of Baxter International Inc. or its subsidiaries



[www.hillrom.com](http://www.hillrom.com)

Liko AB  
Nedre vägen 100  
975 92 Luleå, Sweden  
+46 (0)920 474700

Liko AB is a subsidiary of Hill-Rom Holdings inc.

Enhancing outcomes for patients and their caregivers:

